PRODUCT CARE

Hand wash with mild detergent and water below 30°C without wringing.

Dry in shade on a flat surface.

Wipe the plastic parts with moist lint free cloth and mild detergent

Do not dry-clean.

Do not iron.

Do not bleach.

Close hook loop fasteners before washing.

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How to attach De-rotation Bar, when required

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3. 
 - Place the de-rotation splint against the PF splint while aligning the holes.

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4. 
 - Place the screw from inside and fasten the nut on outer side. Repeat same for fastening other side screw. Tighten both the screws securely.

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2.

- Thread the Leg through the straps. Ensure that the heel rests properly. Tighten the ankle strap along with adjusting the cushion pad.

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English

INTENDED USE :

Tynor PF Night Splint is designed for rehabilitation after injury, fracture, sprains or surgery of foot, ankle or lower leg to prevent plantar flexion, when required. The rigid casing provides support to lift the ankle, constraining the plantar flexion, enabling prevention of conditions like foot drop or contractures on prolonged bed rest. De-rotation Bar can be used for prevention of rotation post femur fracture.

CONSTRUCTION:

Molded foot casing is durable and aesthetically pleasing, it ensures full protection to ankle & foot and maintains the ankle joint in the neutral position. Liner provides effective cushioning for better grip of product. It reduces strapping pressure and discomfort. Extra Pad set can increase degree of dorsiflexion. Bilateral symmetric design allows use for either foot. Can be used as a night splint for early healing and can also be used as a day splint in foot drop. Simple Velcro pull mechanism allows customization of degree of dorsiflexion.

APPLICATIONS:

- Prevention and correction of foot drop contracture.
- Plantar Fasciitis
- Achilles tendonitis
- Prevention of Achilles tendon contracture during prolonged bed rest
- Night splint for rehabilitation.
- Ambulatory, can be used as a day splint
- Perfect post operative immobilization and de-rotation
- Peroneal, pre tibial nerve or muscle damage

CAUTION/RECOMMENDATION : Not for use on patients with circulatory or sensory problems.

INSTRUCTIONS:

- Use this product under strict guidance of a qualified doctor.
- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
 - Pain increases or persists
- Adjust padding and strapping for secure and comfortable fitting.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.