

PRODUCT CARE



Hand wash with mild detergent and water below 30°C without wringing.



Dry in shade on a flat surface.



Do not dry-clean.



Do not Iron.



Do not bleach.

EC REP

MT Promed Consulting GmbH
Altenhofstrasse 80
D-66386 St. Ingbert
Germany
tel.: +49 (0) 6894 581020
fax: +49 (0) 6894 581021
e-mail: info@mt-procons.com
www.mt-procons.com

Symbol Explanation :



Batch code



Manufacturing date



Manufacturer



Catalogue number



Tynor Orthotics Pvt. Ltd.

169-170, Sec 82, Ind. Area, JLPL, Mohali-160 055, (Pb) India
e-mail: marketing@tynorindia.com • www.tynorindia.com
Customer Care : 0091-172-5288111

ISO
9001



W.H.O.
G.M.P.

ISO
13485



US FDA
registered



Patellar support
पैटेलर सपोर्ट
Soporte patelar
Support Rotulien
داعم الرضفة



UM-D19-18-2

REF D 19

www.tynorindia.com

USER MANUAL

How To Use
Cómo Utilizar



कैसे उपयोग करें
Comment utiliser:
كيفية الاستخدام

1.



- Patellar support is worn just below the patella on the patellar tendon, the patella needs to be centered in the silicon pad.
- पटेलर सपोर्ट पटेली की पेशी पर पटेली के ठीक नीचे पहना जाता है, पटेली सिलिकॉन पैड के बीचों-बीच होना चाहिए।
- El soporte patelar se lleva justo debajo de la patela en el tendón patelar; es necesario que la patela esté centrada en la almohadilla de silicona.
- Le support rotulien est porté juste en dessous de la rotule sur le tendon patellaire, la rotule doit être centrée dans le coussinet en silicone.

• یلئیس داعم الرضفة تحديدا أسفل الرضفي على وتر الرضفة، حيث تكون الرضفة مركزة وسط وسادة السيليكون.

2.



- The strap is passed through the buckle for reverse buckling.
- रिवर्स बकलिंग के लिए पट्टी बकल से निकालें।
- La correa se pasa a través de la hebilla para abrocharse de forma inversa.
- La sangle passe à travers la boucle pour le voilement inverse.

• يتم تمرير الحزام عبر الإبزيم ويربط بالاتواء العكسي

How To Use
Cómo Utilizar



कैसे उपयोग करें
Comment utiliser:
كيفية الاستخدام

3.



- Hook Loop closures are closed to a comfortable compression.
- स्विधाजनक दबाव के लिए हुक लूप क्लोजर बंद होते हैं।
- Los cierres de Velcro se cierran para ofrecer una compresión cómoda.
- Les crochets de fermeture de boucle sont fermés à une compression confortable

• يتم إغلاق إبريم الربط هوک لوک من أجل ضغط مريح

Patellar support

English

INTENDED USE :

Patellar Support is designed to lift the patella and alleviate symptoms of pain, inflammation and discomfort associated with knee degeneration, without restricting circulation.

CONSTRUCTION:

The patellar strap is lined with a plush fabric on the inside to maximize skin comfort. The exterior is lined with a jersey fabric for optimum compression and durability.

The anatomic silicone pad ensures the optimal pressure to the patellar tendon, good grip of the product to the body and absorption of vibration. The hook and loop fastening allows for customized pressure. The pad is dimpled to provide enhanced proprioception to reinforce muscular support to the knee joint.

APPLICATIONS:

- Compresses patellar tendon stabilizes patella and offload patella tendon
- Absorption of vibrations (prevention of the effects of patellar tendinitis, jumper knee)
- Femoropatellar syndrome
- Osgood-Schlatter disease
- Patellar Chondromalacia
- Arthritis, wear and tear of aging
- Enhanced Proprioception reinforces muscular support to knee joint
- Patella subluxation and patella dislocation

CAUTION/RECOMMENDATION :

- Use this product under strict guidance of a qualified doctor.
- Make sure your skin is clean and dry before applying product.
- Avoid over tightening, it may restrict blood flow, compress nerve endings, causing swelling, numbness and redness.

INSTRUCTIONS:

- Loosen in case of impaired sensation or impaired blood flow.
- Discontinue use and seek guidance of a qualified doctor, in case of
 - Allergy rash or itching
 - Pain increases or persists
- Consult your doctor in case of concurrent use with any other medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.