PRODUCT CARE

Hand wash with mild detergent and water below 30°C without wringing.

Close hook loop fasteners before washing.

Do not dry-clean.

Dry in shade on a flat surface.

Do not iron.

Do not bleach.

EC REP

MT Promed Consulting GmbH
Altenhofstrasse 80
D-66386 St. Ingbert
Germany
tel.: +49 (0) 6894 581020
fax: +49 (0) 6894 581021
e-mail: info@mt-procons.com
www.mt-procons.com

Symbol Explanation :

Batch code

Manufacturing date

Manufacturer

Catalogue number

Manufactured by:

Tynor Orthotics Pvt. Ltd.

169-170, Sec 82, JLPL Ind. Area, Mahali - 160 055 (Pb) India
Consumer Care Cell: 1800-1212-656
E-mail: care@tynor.in or write to manufacturer
www.tynorindia.com

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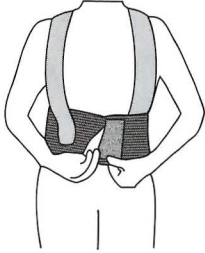
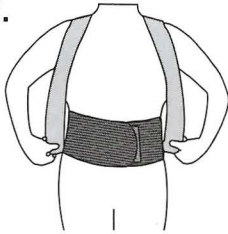
English

INTRODUCTION:

Tynor Posture Corrector is an effective device to maintain correct posture during daily activities and long sitting by applying a correctional force. Customizable design and elasticity of materials ensure optimal correctional force and higher comfort.

CONSTRUCTION:

The PE sheet panel at the back reinforce the back to the correct position. Fabric on the inside is soft and cushiony yet durable. The sleek design ensures good ventilation and user comfort. Carefully chosen elastic bands generate the necessary force for posture correction on one hand and comfortable user experience on the other.

APPLICATIONS:

- Bad Posture
- Posture care for long and demanding work
- Stooping shoulders
- Posture related back pain
- Preventive care in Obesity and Old age

RECOMMENDATIONS:

- For effective use, ensure strong, snug and comfortable fit.

INSTRUCTIONS:

- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
 - Pain increases or persists
- Consult your doctor in case of concurrent use with another medical device
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.

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