

Arch Support Semi Pro
Code- K - 15

Introduction: Arch Support is designed to stretch the plantar fascia ligament & maintain the foot arches. It is anatomically designed to provide relief from arch pains, distribute the pressure on the foot evenly and naturally. It gets placed inside the shoe for effective results.

Construction: Molded out of advanced elastomer which provides optimum support to the arch. It offers unique advantages like, nice cushioning, odorless, non-compressible, is washable. Can flex as you walk. It remains in place at all times due to its unique adhesive surface.

Applications

- **Flat foot / Splay foot**
- **Fallen medial longitudinal arch**
- **Deformed foot or Poor foot biomechanics**
- **Managing pain and disability related to plantar fasciitis**
- **Diabetic foot**
- **Rheumatoid arthritis of the mid foot**
- **Distorted gait**
- **Mid foot pain**

Instructions:

- Use this product under strict guidance of a qualified doctor.
- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
 - Pain increases or persists
- Consult your doctor in case of concurrent use of another medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.

Recommendation:

- Preferably use with a slightly deep or an oversized shoe.
- Accurate placement within the shoe is extremely important.
- Initiate therapy using this semiprofessional product, after a few months shift to the customized professional product for getting 100% results.

Product Care:

1. Wipe with a clean moist lint free cloth.
2. Dry them properly before wearing and after cleaning.

EC Rep:

MT Promedt Consulting GmbH
Altenhofstrasse 80
D-66386 St. Ingbert
Germany
tel.: +49 (0) 6894 581020
e-mail: info@mt-procons.com
www.mt-procons.com

Bullet Points:

- Odorless
- Transparent elastomer
- Easy stick on application
- Aerodynamic & anatomical shape

Contents: Elastomer

Size	European	American	MM
S	29 - 34	11.0 - 3.0	185 - 216
M	34 - 39	3.0 - 7.0	216 - 247
L	39 - 43	7.0 - 10.5	247 - 278

Measure Shoe Size

How to Use:

1. Pick the right size of the product, using the biometric chart provided on the box.
2. Slide the product in the shoe without peeling the release paper and position it accurately, match it with the arch of the foot.
3. Mark this position on the shoe.
4. Remove the release paper and affix the Arch Support on the marked position.
5. Put the feet inside the shoe and ensure that Arch Support is accurately placed.