

PRODUCT CARE



Hand wash with mild detergent and water below 30°C without wringing.



Dry in shade on a flat surface.



Do not dry-clean.



Do not iron.



Do not bleach.



Close hook loop fasteners before washing.

EC REP

MT Promed Consulting GmbH
Altenhofstrasse 80
D-66386 St. Ingbert
Germany
tel.: +49 (0) 6894 581020
fax: +49 (0) 6894 581021
e-mail: info@mt-procons.com
www.mt-procons.com

Symbol Explanation :

LOT

Batch code



Manufacturing date



Manufacturer

REF

Catalogue number



Tynor Orthotics Pvt. Ltd.

169-170, Sec 82, Ind. Area, JLPL, Mohali-160 055, (Pb) India
e-mail: marketing@tynorindia.com • www.tynorindia.com
Customer Care : 0091-172-5288111

ISO
9001



W.H.O.
G.M.P.

ISO
13485



US FDA
registered

www.tynorindia.com



Shoulder Support

शोल्डर सपोर्ट

Épaulière

Soporte De Hombro

دعم الكتف

REF J 14



UM-J14-18-1

USER MANUAL

How To Use
Cómo Utilizar




कैसे उपयोग करें
Comment utiliser
كيفية الاستخدام

1.



- Pull the sleeve up over the arm and shoulder.
- भुजा और कंधे से स्लीव को ऊपर ले जाएं।
- Tirar de la manga por encima del brazo y hombro.
- Tirez la manche par-dessus le bras et l'épaule.
- سحب الأكمام على مدى الذراع والكتف

2.

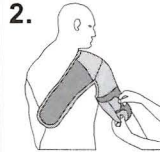

- Close the sleeve hook and loop closures to a comfortable fit.
- स्लीव के हुक और लूप क्लोजर को सुविधाजनक प्रकार से कसते हुए लगाएं।
- Cerrar la cremallera de la manga y ajustar los cierres.
- Fermez les fermetures velcro de la manche pour un ajustement confortable.
- إغلاق رباط الأكمام وكذا الحلقة لوضعية مريحة

How To Use
Cómo Utilizar



कैसे उपयोग करें
Comment utiliser
كيفية الاستخدام

3.



- Apply the strap under the opposite arm and close it in the front.
- दूसरी भुजा में स्ट्रैप लगाएं और आगे की ओर से उसे कस दें।
- Pasar la correa por el brazo opuesto y cerrarla en frente.
- Appliquez la sangle sous le bras opposé et fermez-la à l'avant.
- تطبیق الحزام تحت الذراع المعاكس وإغلاقه إلى الأمام

Shoulder Support

English

Shoulder Support is designed to provide protection and support to the injured or ailing shoulder joint, It keeps the shoulder joint warm and flexible, and aids in quick healing.

CONSTRUCTION:

It is designed anatomically to apply uniform and controlled compression around the 'ball' of the shoulder (Deltoid and Rotator cuff region). It is made out of superior quality Neoprene fabric with nylon lining. The design allows for quick adjustment, controlled pressure, easy application and removal.

APPLICATIONS:

- Post operative management of shoulder dislocation
- Rehabilitation care post fracture of neck of the Humerus
- Arthritis, Bursitis and Tendonitis
- Rehabilitation post shoulder dislocations
- Shoulder inflammation, shoulder pain, Mild shoulder instabilities
- Provides protection, warmth, compression support and flexibility to the shoulder
- Minor sprains, strains and arthritic pain of the shoulder joint
- Ideal for the healing and prevention of sporting injuries



CAUTION/RECOMMENDATION : Fits either left or right arm

INSTRUCTIONS

- Use this product under strict guidance of a qualified doctor.
- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
 - Pain increases or persists
- Consult your doctor in case of concurrent use with any other medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.