English

**POLICY FOR DEFECTIVE GOODS:**

All products manufactured by Tynor Orthotics (P) Limited carry six months warranty against material or manufacturing defect. Defective units must be returned to Tynor Orthotics (P) Limited through the Super Dealer concerned for replacement, along with duly filled " Product Return Form" . Liability of Tynor Orthotics (P) Limited shall be limited to the cost of product only .

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**PRODUCT CARE**

Hand wash with mild detergent and water below 30°C without wringing.

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### Tennis Elbow Support

#### English

Tennis Elbow Support is designed to help provide relief from generalized pain and tenderness in the forearm and elbow caused by repetitive strain injury due to strong grip or active finger movement. Its quite often used by tennis players, golfers, racquet players, industrial workers etc.

#### CONSTRUCTION:

It is made from three layered PU bonded fabric incorporating anatomically contoured silicone pad surrounding the epicondyle silicone pad provide an intermittent compression to the soft tissue of the joint, leading to increased circulation, thus aiding in the resorption of swelling and edema. It also absorbs any vibrations reaching the elbow joint and the tissue around it.

#### APPLICATIONS:

- Osteoarthritis
- Post elbow Arthroscopy
- Chronic swelling and edema
- Pre and Post-operative elbow conditions
- Medial Epicondylitis (golfer's elbow)
- Epicondylitis (tennis elbow)
- Elbow strains or sprains
- Targeted relief from forearm pain

#### INSTRUCTIONS:

- Use this product on recommendation of a doctor.
- Discontinue use and seek guidance of a qualified doctor, in case of
  - Allergy rash or itching
  - Pain increases or persists
- Consult your doctor in case of concurrent use of another medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.