

PRODUCT CARE



Hand wash with mild detergent and water below 30°C without wringing.



Dry in shade on a flat surface.



Do not dry-clean.



Do not Iron.



Do not bleach.

EC REF

MT Promed Consulting GmbH
Altenhofstrasse 80
D-66386 St. Ingbert
Germany
tel.: +49 (0) 6894 581020
fax: +49 (0) 6894 581021
e-mail: info@mt-procons.com
www.mt-procons.com

Symbol Explanation :



Batch code



Manufacturing date



Manufacturer



Catalogue number



Tynor Orthotics Pvt. Ltd.

169-170, Sec 82, Ind. Area, JLPL, Mohali-160 055, (Pb) India
e-mail: marketing@tynorindia.com • www.tynorindia.com
Customer Care : 0091-172-5288111

ISO
9001



W.H.O.
G.M.P.

ISO
13485



US FDA
registered

UM-D14-1B2




High Support

थाई सपोर्ट

Apoyo Para Las Piernas

Support de cuisse

الدعم الفخذ

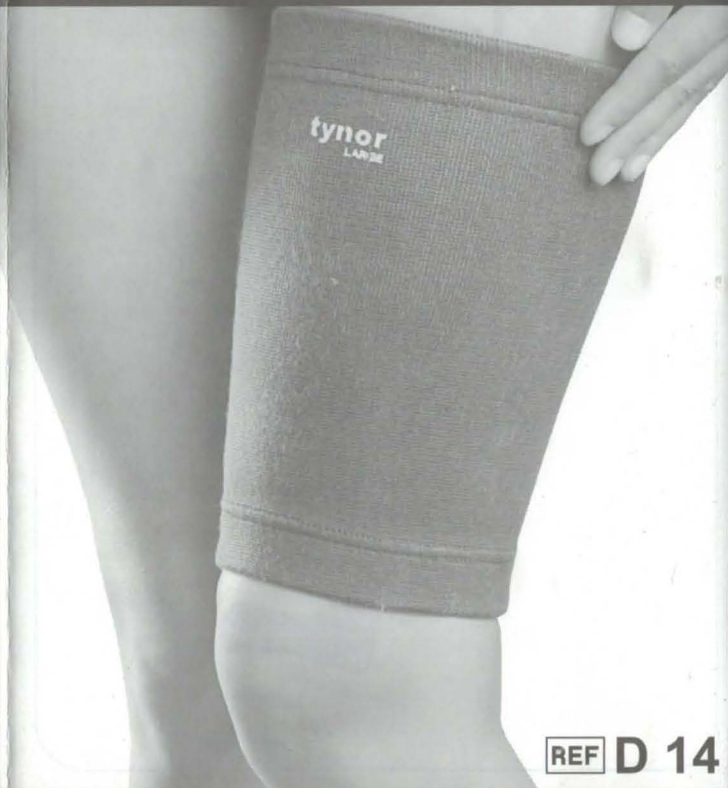

REF D 14

www.tynorindia.com

USER MANUAL

Thigh Support

English

INTENDED USE :

Thigh support is used in orthopedics, trauma and burn management to provide firm compression, warmth & support to the thigh area, to allay pain and inflammation, generally associated with old age, arthritis, sports injury etc.

CONSTRUCTION:

Woven on the state of art circular looms, from high quality, nylon and spandex yarn, to get a four way stretching seamless tube tapered from top to bottom. Elastic thread runs spirally in the matrix of nylon locks to ensure its free movement to allow uniform compression.

APPLICATIONS :

- Gives mild support and immobilization to quadriceps and hamstrings
- Helps reduce inflammation, tenderness and relieves stiffness
- Gives firm compression and therapeutic warmth to inflamed thigh
- Offers protection against further injury while still permitting continued activity
- Psychological support to geriatric patients
- Sports injury sprains and strains
- Post-surgical rehabilitation
- Burn care

RECOMMENDATION :

- Use this product under strict guidance of a qualified doctor.


INSTRUCTIONS:

- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
 - Pain increases or persists
- Consult your doctor in case of concurrent use with any other medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.

How To Use
Cómo Utilizar




कैसे उपयोग करें
Comment utiliser
كيفية الاستخدام

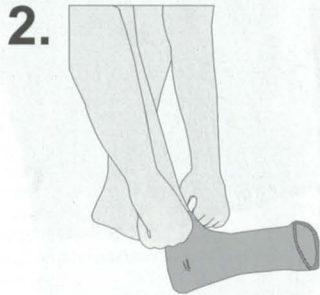

How To Use
Cómo Utilizar



कैसे उपयोग करें
Comment utiliser
كيفية الاستخدام

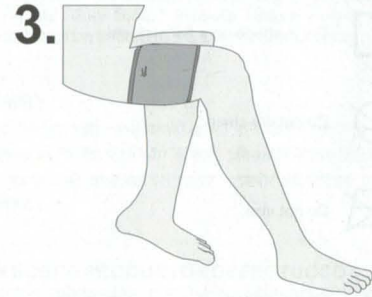

- Simple pull on application like socks.
- उत्पाद को मोड़ने की तरह खींच कर जंग पर पहनें।
- Tracción simple en aplicación como calcetines.
- Tirez simplement lors de l'application comme des chaussettes

• سحب بسيطة على تطبيق مثل الجوارب.