## PRODUCT CARE

Hand wash with mild detergent and water below 30°C without wringing.

Dry in shade on a flat surface.

Do not dry-clean.

Do not Iron.

Do not bleach.

Close hook loop fasteners before washing.

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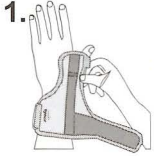
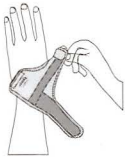
English

### INTENDED USE :

**Thumb spica splint** is a convenient device designed to immobilize the thumb in the abduction (functionally neutral) position. Support the CMC joint and immobilize the MCP joint of the thumb without inhibiting hand movement.

### CONSTRUCTION:

Made out of flexible breathable three layered fabric with sandwiched PU foam contoured to provide close fitting over the thumb.

Semi rigid, removable malleable radial splint supports the thumb in abduction and immobilize the Carpo metacarpo and metacarpal joints of the thumb.

Velcro closures ensure a quick and easy application and removal, of the product and close contoured fitting of the splint around the thumb.

### APPLICATIONS:

- DeQuervain's syndrome
- Ideal for arthritis, skier's thumb and game-keeper's thumb
- Helps reduce pain and inflammation associated with cumulative trauma injuries
- Fracture or suspected fracture of the scaphoid bone in the wrist and soft tissue injuries of the thumb and the wrist
- Treating thumb sprains and strains
- Post cast healing
- Post burn injuries
- Prophylactic protection during certain sports and activities

### RECOMMENDATION:

- Use this product under strict guidance of a qualified doctor.

### INSTRUCTIONS:

- Discontinue use and seek guidance of a qualified doctor, in case of
  - Impaired sensation
  - Impaired blood flow
  - Allergy rash or itching
  - Pain increases or persists
- Consult your doctor in case of concurrent use with any other medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.

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