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POLICY FOR DEFECTIVE GOODS:

All products manufactured by Tynor Orthotics (P) Limited carry six months warranty against material or manufacturing defect. Defective units must be returned to Tynor Orthotics (P) Limited through the Super Dealer concerned for replacement, along with duly filled "Product Return Form". Liability of Tynor Orthotics (P) Limited shall be limited to the cost of product only.

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PRODUCT CARE

Hand wash with mild detergent and water

Air dry in shade

Dust with talc periodically

MT Promed Consulting GmbH
Altenhofstrasse 80
D-66386 St. Ingbert
Germany
tel.: +49 (0) 6894 581020
fax: +49 (0) 6894 581021
e-mail: info@mt-procons.com
www.mt-procons.com

Symbol Explanation :

- LOT Batch code
- Manufacturing date
- Manufacturer
- REF Catalogue number

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Toe Separator (Silicone)

English

Toe Separator (Silicone) is anatomically designed to spread toes, absorb pressure and help with their alignment. Friction between overlapping and pinched toes can cause discomfort, irritation and corns. Toe separator aligns the big toe relieves bunion discomfort, pain and soreness associated with soft corns or toes rubbing together.

CONSTRUCTION:

Made out of non-compressible silicon. The silicone material resists bacterial growth and does not dry out. Silicone is non cellular, has longer functional life, better cushioning coefficient, and hypoallergenic, non toxic, odorless and is washable. Anatomically shaped, so that it can be comfortably placed in between toes.

APPLICATIONS:

- Helps align toes
- Position the great toe
- Pain caused by bunion, callus or corns
- Crooked, pinched toes or overlapping toes
- Correction following bunion surgery
- Nail problems
- Hallux Valgus
- Between-toe friction and chafing

INSTRUCTIONS:

- Use this product under strict guidance of a qualified doctor.
- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
 - Pain increases or persists
- Consult your doctor in case of concurrent use with any other medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children & Pets.
- Do not place product on an open wound.

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