

PRODUCT CARE



Hand wash with mild detergent and water below 30°C without wringing.



Dry in shade on a flat surface.



Do not dry-clean.



Do not Iron.



Do not bleach.



Close hook loop fasteners before washing.

EC REP

MT Promed Consulting GmbH
Altenhofstrasse 80
D-66386 St. Ingbert
Germany
tel.: +49 (0) 6894 581020
fax: +49 (0) 6894 581021
e-mail: info@mt-procons.com
www.mt-procons.com

Symbol Explanation :



Batch code



Manufacturing date



Manufacturer



Catalogue number



Tynor Orthotics Pvt. Ltd.

169-170, Sec 82, Ind. Area, JIPL, Mohali-160 055, (Pb) India
e-mail: marketing@tynorindia.com • www.tynorindia.com
Customer Care : 0091-172-5288111

ISO
9001



W.H.O.
G.M.P.

ISO
13485



US FDA
registered

www.tynorindia.com

tynor

Tummy Trimmer/Abdominal Belt 8"

टमी ट्रिंमर / एबडॉमिनल बेल्ट 8"

Reductor De Pansa/Cinturón Abdominal 8"

أداة شد البطن / حزام البطن 8"

REF A 03



UM-A03-17-1

USER MANUAL

How To Use Cómo Utilizar



कैसे उपयोग करें كيفية الاستخدام



- The belt is wrapped around the waist.
- बेल्ट को कमर के चारों ओर से लपेटा जाता है ।
- Se envuelve el cinturón en torno a la cintura.
- يتم لف الحزام حول الخصر



- Keep it upright and in correct position as per directions of "up inside label".
- इसे सीधा रखते हुए और ठीक स्थिति में "अप इनसाइड लेबल" के निर्देशों के अनुसार लपेटी जाती है ।
- Manteniéndolo derecho y en posición correcta según las instrucciones de la "etiqueta interna".
- وإبقائه في وضع عمودي وصحيح حسب اتجاهات "البطاقة بالداخل".



- Lower edge of the belt should coincide with the base of the spine.
- बेल्ट का निचला किनारा रीढ़ के आधार से मेल खाना चाहिए ।
- El borde inferior del cinturón debería coincidir con la base de la columna.
- وينبغي أن تتوافق الحافة السفلى للحزام مع قاعدة العمود الفقري.



- Apply hook tape to the loop tape to a comfortable compression.
- हुक पट्टे को फीते के फंदे पर आरामदायक दबाव के साथ प्रयोग करें ।
- Ajuste el gancho de cierre para una compresión confortable.
- يجب وضع الشريط الخطافي على شريط في الحلقة للحصول على ضغط مريح.

Tummy Trimmer/ Abdominal Belt 8"

English

INTENDED USE :

Tummy Trimmer/Abdominal belt supports and compresses the abdominal muscles for waist line reduction and is used as a post operative or post delivery aid.

CONSTRUCTION:

Made from extra strong highly porous orthopedic webbing containing heat resistant rubber with high modulus of elasticity. Hook loop closures provide perfect fitting of the belt around the waist, controlled compression, its quick, easy application and removal. Flexi Nylon splints ensure no wrinkles, no buckling and no rolling over of the belt.

APPLICATIONS:

- To protect and bind surgical incisions of the abdomen and the chest.
- Postoperative care of thorax and abdomen.
- Reduce the discomfort of sudden rib cage expansion from deep breaths, cough or sneezing during recovery.
- Used as a general tummy trimmer, it compresses the abdominal wall to give the wearer a coveted slim figure.
- For post delivery mothers, to tone up the mellowed abdominal muscles and set the waistline to its Pre-pregnancy dimensions.
- In umbilical and ventral hernia.

RECOMMENDATION :

- Use this product on recommendation of a doctor.

INSTRUCTIONS:

- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
- Consult your doctor in case of concurrent use of the device with excessive physical exertion, it can lead to restriction of performance of heart and lungs (risk of rise in blood pressure).
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.