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English

POLICY FOR DEFECTIVE GOODS:

All products manufactured by Tynor Orthotics (P) Limited carry six months warranty against material or manufacturing defect. Defective units must be returned to Tynor Orthotics (P) Limited through the Super Dealer concerned for replacement, along with duly filled " Product Return Form" . Liability of Tynor Orthotics (P) Limited shall be limited to the cost of product only .

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PRODUCT CARE

Hand wash with mild detergent and water

Air dry in shade

Dust with talc periodically

EC REP

MT Promed Consulting GmbH
Altenhofstrasse 80
D-66386 St. Ingbert
Germany
tel.: +49 (0) 6894 581020
fax: +49 (0) 6894 581021
e-mail: info@mt-procons.com
www.mt-procons.com

Symbol Explanation :

- LOT Batch code
- Manufacturing date
- Manufacturer
- REF Catalogue number

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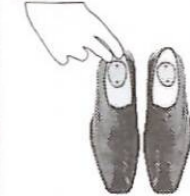
English

INTENDED USE :

Heel Cup/Cushion are designed to provide soft cushioning to the foot heel. Gel spot at the site of calcaneus reduces/distributes pressure and relieves pain.

CONSTRUCTION:

It is made out of non-compressible silicone. Designed with super soft zones in the heel. Silicon resists bacterial growth, has longer functional life, better cushioning coefficient, is hypoallergenic.

While Heel Cup is thin and sleek, so fits easily into a normal shoe. The honey comb structure at the base of the Heel Cushion further enhances the degree of cushioning.

APPLICATIONS:

- Diabetic foot, general heel pain.
- Calcaneal spur, heel pain.
- Achilles heel.
- Plantar fasciitis.
- Vascular Claudication.
- Diminished sensation.
- Prophylactic use in sports.
- Active geriatrics.
- Indirect relief in heel, joints, knee and back pain through shock absorption.

RECOMMENDATION:

- Use this product under strict guidance of a qualified doctor.

INSTRUCTIONS:

- Discontinue use and seek guidance of a qualified doctor, in case of
 - Allergy rash or itching
 - Pain increases or persists
- Keep out of reach of children.

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