PRODUCT CARE

Hand wash with mild detergent and water below 30°C without wringing.

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Universal Shoulder Immobilizer

English

Universal Shoulder immobilizer is one of the most effective designs to immobilize and support the dislocated shoulder. This is also used in the fractures of clavicle bone and the shoulder joint where immobilization of the region is indispensable.

CONSTRUCTION:

Both pouch and arm restraint band are made out of three layered PU bonded fabric, each layer imparting to the fabric its unique properties. Outer layer with raising provides soft feel and plush looks. Middle layer imparts cushioning effect and a wrinkle free property to the fabric. Inner most layer gives the fabric its required strength and non tear ability.

The pouch holds and provides a comfortable rest to the arm. Arm Restraint Band designed with a large loop area ensures rigid immobilization of arm and the shoulder joint on one hand and ensures that one size can fit diverse body sizes. Unique design with reverse buckle mechanism makes it easy to apply and one size can fit diverse body sizes.

APPLICATIONS:

- Shoulder dislocation
- Fractures of the shoulder joint
- Fractures of clavicle bone, humerus or scapula
- Post surgical care of the shoulder area

CAUTION/RECOMMENDATION:

Use other design in case of open wounds or nervous injury in the shoulder area.

INSTRUCTIONS:

- Use this product under strict guidance of a qualified doctor.
- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
 - Pain increases or persists
- Consult your doctor in case of concurrent use with any other medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.
- Adjust padding and strapping for a secure and comfortable fitting.