## PRODUCT CARE

Hand wash with mild detergent and water below 30°C without wringing.

Dry in shade on a flat surface.

Do not dry-clean.

Do not Iron.

Do not bleach.

Close hook loop fasteners before washing.

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English

### INTENDED USE :

Tynor Wrist Splint with thumb is an ideal brace for conditions where immobilization of wrist and thumb is required. Wrist can be maintained in the dorsiflexion and thumb in abduction.

### CONSTRUCTION:

Wrist Splint with Thumb is made from a strong three layered bonded fabric with the middle layer made of open celled foam to ensure good cushioning. The rigid support is provided by aluminum splints contained in Nylon pockets. Both the splints can be customized to the individual's needs precisely, thus ensuring better immobilization and improved comfort.

Lace design ensures easy application and an intimate fit. Velcro fasteners with reverse buckle mechanism ensure close and firm fitting of the brace around the wrist.

### APPLICATIONS:

- Wrist and Thumb injuries
- Injuries of the metacarpal
- Early cast removal
- Fracture management
- Post surgical protection and rehabilitation
- Management of arthritic conditions of the wrist
- Bursitis, Tendonitis, Tenosynovitis
- Psychological assurance
- Post extensor tendon repair surgery

### RECOMMENDATION:

- Use this product under strict guidance of a qualified doctor.
- Customize the aluminum splints for proper fitting, better immobilization and improved comfort. Degree of dorsiflexion of the wrist or the abduction of thumb can be altered as per the need.
- Can be used for the left or right wrist.

### INSTRUCTIONS: [1]

- Discontinue use and seek guidance of a qualified doctor, in case of
  - Allergy rash or itching
  - Pain increases or persists
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.
- Remove, customize and reinsert the splint for comfortable immobilization.
- Adjust padding and strapping for a secure and comfortable fitting.

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