PRODUCT CARE

Hand wash with mild detergent and water below 30°C without wringing.

Dry in shade on a flat surface.

Do not dry-clean.

Do not Iron.

Do not bleach.

Close hook loop fasteners before washing.

EC REP

MT Promed Consulting GmbH
Altenhofstrasse 80
D-66386 St. Ingbert
Germany
tel.: +49 (0) 6894 581020
fax: +49 (0) 6894 581021
e-mail: info@mt-procons.com
www.mt-procons.com

Symbol Explanation :

Batch code

Manufacturing date

Manufacturer

Catalogue number

Ref. Code: E03300520

tynor

Manufactured by:

Tynor Orthotics Pvt. Ltd.

169-170, Sec 82, JLPL Ind. Area, Mohali - 160 055 (Pb) India
Consumer Care Cell: 1800-1212-656
E-mail: care@tynor.in or write to manufacturer
www.tynorindia.com

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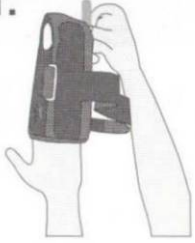
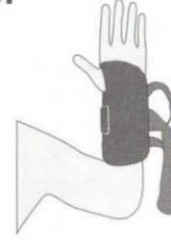
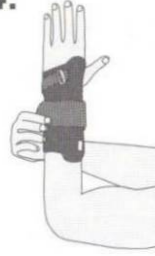
English

INTENDED USE :

Wrist and Forearm Splint is designed to immobilize and provide firm but comfortable support to hand and wrist in various orthopedic conditions. It maintains the wrist in the neutral/functional position.

CONSTRUCTION:

Wrist and Forearm splint is made from a strong three layered bonded fabric with the middle layer made of open celled foam to ensure good cushioning. The rigid support is provided by aluminum splints contained in a Nylon pocket. Splint can be customized to fit the individual patient precisely, which ensures better immobilization and improved comfort. Hook loop closure provides, snug fitting of the product, its easy application and removal.

Wrist and Forearm splint is designed to effectively maintain wrist in dorsi flexion or neutral position, while allowing full range of motion to fingers and thumb. It allows slight thumb abduction for its comfort. It compresses and retains the body heat to allay the local pain and inflammation. Wrist and Forearm splint can be removed to permit, therapy sessions, washing and hygiene.

APPLICATIONS:

- Sprains and strains
- Early cast removal
- Fracture management
- Post surgical protection and rehabilitation
- Rheumatic disease of the wrist and arthritic hand positioning
- Bursitis, Tendonitis, Tenosynovitis
- Soft tissue injury of the wrist (conservative, postoperative)
- Carpal tunnel syndrome
- Psychological assurance
- Post extensor tendon repair surgery
- Conservative management of Keinbock's disease

CAUTION/RECOMMENDATION :

- Use this product on recommendation of a doctor.
- Customize the aluminum splint for proper fitting, better immobilization and improved comfort.

INSTRUCTIONS:

- If pain increases or persists, discontinue use and consult your physician.
- Discontinue use in case of allergy rash or itching.
- Remove, customize and reinsert the splint for comfortable immobilisation.
- Adjust splinting and strapping for a secure and comfortable fitting.
- Follow instructions mentioned on the box carefully.
- Can use Ortho Stockinet as an under lining before wearing the product.
- Keep out of reach of children.

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