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PRODUCT CARE

Hand wash with mild detergent and water below 30°C without wringing.

Dry in shade on a flat surface.

Do not dry-clean.

Do not Iron.

Do not bleach.

Close hook loop fasteners before washing.

MT Promed Consulting GmbH
Altenhofstrasse 80
D-66386 St. Ingbert
Germany
tel.: +49 (0) 6894 581020
fax: +49 (0) 6894 581021
e-mail: info@mt-procons.com
www.mt-procons.com

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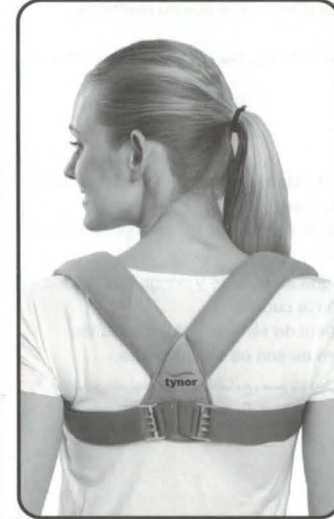
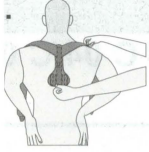
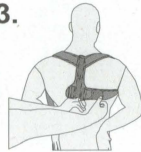
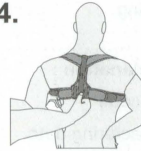
Clavicle Brace

English

Clavicle Brace is a convenient, scientifically designed brace, meant to immobilize, compress the clavicle bone. It ensures linear union and no telescoping of the fractured clavicle bone. It is used where neither neuro vascular disturbances are severe, nor surgery is required.

CONSTRUCTION:

Made from raised fabric with soft cushiony PU foam inside. The elastic nature of raw materials used, allow the brace to breathe with the body. It is constructed to provide lateral compression to the fractured clavicle bone, which gives a traction force which ensures linear union and no telescoping of the clavicle bone. Back pad forms a fulcrum for the four way pull, The splint in the back pad ensures no pinching of the sensitive axilla.

Adjustable straps and special stainless steel claw buckles help to get a very strong fitting of the brace around the clavicle region and prevent its loosening, in Tynor Clavicle brace with buckle design.

Reverse buckle mechanism and hook loop tapes ensure flexibility in positioning, strong fitting and no loosening of the brace in Tynor Clavicle Brace with Velcro design.

APPLICATIONS:

- Clavicle bone fractures.
- Post operative care of clavicular region to support the bandage & bind the incision.
- For curing the problem of stooping shoulders & hump back.

CAUTION/RECOMMENDATION :

- Clavicle brace should be used where neither neurovascular disturbances are severe, nor surgery is required.
- If pain persists or pain increases, look for over-tightening and consult your doctor.
- Rule out vaso constriction.

INSTRUCTIONS: [1]

- Use this product under strict guidance of a qualified doctor.
- Discontinue use and seek guidance of a qualified doctor, in case of
 - Impaired sensation
 - Impaired blood flow
 - Allergy rash or itching
 - Pain increases or persists
- Consult your doctor in case of concurrent use with any other medical device.
- Misuse or improper use of the device can lead to adverse effects and decreased effectiveness.
- Follow instructions mentioned on the box carefully.
- Keep out of reach of children.