

# TRILOC IMPLANT ANALOG 3D PRINTED MODEL INSTRUCTION

CreoDent.

TRiLOC



ANALOG®



1

Creodent will provide you with a testing Jig File. This Jig needs to be printed.



2

Place the Triloc Analog in the various fit locations and see which is the best fit.

3

Contact us again and let us know the result. We will load the correct Library so that you can correctly generate the 3D model in your CAD software.



4

Our Triloc Analogs are push and secure type system. You will need a scan body to securely push in the analog into the 3D printed model.



5

Use a screw driver to push out the analog on the bottom to remove it.



If you have any issues, please contact us.