



THUASNE

**LYMPHEDEMA
AND SCARS
TREATMENTS**



2021
CATALOG

Take care differently
with Thuasne solutions

In everything we do, we believe in challenging the status quo. We believe that we must think differently.

Our way of challenging the status quo is to manufacture our solutions by ensuring that they are **efficient, easy to use** and **comfortable**.

- 40 years of medical and technical expertise in compression devices
- 2 made-to-measure production sites
- Network of specialized fitters
- Expertise of our made-to-measure customer services
- Recognized brands
- Efficiency proven solutions
- Individualized solutions all along the patient's treatment

Quality standards



Our products comply with Oeko-Tex 100 standard⁽¹⁾. Thus Thuasne guaranty that materials used for the manufacturing of our products are safe pour the health and the environnement.





Lymphatrex Essential and Lymphatrex Expert are ASQUAL certified and thus comply with ASQUAL standards.



Lymphatrex is RAL certified and thus complies with RAL standards.

(1) The list of products certified Oeko-tex is available on demand.

LYPHHEDEMA TREATMENT	4
The lymphedema	5
The reduction phase	8
- MOBIDERM devices	10
- KIT BIFLEX	12
- Multi-type bandages	14
- Accessories	17
The maintenance phase	18
 Contain	22
- LYMPHATREX Essential	22
- VENO Elegance	26
- LYMPHATREX Expert	28
- LYMPHATREX	32
- Fitting and care instructions	36
 Mobilize	37
- MOBIDERM autofit	38
- MOBIDERM Made-to-Measure Patient	42
- MOBIDERM Standard	43
- Fitting and care instructions	44
SCARS TREATMENT	45
Cicatrex	46
Fitting and care instructions	52
Cicatrex Standard	53
Cicatrex Made-to-Measure Patient	56
Additional accessories	58
MEASUREMENT FORMS	60




USEFUL THINGS
TO KNOW




CARE
INSTRUCTIONS



STANDARD
DEVICE

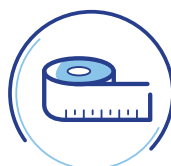

MADE-TO-MEASURE
PATIENT DEVICE

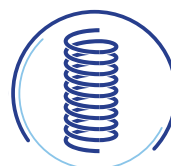

CIRCULAR
KNIT



FLAT
KNIT




FITTING
INSTRUCTIONS

Lymphatic system

The lymphatic system is made up of a series of canals called lymphatic vessels.

It helps to protect the body against diseases and infections. It is the most important part of the immune system.

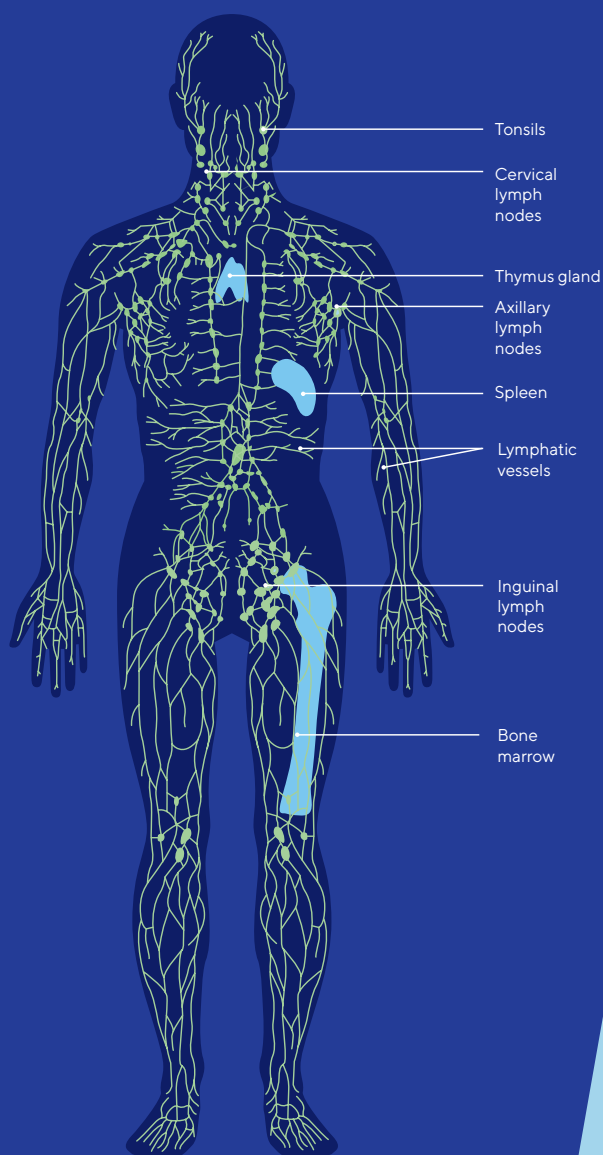

Lymphatic Circulation

The lymphatic vessels carry the lymph throughout the body.

Body movement and muscles contraction ensure circulation of the lymph in addition to contractile units within the lymphatic vessels.

Lymph is filtered through the lymph nodes and several organs to remove bacteria, viruses and other foreign substances.

Lymphedema

Lymphedema is a chronic disease related to a malfunction of the lymphatic system.⁽¹⁾

This results from a blockage or a slowing down of the circulation of this liquid between the cells and tissues of the body.

Lymph accumulated in the subcutaneous tissues causes swelling that **produces discomfort, heaviness and loss of mobility in the affected part of the body.**

Lymphedema Stages

There are **4 stages of lymphedema**.⁽¹⁾
Those stages can be grouped in **3 levels of severity**:

Mild

STAGE 0

- Swelling is hardly visible
- Symptoms are: heaviness of the affected limb, slight tingling and fatigue
- Takes several months or even years for the edema to appear

STAGE I

- Positive pitting sign possible
- Skin and tissue are not permanently damaged
- Reversible lymphedema
- Volume excess is below 20%

Moderate

STAGE II

- Positive pitting sign
- Skin folds, inflammations and cutaneous modifications
- Non reversible edema
- Volume excess is between 20 and 40%
- Increased risk of skin infection

LATE STAGE II

- Positive Stemmer Kaposi sign
- Tissue fibrosis
- Permanent edema
- Skin folds, inflammations and cutaneous modifications
- Volume excess is between 20 and 40%
- Increased risk of skin infection

Severe

STAGE III


- Very voluminous limb
- Non regular limb shape with skin folds
- Permanent edema
- Negative pitting sign
- Dermatological complications
- Volume excess is above 40%
- Mobility decrease

(1) ILF - Best practice for the management of LO - 2006 - Page 7

Origin of Lymphedema

There are 2 types of lymphedema:


Primary lymphedema

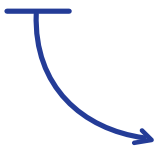

Related to a constitutional defect of the lymphatic system. Congenital or familial, it mainly affects the lower limbs. In the majority of cases, it is necessary to initiate management of the condition from childhood. Primary lymphedema can also start in young adults.





Secondary lymphedema

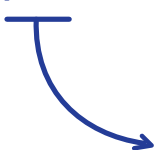

Results from a damage of the lymphatic system associated with a trauma, a disease like cancer and its treatment (lymph node dissection, radiotherapy...) or a contamination.

Secondary lymphedema concerns

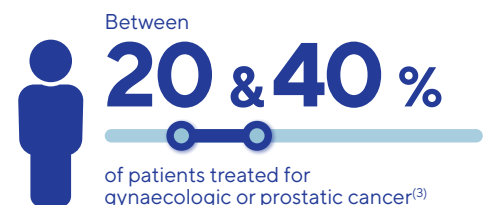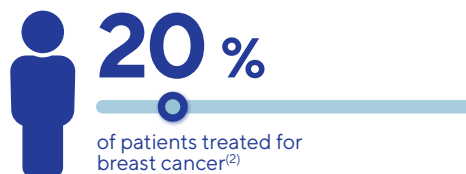

(1) Orphanet, webportal on rare diseases and orphan drugs, www.orpha.net : Primary lymphedema – November 2007 – Document prepared in collaboration with: Dr Stephane Vignes /Consultation of the reference center for rare vascular diseases, Cognacq-Jay Hospital, Paris.

(2) Disipio T et al. Lancet Oncol 2013; 14: 500 Incidence of unilateral arm Lymphedema after breast cancer: a systematic review and meta-analysis

(3) Beesley V et al. Cancer 2007 Jun 15;109(12):2607-14. Lymphedema after gynecological cancer treatment : prevalence, correlates, and supportive care needs. Rasmusson E. et al. Radiat Oncol. 2013 Nov 19;8:271 Low rate of lymphedema after extended pelvic lymphadenectomy followed by pelvic irradiation of node-positive prostate cancer.

Management of Lymphedema

Patients presenting lymphedema should benefit from coordinated management and receive information adapted to their needs.

The main objectives of the treatment are:

- Reduction, then stabilization of the volume
- Improvement of the limb function
- Prevention of complication risks
- Patient education and psycho-social support

This chronic disease therefore requires continuous management, which can be split into two essential and complementary phases:



The reduction
phase



The maintenance
phase

that will follow one another depending on the evolution of the lymphedema.

*Medical compression is
one of the key elements
of the treatment.*

The **reduction** phase


The reduction phase is also known as the intensive decongestive treatment.

Its goal is to achieve maximum reduction of the edema to come as close as possible to the volume of the opposite healthy limb.

The main treatments used are:

- **Manual lymphatic drainage (MLD)**, carried out by a specifically trained physiotherapist, to stimulate and facilitate the circulation of the lymph by promoting the unblocking of the lymph nodes.
- **Reducing and draining bandages** requiring the use of multi-component bandages which can be adapted to the volume and repositioned at will.
- **Coached physical activity** under bandages to improve lymph circulation, regain mobility and flexibility.
- **Skin care** to prevent complications related to lymph accumulation.

This phase may vary from one to several weeks, depending on the severity of the lymphedema, and can be renewed later.

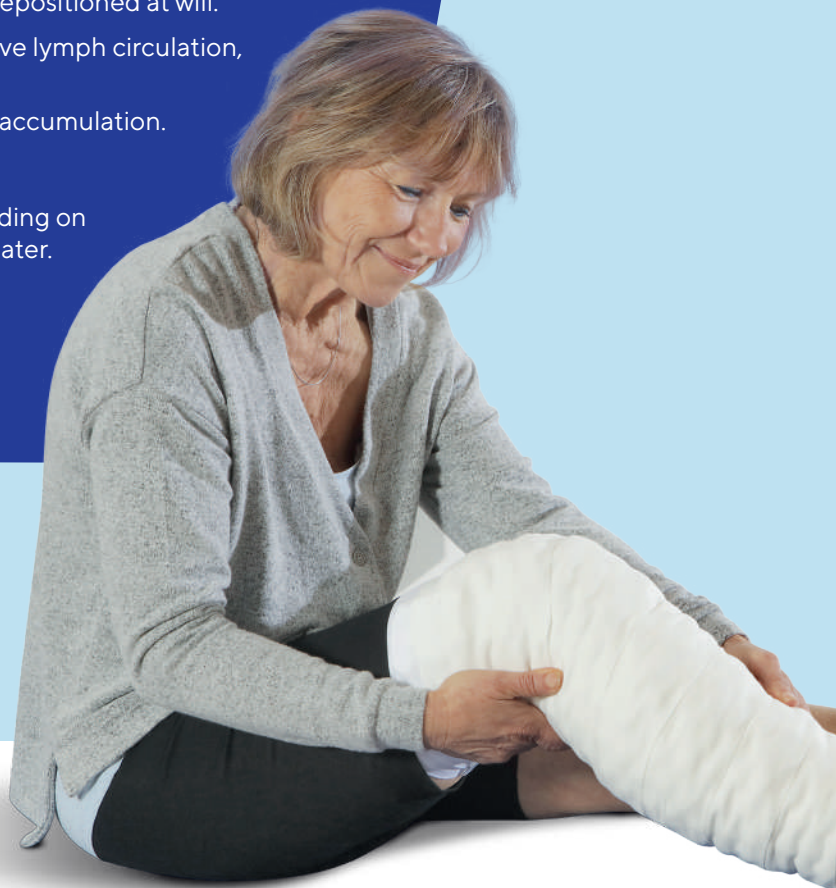

THUASNE Protocol

The reduction phase is usually performed using multi-component bandages⁽¹⁾

The advantage of bandages is that they can be repositioned over and over to ensure the best possible fit to successive variations in edema volume.

There are different sorts of bandages and it is their combined use that reduces edema.



1

PROTECT the skin

SHORT-STRETCH COTTON BANDAGE

In direct contact with the skin, this tubular cotton bandage provides tight containment of the limb. Made of cotton, it provides soft and lined protection of the skin weakened by the lymphedema.



2

MOBILIZE the edema

MOBIDERM BANDAGE

Used under a reducing bandage, the Mobiderm technology mobilizes edematous or indurated subcutaneous tissue.



3

CONTAIN the edema

BIFLEXIDEAL® SHORT-STRETCH BANDAGE

A short-stretch bandage (<100% stretch) as the third layer helps containment of the edema, as well as drainage in case of physical activity (high working pressure)⁽²⁾.



4

COMPRESS the edema

BIFLEX MEDIUM-STRETCH BANDAGE

A medium-stretch bandage as last layer applies constant pressure, even at rest.

BIFLEX 16+ (light) is recommended for mild to moderate lymphedema.

BIFLEX 17+ (strong) is recommended for moderate to severe lymphedema.




Discover video tutorials presenting multi-component bandage techniques **on our Thuasne YouTube channel!**

The MOBIDERM Technology

MOBIDERM is a medical device made up of foam blocks encased between two non-woven bandages.

The **MOBIDERM** Technology can be used under a reducing bandage (in its bandage or pad forms), or incorporated into mobilizing garments.

Due to the difference in pressure created between the support area of the blocks and their surrounding area, **it promotes the flow of lymphatic fluid and optimizes the drainage efficacy⁽¹⁾**.

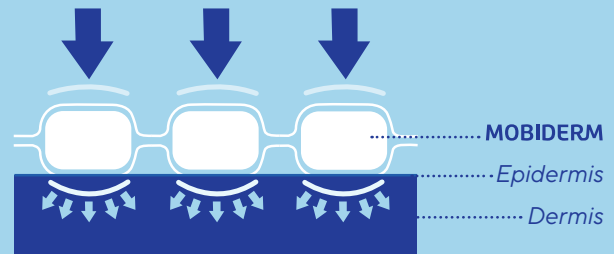

EFFECTIVE

- Rapid volume reduction of the edema⁽²⁾.

PRACTICAL

- Light and comfortable, it enables the patient to keep their freedom of movement⁽⁴⁾.

COMFORTABLE

- Soft on the skin
- Ability of the product to contour the extremity provides a superior anatomical fit.

(1) Todd - Mobiderm Autofit: an adjustable sleeve that enables patients to self-manage lymphedema - Chronic edema - April 2018 - Page 6
 (2) Quéré - Prospective multicenter observational study of lymphedema therapy: POLIT study - Vascular Disease Journal - 2014 - Pages 4 and 5
 (3) Only in following countries: Algeria, Austria, Belgium, Canada, China, Czech Republic, Denmark, Finland, France, Germany, Hungary, India, Morocco, Italy, Netherlands, Poland, Portugal, Romania, Russian Federation, Slovakia, Slovenia, Spain, Sweden, Switzerland, Tunisia, United-Kingdom
 (4) Mestre - Interest of an auto-adjustable night-time compression sleeve (MOBIDERM Autofit) in maintenance phase of upper limb lymphedema: the MARILYN pilot RCT Support Care Cancer (25:2455-2462) - 2017 - Page 7

MOBIDERM Pads

Dimensions	Designation	Reference
25 cm x 25 cm	Small blocks 5 mm x 5 mm	371001 02502501
25 cm x 1 m		371001 03310001
20 cm x 1 m	Big blocks 15 mm x 15 mm	371001 03310002



MOBIDERM Bandages

Dimensions	Designation	Reference
10 cm x 3 m	Small blocks 5 mm x 5 mm	372001 01030001
	Big blocks 15 mm x 15 mm	372001 01030002



MOBIDERM Kits

MOBIDERM KIT n°1

Reduction kit for the treatment of upper limb edema



Content	Reference
• 1 Biflex 16+ bandage, 8 cm x 5 m	370090 29999901
• 1 Biplast bandage, 3 cm x 2.5 m	
• 1 Short-stretch cotton bandage, 11 cm x 4 m	
• 2 Mobiderm bandages, 10 cm x 3 m	
• 1 Biflexideal® bandage, 10 cm x 5 m	

MOBIDERM KIT n°2


Reduction kit for the treatment of lower limb edema



Content	Reference
• 1 Biflex 16+ bandage, 10 cm x 3 m	370090 29999902
• 1 Biflex 16+ bandage, 10 cm x 5 m	
• 1 Biplast bandage, 3 cm x 2.5 m	
• 2 Short-stretch cotton bandages, 11 cm x 4 m	
• 3 Mobiderm bandages, 10 cm x 3 m	
• 2 Biflexideal® bandages, 10 cm x 5 m	



2 block sizes

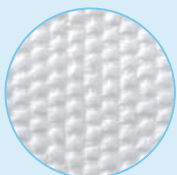

SMALL BLOCKS

- More effective in highly induced areas
- Better adapted to small anatomies (pediatric patients, hands/foot, breast, ...)




BIG BLOCKS

- Better adapted along the limb

KITBIFLEX


Kit of 2 short-stretch washable elastic compression bandages

Health all squared up

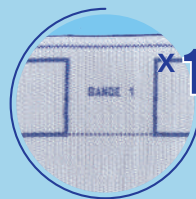

x1 FLEECE NON-ELASTIC BANDAGE (33.2% maximum stretch)

- Soft on the skin thanks to microfiber polyamid and viscose components
- Protects the skin and bony prominences
- Contains the edema

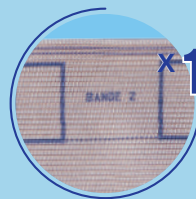


x1 TWO-WAYS ELASTIC BANDAGE (81.6% maximum stretch)

- Brings required level of pressure
- Maintains pressure over time



x6 SELF-GRIPPING FASTENERS

- Ensure good hold of the bandages
- Do not hurt the patient's skin

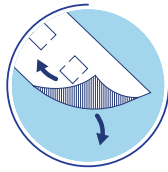

REPOSITIONNABLE

- The two bandages can be repositioned to follow lymphedema volume variation.



WASHABLE

- The two bandages are machine washable 3 times at 40°C.



EASE OF USE

- The two bandages are numbered and calibrated to ease the application.
- The patented calibration system allows to control the tension of the bandages and ensure the reproductibility of application.



3 SIZES TO ADAPT TO EACH PATIENT BODY SHAPE

Size	Ankle circumference	Bandage 1	Bandage 2	Bandages width	Reference
1	< 25 cm	4 m	3.5 m		170050 20199999
2	25 - 30 cm	4.5 m	4 m	10 cm	170050 20299999
3	> 30 cm	5 m	4.5 m		170050 20399999

Compression bandages



Cutaneous protection

SHORT-STRETCH COTTON BANDAGE

Non-elastic bandage

Tubular bandage. Composition 100% cotton.
Protection of skin surface under non-adhesive bandage.

Width	Length	Color	Reference
11 cm	4 m	White	010101 01140000



PROTECTIVE UNDERWRAP

Thin stretch foam bandage

Protection of skin surface under adhesive bandage.

Width	Length	Color	Reference
7 cm	27 m	White	550709 1000

Containment




BIFLEXIDEAL®

Short-stretch compression bandage

Made of cotton and viscose, it is a soft and breathable bandage.
51.6% maximum stretch.⁽¹⁾

Width	Length	Color	Reference
3 cm	5 m	White	110002 00350000
6 cm			110002 00650000
8 cm			110002 00850000
10 cm			110002 01050000

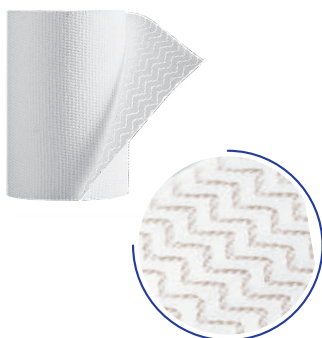

BIPLAST

Adhesive short-stretch compression bandage

Herringbone adhesive application. Multidirectional adhesive-ness right to the edge of the bandage. Flexible and adjustable.
50.2% maximum stretch.⁽²⁾

Width	Length	Color	Reference
3 cm	2.5 m	White	200001 00325000
6 cm			200001 00625000
8 cm			200001 00825000
10 cm			200001 01025000
15 cm			200001 01525000
20 cm			200001 02025000

Compression bandages

Compression



BIFLEX+ GRADUATED Medium-stretch graduated elastic compression bandage

Two-ways elastic bandage. Elastic in length and extensible in width.

Precise and repeatable control of the tension exerted thanks to the fitting indicator. Multiple overlappings possible to achieve multiple pressures level. 150.9% maximum stretch.⁽¹⁾

2 versions

BIFLEX 16+ (Light)




BIFLEX 17+ (Strong)



Width	Length	Color	Reference
8 cm	3 m	Beige	160002 00830002
	3.5 m		160002 00835002
	4 m		160002 00840002
	5 m		160002 00850002
	3 m		160002 01030002
10 cm	3.5 m	Beige	160002 01035002
	4 m		160002 01040002
	5 m		160002 01050002
	3 m		170002 00830002
	4 m		170002 00840002
8 cm	3 m	Beige	170002 01030002
	3.5 m		170002 01035002
	4 m		170002 01040002
	5 m		170002 01050002
	3 m		170002 01030002
10 cm	3.5 m	Beige	170002 01035002
	4 m		170002 01040002
	5 m		170002 01050002
	3 m		170002 01030002
	4 m		170002 01040002

BIFLEX application in 2 simple steps


1/ STRETCHING

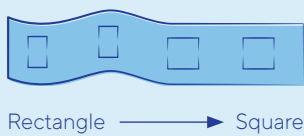

2/ OVERLAPPING

BIFLEX 16+

Pressure □□□■ Pressure □□■□

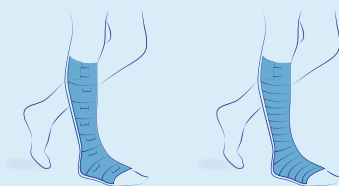

Half-square Overlapping

Full-square Overlapping

BIFLEX 17+

Pressure □■□■ Pressure ■□■□



Half-square Overlapping

Full-square Overlapping

(1) Internal test report 2017-064 D

Compression bandages




Compression

BIFLEX+ PRATIC Medium-stretch graduated elastic compression bandage with stirrup

Fitting stirrup makes this bandage easy to use. Enhances patient autonomy. Time saving for caregivers at home or at the hospital. 150.9% maximum stretch.⁽¹⁾

Designation	Width	Length	Color	Reference
BIFLEX 16+ PRATIC (Light)	8 cm	3 m	Beige	160003 00830002
		4 m		160003 00840002
BIFLEX 17+ PRATIC (Strong)	10 cm	3 m	Beige	160003 01030002
		4 m		160003 01040002
BIFLEX 17+ PRATIC (Strong)	10 cm	3 m	Beige	170003 01030002
		4 m		170003 01040002

Pratic stirrup application

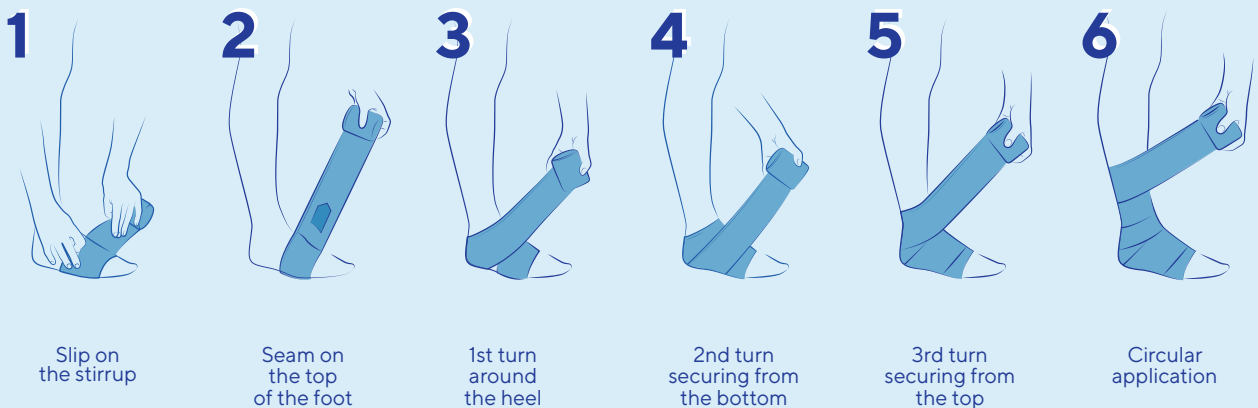

1 Slip on the stirrup

2 Seam on the top of the foot

3 1st turn around the heel

4 2nd turn securing from the bottom

5 3rd turn securing from the top

6 Circular application

(1) Internal test report 2017-064 D

Accessories



FOAM BANDAGE

Emulsified latex foam

Optimization of compression treatment by levelling out of anatomical relief.

The bevelled cut makes it possible to overlap the bandage as it is wrapped around without making it too thick.

Thanks to open cells foam, the bandage is permeable to air and helps preventing the risk of wound maceration.

Thickness	Width	Length	Reference
8 mm	8 cm	1 m	530809 1100
	10 cm		531009 1100
12 mm	10 cm	1 m	541009 1100



VARICO

Latex foam pad

Optimization of compression treatment by levelling out of anatomical relief.

Thanks to open cells foam, the pad is permeable to air and helps preventing the risk of wound maceration.

Model	Dimensions	Packaging	Reference
0	63 mm x 87 mm	x 2 units	520009 1000
1	70 mm x 132 mm	x 1 unit	520009 1100
2	125 mm x 164 mm	x 1 unit	520009 1200
3	200 mm x 250 mm	x 1 unit	520009 1300



VELCRO FASTENERS

Packaging	Color	Reference
x 4 units	White	590309 1000



ELASTIC FASTENERS

Packaging	Color	Reference
x 4 units	Beige	590209 1000

The **maintenance** phase

As soon as a satisfactory volume reduction is achieved, the maintenance phase of the lymphedema starts.

Its role is to stabilize the volume of the reduced limb in the initial phase, or even to continue the volume reduction, but also to avoid relapses and enable the patient to recover normal mobility.

This phase is conducted first with elastic compression garments which will be renewed regularly under the supervision of a healthcare professional.



Contain

Worn daily from waking up to bedtime, day-time compression garments ensure both:

- **compression** aimed at easing the circulation of the lymph to the heart, thus preventing the formation of edema.
- **containment** preventing the edema from recovering more than a certain percentage of volume, irrespective of its intrinsic expansion force.

A variety of garment models are available (stockings, sleeves...) in several compression classes.

Class	France (AFNOR G30 102B STANDARD)	Germany (RAL-GZ 387:2000 STANDARD)
Class 1	10 - 15 mmHg	18 - 21 mmHg
Class 2	15 - 20 mmHg	23 - 32 mmHg
Class 3	20 - 36 mmHg	34 - 46 mmHg
Class 4	> 36 mmHg	> 49 mmHg



Mobilize

To optimize treatment efficiency, it is recommended to wear light compression device during rest and night-time.

MOBIDERM garments have been developed to mobilize the edema during rest and night-time. They offer an efficient, easy to use and comfortable alternative to self-bandage.⁽¹⁾

The maintenance phase also includes manual lymphatic drainage at a lesser frequency, as well as the learning of self-massaging and self-bandaging as needed. Lastly, it requires the practice of an adapted physical activity, skin cares and regular medical monitoring.



CONTAIN

UPPER & LOWER LIMB

THORAX/PELVIS/HEAD

SEVERE LYMPHEDEMA

Stage III:
Non regular limb shape
Skin folds
Fibrotic consistency
>40% volume excess

LYMPHATREX EXPERT



LYMPHATREX



MODERATE LYMPHEDEMA

Stage II & Late II:
Regular limb shape
Liquid consistency
20-40% volume excess

LYMPHATREX ESSENTIAL



CICATREX

MADE-TO-MEASURE PATIENT



MILD LYMPHEDEMA

Stage 0 & I:
Regular limb shape
Liquid consistency
No skin folds
<20% volume excess

VENO ELEGANCE



CICATREX STANDARD





MOBILIZE

UPPER LIMB

LOWER LIMB

MOBIDERM MADE-TO-MEASURE PATIENT



MOBIDERM autofit



MOBIDERM STANDARD



Lymphedema treatment:
the lymphedema

Lymphedema Treatment:
the reduction phase


Lymphedema Treatment:
the maintenance phase

Scars Treatment

Measurement Forms

LYMPHATREX ESSENTIAL

Made-to-measure patient circular knit
compression garments

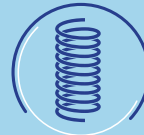

For which patients

LYMPHATREX Essential garments are intended for use in the treatment of lymphedema in maintenance phase following a reduction phase.

The range is recommended for patients presenting:

- Mild to moderate lymphedema⁽¹⁾
- Regular limb shape without skin folds⁽¹⁾



In case of severe lymphedema

Flat knitted garment may be more appropriate providing stronger containment of the edema.⁽¹⁾

Principle of action

Compression: to help lymphatic circulation and prevent the formation of edema⁽²⁾

Characteristics

Composition⁽³⁾: polyamide, elastane

Colors: beige , black 

Compression classes:
class 2 (15 – 20 mmHg), class 3 (20 – 36 mmHg)

(1) SFMV – Compression et lymphoedème du membre inférieur – Comité Interface Compression – Mars 2015 – Page 2

(2) SNITEM – Coupé – Compression et contention : quelles différences ?, Historique et Innovation en Lymphologie Dec 2015 – Pages 4 & 5

(3) Please refer to the Instructions for Use of the product for the full list of components

(4) Compared to Thuasne Lymphology. IFTH sensorial analysis circular knit garments – Nov 2019. Inside softness: 4,88/10 for Lymphatrex Essential vs 0,83/10 for Thuasne Lymphology. Outside softness: 7,62/10 for Lymphatrex Essential vs 1,16/10 for Thuasne Lymphology. Suppleness: 6,89/10 for Lymphatrex Essential vs 2,91/10 for Thuasne Lymphology. Elasticity in height: 4,36/10 for Lymphatrex Essential vs 1,97/10 for Thuasne Lymphology. Thermal feeling: 2,85/10 for Lymphatrex Essential vs 3,73 to 7,59/10 for tested competitive products (SIGVARIS Coton 3, VARISAN Circulaire, JUZO Dynamic, JUZO Dynamic Cotton, RADIANTE Qoton and JOBST Bellavar)



IMPROVED SOFTNESS⁽⁴⁾




- 5 times softer inside the garment
- 7 times softer outside the garment **1**

IMPROVED COMFORT⁽⁴⁾

- 2 times more supple knit for a second skin feeling
- Cooler thermal feeling wearing the garment than competitive products **2**
- 2 times more elastic in height for a better hold
- Enhanced mitten finishing for more comfort at the thumb commissure **3**
- Flat seams **4**

MORE COLOR CHOICE⁽⁴⁾

- Available in beige and black **5**

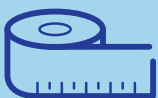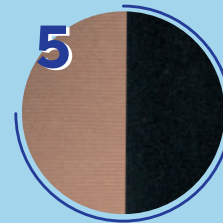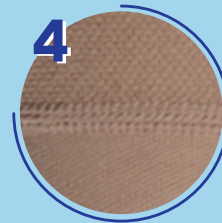




Made-to-measure patient devices require the involvement of a trained healthcare professional.
Measurement forms available on pages 68 to 71

LYMPHATREX ESSENTIAL


Made-to-measure patient circular knit compression garments

Models and Options

Upper limb

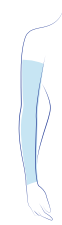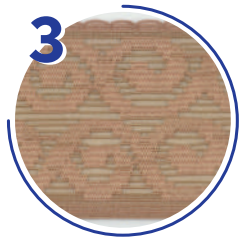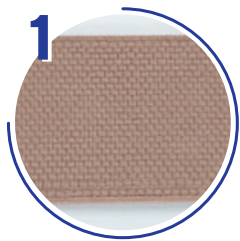

Armsleeve

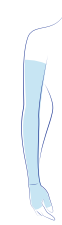

Armsleeve with mitten



Mitten

Compression classes

	Armsleeve	Armsleeve with mitten	Mitten
Class 2 (15 - 20 mmHg)	●	●	●
Class 3 (20 - 36 mmHg)	●	●	●

Colors

	Armsleeve	Armsleeve with mitten	Mitten
Beige	●	●	●
Black	●	●	●

Proximal end options

	Armsleeve	Armsleeve with mitten	Mitten
Without anti-slip (ribbed edge)	-	-	●
Elastic plain braid 3 cm ^①	●	●	-
Anti-slip with silicone dots 3 cm ^②	●	●	-
Silicone anti-slip for women 5 cm ^③	●	●	-
Shoulder cover	●	●	-

Distal end options

	Armsleeve	Armsleeve with mitten	Mitten
With thumb	-	●	●
Without thumb ^④	-	●	●

Lower limb



Below-knee



Thigh-high



Tights

Compression classes

	Below-knee	Thigh-high	Tights
Class 2 (15 - 20 mmHg)	●	●	●
Class 3 (20 - 36 mmHg)	●	●	●

Colors

	Below-knee	Thigh-high	Tights
Beige	●	●	●
Black	●	●	●

Proximal end options

	Below-knee	Thigh-high	Tights
Without anti-slip (ribbed edge)	●	-	-
Elastic plain braid 5 cm ^①	-	-	●
Anti-slip with silicone dots 3 cm ^②	●	-	-
Anti-slip with silicone dots 5 cm ^②	●	●	-
Silicone anti-slip for women 5 cm ^③	-	●	-
Hip cover	-	●	-

Brief options


	Below-knee	Thigh-high	Tights
Compressive brief	-	-	●
Non compressive brief	-	-	●

Distal end options

	Below-knee	Thigh-high	Tights
Open toe	●	●	●
Closed toe ^⑤	●	●	●
Compression pad	●	●	●

VENO ELEGANCE

Standard circular knit
compression garments

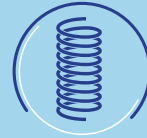

Standard products: **faster availability!**

For which patients

VENO ELEGANCE armsleeves are intended for use in the treatment of lymphedema in maintenance phase following a reduction phase.

The product is recommended for patients presenting:


- Mild to moderate lymphedema ⁽¹⁾
- Regular limb shape without skin folds ⁽¹⁾

Principle of action

Compression: to help lymphatic circulation and prevent the formation of edema ⁽²⁾


Characteristics

Composition⁽³⁾: polyamide microfibre, elastane

Color: beige 

Compression classes: class 2 (23 - 32 mmHg)

Sizes (in cm)

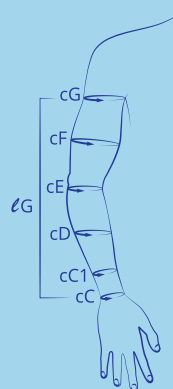

	SMALL	MEDIUM	LARGE
cG	25 - 29.5	30 - 35.5	36 - 42
cF	23 - 26.5	27 - 31	31.5 - 36
cE	21 - 24.5	25 - 28.5	29 - 33
cD	20 - 23.5	24 - 27.5	28 - 32
cC1	13.5 - 16	16.5 - 19	19.5 - 22.5
cC	13 - 15.5	16 - 18.5	19 - 22
Length SHORT			
lG	38 - 44	40 - 46	42 - 48
Length LONG			
lG	42 - 48	44 - 50	46 - 52

(1) SFMV - Compression et lymphœdème du membre inférieur - Comité Interface Compression - Mars 2015 - Page 2







(2) SNITEM - Coupé - Compression et contention : quelles différences ?, Historique et Innovation en Lymphologie Dec 2015 - Page 5

(3) Please refer to the Instructions for Use of the product for the full list of components

Models and Options

ARMSLEEVE & ARMSLEEVE WITH MITTEN

Class 2 (23 - 32 mmHg)

	DESIGNATION	Ref Short	Ref Long
	Armsleeve with anti-slip		
	Small	2002D221102561	2002D221102563
	Medium	2002D221202561	2002D221202563
	Large	2002D221302561	2002D221302563
	Armsleeve with mitten and anti-slip		
	Small	2002D221102551	2002D221102553
	Medium	2002D221202551	2002D221202553
	Large	2002D221302551	2002D221302553
	Armsleeve with shoulder cover - RIGHT		
	Small	2002E220102561	2002E220102563
	Medium	2002E220202561	2002E220202563
	Large	2002E220302561	2002E220302563
	Armsleeve with shoulder cover - LEFT		
	Small	2002F220102561	2002F220102563
	Medium	2002F220202561	2002F220202563
	Large	2002F220302561	2002F220302563
	Armsleeve with mitten and shoulder cover - RIGHT		
	Small	2002E220102551	2002E220102553
	Medium	2002E220202551	2002E220202553
	Large	2002E220302551	2002E220302553
	Armsleeve with mitten and shoulder cover - LEFT		
	Small	2002F220102551	2002F220102553
	Medium	2002F220202551	2002F220202553
	Large	2002F220302551	2002F220302553

LYMPHATREX EXPERT

Made-to-measure patient flat knit compression garments



For which patients


LYMPHATREX Expert garments are intended for use in the treatment of lymphedema in maintenance phase following a reduction phase.

The range is recommended for patients presenting:

- Moderate to severe lymphedema⁽¹⁾
- Non-regular limb shape with skin folds⁽¹⁾
- Spreading to fingers or toes⁽¹⁾
- Large limb circumference⁽¹⁾

Principle of action

Containment: a short-stretch and robust technical knit allows **LYMPHATREX Expert** garments to apply a high working pressure and a low resting pressure on the limb.⁽²⁾

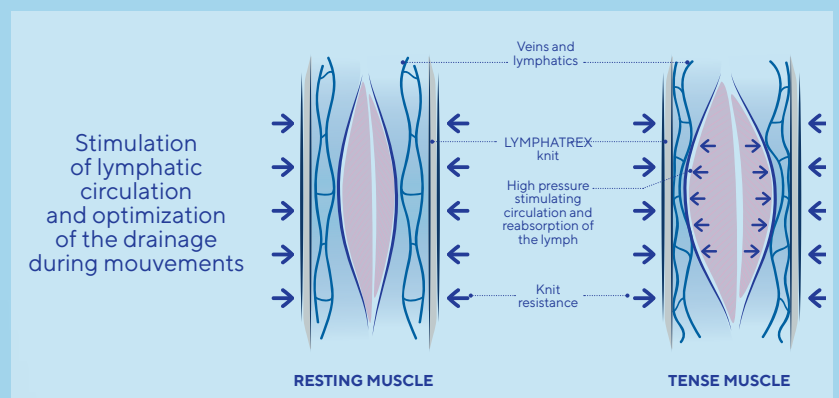

Characteristics

Composition⁽³⁾: polyamide, elastane

Colors: beige , tanning beige , black 




Compression classes:

class 2 (15 – 20 mmHg), class 3 (20 – 36 mmHg), class 4 (> 36 mmHg)

(1) SFMV – Compression et lymphoedème du membre inférieur – Comité Interface Compression – Mars 2015 – Page 2

(2) SNITEM – Coupé – Compression et contention : quelles différences ?, Historique et Innovation en Lymphologie – Dec 2015 – Page 4

(3) Please refer to the Instructions for Use of the product for the full list of components

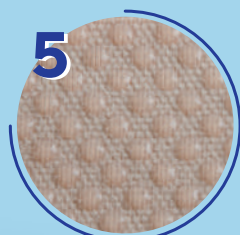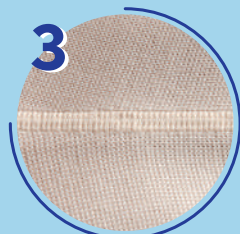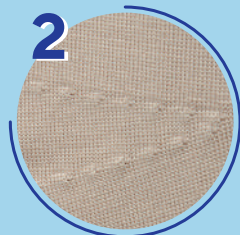


SOFTNESS

- Knitted from **polyamide microfibre yarn**, naturally very soft⁽¹⁾.
- Very fine weft of knitting and not in contact with the skin, in order to keep the **softness of the mesh.** ①

COMFORT

- **Anatomical adjustment**, even at the most sensitive areas such as elbow folds or knee. ②
- **Thermoregulation** by double-sided knitting: absorption and then removal of moisture outwards⁽¹⁾.

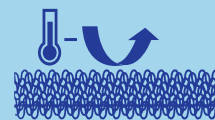

- **Flat seams.** ③
- **Sheathing knit**, does not penetrate skin folds.


DISCRETION AND DRESS

- Available in **timeless and sober colors**: black, beige and tanning beige to adapt to different skin carnations. ④
- Controlled balance between the knitting parameters and the anti-slip device helps **proper hold on the limb.** ⑤



Knit adaptation to limb volume variation

The flat knitted technology allows to vary the number of stitch loop on each row based on the precise patient measurements.

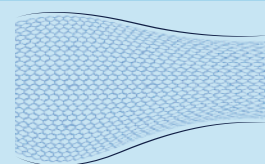

Made-to-measure patient devices require the involvement of a trained healthcare professional.

Measurement forms available on pages 72 to 78




(1) ETN 280 - Dossier Textile Polyamide - July Aug 2017 - Page 1

LYMPHATREX EXPERT

Made-to-measure patient flat knit compression garments

Models and Options

Upper limb

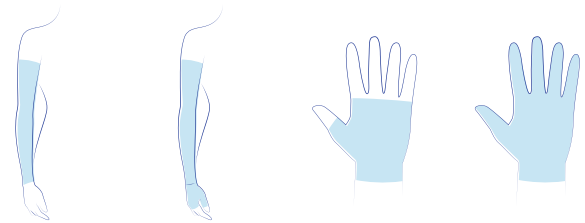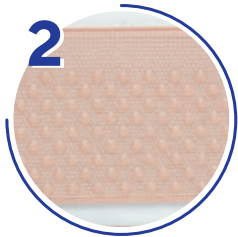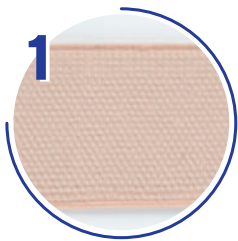

Armsleeve Armsleeve with mitten Mitten Glove

Compression classes

	Armsleeve	Armsleeve with mitten	Mitten	Glove
Class 2 (15 - 20 mmHg)	●	●	●	●
Class 3 (20 - 36 mmHg)	●	●	●	●

Colors

	Armsleeve	Armsleeve with mitten	Mitten	Glove
Beige	●	●	●	●
Tanning Beige	●	●	●	●
Black	●	●	●	●

Proximal end options

	Armsleeve	Armsleeve with mitten	Mitten	Glove
Without anti-slip (ribbed edge)	-	-	●	●
Angled proximal edge	●	●	-	-
Elastic plain braid 3 cm ^①	●	●	●	●
Anti-slip with silicone dots 3 cm ^②	●	●	●	●
Shoulder attachment	●	●	-	-
Shoulder cover	●	●	-	-
Non compressive cuff	-	-	●	●
Compressive cuff	-	-	●	●
Extended cuff	-	-	●	●

Distal end options

	Armsleeve	Armsleeve with mitten	Mitten	Glove
Zipper opening hand top ^③	-	-	●	●
Foam compression pad	-	-	●	●

Lower limb



Compression classes

	Below-knee	Thigh-high	Half-Tight	Tights	Panty	Toe Cap
Class 3 (20 - 36 mmHg)	●	●	●	●	●	●
Class 4 (>36 mmHg)	●	●	●	●	●	●

Colors

	Below-knee	Thigh-high	Half-Tight	Tights	Panty	Toe Cap
Beige	●	●	●	●	●	●
Tanning Beige	●	●	●	●	●	●
Black	●	●	●	●	●	●

Proximal end options

	Below-knee	Thigh-high	Half-Tight	Tights	Panty	Toe Cap
Without anti-slip (ribbed edge)	●	-	-	-	-	-
Elastic plain braid 3 cm ^①	●	-	-	-	-	-
Anti-slip with silicone dots 3 cm ^②	●	-	-	-	-	-
Anti-slip with silicone dots 5 cm ^②	-	●	-	-	-	-
Angled thigh end	-	●	-	-	-	-
Hip cover	-	●	-	-	-	-
Elastic belt 5 cm	-	-	●	●	●	-
Adjustable elastic belt	-	-	●	●	●	-
Without belt	-	-	●	●	●	-

Brief options

	Below-knee	Thigh-high	Half-Tight	Tights	Panty	Toe Cap
Compressive brief	-	-	●	●	●	-
Non compressive brief	-	-	●	●	●	-
Hygiene tab	-	-	●	●	●	-
Opening zipper	-	-	●	●	●	-
Opening Velcro	-	-	●	●	●	-
Without opening	-	-	●	●	●	-

Distal end options

	Below-knee	Thigh-high	Half-Tight	Tights	Panty	Toe Cap
Open toe ^④	●	●	●	●	-	-
Closed toe	●	●	●	●	-	-
With toes ^⑤	●	●	●	●	-	●
Compression pad	●	●	●	●	-	-

Other options

	Below-knee	Thigh-high	Half-Tight	Tights	Panty	Toe Cap
Combination with thigh-high	-	-	●	-	●	-

LYMPHATREX

Made-to-measure patient flat knit compression garments



For which patients


LYMPHATREX garments are intended for use in the treatment of lymphedema in maintenance phase following a reduction phase.

The range is recommended for patients presenting:

- Moderate to severe lymphedema ⁽¹⁾
- Non-regular limb shape with skin folds ⁽¹⁾
- Spreading to fingers or toes ⁽¹⁾
- Large limb circumference ⁽¹⁾

Principle of action

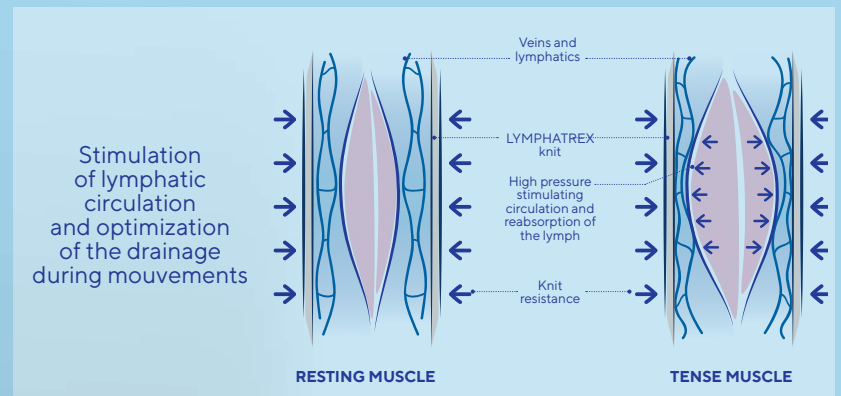

Containment: a short-stretch and robust technical knit allows to **LYMPHATREX** garments to apply a high working pressure and a low resting pressure on the limb. ⁽²⁾



Characteristics

Composition ⁽³⁾:

- Fine: polyamide, tergal, elastane
- Poly & Poly Strong: polyamide, elastane

Colors: beige , tanning beige ⁽⁴⁾ , black 

Compression classes:

class 2 (23 - 32 mmHg), class 3 (34 - 46 mmHg) ⁽⁵⁾


⁽¹⁾ SFMV - Compression et lymphoedème du membre inférieur - Comité Interface - Compression - Mars 2015 - Page 2

⁽²⁾ SNITEM - Coupé - Compression et contention : quelles différences ?, Historique et Innovation en Lymphologie - Dec 2015 - Pages 4

⁽³⁾ Please refer to the Instructions for Use of the product for the full list of components

⁽⁴⁾ Only available in Poly and Poly Strong

⁽⁵⁾ Only available in Poly Strong

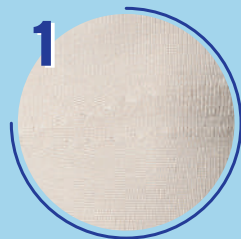

COMFORT

- **Anatomic adjustment**, even at the most sensitive areas such as elbow folds or knee. ① ②
- **Flat seams**. ③

- **Sheathing knit**, does not penetrate skin folds.


PERFECT FIT

- **Large variety of options** to adapt most of specific needs.
- **Large variety of models** to adapt most of lymphedema localizations.

LARGE MATERIAL CHOICE

3 different materials with specific properties:

- **Fine**, offers **good tolerance for sensitive skins** and helps maintaining skin temperature - ideal for elderly patients.
- **Poly**, knitted from polyamide microfibers, offers **softness** and **good thermoregulation** - ideal for more active patients. ④


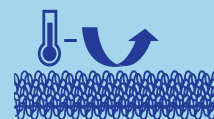


- **Poly Strong**, in addition to Poly properties, offers **stronger compression and rigidity** thanks to higher elastane content - ideal for severe lymphedemas.



Knit adaptation to limb volume variation

The flat knitted technology allows to vary the number of stitch loop on each row based on the precise patient measurements.

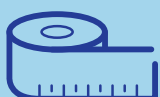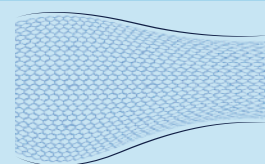

Made-to-measure patient devices require the involvement of a trained healthcare professional.
Measurement forms available on pages Thuasne website.

LYMPHATREX

Made-to-measure patient flat knit
compression garments

Models and Options

Upper limb

Thorax

	Armsleeve			Mitten & glove		Armsleeve with mitten or glove		Vest	Jacket
	CG	CG1	CE	AC1	AE	AG			
Material									
Fine	•	•	•	•	•	•	•	•	•
Poly	•	•	•	•	•	•	•	•	•
Compression classes									
Class 2 (23 - 32 mmHg)	•	•	•	•	•	•	•	•	•
Colors									
Beige	•	•	•	•	•	•	•	•	•
Tanning Beige	•	•	•	•	•	•	•	•	•
Black	•	•	•	•	•	•	•	•	•
Proximal end options									
Aslope edge	•	•	•	-	•	•	-	-	
Anti-slip with silicone dots	•	•	•	•	•	•	•	•	
Pieces of anti-slip with silicone dots	•	•	•	•	•	•	•	•	
Elastic plain braid	•	•	•	•	•	•	•	•	
Knitted anti-slip tape	•	•	•	•	•	•	•	•	
Silicone pads	•	•	•	•	•	•	-	-	
Shoulder cover	•	•	•	-	•	•	-	-	
Shoulder attachment	•	•	-	-	•	•	-	-	
Knitted anti-slip tape	•	•	-	-	•	•	-	-	
Raglan arm	-	-	-	-	-	-	•	•	
T-shirt arm	-	-	-	-	-	-	•	•	
Distal end options									
Distal Porous	•	•	•	-	•	•	•	-	
Distal full compression	•	•	•	•	•	•	-	-	
With fingers	-	-	-	•	•	•	-	-	
Without fingers	-	-	-	•	•	•	-	-	
Cup options									
SilonTex epithesis pocket right/left	-	-	-	-	-	-	•	•	
Cup right/left	-	-	-	-	-	-	•	•	
Openings options									
Zipper with hooks and eyes	•	•	•	•	•	•	•	•	
Zipper	•	•	•	•	•	•	•	•	
Velcro	•	•	•	•	•	•	•	•	
Hooks and eyes	•	•	•	•	•	•	•	•	
Inside/outside	•	•	•	•	•	•	•	•	
Small finger side	-	-	-	•	•	•	-	-	
Thumb side	-	-	-	•	•	•	-	-	
Openings options									
Anatomical elbow 20/40/60/90°	•	•	-	-	•	•	•	-	
Lining	•	•	•	•	•	•	-	-	
Buffer material	•	•	•	•	•	•	-	-	
In combination with sleeve	-	-	-	•	•	•	•	•	
Compression pad	-	-	-	•	•	•	-	-	

Lower limb



	Below-knee		Thigh-high		Panty		Biker short	Bermuda	Capri pant	Legging		
	AB1	AD	AF	AG	AT	ATE	FT	ET	CT	CTE	BT	Toe Cap
Material												
Poly	•	•	•	•	•	•	•	•	•	•	•	•
Poly strong	•	•	•	•	•	•	•	•	•	•	•	•
Compression classes												
Class 2 (23 - 32 mmHg)	•	•	•	•	•	•	•	•	•	•	•	•
Class 3 (34 - 46 mmHg)	•	•	•	•	•	•	•	•	•	•	•	-
Colors												
Beige	•	•	•	•	•	•	•	•	•	•	•	•
Tanning beige	•	•	•	•	•	•	•	•	•	•	•	•
Black	•	•	•	•	•	•	•	•	•	•	•	•
Proximal end options												
Waist belt	-	-	-	-	•	•	•	•	•	•	•	-
Perforated elastic tape	•	•	•	•	•	•	•	•	•	•	•	-
Hip attachment	-	-	-	•	-	-	-	-	-	-	-	-
Suspenders	•	•	•	•	•	•	•	•	•	•	•	-
Porous edge	•	•	•	•	•	•	•	•	•	•	•	-
Anti-slip with silicone dots	•	•	•	•	•	•	•	•	•	•	•	-
Pieces of anti-slip with silicone dots	•	•	•	•	•	•	•	•	•	•	•	-
Elastic plain braid	•	•	•	•	•	•	•	•	•	•	•	-
Knitted anti-slip tape	•	•	•	•	•	•	•	•	•	•	•	-
Aslope edge	•	•	•	•	•	•	•	•	•	•	•	-
Anti-slip tape 3/4 inside	•	•	•	•	•	•	•	•	•	•	•	-
Silicone pads	•	•	•	•	•	•	•	•	•	•	•	-
Distal end options												
Open toe straight	•	•	•	•	•	•	-	-	-	-	-	•
Open toe angled	•	•	•	•	•	•	-	-	-	-	-	•
Closed toe straight	•	•	•	•	•	•	-	-	-	-	-	•
Closed toe angled	•	•	•	•	•	•	-	-	-	-	-	•
With toes	•	•	•	•	•	•	-	-	-	-	-	•
Heel with compression	•	•	•	•	•	•	-	-	-	-	-	•
Heel without compression	•	•	•	•	•	•	-	-	-	-	-	•
Foot with compression	•	•	•	•	•	•	-	-	-	-	-	•
Foot without compression	•	•	•	•	•	•	-	-	-	-	-	•
Y-heel compressed	•	•	•	•	•	•	-	-	-	-	-	•
Porous edge	•	•	•	•	•	•	•	•	•	•	•	•
Anti-slip with silicone dots	•	•	•	•	•	•	•	•	•	•	•	•
Pieces of anti-slip with silicone dots	•	•	•	•	•	•	•	•	•	•	•	•
Elastic plain braid	•	•	•	•	•	•	•	•	•	•	•	•
Knitted anti slip tape	•	•	•	•	•	•	•	•	•	•	•	•
Silicone pads	•	•	•	•	•	•	•	•	•	•	•	•
Openings options												
Zipper with hooks and eyes	•	•	•	•	•	•	•	•	•	•	•	-
Zipper	•	•	•	•	•	•	•	•	•	•	•	-
Front/back	•	•	•	•	•	•	•	•	•	•	•	-
Right/left	•	•	•	•	•	•	•	•	•	•	•	-
Gusset options												
With compression	-	-	-	-	•	•	•	•	•	•	•	-
Without compression	-	-	-	-	•	•	•	•	•	•	•	-
Stiff	-	-	-	-	•	•	•	•	•	•	•	-
Porous	-	-	-	-	•	•	•	•	•	•	•	-
Small/medium/large width	-	-	-	-	•	•	•	•	•	•	•	-
With relief opening	-	-	-	-	•	•	•	•	•	•	•	-
Other options												
With stocking/pants separate	•	•	•	•	•	•	•	•	•	•	•	-
Standard knee	-	-	•	•	•	•	•	•	•	•	•	-
Anatomical knee 20/40/60/90°	-	-	•	•	•	•	•	•	•	•	•	-

Lymphedema treatment:
the lymphedema

Lymphedema Treatment:
the reduction phase

Lymphedema Treatment:
the maintenance phase

Scars Treatment

Measurement Forms



LYMPHATREX ESSENTIAL, VENO ELEGANCE, LYMPHATREX EXPERT & LYMPHATREX



Fitting Instructions


SLEEVES

- Place your hand inside the garment and turn it back on itself to the hand opening. **1**
- Place your hand inside the garment **2**, then carefully slide it over the limb to under the armpit (armsleeve and armsleeve with mitten), or to the wrist (mitten and glove). **3**
- Avoid pulling the garment any more than necessary.
- Adjust the garment to ensure there are no wrinkles along the length of the limb.
- Once the garment is properly in place, massage the limb to ensure uniform compression. **4**



STOCKINGS

- Place your hand inside the garment and turn it back on itself to the heel. **5**
- Slip your foot into the garment until the heel is correctly positioned. **6**
- Carefully slide the garment over the limb to below the knee (below-knee), below the gluteal fold (thigh-high), or to the hip (tights or half-tight). **7**
- Avoid pulling the garment any more than necessary.
- Adjust the garment to ensure there are no wrinkles along the length of the limb.
- Stockings with anti-slip system: when putting on the garment, be careful not to grip the anti-slip part to avoid any damage.
- Once the garment is properly in place, massage the limb to ensure uniform compression. **8**

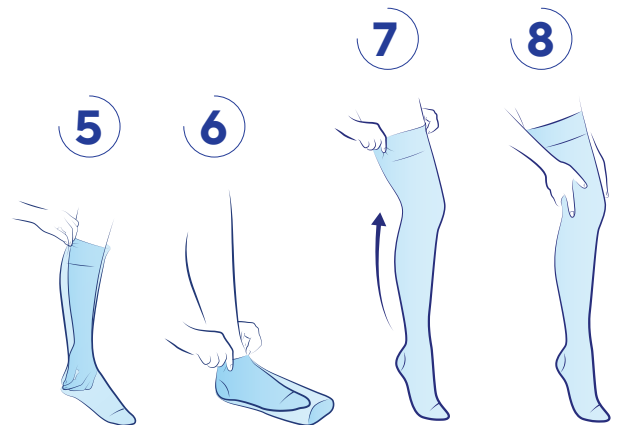

Care Instructions

It is recommended to wash the garment before first use.
Wash your garment every day, or after each use.



- Machine wash at 40°C.
- Use a washing product without softener. Do not use detergents, fabric softeners or aggressive products (products containing chlorine).
- Place the garment in a wash bag to make sure other garments (particularly those with touch fasteners or zips) do not damage the knit of your garment.
- Dry well away from any direct heat source (radiator, sun, etc.).
- Do not tumble-dry.
- Do not iron.



For rest and night-time treatment

MOBIDERM GARMENTS

In order to maintain results obtained during the intensive phase of multi-component bandages application, the patient must have access, during the maintenance phase, to a permanent and comfortable treatment.

Patient's autonomy will contribute to observance of the treatment prescribed and can guarantee its long-term success.

Taking over from day-time compression and as an alternative to self-bandage, **MOBIDERM** garments have been developed to continue mobilizing the edema during the night, and thereby reinforce the effects of the day-time treatment.⁽¹⁾⁽²⁾

These garments are made of a soft elastic material designed to deliver a soft pressure, well-accepted by patients in the rest and night-time phase⁽²⁾. They are fitted with the **MOBIDERM** Technology on the inside, which helps promoting lymphatic circulation with micro-massages.

Effective

- Reinforce the effects of the day-time treatment⁽¹⁾⁽²⁾
- Maintains the volume reduction of the edema⁽²⁾

Ease of use

- Enhances patient's autonomy and self-management
- A single garment to put on

Comfortable

- Unintrusive and discreet treatment⁽²⁾
- Do not disturb the sleep⁽²⁾

(1) Todd - Mobiderm Autofit: an adjustable sleeve that enables patients to self-manage lymphedema - Chronic edema - April 2018 - Page 6

(2) Mestre - Interest of an auto-adjustable night-time compression sleeve (MOBIDERM Autofit) in maintenance phase of upper limb lymphedema: the MARILYN pilot RCT - Support Care Cancer (25:2455-2462) - 2017 - Page 6 & 7



Lymphedema treatment:
the lymphedema

Lymphedema Treatment:
the reduction phase

Lymphedema Treatment:
the maintenance phase

Scars Treatment

Measurement Forms



MOBIDERM autofit

Standard adjustable mobilizing garments



Standard products:
faster availability!

Description

The **MOBIDERM autofit** garments are designed to mobilize edematous or indurated subcutaneous tissues.

They are intended for the treatment of lymphedema in maintenance phase for day and night to maintain the volume reduction.

The standard, self-adjustable garment adapts to most of patient's individual body shape and evolution overtime.

They are available in multiple models:

- For upper limb: **MOBIDERM autofit** armsleeve with mitten.
- For lower limb: **MOBIDERM autofit** below-knee or thigh-high.

For which patients


MOBIDERM autofit garments are recommended for patients presenting:

- Mild to moderate lymphedema
- Regular or non-regular limb shape
- Stable or evolutive edema

Principle of action

The **MOBIDERM** autofit garments incorporate **MOBIDERM** technology that is made up of foam blocks encased between two non-woven bandages.

Due to the difference in pressure it creates between the support area of the blocks and their surrounding area, it helps stimulate the flow of lymphatic fluid and optimizes drainage efficacy⁽¹⁾.

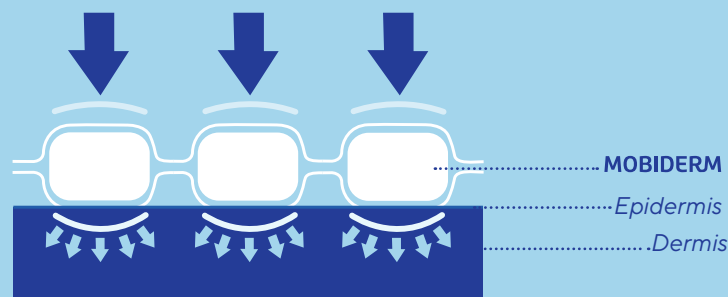

Characteristics

Composition⁽²⁾:

- Outer layer: polyester, polyamide, elastane
- Inner layer along the leg/arm: adhesive non-woven, polyurethane foam (15 x 15 mm blocks)
- Inner layer foot/hand level: adhesive non-woven, polyethylene foam (5 x 5 mm blocks)

Color: beige 

Compression class: around 15 mmHg⁽³⁾


(1) Todd – Mobiderm Autofit: an adjustable sleeve that enables patients to self-manage lymphedema – Chronic edema – April 2018 – Page 6

(2) Please refer to the Instructions for Use of the product for the full list of components

(3) Mestre – Interest of an auto-adjustable night-time compression sleeve (MOBIDERM Autofit) in maintenance phase of upper limb lymphedema: the MARILYN pilot RCT – Support Care Cancer (25:2455-2462) – 2017

MOBIDERM autofit

Standard adjustable mobilizing garments

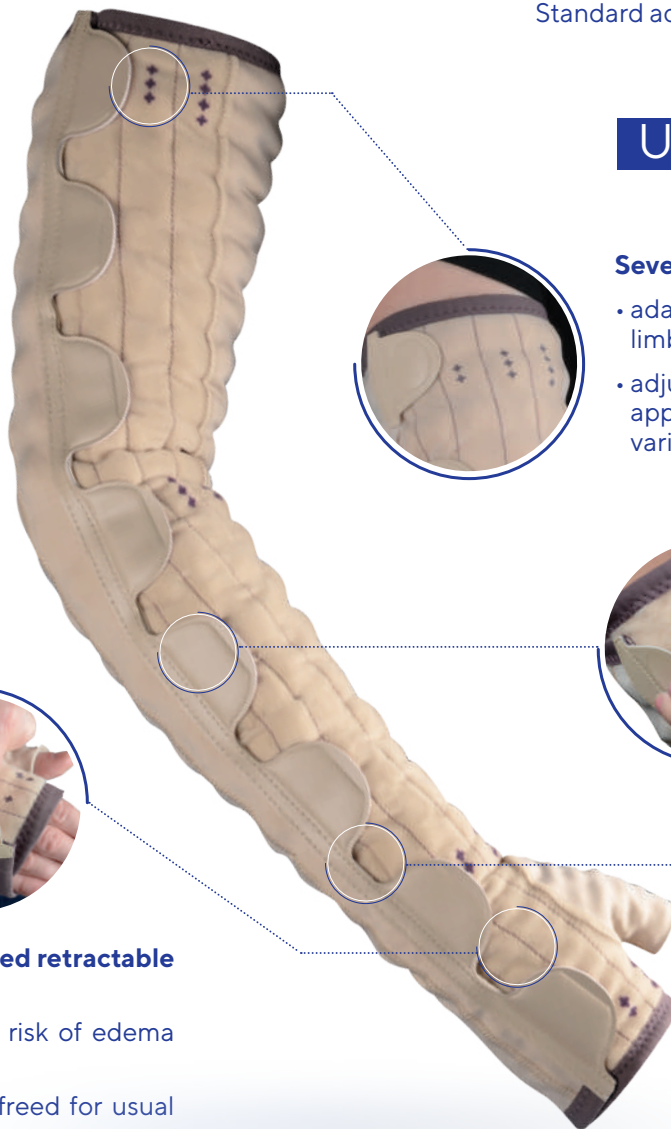

Upper limb

Several tightening notches

- adaptation to most of patient's individual limb shape
- adjustment marks: precise and repeatable application, control of the edema volume variation

Velcro fasteners

easy and quick set up⁽²⁾


Semi-open garment

easy to put on, easy to take off⁽²⁾

Sleeve with attached retractable mitten

- helps limiting the risk of edema migration
- the hand can be freed for usual tasks

Sizes

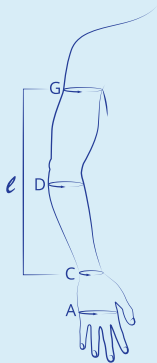

ℓ: Length from wrist to axillary
 cA: Hand circumference
 cC: Wrist circumference
 cD: Forearm circumference (at the widest point)
 cG: Axillary arm circumference

Size	cA (in cm)	cC (in cm)	cD (in cm)	cG (in cm)	Reference normal ℓ = 40 - 45 cm	Reference long ℓ = 45 - 50 cm
Right armsleeve						
1	17 - 21	15 - 19	22 - 28	25 - 32	373302 20100261	373302 20100271
2	17 - 21	16 - 20	24 - 30	29 - 36	373302 20200261	373302 20200271
3	18 - 22	17 - 21	26 - 32	33 - 40	373302 20300261	373302 20300271
4	18 - 22	17 - 21	28 - 34	37 - 44	373302 20400261	373302 20400271
5	19 - 23	18 - 22	30 - 36	41 - 48	373302 20500261	373302 20500271
6	20 - 24	19 - 23	31 - 37	45 - 52	373302 20600261	373302 20600271
Left armsleeve						
1	17 - 21	15 - 19	22 - 28	25 - 32	373302 20100262	373302 20100272
2	17 - 21	16 - 20	24 - 30	29 - 36	373302 20200262	373302 20200272
3	18 - 22	17 - 21	26 - 32	33 - 40	373302 20300262	373302 20300272
4	18 - 22	17 - 21	28 - 34	37 - 44	373302 20400262	373302 20400272
5	19 - 23	18 - 22	30 - 36	41 - 48	373302 20500262	373302 20500272
6	20 - 24	19 - 23	31 - 37	45 - 52	373302 20600262	373302 20600272

(1) Only in following countries: Algeria, Austria, Belgium, Canada, China, Czech Republic, Denmark, Finland, France, Germany, Hungary, India, Morocco, Italy, Netherlands, Poland, Portugal, Romania, Russian Federation, Slovakia, Slovenia, Spain, Sweden, Switzerland, Tunisia, United-Kingdom
 (2) Mestre - Interest of an auto-adjustable night-time compression sleeve (MOBIDERM Autofit) in maintenance phase of upper limb lymphedema: the MARILYN pilot RCT - Support Care Cancer (25:2455-2462) - 2017 - Page 6

Lower limb



Several tightening notches

- adaptation to most of patient's individual limb shape
- adjustment marks: precise and repeatable application, control of the edema volume variation

Semi open garment

easy to put on, easy to take off⁽²⁾

Velcro fasteners

easy and quick set up⁽²⁾

Double row

- wide adaptation range
- allows right or left leg use

Anti-slip sole


limits slippage when walking

Lymphedema treatment:
the lymphedema

Lymphedema Treatment:
the reduction phase

Lymphedema Treatment:
the maintenance phase

Sizes

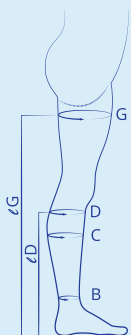

Below-knee

Size	cB (in cm)	cC (in cm)	cD (in cm)	Reference (right/left)			
				Short ℓD = 38 - 40 cm	Normal ℓD = 40 - 42 cm	Long ℓD = 42 - 44 cm	Xtra Long ℓD = 44 - 46 cm
1	16 - 24	24 - 35	25 - 36	37340120100231	37340120100232	37340120100233	37340120100234
2	20 - 28	32 - 43	32 - 43	37340120200231	37340120200232	37340120200233	37340120200234
3	24 - 32	40 - 51	39 - 50	37340120300231	37340120300232	37340120300233	37340120300234
4	28 - 36	48 - 59	46 - 57	37340120400231	37340120400232	37340120400233	37340120400234

Thigh-high

Size	cB (in cm)	cC (in cm)	cG (in cm)	Reference (right/left)			
				Short ℓG = 67 - 72 cm	Normal ℓG = 72 - 77 cm	Long ℓG = 77 - 82 cm	Xtra Long ℓG = 82 - 87 cm
1	16 - 24	24 - 35	46 - 57	37350120100231	37350120100232	37350120100233	37350120100234
2	20 - 28	32 - 43	53 - 65	37350120200231	37350120200232	37350120200233	37350120200234
3	24 - 32	40 - 51	61 - 72	37350120300231	37350120300232	37350120300233	37350120300234
4	28 - 36	48 - 59	68 - 80	37350120400231	37350120400232	37350120400233	37350120400234

ℓD: Floor-to-knee height (measurement taken below the bend of the knee, back side)
 ℓG: Floor-to-thigh height (measurement at the gluteal fold)
 cB: Ankle circumference (at the narrowest point)
 cC: Calf circumference (at the widest point)
 cD: Below knee circumference (on the head of the fibula)
 cG: Thigh circumference (at the widest point)

Scars Treatment

Measurement Forms

MOBIDERM MADE-TO-MEASURE PATIENT

Made-to-measure patient mobilizing garments



Description

The **MOBIDERM** made-to-measure patient garments are designed to mobilize edematous or indurated subcutaneous tissues.

They are intended for the treatment of lymphedema in maintenance phase for day and night to maintain the volume reduction.

They are available in multiple models:

- For upper limb: armsleeve (1), armsleeve with mitten and without thumb (2) or armsleeve with mitten and thumb (3)
- For lower limb: below-knee (4) or thigh-high (5)

For which patients


MOBIDERM made-to-measure patient garments are recommended for patients presenting:

- Moderate to severe lymphedema
- Non-regular limb shape
- Out of standard sizes
- Stable lymphedema volume


Characteristics

Composition⁽¹⁾:

- Outer layer: polyamide, elastane
- Inner layer: adhesive non-woven, polyurethane foam (15 x 15 mm blocks) or polyethylene foam (5 x 5 mm blocks)

Color: beige 

Compression class: around 15 mmHg⁽²⁾

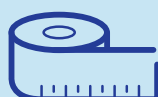

Made-to-measure patient devices require the involvement of a trained healthcare professional.

Measurement forms available on pages 64 to 67

(1) Please refer to the Instructions for Use of the product for the full list of components.

MOBIDERM STANDARD

Standard mobilizing garments

MOBILIZE



Standard products: **faster availability!**



Description

The **MOBIDERM** standard garments are designed to mobilize edematous or indurated subcutaneous tissues.

They are intended for the treatment of lymphedema in maintenance phase for day and night to maintain the volume reduction.

They are available in 3 models: armsleeve (1), mitten (2) or glove (3)

For which patients

MOBIDERM standard garments are recommended for patients presenting:

- Mild to moderate lymphedema
- Regular limb shape
- Stable lymphedema volume

Characteristics


Composition⁽¹⁾:

- Outer layer: polyamide, elastane
- Armsleeve inner layer: adhesive non-woven, polyurethane foam (15 x 15 mm blocks)
- Mitten and glove inner layer: adhesive non-woven, polyethylene foam (5 x 5 mm blocks)

Color: beige 

Compression class: around 15 mmHg⁽²⁾

Sizes

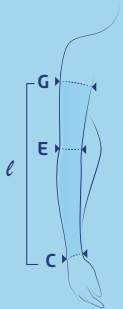


Armsleeve

Size	cC	cE	cG	Reference (right / left)	
				Normal	Long
				ℓ = 40 - 45 cm	ℓ = 45 - 50 cm
1	14 - 16 cm	22 - 25 cm	25 - 29 cm	373002 20100232	373002 20100233
1+		24 - 27 cm	28 - 32 cm	373002 40100232	373002 40100233
2	16 - 18 cm	25 - 28 cm	29 - 33 cm	373002 20200232	373002 20200233
2+		27 - 30 cm	31 - 35 cm	373002 40200232	373002 40200233
3	18 - 20 cm	28 - 31 cm	32 - 36 cm	373002 20300232	373002 20300233
3+		30 - 33 cm	35 - 39 cm	373002 40300232	373002 40300233
4	20 - 22 cm	31 - 34 cm	35 - 39 cm	373002 20400232	373002 20400233
4+		33 - 36 cm	38 - 42 cm	373002 40400232	373002 40400233
5	22 - 24 cm	34 - 37 cm	38 - 42 cm	373002 20500232	373002 20500233
5+		36 - 39 cm	41 - 45 cm	373002 40500232	373002 40500233

Glove and Mitten

Size	cC	cA	Glove		Mitten
			Reference right	Reference left	Ref (right / left)
1	14 - 16 cm	15 - 17 cm	373201 20100201	373201 20100202	373101 20100299
2	16 - 18 cm	17 - 19 cm	373201 20200201	373201 20200202	373101 20200299
3	17 - 19 cm	19 - 21 cm	373201 20300201	373201 20300202	373101 20300299
4	19 - 21 cm	21 - 23 cm	373201 20400201	373201 20400202	373101 20400299
5	20 - 22 cm	23 - 25 cm	373201 20500201	373201 20500202	373101 20500299
6	21 - 23 cm	25 - 27 cm	373201 20600201	373201 20600202	373101 20600299

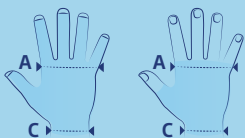

A: Hand circumference (head of metacarpal bones)
 C: Wrist circumference
 E: Elbow circumference
 G: Axillary arm circumference

(1) Please refer to the Instructions for Use of the product for the full list of components.

(2) Mestre - Interest of an auto-adjustable night-time compression sleeve (MOBIDERM Autofit) in maintenance phase of upper limb lymphedema: the MARILYN pilot RCT - Support Care Cancer (25:2455-2462) - 2017




Fitting Instructions

MOBIDERM garments can be worn directly on the skin or with a cotton layer in between in case of sensitive skin.


MOBIDERM MADE-TO-MEASURE PATIENT OR STANDARD

- Turn the garment over onto itself ①
- Slide the garment along the limb to the desired position ②
- Massage the limb to ensure an even distribution of foam blocks ③
- *For Mitten and Glove* - Make sure the foam blocks are positioned on top of the hand
- *For Mitten and Glove* - If combined with an armsleeve, position it over the armsleeve
- *For Mitten* - The garment is ambidextrous, simply flip it inside

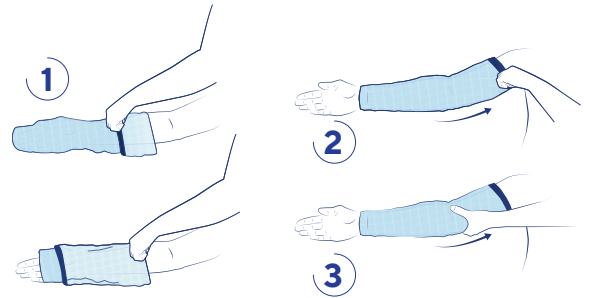

MOBIDERM autofit

- Open all hook-and-loop fasteners ①
- Slide the garment along the limb to the desired position ②
- Close the hook-and-loop fasteners by starting with the distal extremity and moving up ③
- Proceed in several steps until you reach the setting recommended by your healthcare professional (+/++/+++/++++/+++++)
- Massage the limb to ensure an even distribution of foam blocks ④

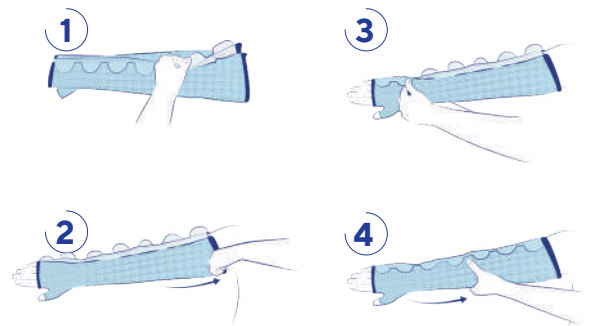

Care Instructions

It is recommended to wash your **MOBIDERM** garment daily (or after each use).



ALL MOBIDERM GARMENTS

- Machine wash at wool cycle or cold temperature.
- Use a washing product without softener. Do not use detergents, fabric softeners or aggressive products (products containing chlorine).
- Place the garment in a wash bag to make sure other garments (particularly those with touch fasteners or zips) do not damage it.
- Dry well away from any direct heat source (radiator, sun, etc.).
- Do not tumble-dry.
- Do not iron.

MOBIDERM autofit

When not used, the garment should be kept flat, with the velcro fasteners closed. If it must be folded, position the hooks pointing outward.



Discover all available video tutorials presenting **MOBIDERM autofit** fitting instructions on our **Thuasne YouTube channel!**

MAJOR BURNS

Burns are a pathology characterized by a very wide spectrum of potential severity.

There is a large difference between a superficial burn of a few square centimeters (which heals without sequelae within a few days) and a deep burn covering most of the skin, which is both life-threatening and compromising to the patient's functions.⁽¹⁾

Burns are very frequent cause of injury. Worldwide, an estimated 6 million patients seek medical help for burns annually.⁽¹⁾


The decision to treat in a specialized unit depends on the type of burns and their location, the depth of the lesions and the surface area burnt on the body⁽²⁾.

The treatment of scars with compression

With a second degree deep burn, the patient is confronted to a major risk of having a hypertrophic scar possibly escalating into a keloid⁽²⁾.

Wearing compression garments reduces⁽²⁾:

- Blood flow (reduced flow of oxygen)
- Fibroblast proliferation
- Collagen synthesis
- The risk of local inflammation (prurit, edema...)

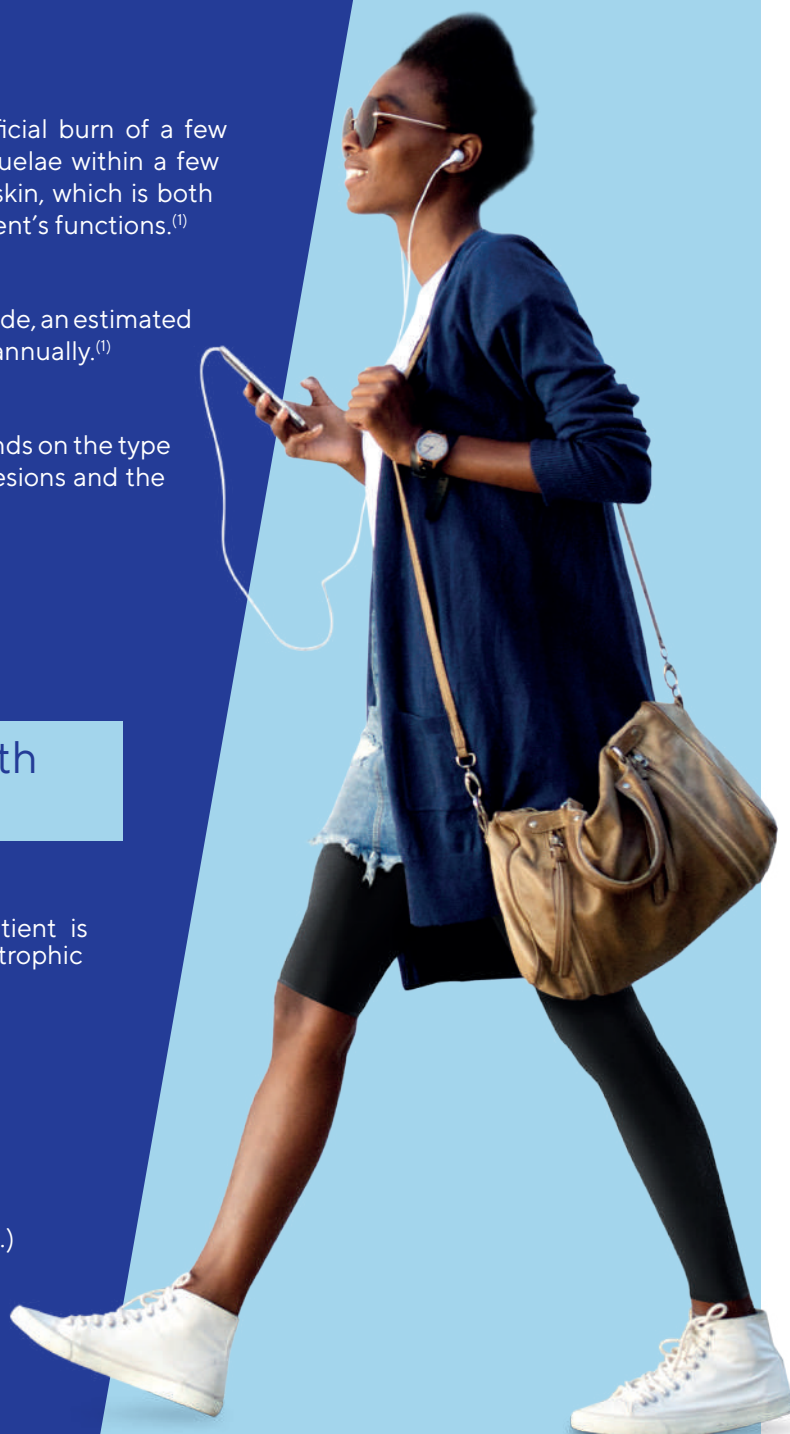

Lymphedema treatment:
the lymphedema

Lymphedema Treatment:
the reduction phase

Lymphedema Treatment:
the maintenance phase

Scars Treatment

Measurement Forms

(1) Brusselselaers et al. - Severe burn injury in Europe: A systematic review of the incidence, etiology, morbidity, and mortality - Critical Care (14):R188 - 2010 - Page 1
(2) Rochet - Rééducation et réadaptation de l'adulte brûlé - Encyclopédie Médico Chirurgicale (26-280-C-10) - 1998 - Page 1 & 5

CICATREX

Compression garments

Description


Cicatrex garments offer:

- A large range of fabrics meeting most of patients' needs.
- A precise fabrication process thanks to digitalization of measurement methods for making the garments.
- Training methods adapted to the prescription and to taking the patient's measurements.


Principle of action⁽¹⁾

The Cicatrex compression principle contributes to a reorganization of collagen fibers in a plane parallel to the skin, thus limiting uncontrolled scarring of the conjunctive tissue.


Inaesthetic scars and dysfunctional adhesions are reduced by compression.

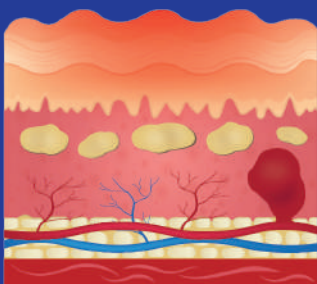

• Hypertrophic scar before treatment

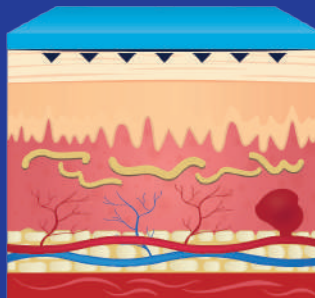

• Effect of compression during treatment
• Reduction of hypervascularisation
• Reorganisation of fibroblasts

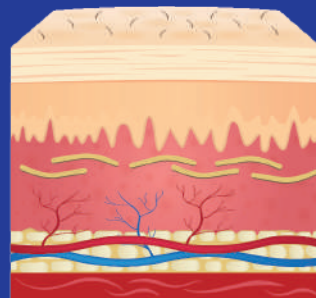

• Flatter scar after treatment

🔍 Compression garment usage

It is during the scar maturation phase that the effectiveness of compression garments like Cicatrex will be the most adapted.

Compression garments should be used as soon as healing permits.

For which patients

Cicatrex compression garments are a treatment method in the management of major burns and help limiting recourse to reconstructive surgery.

However, the benefits are not restricted to burn healing, they can also be used in the treatment of other conditions such as lymphology or Ehlers-Danlos syndrome.

Burns⁽¹⁾

- Prevention and treatment of hypertrophic scars, keloids and scar retractions.
- Treatment of thermal, chemical or electrical burns.
- Compression of skin graft zones.

Lymphology⁽²⁾

- Pubian, genital lymphedema.
- Primary lymphedema in children
- Maintenance of lymphedema volume reduction

Plastic and reconstructive surgery

- Prevention of hypertrophic scars.⁽¹⁾

Ehlers-danlos syndrome⁽³⁾

- Improved proprioception thanks to compression of the limbs.
- Pain alteration.
- Compression from 6 to 13.5 mmHg.



(1) Monstrey S, et al. - Updated Scar Management Practical Guidelines: Noninvasive and invasive measures - Journal of Plastic, Reconstructive & Aesthetic Surgery - 2014

(2) HAS Report - Medical compression in lymphedema therapy - 2010


(3) HAS Report - Compressive garments in the treatment of Ehlers-Danlos syndrome - Jan 2010 - Page 4

Fabrics


Available from Thuasne France only

CICATREX FILIFINE


Cicatrex Filifine compression garments are made of extra-fine, high-tech fibres. The fineness of the Cicatrex Filifine fabric makes it easier to fit garments to fingers and toes.

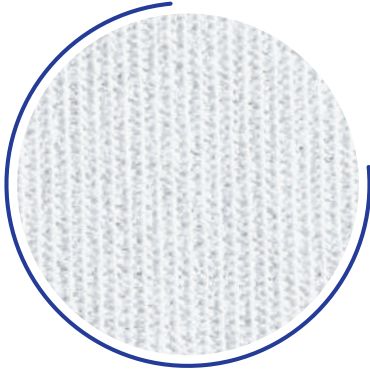

Made-to-measure

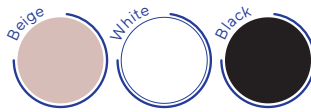


Standard

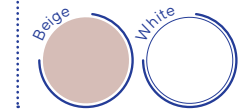


Available from Thuasne France and Thuasne Germany

CICATREX NATURE


Cicatrex Nature compression garments contain a high proportion of viscose. Made from natural raw materials (wood), this fibre is soft on the skin, permeable and breathable, making garments very comfortable to wear.

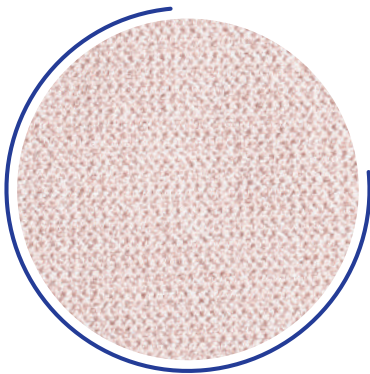

Made-to-measure

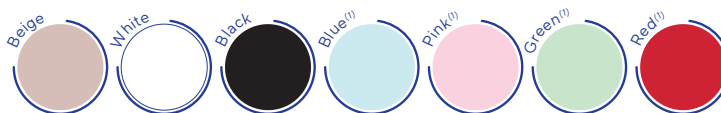

Standard

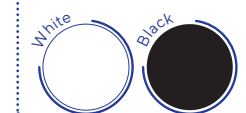


CICATREX AIRSKIN/SKIN


The quality of Cicatrex AirSkin and Cicatrex Skin stands out thanks to its fine, smooth, light knit, with silky tones.

The composition of Cicatrex AirSkin and Cicatrex Skin compression garments makes them both aesthetically pleasing and easy to care for.


For EDS, only the glove, foot glove and ankle support models are available.

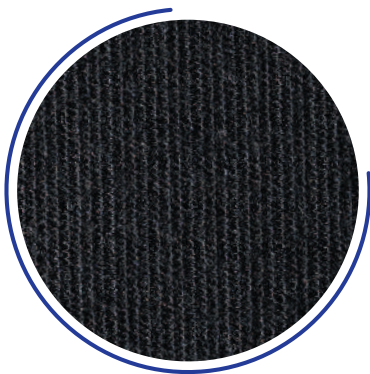


Made-to-measure

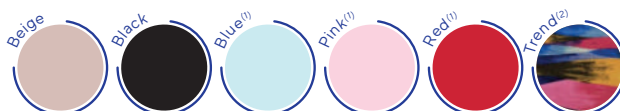


Standard

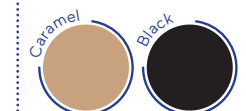


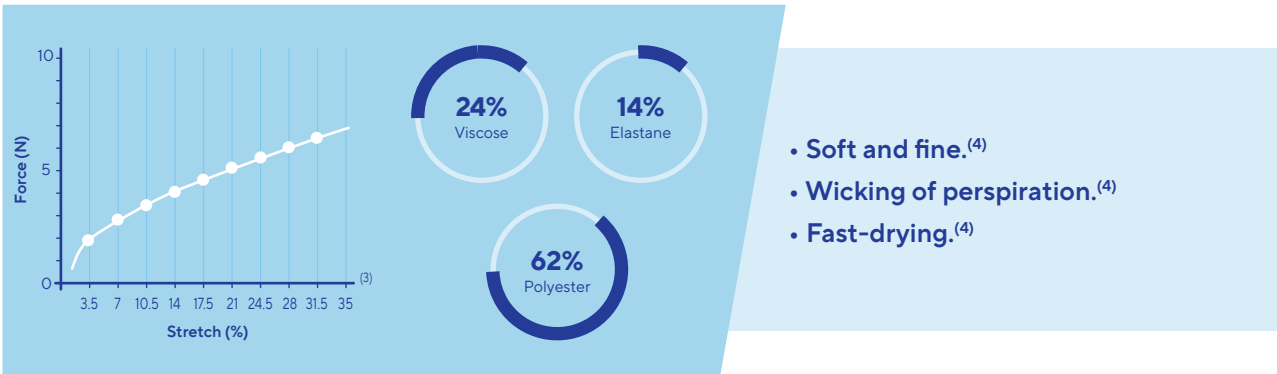

- **Soft and fine.**⁽⁴⁾
- **Wicking of perspiration.**⁽⁴⁾
- **Fast-drying.**⁽⁴⁾

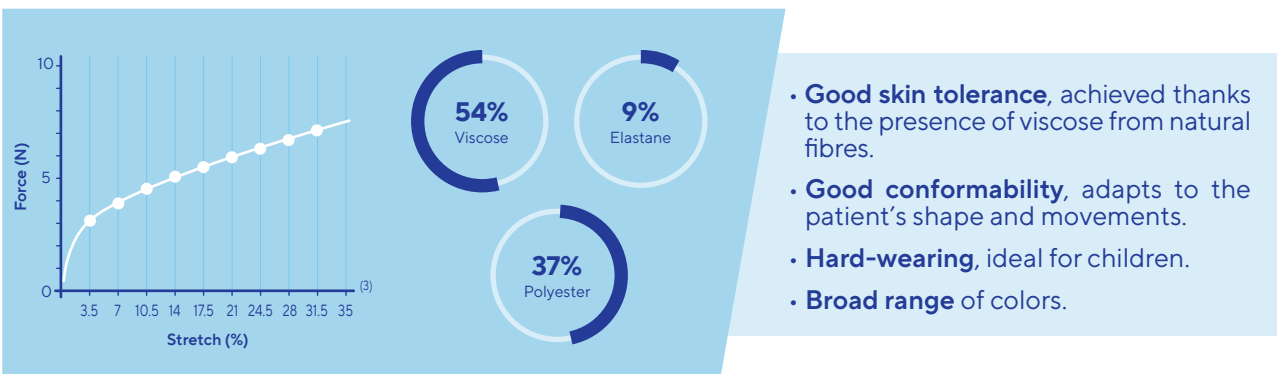

- **Good skin tolerance**, achieved thanks to the presence of viscose from natural fibres.
- **Good conformability**, adapts to the patient's shape and movements.
- **Hard-wearing**, ideal for children.
- **Broad range** of colors.

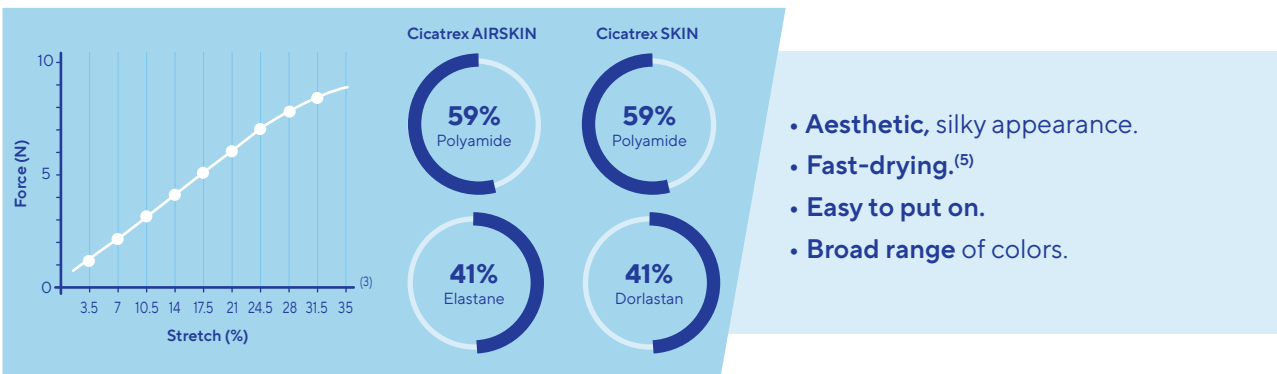

- **Aesthetic**, silky appearance.
- **Fast-drying.**⁽⁵⁾
- **Easy to put on.**
- **Broad range** of colors.



(1) Some products and Colors are only available from Thuasne Germany. Please consult your Thuasne partner.
 (2) For Cicatrex Skin only
 (3) Internal tests
 (4) ETN 274 - Dossier Textile Elasthane - July Aug 2016 - Page 1
 (5) ETN 280 - Dossier Textile Polyamide - July Aug 2017 - Page 1

Fabrics

Available from Thuasne Germany only


CICATREX RIGID

Cicatrex Rigid is a flat-knit fabric that is resistant to wear thanks to its composition. Consequently, it is particularly suitable for children or very active people.

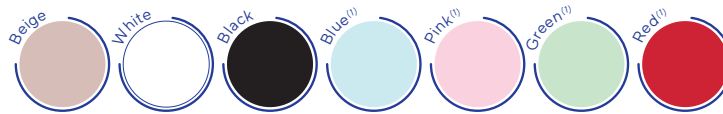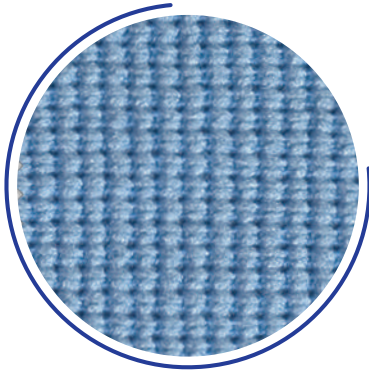

CICATREX SILVER

Cicatrex Silver is a flat-knit fabric incorporating a silver yarn. Silver naturally eliminates bacteria from the skin and protects against infections.⁽¹⁾

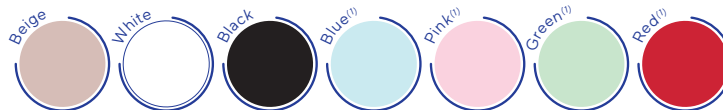


The breadth of the Cicatrex range ensures it meets the very different needs of patients and the requirements of healthcare professional. Whether the aim is to produce a fine, aesthetic compression garment or a very stretchy garment that is easy to put on, whether hard-wearing properties are a priority or a specific color is sought, it is possible to adapt the compression garment to optimally meet patients' requirements without compromising the treatment.




Cicatrex garments are made of long stretch elastic fabrics.

This high stretch capacity makes the garments easy to put on and remove. In addition, it helps freedom of movement and offers a good level of adaptation to variations in circumferences.

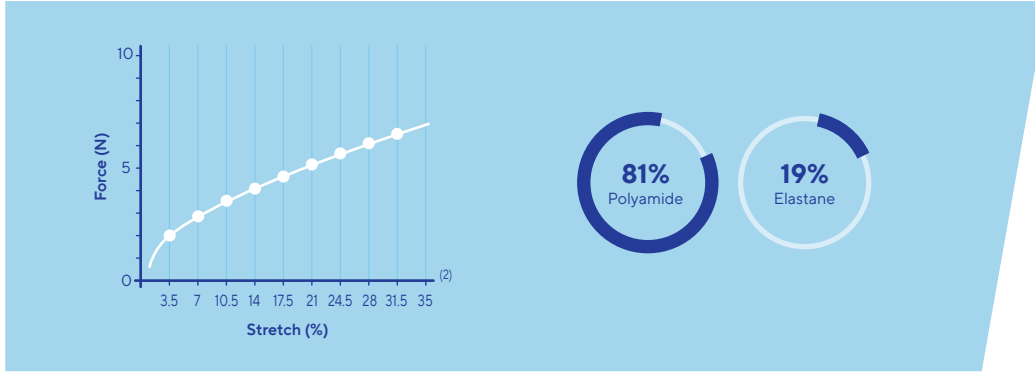


Lymphedema treatment:
the lymphedema

Lymphedema Treatment:
the reduction phase

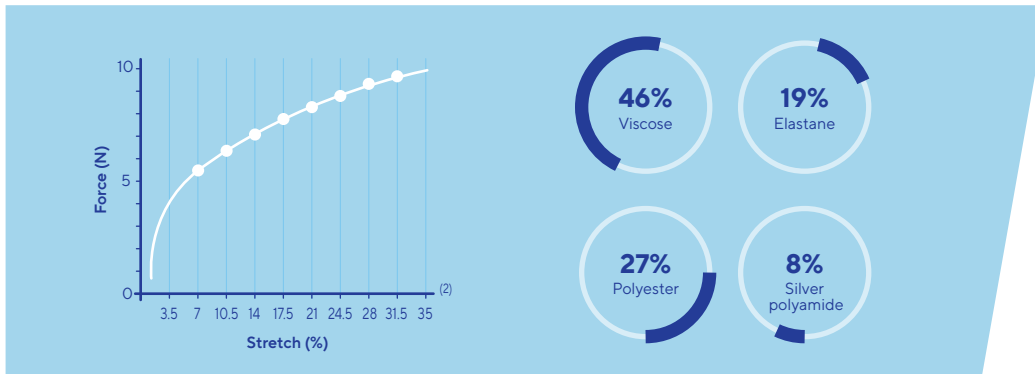

Lymphedema Treatment:
the maintenance phase

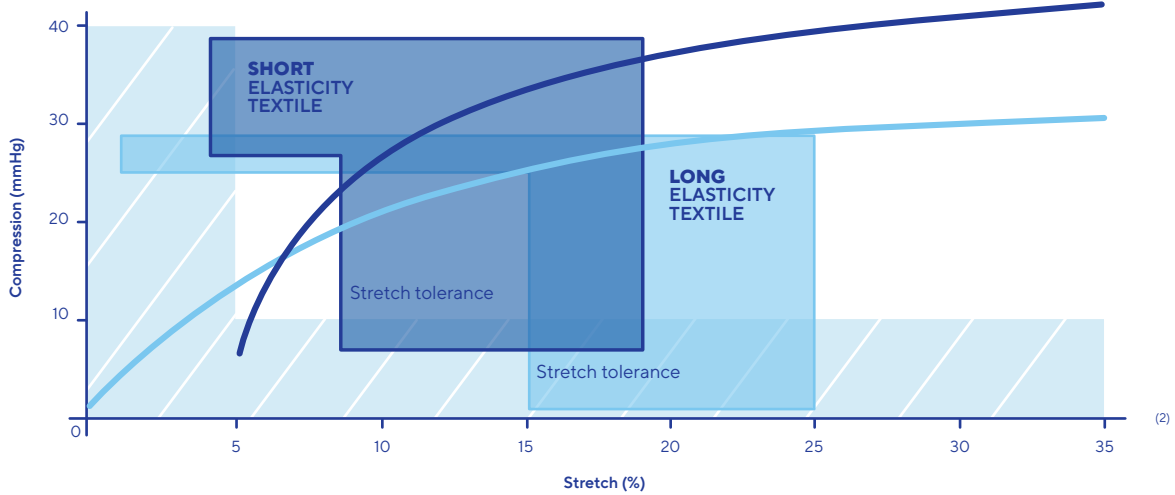

Scars Treatment

Measurement Forms



Fitting Instructions



- Stretch the elastic fabric as much as possible so that fragile skin is not irritated while the garment is being pulled on.
- Help the fabric to fit all the contours of the covered body parts evenly, and to be properly positioned without getting in the way of clothing and without being too tight.
- Pull up sleeves if they are too long.
- Make sure there is freedom of movement at the elbow and shoulder area.

Care Instructions



- Always wash Cicatrex by hand, as often as you want, using soap or a product designed for delicate fabrics. Do not add bleach or stain-remover. Rinse thoroughly and squeeze out excess water without twisting.
- Dry flat away from any direct heat (do not dry on a radiator, for example).
- Do not iron.



Advise 2 Cicatrex garments

If possible, advise two Cicatrex garments of the same form available so that the patient can continue to wear one when the other is being washed.

CICATREX STANDARD

Compression garments




Standard products:
faster availability!

LOWER LIMB MEASUREMENT TABLE

(panty girdle, panty, capri pants, trousers)

The measurement line for the pelvis circumference (A) must be located 3 or 4 cm above the pubic crest at the front and at the level of the upper median of the buttocks at the back.

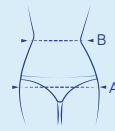

Size	Pelvis circumference (A)	Waist circumference (B)
1	88 - 94 cm	62 - 72 cm
2	95 - 101 cm	67 - 79 cm
3	102 - 108 cm	73 - 85 cm
4	109 - 115 cm	80 - 93 cm
5	116 - 122 cm	90 - 104 cm
6	123 - 129 cm	97 - 113 cm

PANTY GIRDLE - Cicatrex Nature

Size	Reference White	Reference Black
1	341501 20100099	
2	341501 20200099	
3	341501 20300099	
4	341501 20400099	
5	341501 20500099	
6	341501 20600099	

PANTY - Cicatrex Nature

Size	Reference White	Reference Black
1	340301 20100099	340301 20100399
2	340301 20200099	340301 20200399
3	340301 20300099	340301 20300399
4	340301 20400099	340301 20400399
5	340301 20500099	340301 20500399
6	340301 20600099	340301 20600399

CAPRI PANTS - Cicatrex Nature

Size	Reference White	Reference Black
1	341201 20100099	341201 20100399
2	341201 20200099	341201 20200399
3	341201 20300099	341201 20300399
4	341201 20400099	341201 20400399
5	341201 20500099	341201 20500399
6	341201 20600099	341201 20600399

TROUSERS - Cicatrex Nature

Size	Reference White	Reference Black
1	342001 20100099	342001 20100399
2	342001 20200099	342001 20200399
3	342001 20300099	342001 20300399
4	342001 20400099	342001 20400399
5	342001 20500099	342001 20500399
6	342001 20600099	342001 20600399

HOOD - Cicatrex Nature

Size	Head circumference (A)	Reference White
0	51 - 52 cm	340201 20000099
1	53 - 55 cm	340201 20100099
2	56 - 57 cm	340201 20200099
3	58 - 59 cm	340201 20300099

PARTIAL HOOD - Cicatrex Nature

Size	Neck circumference (B)	Reference White
1	27 - 30 cm	341001 20100099
2	31 - 36 cm	341001 20200099
3	37 - 45 cm	341001 20300099

OPEN HOOD - Cicatrex Nature

Size	Head circumference (A)	Reference White
0	51 - 52 cm	341601 20000099
1	53 - 55 cm	341601 20100099
2	56 - 57 cm	341601 20200099
3	58 - 59 cm	341601 20300099

EAR BAND - Cicatrex Nature

Size	Head circumference (A)	Reference White
1	40 - 52 cm	341109 1100
2	52 - 64 cm	341109 1200

Lymphedema treatment:
the lymphedema

Lymphedema Treatment:
the reduction phase

Lymphedema Treatment:
the maintenance phase

Scars Treatment

Measurement Forms




CICATREX STANDARD

Compression garments

Standard products:
faster availability!


BELT - Cicatrex Nature

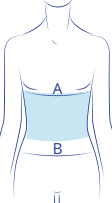

Size	Thoracic or abdominal circumference (A or B) ⁽¹⁾	Reference White
1	70 - 92 cm	340109 1100
2	93 - 114 cm	340109 1200


(1) Measurement taken below the chest or at the hips

SLEEVELESS VEST - Cicatrex Nature

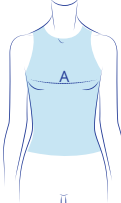


Size	Chest circumference (A)	Reference White
1	74 - 82 cm	342401 20100099
2	83 - 91 cm	342401 20200099
3	92 - 98 cm	342401 20300099
4	99 - 115 cm	342401 20400099
5	116 - 120 cm	342401 20500099
6	121 - 135 cm	342401 20600099

LONG SLEEVE - Cicatrex Nature

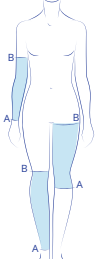

Size	Distal circumference (A)	Proximal circumference (B)	Height (AB)	Reference White
1	9 - 11 cm	19 - 23 cm	35 cm	342501 20100099
2	11 - 14 cm	23 - 28 cm		342501 20200099
3	14 - 16 cm	28 - 32 cm		342501 20300099
4	16 - 20 cm	32 - 37 cm		342501 20400099
5	20 - 25 cm	37 - 42 cm	45 cm	342501 20500099
6	25 - 30 cm	42 - 48 cm		342501 20600099
7	30 - 40 cm	48 - 58 cm		342501 20700099
8	40 - 50 cm	58 - 67 cm		342501 20800099


SLEEVE - Cicatrex Nature

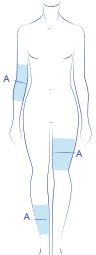

Size	Medial circumference (A) ⁽²⁾	Height	Reference White
1	12 - 14 cm	20 cm	340001 20100099
2	15 - 17 cm		340001 20200099
3	18 - 22 cm	26 cm	340001 20300099
4	23 - 29 cm		340001 20400099
5	30 - 37 cm		340001 20500099
6	38 - 45 cm		340001 20600099
7	46 - 54 cm		340001 20700099
8	55 - 63 cm		340001 20800099


(2) Measurement taken at the middle of the conical sleeve.

SLEEVE FOR LYMPHEDEMA - Cicatrex Skin

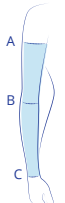

Size	Distal circumference (A)	Medial circumference (B)	Proximal circumference (C)	Height (AC)	Reference Caramel
1	13 - 15 cm	20 - 25 cm	22 - 28 cm	39 cm	344902 20100299
2	15 - 17 cm	23 - 28 cm	25 - 31 cm	39 cm	344902 20200299
3	17 - 19 cm	26 - 31 cm	29 - 35 cm	39 cm	344902 20300299
4	19 - 21 cm	29 - 34 cm	33 - 39 cm	39 cm	344902 20400299
5	21 - 23 cm	32 - 37 cm	37 - 43 cm	39 cm	344902 20500299
6	23 - 26 cm	35 - 41 cm	41 - 48 cm	39 cm	344902 20600299


SKIN LADDER-PROOF GLOVE MEASUREMENT TABLE

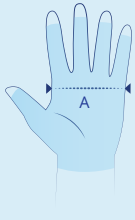

Size	Hand circumference (A)
1	13 - 14.5 cm
2	15 - 16.5 cm
3	17 - 18.5 cm
4	19 - 20.5 cm
5	21 - 22.5 cm
6	23 - 24.5 cm
7	25 - 26.5 cm
8	27 - 28.5 cm

SKIN LADDER-PROOF GLOVE - Cicatrex Skin

Closed-finger long sleevelet (elbow)

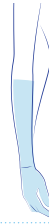

Size	Reference Black	Reference Caramel
1	344805 20100399	344805 20100299
2	344805 20200399	344805 20200299
3	344805 20300399	344805 20300299
4	344805 20400399	344805 20400299
5	344805 20500399	344805 20500299
6	344805 20600399	344805 20600299
7	344805 20700399	344805 20700299
8	344805 20800399	344805 20800299

SKIN LADDER-PROOF GLOVE - Cicatrex Skin

Closed-finger short sleevelet (wrist)



Size	Reference Black	Reference Caramel
1	344806 20100399	344806 20100299
2	344806 20200399	344806 20200299
3	344806 20300399	344806 20300299
4	344806 20400399	344806 20400299
5	344806 20500399	344806 20500299
6	344806 20600399	344806 20600299
7	344806 20700399	344806 20700299
8	344806 20800399	344806 20800299

SKIN LADDER-PROOF GLOVE - Cicatrex Skin


Open-finger short sleevelet (wrist)



Size	Reference Black	Reference Caramel
1	344807 20100399	344807 20100299
2	344807 20200399	344807 20200299
3	344807 20300399	344807 20300299
4	344807 20400399	344807 20400299
5	344807 20500399	344807 20500299
6	344807 20600399	344807 20600299
7	344807 20700399	344807 20700299
8	344807 20800399	344807 20800299

FILIFINE MICROFIBRE GLOVE - Cicatrex Filifine


Open fingers

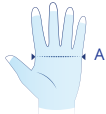

Size	Hand circumference (A)	Reference White	Reference Beige
1	13 - 14.5 cm	344701 20100099	344701 20100299
2	15 - 16.5 cm	344701 20200099	344701 20200299
3	16.5 - 18 cm	344701 20300099	344701 20300299
4	18.5 - 20 cm	344701 20400099	344701 20400299
5	20.5 - 22 cm	344701 20500099	344701 20500299
6	22.5 - 24 cm	344701 20600099	344701 20600299

FINGER SEPARATOR

51% Polyester/45% Polyamide/4% Elastane

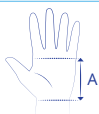

Size	Height (A) ⁽¹⁾	Reference White
1	< 9 cm	341309 1100
2	> 9 cm	341309 1200

(1) Measurement taken from wrist fold to middle finger base

FINGER PROTECTOR

Cicatrex Skin



Size	Finger circumference (A) ⁽²⁾	Height	Reference Caramel
1	25 - 27 mm	100 - 110 mm	344601 20100299
8	29 - 31 mm		344601 20200299

(2) Measurement taken at the base of the finger

OPEN FOOT GLOVE - Cicatrex Skin



Size	Circumference (A) ⁽³⁾	Reference Black	Reference Caramel
1	21 - 22 cm	340603 20100399	340603 20100299
2	23 - 24 cm	340603 20200399	340603 20200299
3	25 - 26 cm	340603 20300399	340603 20300299
4	27 - 28 cm	340603 20400399	340603 20400299

(3) Measurement taken at the head of the metatarsal bones.



Some models are also available in the made-to-measure patient range.




CICATREX

Made-to-Measure Patient


Compression garments

Torso	
Models	Vest Body Shoulder pad
Fabrics	Cicatrex Nature Cicatrex Filifine Cicatrex AirSkin
Collars	V-neck Round neck Stand-up collar
Openings	Zipper Hooks to help fasten the vest
Other options	Compression pad Crotch strap Cup with or without compression Armpit with or without compression Belt Additional openings (zipper, Velcro fastening) Velcro for attaching to panty Soft fabric lining

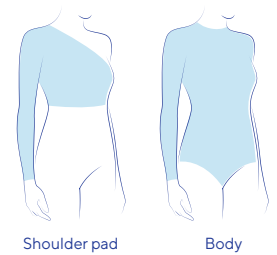


Shoulder pad

Body

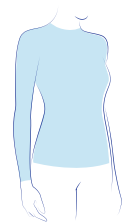

Vest

Hand	
Models	Mitten (<i>length on wrist 5 cm max</i>) Glove (<i>length on wrist 5 cm max</i>) Finger protector
Fabrics	Cicatrex Filifine Cicatrex AirSkin
Proximal end options	With sleeve (<i>beyond 5 cm and up to the armpit with length on wrist 5 cm max</i>): • attached • separate
Openings	Zipper/Velcro: • palm of hand • back of hand • little finger side
Finger protector options	With strap Without strap
Other options	Compression pad: • palm of hand • back of hand • fixed • removable Palm support

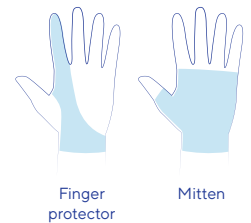


Finger protector

Mitten



Glove


Arm	
Model	Armsleeve
Fabrics	Cicatrex Nature Cicatrex Filifine Cicatrex AirSkin
Fasteners	Without anti-slip Elastic plain braid Anti-slip with silicone dots 3 cm Silicone anti-slip for men 5 cm Silicone anti-slip for women 5 cm Shoulder cover Shoulder attachment Up to the armpit
Openings	Zipper/Velcro • join side • back of hand side • palm of hand side
Other option	Compression pad

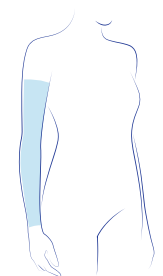

Armsleeve



Head

Models	Chin band Hood
Fabrics	Cicatrex Nature Cicatrex AirSkin
Face	Uncovered Covered
Ridge of nose	Uncovered Covered
Ears	Uncovered Covered
Other option	Compression pad

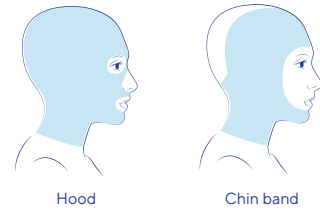

Leg

Models	Sock Panty Leg only Half-tight
Fabrics	Cicatrex Nature Cicatrex Filifine Cicatrex AirSkin
Stomach openings	Zipper/Velcro: • front/back • right/left
Crotches	Open with or without hygiene tab Closed with or without gusset Kangaroo pouch
Leg openings	Zipper/Velcro: • front/back • inner/outer
Fasteners	Belt Braces Without anti-slip Elastic plain braid 3 cm Anti-slip with silicone dots 3 cm Silicone anti-slip for men 5 cm Silicone anti-slip for women 5 cm Vest over panty Velcro Panty over vest Velcro
Foot	Attached/separated Open/closed toe
Other options	Compression pad Soft fabric lining (knee/heel)

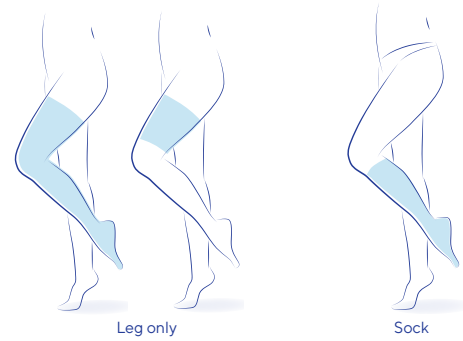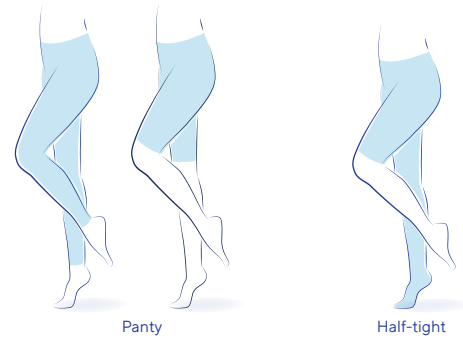

Foot

Model	Foot glove
Fabrics	Cicatrex Filifine Cicatrex AirSkin
Toes options	Open Closed
Other option	Compression pad




Lymphedema treatment:
the lymphedema

Lymphedema Treatment:
the reduction phase

Lymphedema Treatment:
the maintenance phase



Scars Treatment

Measurement Forms

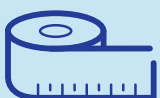

Made-to-measure patient devices require the involvement of a trained healthcare professional.

Measurement forms available on pages 80 to 87

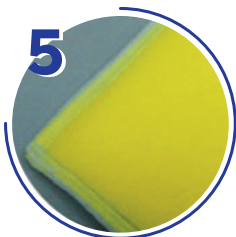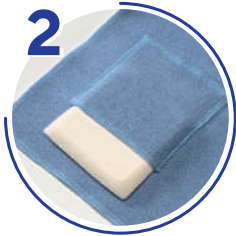

CICATREX

Made-to-Measure Patient

Compression garments

Zoom on some options

1 Fastenings

- Make it easier to put on and take off the garment, particularly for immobile patients.
- Their position is customized depending on the shape of the garment.

2 Compression pad

- Used to locally increase the pressure exerted on the skin.
- Sewn into the garment or removable.

3 Non-compressive fabric

- For more comfort, zones of non-compressive fabric are added at the armpits, chest but also between fingers if treatment does not require any compression at these regions.

4 Seams

- The flat seams are on the outside, to limit irritation of sensitive skin.

5 Memory foam

- Compresses inequalities and distributes pressure.
- Located into a pocket

6 Finger separators


- Reinforced compression between the fingers.
- Suitable for use with a glove.

7 Silicon-tex and sewn-in silicone sheets

- Soften scar tissue

8 Iron-on patches ⁽¹⁾

- To customize your garment.
- Full list of iron-on patches available on request.

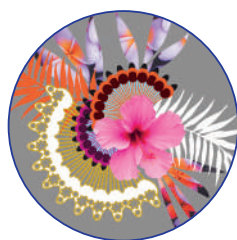

(1) Non-contractual photographs. Non-exhaustive list.

Made-to-measure patient by Thuasne in few figures



85 persons

dedicated to give you and your patients satisfaction

Fast delivery time with



98%

of orders shipped within 3 days



About

80,000

products made every year




All measurement forms available from www.thuasne.com



CAMMK technology


Computer Assisted Made-to-Measure knitting provides devices fully adapted to patient body shape

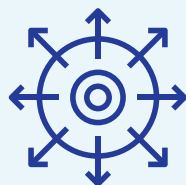

A large offering of

400

product references (all ranges considered)

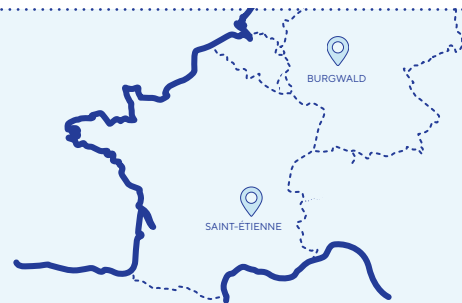

2 dedicated manufacturing sites

IN FRANCE (St-Etienne)

Mobiderm - Cicatrex
Lymphatrex Essential
Lymphatrex Expert

IN GERMANY (Burgwald)

Cicatrex
Lymphatrex

Measurement

A few essential reminders

- Measurements must be taken by a healthcare professional trained in made-to-measure lymphology devices and to the specificities of Thuasne measurements for the delivery of our devices.
- Proper adjustment of the garment on the patient will depend on the transmission of correct measurements.
- Measurements are preferably made in the morning or after manual lymphatic drainage and at the end of the intensive treatment in case of first use of the device.

General protocol

- 1 - Select the measurement form corresponding to the desired range and type of garment.
- 2 - Read the whole form.
- 3 - Complete the header information.
- 4 - Select the desired model, draw it on the form and cross out useless measures.
- 5 - Select the desired options.
- 6 - Draw all anatomical markers on the patient based on the measurement form guidelines.
- 7 - Measure circumferences at these various markers.
- 8 - Measure heights associated with the markers.

Take the time to take those measurements and do not hesitate to attach photographs or additional measures to your request.

Tools required

- Measurement forms
- Dermatological felt-tip pen, or kohl pencil to point the markers
- Measuring tape (for arm/leg)
- "Lasso" tape (for fingers/toes)
- Flexible ruler
- Wipes



From your order intake
to delivery to the patient

*Send your request
(order or simple quotation)
to your regular Thuasne distributor*

Important

Upon reception of your request, it is analyzed to ensure that no information is missing. In case of missing information, you will be contacted by your distributor.

In case of a simple quotation, please tick the "Quotation" box on the selected form.

When you receive your order

Try the garment on your patient to validate:

- that it is adapted to his/her body shape
- the method for putting on and removing the device

Do not forget to instruct the patient on recommended care.



The dimensions of the flat-lying product may be misleading, **the only way to confirm that it is properly adapted is by putting it on.**

There may be differences between several products knitted according to the same measurements at different periods (e.g. in case of renewal).

This is a normal phenomenon as several parameters have an influence on the dimensions of the product: temperature, hydrometry during the fabrication process, the type of knitting machine used,...

Content

MOBIDERM

Sleeves.....	64-65
Stockings.....	66-67

LYMPHATREX Essential

Sleeves.....	68-69
Stockings.....	70-71

LYMPHATREX Expert

Armsleeve.....	72-73
Mitten & glove.....	74-75
Stockings.....	76-77
Toe cap.....	78

Cicatrex

Torso.....	80-81
Hand.....	82
Arm.....	83
Leg.....	84-85
Foot.....	86
Head.....	87

Lymphedema treatment:
the lymphedema

Lymphedema Treatment:
the reduction phase

Lymphedema Treatment:
the maintenance phase

Scars Treatment

Measurement Forms

MOBIDERM


SLEEVES

Anatomical Markers

- G** corresponds to the axilla (armpit)
- F** corresponds to the widest point of the biceps
- E** is located at the level of the olecranon
- D** equidistant between A and C or at the starting site of the edema
- C** to be measured at the narrowest point of the wrist external face in the middle of the radial ulnar joint


The arm must be slightly bended when taking measures.

→ *Positioning of the measuring tape to measure heights.*

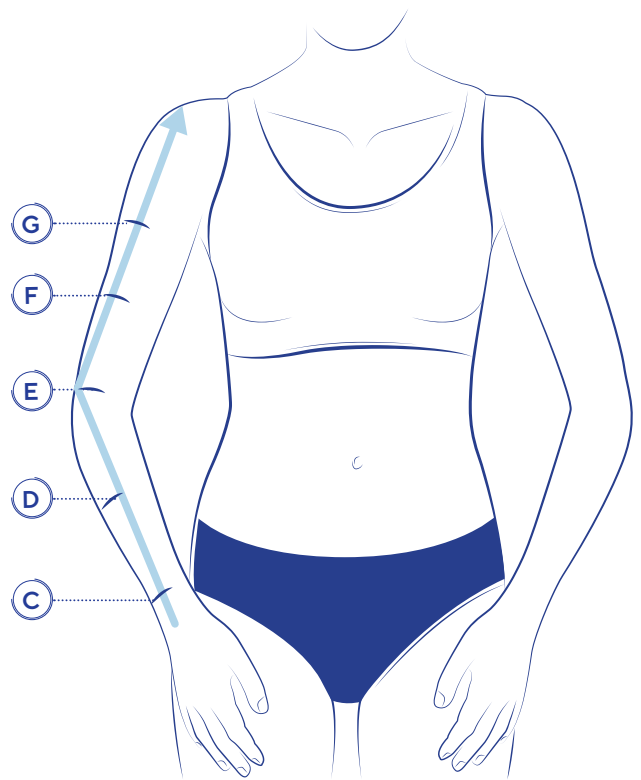


Measurement Instructions

ARMSLEEVE

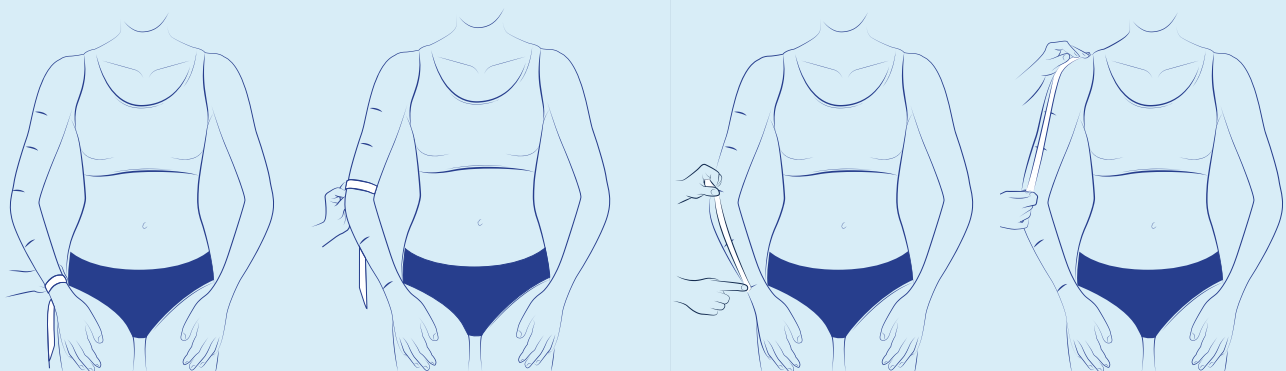


The lengths must be measured from the middle of the wrist to the tip of the elbow, then at the outside of the arm up to the desired finished height.

MITTEN

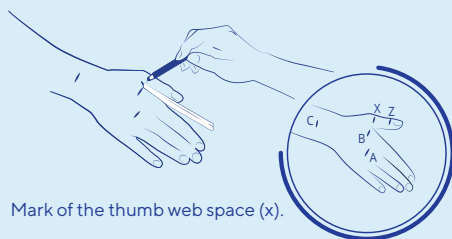


Mark of the thumb web space (x).

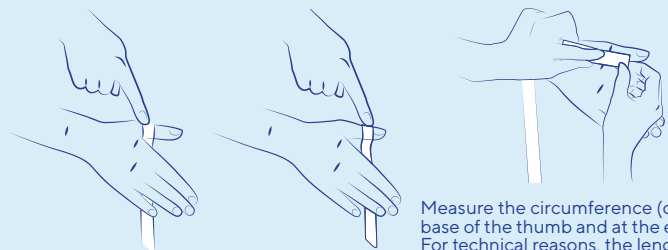

Measure the circumference (cX) at the base of the thumb and at the end (cZ). For technical reasons, the length of the thumb (XZ) cannot be less than 2 cm.



The circumference at the end of the mitten (cA) is located at the head of the metacarpal bones.

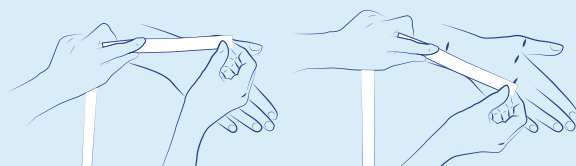

Measure the length of the mitten:
 1/ from the wrist (C) to the base of the thumb (X)
 2/ from the wrist (C) to the finished height (A)



THUASNE

MOBIDERM



SLEEVES

ORDER (by default) **QUOTATION** **RENEWAL**

Customer code

Case No. for renewal

1st treatment

Date: Quantity:

Patient's surname:

Patient's first name:

Gender: M F Child Patient's height:

I authorize my health care professional to collect my data and to communicate them to Thuasne company as part of the processing of my made to measure medical device in accordance with Law No 78-17 of 6 January 1978, and European Regulation No 2016/679/EU of 27 April 2016. I have rights including in particular the rights of access, rectification, portability and deletion of my data. I can exercise these rights by contacting the health care professional from whom I ordered my medical device.

PATIENT SIGNATURE

RETAILER IDENTIFICATION

**If possible, please enclose photos of the limb to be fitted.
Please draw in the contours of the garment on the diagram and cross unnecessary measures.**

RIGHT ARM **LEFT ARM**

Fill out one form for each side

FACING VIEW

Models

- Armsleeve (big blocks only)
- Armsleeve with mitten with thumb (big blocks on the arm and small blocks on the hand)
- Armsleeve with mitten without thumb (big blocks on the arm and small blocks on the hand)

Sleeve options

- Anti-slip with silicone dots 3 cm
- Velcro opening

Comments

.....



.....

.....

.....

.....

.....

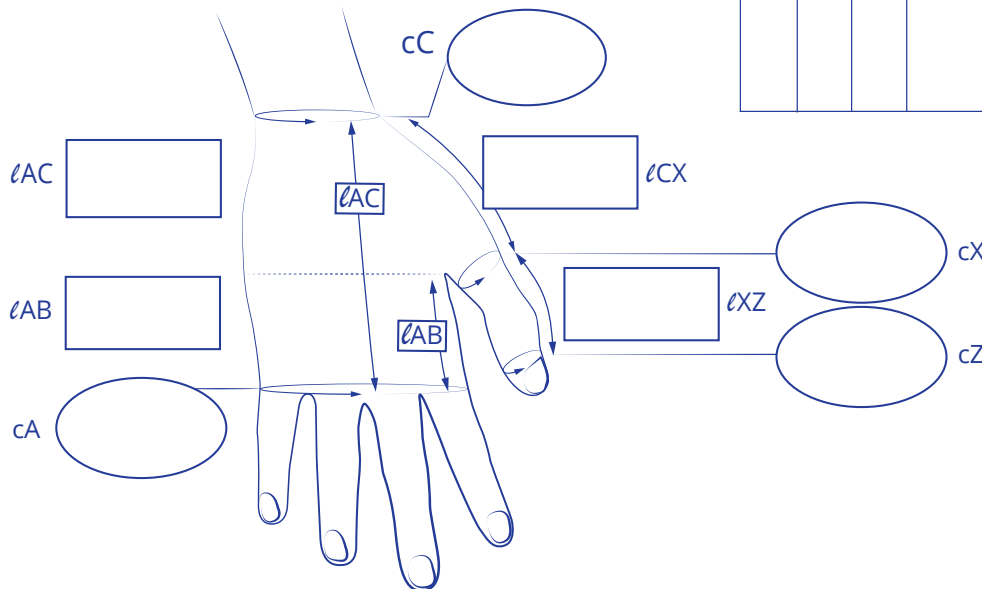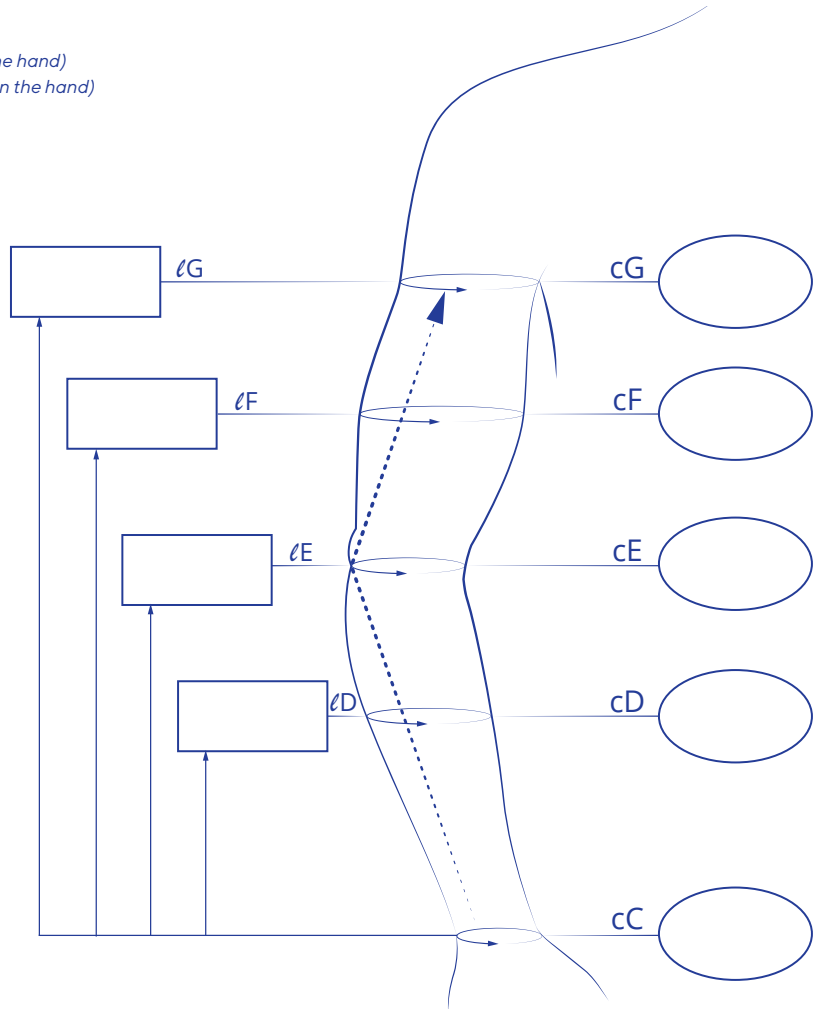

---> Positioning the measuring tape to measure heights

Length in cm

Circumference in cm


MOBIDERM

STOCKINGS

Anatomical Markers

- Ⓒ is located on the gluteal fold (for thigh-high)
- Ⓓ equidistant point between G and E
- Ⓔ corresponds to the middle of the patella
- Ⓕ is located on the head of the fibula (3 - 4 cm under the patella)
- Ⓖ corresponds to the widest point of the calf
- Ⓖ1 corresponds to the start of the calf muscles
- Ⓗ to be measured at the narrowest point of the ankle
- Ⓘ goes through the instep and the tuberosity of the calcaneus
- Ⓙ goes through the head of the metatarsal bones (at the widest point)

All markers must be positioned on the external face of the leg.

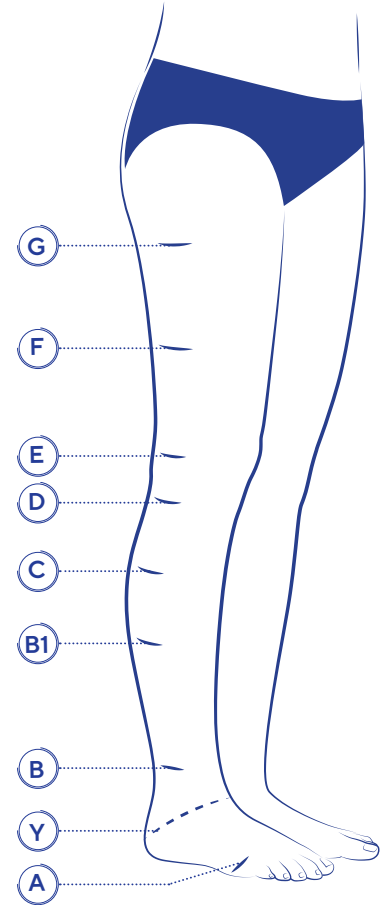

Measurement Instructions



Circumference of the head of the metatarsal bones




Circumference at the instep



Ankle circumference (at the narrowest point)



Circumference of the thigh (at the gluteal fold)

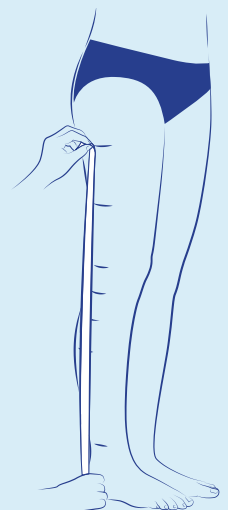

Height measurement (position of measuring tape on the external face of the leg)



THUASNE

MOBIDERM



STOCKINGS

ORDER (by default) **QUOTATION** **RENEWAL**

Customer code

Case No. for renewal

1st treatment

Date: Quantity:

Patient's surname:

Patient's first name:

Gender: M F Child Patient's height:

I authorize my health care professional to collect my data and to communicate them to Thuasne company as part of the processing of my made to measure medical device in accordance with Law No 78-17 of 6 January 1978, and European Regulation No 2016/679/EU of 27 April 2016, I have rights including in particular the rights of access, rectification, portability and deletion of my data. I can exercise these rights by contacting the health care professional from whom I ordered my medical device.

PATIENT SIGNATURE

RETAILER IDENTIFICATION

**If possible, please enclose photos of the limb to be fitted.
Please draw in the contours of the garment on the diagram and cross unnecessary measures.**

Models

- Below-knee
- Thigh-high

LEFT

-
-

RIGHT

-
-

Distal options

- Open toe
- Closed toe


Foam options

- All small blocks
- All big blocks
- Big blocks on the leg and small blocks on the feet

Comments


Proximal options

- Anti-slip with silicone dots 3 cm
- Velcro opening

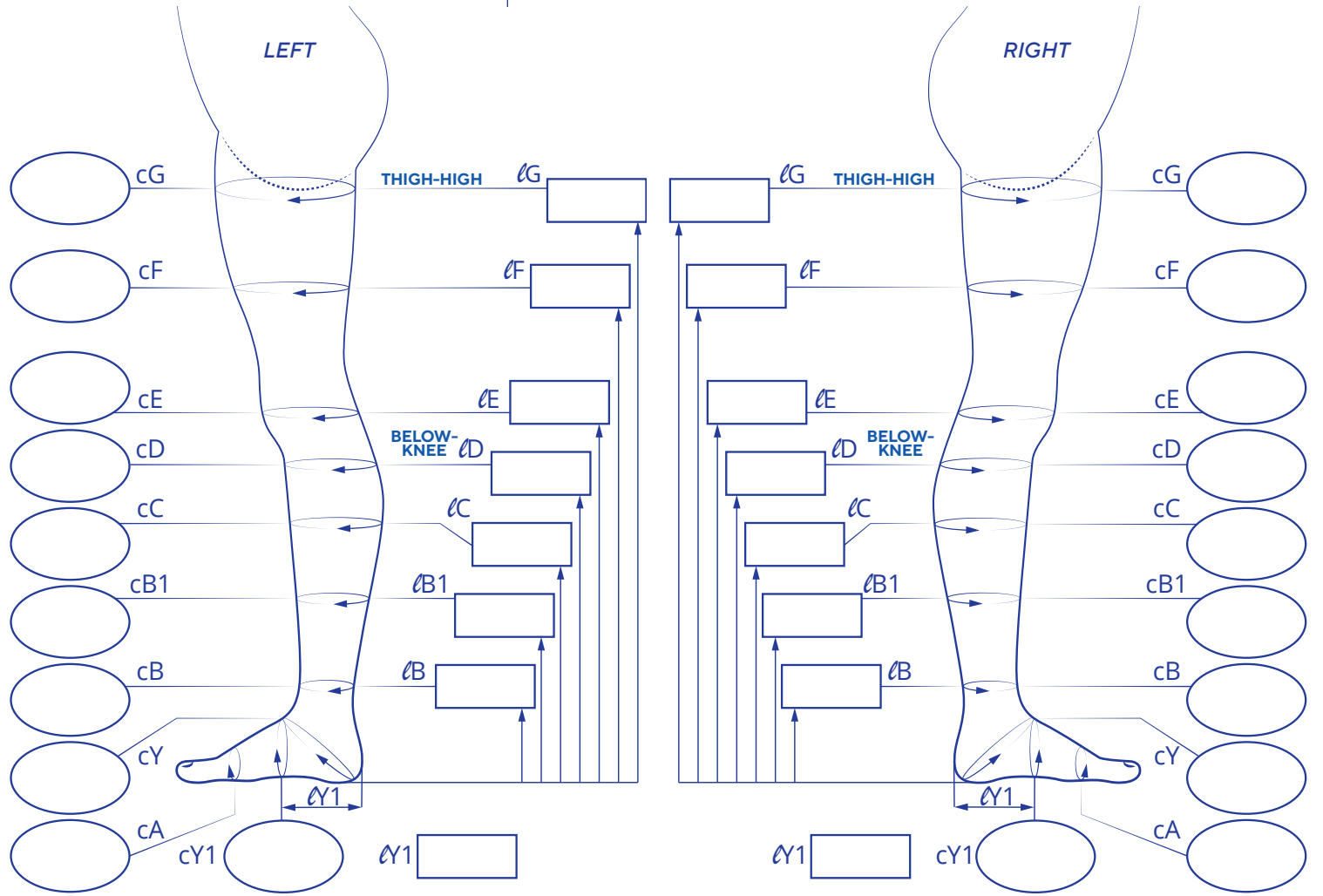

Desired foot length:

Inner (lA)
Outer (lA1)

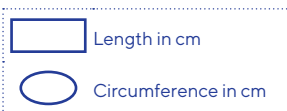

Desired foot length:

Inner (lA)
Outer (lA1)

Please contact your regular Thuasne distributor

LYMPHATREX Essential


SLEEVES

Anatomical Markers

- G** corresponds to the axilla (armpit)
- F** corresponds to the widest point of the biceps
- E** is located at the level of the olecranon
- D** equidistant between A and C or at the origin of the edema
- C1** positioned at 6 cm from C
- C** to be measured at the narrowest point of the wrist external face in the middle of the radial-ulnar joint


The arm must be slightly bended when taking measures.

→ *Positioning of the measuring tape to measure heights.*

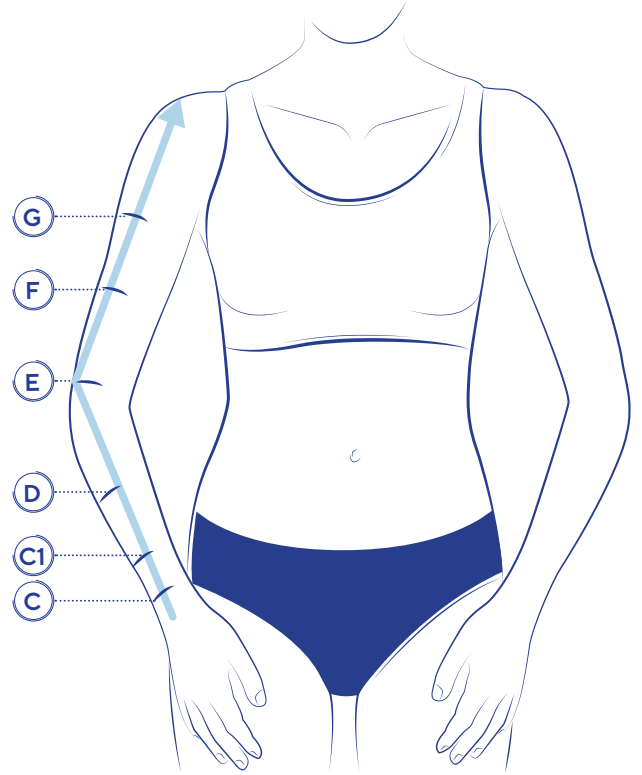

Measurement Instructions

ARMSLEEVE

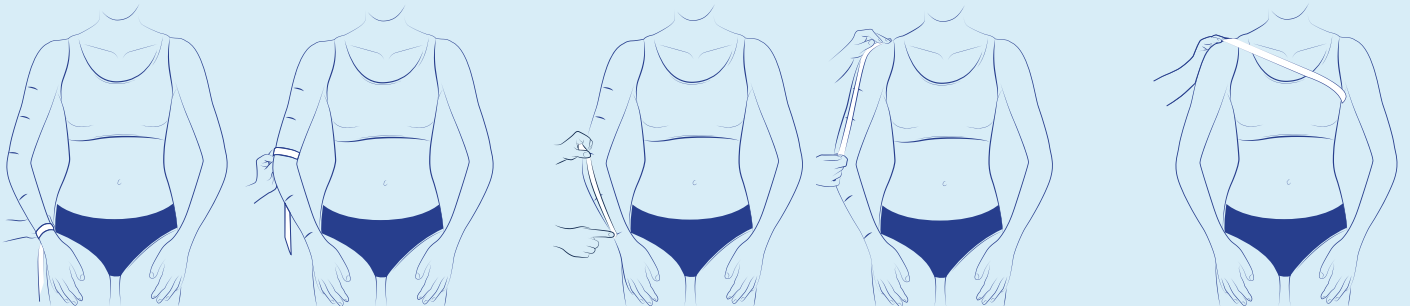


Measure all the circumferences.


The lengths must be measured from the middle of the wrist to the tip of the elbow, then at the outside of the arm up to the desired finished height.

For the **shoulder cover**, measure the chest circumference from the acromion process and going under the opposite armpit.

MITTEN

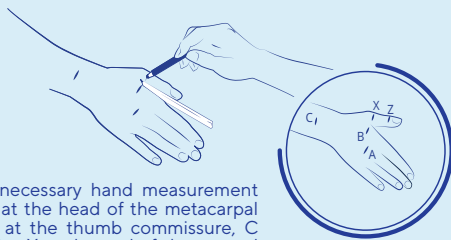


Mark all necessary hand measurement points: A at the head of the metacarpal bones, B at the thumb commissure, C at the wrist, X at the end of the second thumb phalanx, Z at the first thumb nail base.

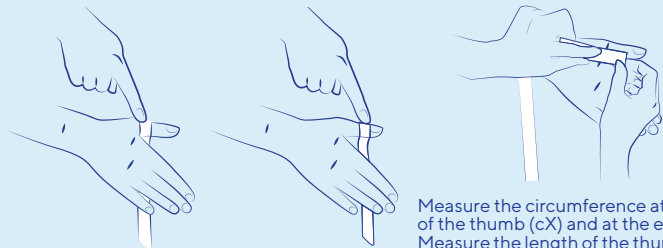

Measure the circumference at the base of the thumb (cX) and at the end (cZ). Measure the length of the thumb (ℓXZ). For technical reasons, the length of the thumb (ℓXZ) cannot be less than 2 cm.



The circumference at the end of the mitten (cA) is located at the head of the metacarpal bones.

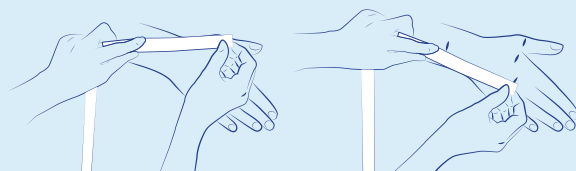

Measure the length of the mitten:
 1/ℓCX: from the wrist (C) to the base of the thumb (X)
 2/ℓAC: from the wrist (C) to the finished height (A)
 3/ℓAB: from the finished height (A) to the thumb commissure (B)



THUASNE

LYMPHATREX Essential



SLEEVES - CIRCULAR KNIT

ORDER (by default) **QUOTATION** **RENEWAL**

Customer code

Case No. for renewal

1st treatment

Date: Quantity:

Patient's surname:

Patient's first name:

Gender: M F Child Patient's height:

I authorize my health care professional to collect my data and to communicate them to Thuasne company as part of the processing of my made to measure medical device in accordance with Law No 78-17 of 6 January 1978, and European Regulation No 2016/679/EU of 27 April 2016. I have rights including in particular the rights of access, rectification, portability and deletion of my data. I can exercise these rights by contacting the health care professional from whom I ordered my medical device.

PATIENT SIGNATURE

RETAILER IDENTIFICATION

If possible, please enclose photos of the limb to be fitted. Please draw in the contours of the garment on the diagram and cross unnecessary measures. We do not make fingers, except for the thumb. If you want fingers, please change your order over to Lymphatrex.

RIGHT ARM **LEFT ARM**

Fill out one form for each side

Models

Armsleeve Armsleeve with mitten Mitten

Compression

Class 2 (15 - 20 mmHg)
 Class 3 (20 - 36 mmHg)

Colors

Beige
 Black

Armsleeve options


Elastic plain braid 3 cm
 Anti-slip with silicone dots 3 cm
 Silicone anti-slip for women 5 cm
 Shoulder cover (indicate measures ℓGH and cH)

Mitten options


With thumb
 Without thumb

Comments

FACING VIEW ARMSLEEVE

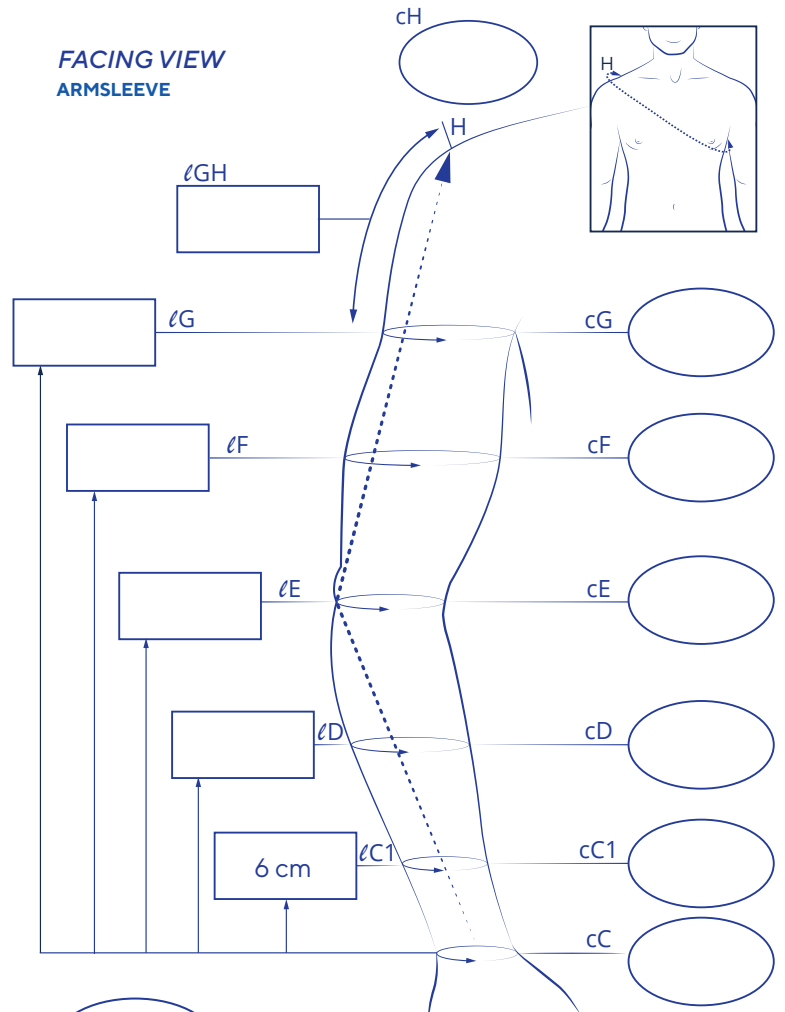

MITTEN

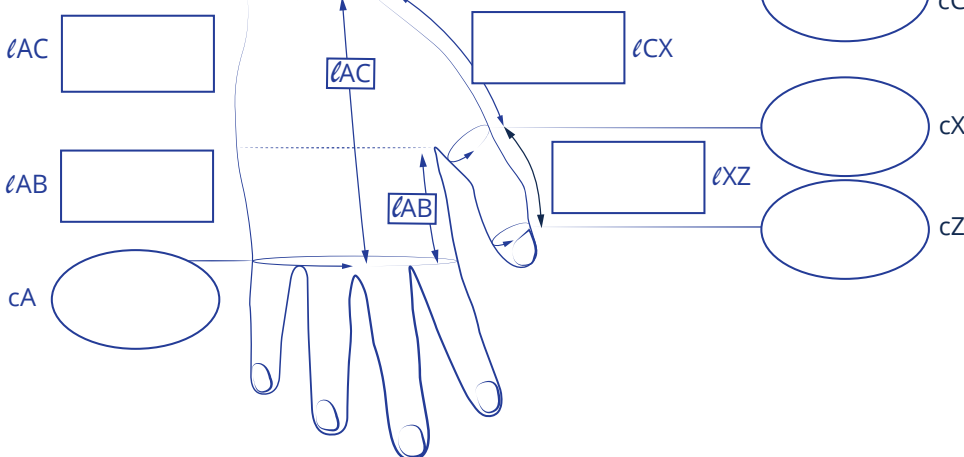


LYMPHATREX Essential

STOCKINGS

Anatomical Markers

- Ⓒ is located at the gluteal fold (for the tights)
- Ⓓ is equidistant between G and E
- Ⓔ corresponds to the middle of the patella
- Ⓕ is located on the head of the fibula (3 - 4 cm below the patella)
- Ⓖ corresponds to the widest point of the calf
- Ⓖ1 corresponds to the start of the calf muscles
- Ⓗ to be measured at the narrowest point of the ankle
- Ⓘ goes through the instep and the tuberosity of the calcaneus
- Ⓙ goes through the head of the metatarsal bones (at the widest point)

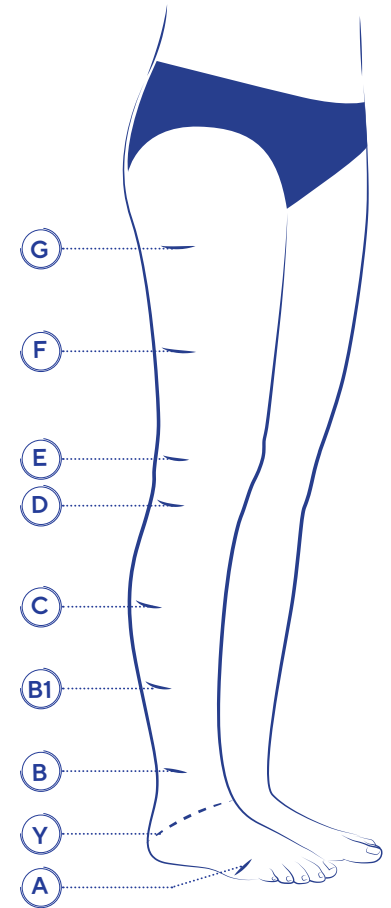

All markers must be positioned on the external face of the leg.

Measurement Instructions



Metatarsal head circumference



Instep circumference



Circumference of the ankle (narrowest point)




Circumference of the thigh (at gluteal fold)




Measuring heights (position of measuring tape on the external face of the leg)




Buttocks circumference

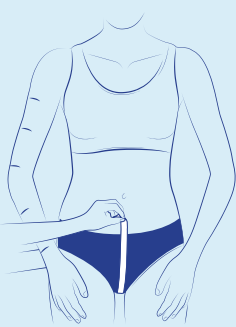

Crotch:
Distance at the front (ⒺK1T)/Distance at the back (ⒺK2T)

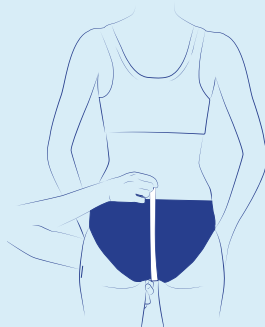

Hip circumference

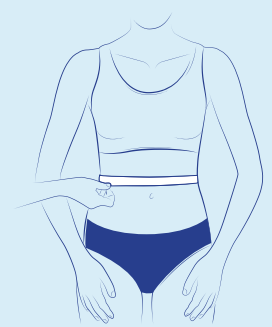

Waist circumference (or indicate the circumference at the desired height)



THUASNE

LYMPHATREX Essential



STOCKINGS - CIRCULAR KNIT

ORDER (by default) **QUOTATION** **RENEWAL**

Customer code

Case No. for renewal

1st treatment

Date: Quantity:

Patient's surname:

Patient's first name:

Gender: M F Child Patient's height:

I authorize my health care professional to collect my data and to communicate them to Thuasne company as part of the processing of my made to measure medical device in accordance with Law No 78-17 of 6 January 1978, and European Regulation No 2016/679/EU of 27 April 2016. I have rights including in particular the rights of access, rectification, portability and deletion of my data. I can exercise these rights by contacting the health care professional from whom I ordered my medical device.

PATIENT SIGNATURE

RETAILER IDENTIFICATION

If possible, please enclose photos of the limb to be fitted. Please draw in the contours of the garment on the diagram and cross unnecessary measures.

Models

- Below-knee
- Thigh-high
- Tights

Compression

- Class 2 (15 - 20 mmHg)
- Class 3 (20 - 36 mmHg)
- No compression

* If Tight compressive briefs desired, compression level of the briefs must be identical to one of the two legs.

Colors

- Beige
- Black

Below-knee options

- Without anti-slip (ribbed edge)
- Anti-slip with silicone dots 3 cm
- Anti-slip with silicone dots 5 cm

LEFT RIGHT

LEFT RIGHT BRIEFS*

LEFT RIGHT

LEFT RIGHT

Thigh-high options

- Anti-slip with silicone dots 5 cm
- Silicone anti-slip for women 5 cm
- Hip cover

(indicate measures cT, cH, lKIT and lK2T)

General options

Toe

- Open toe
- Closed toe

Compression pad

- Instep compression pad
- Medial malleolus compression pad
- Lateral malleolus compression pad

LEFT RIGHT



LEFT RIGHT

Dimensions (l x h) cm

Foam type

- Soft 7 mm
- Hard 5 mm

Comments

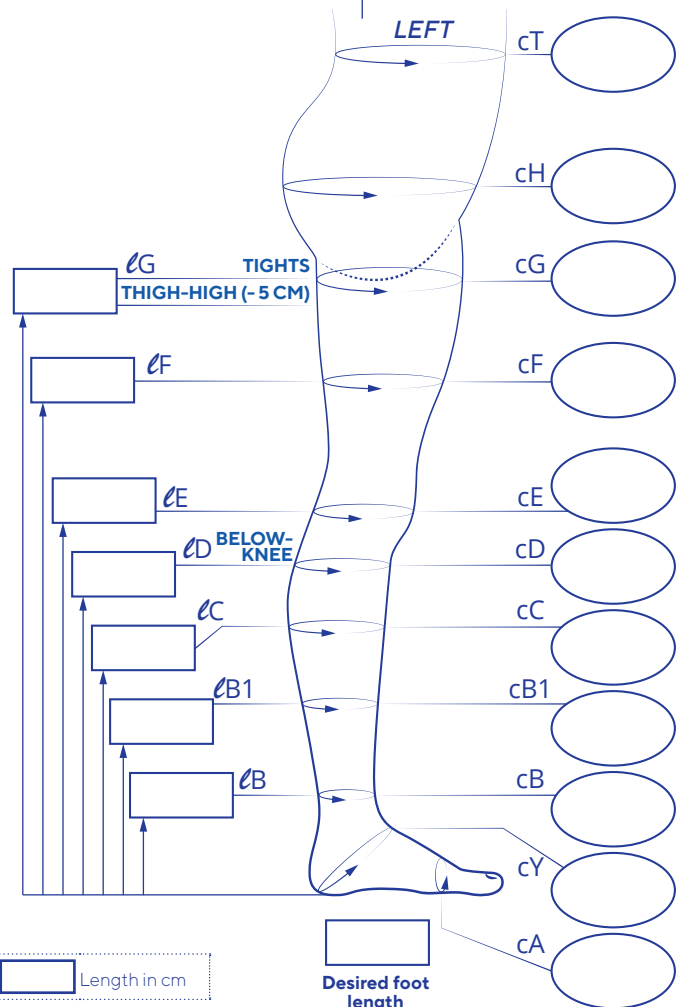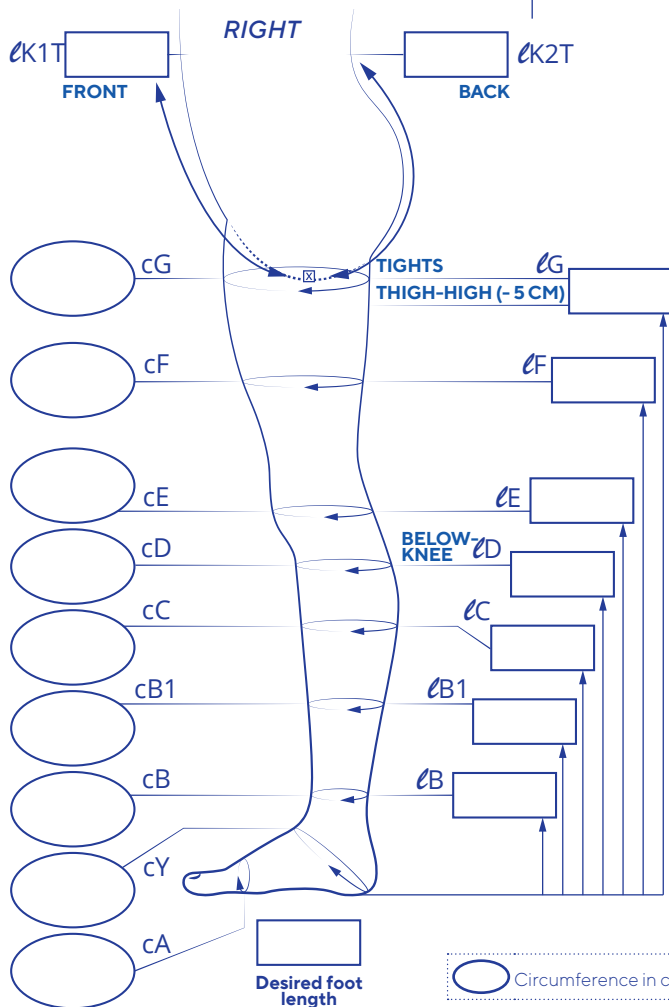

Circumference in cm Length in cm

Please contact your regular Thuasne distributor


LYMPHATREX Expert

ARMSLEEVE


Anatomical Markers

- G** corresponds to the axilla (armpit)
- F** corresponds to the widest point of the biceps
- E** localized at the antecubital fossa
- D** corresponds to the widest point of the forearm
- C1** positioned at 6 cm from C
- C** corresponds to the narrowest point of the wrist



*The arm must be in a relaxed position.
Measure the heights on the inner side of the arm.*

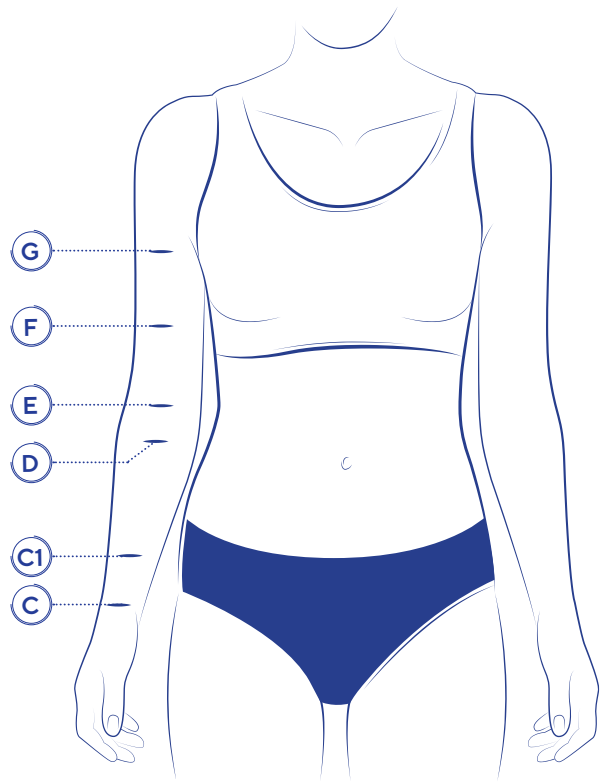


Measurement Instructions

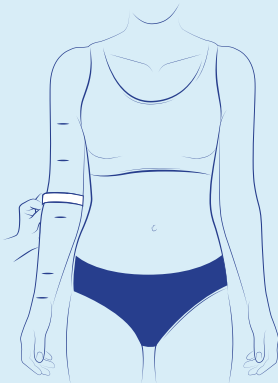


Measurement of circumferences (are illustrated here the fold of the elbow and the axilla)

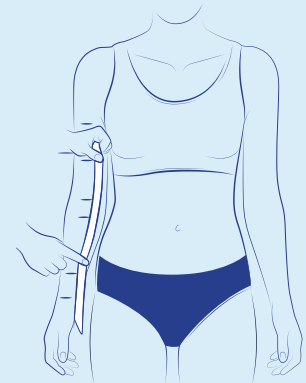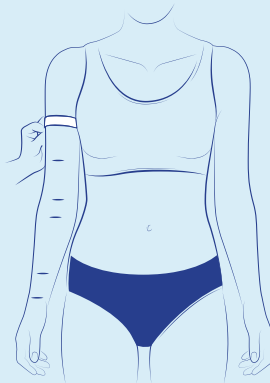

Important: the heights are taken on the inside of the arm

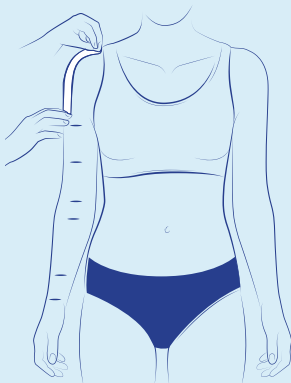

For a shoulder attachment, take the \angle GH measurement that goes from the axilla (projected on the outside of the arm) to the bra shoulder harness.

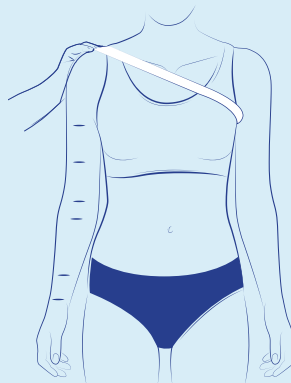

For a shoulder cover, take the chest circumference (H), from the acromion, passing through the opposite axilla.



THUASNE

LYMPHATREX Expert



ARMSLEEVE - FLAT KNIT

ORDER (by default) QUOTATION RENEWAL

Customer code

Case No. for renewal

1st treatment

Date: Quantity:

Patient's surname:

Patient's first name:

Gender: M F Child Patient's height:

I authorize my health care professional to collect my data and to communicate them to Thuasne company as part of the processing of my made to measure medical device in accordance with Law No 78-17 of 6 January 1978, and European Regulation No 2016/679/EU of 27 April 2016. I have rights including in particular the rights of access, rectification, portability and deletion of my data. I can exercise these rights by contacting the health care professional from whom I ordered my medical device.

PATIENT SIGNATURE

RETAILER IDENTIFICATION

If possible, please enclose photos of the limb to be fitted.
Please draw in the contours of the garment on the diagram and cross unnecessary measures.

RIGHT ARM LEFT ARM

Fill in one form for each side

Models

- Armsleeve
- Armsleeve with mitten*

* Fill in mitten measurement form in addition

Compression

- Class 2 (15 - 20 mmHg)
- Class 3 (20 - 36 mmHg)

Colors


- Beige
- Tanning beige
- Black

Options


- Angled proximal edge
- Elastic plain braid 3 cm
- Anti-slip with silicone dots 3 cm
- Shoulder attachment (indicate measure ℓGH)
- Shoulder cover (indicate measures $\ell GH + H$)

Comments

.....
.....
.....
.....
.....
.....
.....
.....
.....
.....

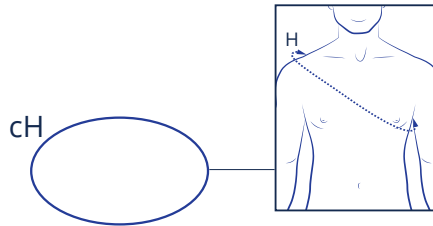

FACING VIEW

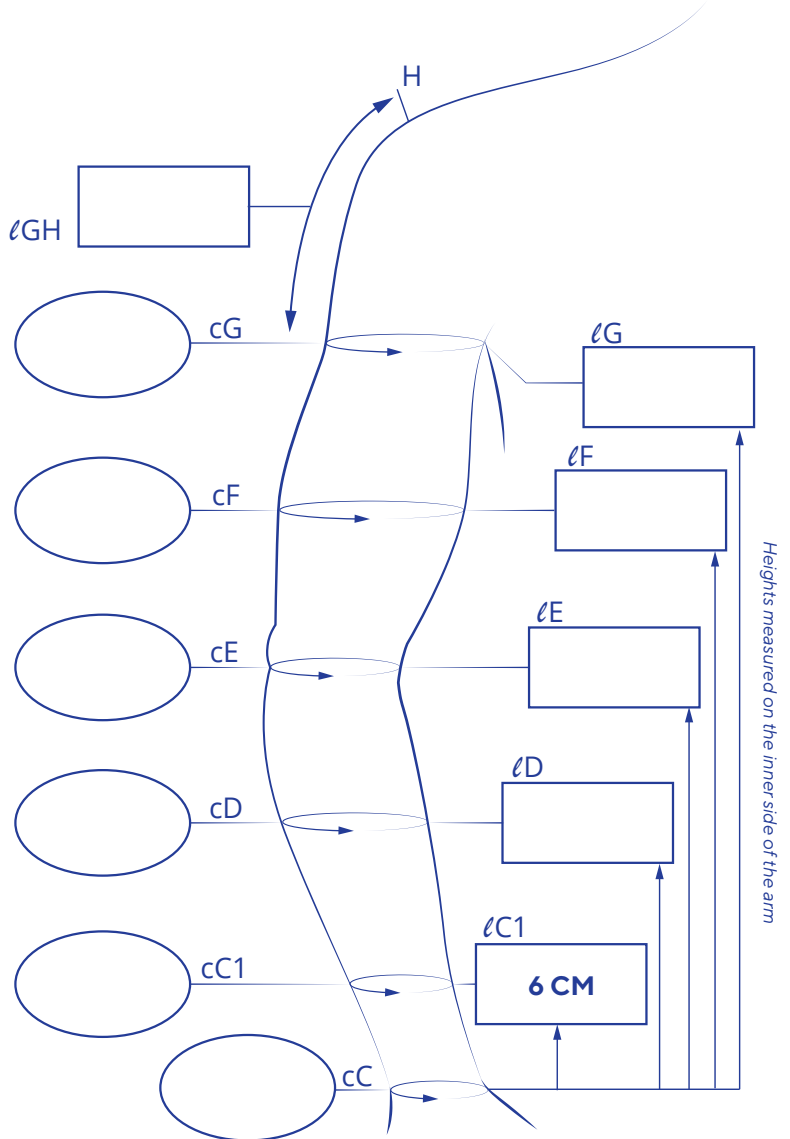

LYMPHATREX Expert

MITTEN & GLOVE

Anatomical Markers


Major-annular commissure

- (A) located at the highest level of the metacarpal head
- (B) located in the axis of the thumb commissure
- (C) wrist bending fold (at the narrowest point of the wrist)
- (C1) where ever from wrist if arm is regular, or at the birth of edema (minimum 2 cm and maximum 6 cm from C).


For an extended cuff, place a D marker corresponding to the desired length (minimum 6 cm from wrist and maximum to the elbow).

The different markers must be positioned on the median axis of the hand, on the palmar side.


Measure the heights on the palmar side of the hand.

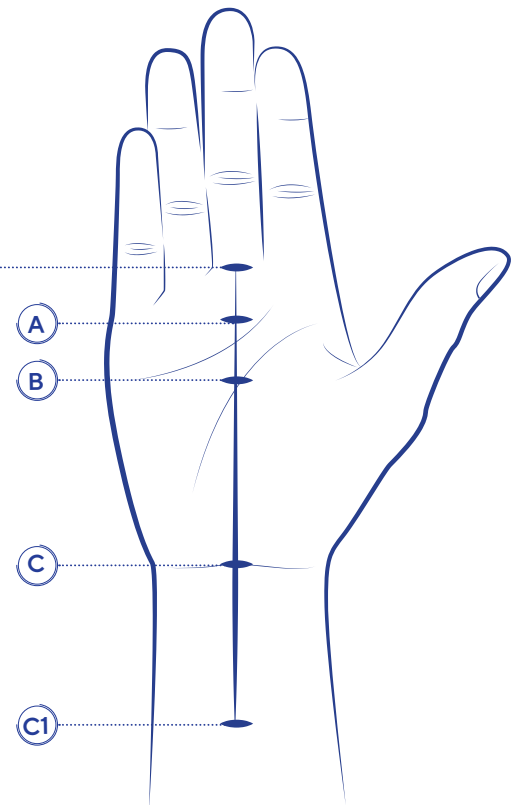


Measurement Instructions

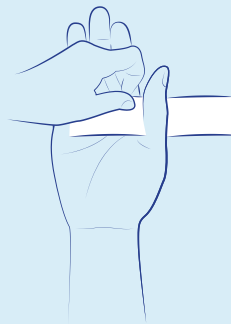

Identification of the axis of the commissure of the thumb

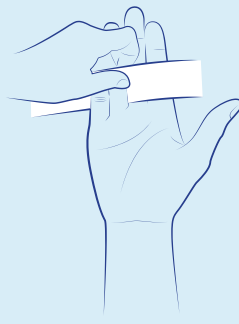


Identification of the commissure major - annular

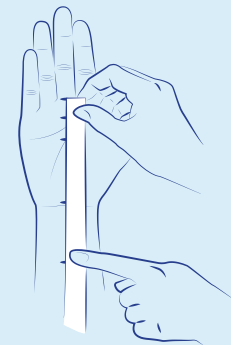


Lengths between the bending fold of the wrist and:

- metacarpo-phalangeal joint (ℓ A)
- commissure of thumb (ℓ B)
- annular - major commissure (ℓ C)

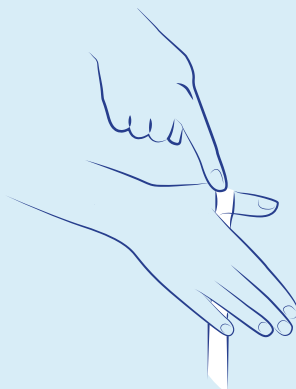

Circumferences of the fingers are measured:

- in the middle of the first phalanx
- at the level where you want to stop the finger (preferably use a lasso meter)

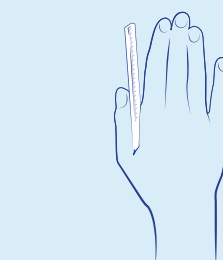

Lengths are preferably measured with a rigid ruler, to position in the commissures, without pressing it



THUASNE

LYMPHATREX Expert



MITTEN & GLOVE - FLAT KNIT

ORDER (by default) **QUOTATION** **RENEWAL**

Customer code

Case No. for renewal

1st treatment

Date: Quantity:

RETAILER IDENTIFICATION

Patient's surname:

Patient's first name:

Gender: M F Child Patient's height:

I authorize my health care professional to collect my data and to communicate them to Thuasne company as part of the processing of my made to measure medical device in accordance with Law No 78-17 of 6 January 1978, and European Regulation No 2016/679/EU of 27 April 2016. I have rights including in particular the rights of access, rectification, portability and deletion of my data. I can exercise these rights by contacting the health care professional from whom I ordered my medical device.

PATIENT SIGNATURE

If possible, please enclose photos of the limb to be fitted. Please draw in the contours of the garment on the diagram and cross unnecessary measures. Available with open fingers only. Mitten only includes thumb finger. Glove includes all fingers.

RIGHT HAND **LEFT HAND**

Fill in one form for each side

Models

- Mitten with thumb
- Glove

(If associated with an armsleeve, fill in ARMSLEEVE measurement form)

Compression

- Class 2 (15 - 20 mmHg)
- Class 3 (20 - 36 mmHg)

Colors

- Beige
- Tanning beige
- Black


Cuff options

- Non-compressive cuff (recommended in case of overlapping with armsleeve)
- Compressive cuff (by default)
- Extended cuff (indicate measures ℓD and cD)
- Without anti-slips
- Elastic plain braid 3 cm
- Anti-slip with silicone dots 3 cm


Hand top options

- Opening zipper Length cm
- Foam compression pad Dimensions ($\ell \times h$) cm

Comments

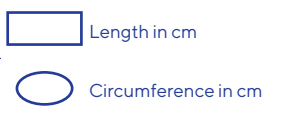

Palmar side

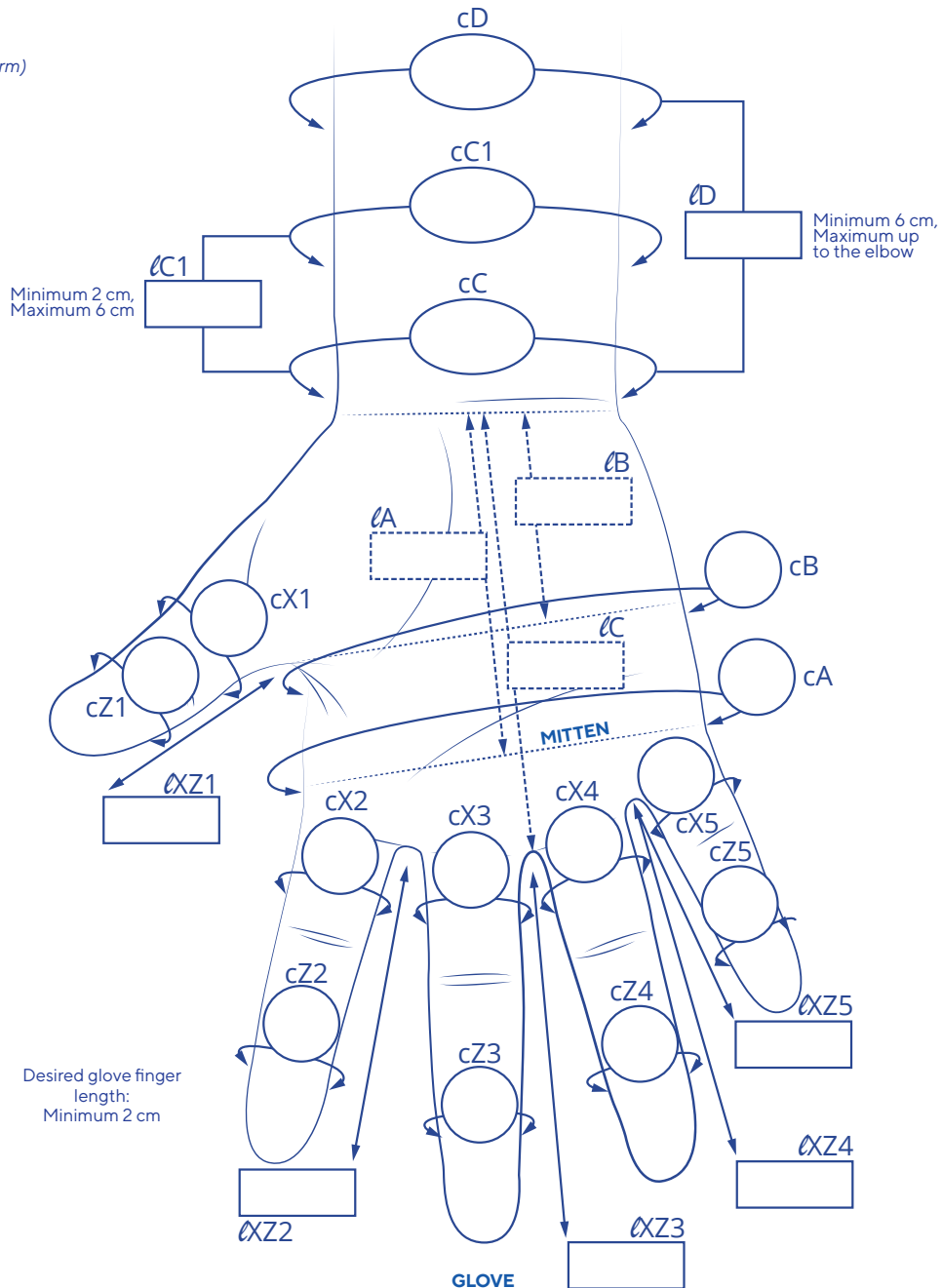


LYMPHATREX Expert

STOCKINGS


Anatomical Markers

- (K) is located at the crotch (pantys/tights)
- (H) corresponds to the widest point of the buttocks (pantys/tights)
- (G) is located 5 cm under the gluteal fold
- (F) equidistant point between G and E
- (E) corresponds to the middle of the patella
- (D) is located on the head of the fibula (3 - 4 cm under the patella)
- (C) corresponds to the widest point of the calf
- (B1) corresponds to the start of the calf muscles
- (B) to be measured at the narrowest point of the ankle
- (Y) goes through the instep and the tuberosity of the calcaneus
- (A) goes through the heads of the metatarsal bones (at the widest point)

*All markers must be positioned on the external face of the leg.
Do not forget to measure and indicate desired foot lengths (inner/outer).
For a garment with open toe: please provide heel to toe tip measure.
For a garment with closed toe: please provide heel to desired knitting stop measure.*

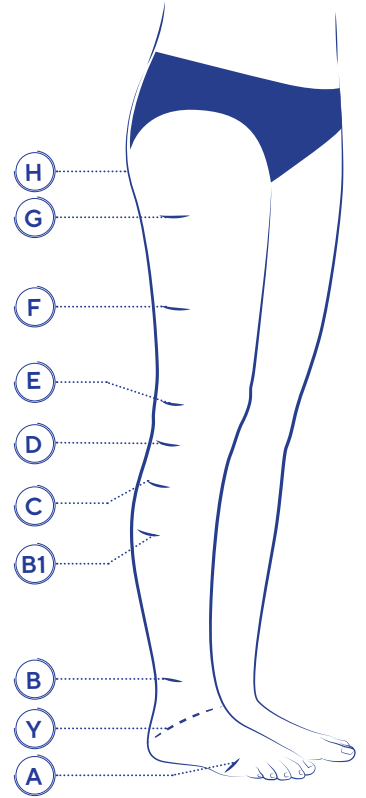

Measurement Instructions

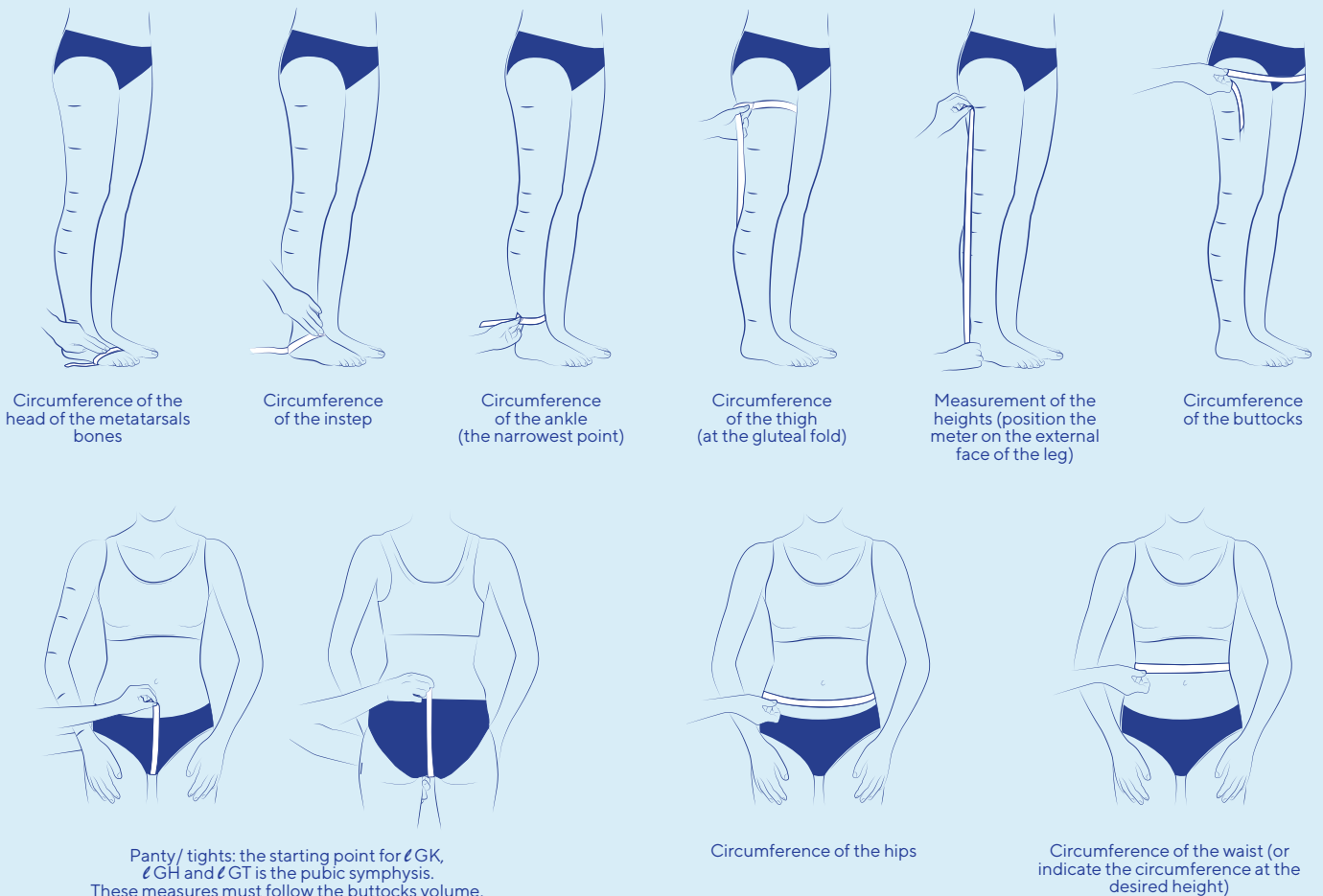



THUASNE

LYMPHATREX Expert



STOCKINGS - FLAT KNIT

ORDER (by default) **QUOTATION** **RENEWAL**

Customer code

Case No. for renewal

1st treatment

Date: _____ Quantity: _____

Patient's surname: _____

Patient's first name: _____

Gender: M F Child Patient's height: _____

I authorize my health care professional to collect my data and to communicate them to Thuasne company as part of the processing of my made to measure medical device in accordance with Law No 78-17 of 6 January 1978, and European Regulation No 2016/679/EU of 27 April 2016, I have rights including in particular the rights of access, rectification, portability and deletion of my data. I can exercise these rights by contacting the health care professional from whom I ordered my medical device.

PATIENT SIGNATURE

RETAILER IDENTIFICATION

If possible, please enclose photos of the limb to be fitted.

Please draw in the contours of the garment on the diagram and cross unnecessary measures.

Models

- | | | |
|--|--------------------------|--------------------------|
| <input type="checkbox"/> Below-knee | LEFT | RIGHT |
| <input type="checkbox"/> Thigh-high | <input type="checkbox"/> | <input type="checkbox"/> |
| <input type="checkbox"/> Tights | <input type="checkbox"/> | <input type="checkbox"/> |
| <input type="checkbox"/> Half-tight (short leg up to E marker) | | |
| <input type="checkbox"/> Panty | | |

Compression

- | | | | |
|---|--------------------------|--------------------------|--------------------------|
| <input type="checkbox"/> Class 3 (20 - 36 mmHg) | LEFT | RIGHT | BRIEF |
| <input type="checkbox"/> Class 4 (> 36 mmHg) | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> |
| <input type="checkbox"/> No compression | - | - | <input type="checkbox"/> |

Colors

- Beige
- Tanning beige
- Black

Below-knee options

- | | | |
|--|--------------------------|--------------------------|
| <input type="checkbox"/> Elastic plain braid 3 cm | LEFT | RIGHT |
| <input type="checkbox"/> Anti-slip with silicone dots 3 cm | <input type="checkbox"/> | <input type="checkbox"/> |
| <input type="checkbox"/> Without anti-slip (by default) | <input type="checkbox"/> | <input type="checkbox"/> |

Thigh-high options

- | | | |
|--|--------------------------|--------------------------|
| <input type="checkbox"/> Anti-slip with silicone dots 5 cm | LEFT | RIGHT |
| <input type="checkbox"/> Angled thigh end | <input type="checkbox"/> | <input type="checkbox"/> |
| <input type="checkbox"/> Hip cover | <input type="checkbox"/> | <input type="checkbox"/> |

Tights, half-tight & panty options

Belt

- Elastic belt 5 cm
- Adjustable elastic belt
- Without belt

Opening

- | | | | |
|--|--------------------------|--------------------------|--------------------------|
| <input type="checkbox"/> Opening zipper | LEFT | RIGHT | MIDDLE FRONT* |
| <input type="checkbox"/> Opening Velcro | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> |
| <input type="checkbox"/> Without opening | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> |

* Only for women

Length cm

Other

- Hygiene tab (soft fabric)
- Combination with thigh-high (for half-tight and panty only)

General option

Toe LEFT RIGHT

- | | | |
|-------------------------------------|--------------------------|--------------------------|
| <input type="checkbox"/> Open toe | <input type="checkbox"/> | <input type="checkbox"/> |
| <input type="checkbox"/> Closed toe | <input type="checkbox"/> | <input type="checkbox"/> |
| <input type="checkbox"/> With toes | | |


(Please fill in toe cap measurement form)

Compression pad

- | | | |
|--|--------------------------|--------------------------|
| <input type="checkbox"/> Instep compression pad | LEFT | RIGHT |
| <input type="checkbox"/> Medial malleolus compression pad | <input type="checkbox"/> | <input type="checkbox"/> |
| <input type="checkbox"/> Lateral malleolus compression pad | <input type="checkbox"/> | <input type="checkbox"/> |

Dimensions (l x h):

Comments

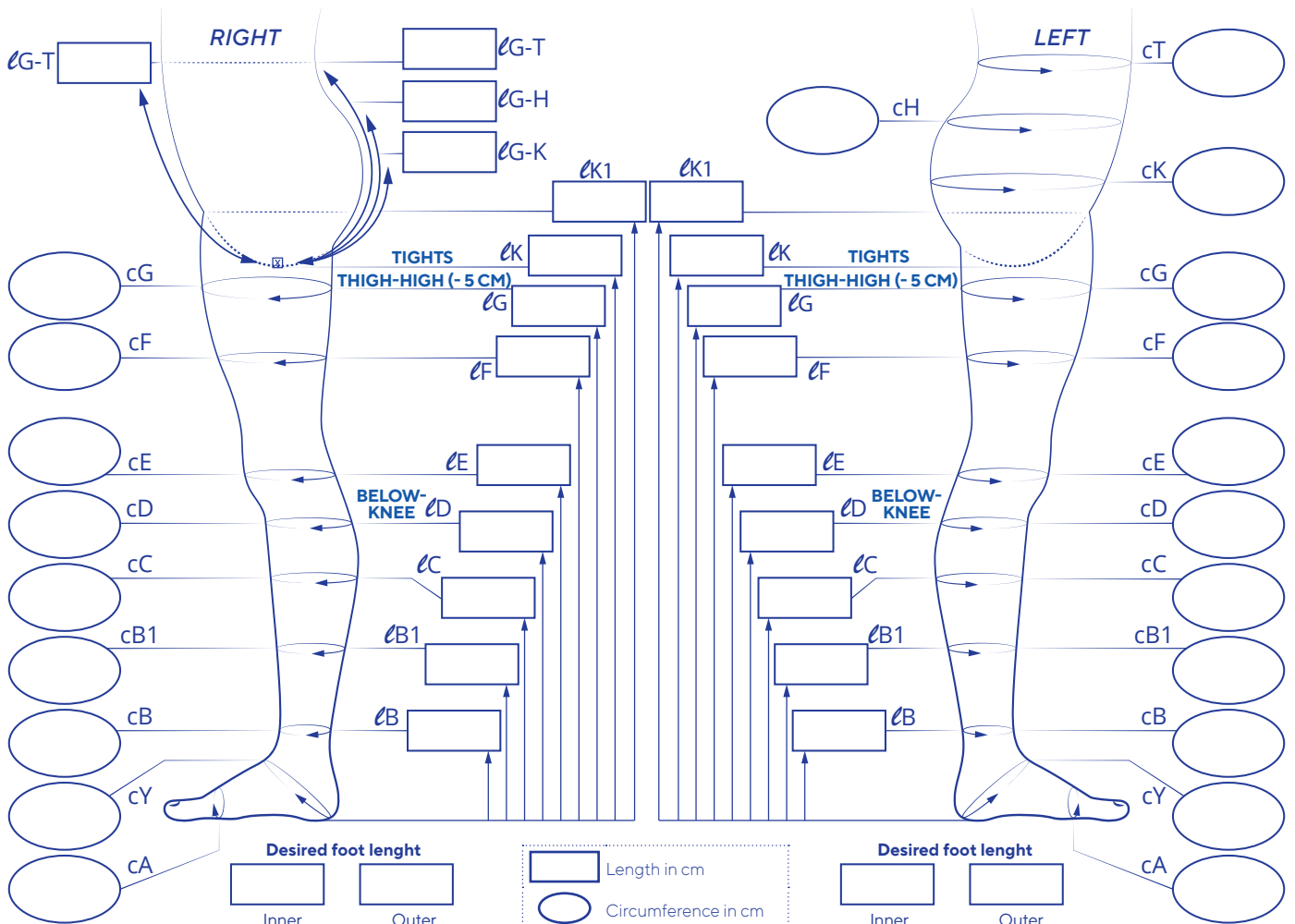

Please contact your regular Thuasne distributor



THUASNE

LYMPHATREX Expert



TOE CAP - FLAT KNIT

ORDER (by default) **QUOTATION** **RENEWAL**

Customer code

Case No. for renewal

1st treatment

Date: Quantity:

Patient's surname:

Patient's first name:

Gender: M F Child Patient's height:

I authorize my health care professional to collect my data and to communicate them to Thuasne company as part of the processing of my made to measure medical device in accordance with Law No 78-17 of 6 January 1978, and European Regulation No 2016/679/EU of 27 April 2016. I have rights including in particular the rights of access, rectification, portability and deletion of my data. I can exercise these rights by contacting the health care professional from whom I ordered my medical device.

PATIENT SIGNATURE

RETAILER IDENTIFICATION

If possible, please enclose photos of the limb to be fitted.

Please draw in the contours of the garment on the diagram and cross unnecessary measures. With open toes only.

Models

- Toe cap
- Attached to below-knee/thigh-high/tights/half-tight
(please fill in stockings measurement form)

Compression

- Class 3 (20 - 36 mmHg)
- Class 4 (> 36 mmHg)


LEFT RIGHT

Colors


- Beige
- Tanning beige
- Black

Comments

LEFT RIGHT

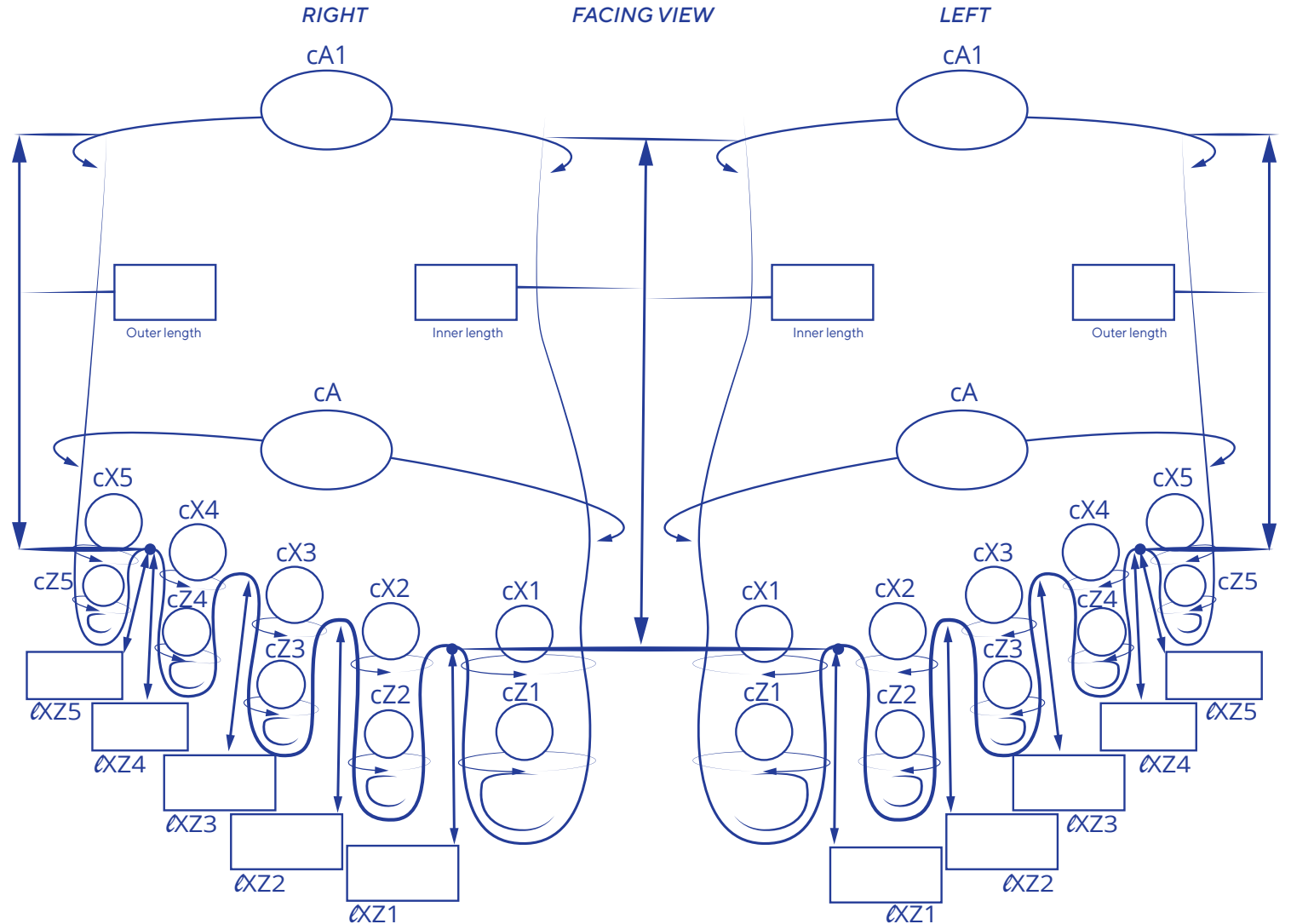

Indicate desired toes length (not total length)

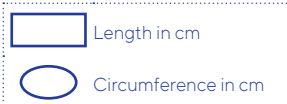

Indicate desired toes length (not total length)

Please contact your regular Thuasne distributor

A series of horizontal dotted lines for taking notes, organized into four sections by vertical dashed lines.

Lymphedema treatment:
the lymphedema

Lymphedema Treatment:
the reduction phase

Lymphedema Treatment:
the maintenance phase

Scars Treatment



THUASNE

CICATREX



TORSO n°1a

ORDER (by default) **QUOTATION** **RENEWAL**

Customer code

Case No. for renewal

1st treatment

Date: Quantity:

RETAILER IDENTIFICATION

Patient's surname:


Patient's first name:

Gender: M F Child Patient's height:

I authorize my health care professional to collect my data and to communicate them to Thuasne company as part of the processing of my made to measure medical device in accordance with Law No 78-17 of 6 January 1978, and European Regulation No 2016/679/EU of 27 April 2016, I have rights including in particular the rights of access, rectification, portability and deletion of my data. I can exercise these rights by contacting the health care professional from whom I ordered my medical device.

PATIENT SIGNATURE

**If possible, please enclose photos of the limb to be fitted.
Please draw in the contours of the garment on the diagram and cross unnecessary measures.
Please also fill in torso form No. 1b.**

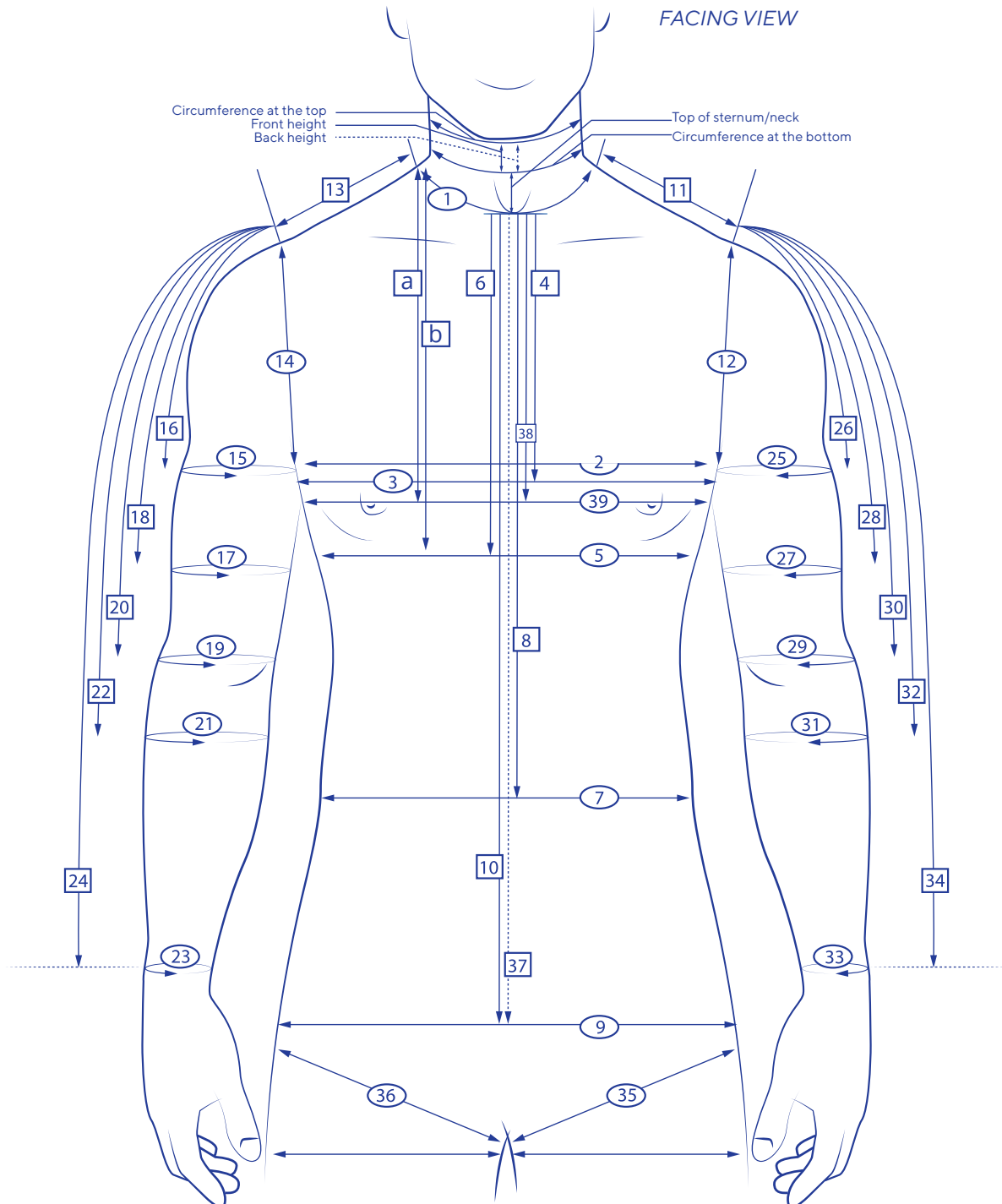

circumference in cm length in cm half-circumference in cm

35 and 36: For the body only
Measurement taken at an angle for a brief-cut body
Measurement taken straight for a short-cut body

Please contact your regular Thuasne distributor



THUASNE

CICATREX



TORSO °1b

ORDER (by default) **QUOTATION** **RENEWAL**

Customer code

Case No. for renewal

1st treatment

Date: Quantity:

RETAILER IDENTIFICATION

Patient's surname:

Patient's first name:

Gender: M F Child Patient's height:

I authorize my health care professional to collect my data and to communicate them to Thuasne company as part of the processing of my made to measure medical device in accordance with Law No 78-17 of 6 January 1978, and European Regulation No 2016/679/EU of 27 April 2016, I have rights including in particular the rights of access, rectification, portability and deletion of my data. I can exercise these rights by contacting the health care professional from whom I ordered my medical device.

PATIENT SIGNATURE

**If possible, please enclose photos of the limb to be fitted.
Please draw in the contours of the garment on the diagram and cross unnecessary measures.
Please also fill in torso form No. 1a.**

Indications

- Major burn (by default)
- EDS (Ehlers-Danlos Syndrome)

Models

- Shoulder pad
- Body
- Vest

Fabrics

- Cicatrex Nature
- Cicatrex Filifine
- Cicatrex AirSkin

	WHITE	BEIGE	BLACK
<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<input type="checkbox"/>	-	<input type="checkbox"/>	<input type="checkbox"/>

Options

- Center opening
- Zipper
 - Front position
 - Back position
- Hook to help fasten the vest (by default)

Armpit compression

- Right Yes No*
- Left Yes No*

*Soft fabric by default

Collar

- Round neck
- V-neck

Front collar cut height (from the sternum): cm

Back collar cut height (from the 7th cervical vertebra): cm

- Stand-up collar

Circumference at top: cm

Circumference at bottom: cm

Front height: cm

Back height: cm

Top of sternum/neck height: cm

Cups

- With compression
- Without compression
- Depth: A B C D E F

Velcro fastening (for panty)

- Vest over panty
- Panty over vest

Additional opening**

Position: (to be drawn in)
 Zipper Velcro Length: cm

Soft fabric**

- Elbow
- Other

Other**

Compression pad
Position: (to be drawn in)
Dimensions: cm
 Belt

** Extras, not included in the cost

RIGHT SIDE

13 Shoulder length from the acromion process to neck circumference No.1

14 Armhole circumference

16 Top of sleeve

15 Arm circumference at the armpit

18 Length to the middle of the upper arm

17 Upper arm circumference (widest part)

20 Length to the elbow

19 Elbow circumference

22 Length to the middle of the forearm

21 Forearm circumference (widest part)

24 Length to the wrist

23 Wrist circumference

36 Thigh circumference*

1 Neck circumference from the 7th cervical vertebra to the top of the sternum

2 Chest width, breathing in (armpit to armpit)

4 Top of sternum - armpit line length

3 Chest perimeter, breathing in

6 Top of sternum - under the bust length

5 Circumference under the bust, breathing in

8 Top of sternum - waist length

7 Waist circumference

10 Top of sternum - bottom of vest length (for a body, length to the crotch)

9 Hip circumference

37 Back length (7th cervical vertebra - bottom of vest). For a body, length to the crotch, measure with the tape measure flat against the skin

39 Circumference tip of the breast breathing in

38 Length top of sternum - tip of the breast

a** Length neck base - tip of the breast

b** Length neck base - under the bust

LEFT SIDE

12 Armhole circumference

11 Shoulder length from the acromion process to neck circumference No.1

25 Arm circumference at the armpit

26 Top of sleeve

27 Upper arm circumference (widest part)

28 Length to the middle of the upper arm

29 Elbow circumference

30 Length to the elbow

31 Forearm circumference (widest part)

32 Length to the middle of the forearm

33 Wrist circumference

34 Length to the wrist

35 Thigh circumference*

* For the body only
** To fill out only from D cup

Please contact your regular Thuasne distributor



THUASNE

CICATREX



HAND n°2

ORDER (by default) **QUOTATION** **RENEWAL**

Customer code

Case No. for renewal

1st treatment

Date: Quantity:

Patient's surname:

Patient's first name:

Gender: M F Child Patient's height:

I authorize my health care professional to collect my data and to communicate them to Thuasne company as part of the processing of my made to measure medical device in accordance with Law No 78-17 of 6 January 1978, and European Regulation No 2016/679/EU of 27 April 2016, I have rights including in particular the rights of access, rectification, portability and deletion of my data. I can exercise these rights by contacting the health care professional from whom I ordered my medical device.

PATIENT SIGNATURE

RETAILER IDENTIFICATION

**If possible, please enclose photos of the limb to be fitted.
Please draw in the contours of the garment on the diagram and cross unnecessary measures.
For open fingers, please indicate the desired length for each finger.**

RIGHT HAND **LEFT HAND**

Fill in one form for each side

Indications

- Major burn (by default)
- EDS (Ehlers-Danlos Syndrome)

Fabrics

- Cicatrex Filifine* WHITE BEIGE BLACK
- Cicatrex AirSkin** -

*Minimum circumferences of 4.6 cm, minimum length of fingers of 2 cm.

**Minimum circumferences of 2.6 cm, minimum length of fingers of 1 cm.

Options

Proximal end

- With length on wrist (5 cm max)
- With sleeve beyond 5 cm and up to the armpit as maximum (fill in form Arm No. 2) Separate Attached
- Finger protector
- Finger protector with strap
- Finger protector without strap

Opening

Type:

- Zipper Velcro Dimensions: cm

Position:

- Palm of hand Back of hand Little finger side

Compression pad

- Compression pad only
- Pocket for ompression pad

Dimensions (l x h): cm


- Position: to be drawn in Palm of hand Back of hand

Other

- Palm support

Comments

Circumferences		Measures in cm
22	Wrist circumference + 5 cm max	
23	Wrist circumference	
E	Circumference of the hand at the head of the metacarpal bones	
F	Circumference of the little finger at the 1st phalanx	
G	Circumference of the little finger at the 3rd phalanx	
H	Circumference of the ring finger at the 1st phalanx	
I	Circumference of the ring finger at the 3rd phalanx	
J	Circumference of the middle finger at the 1st phalanx	
K	Circumference of the middle finger at the 3rd phalanx	
L	Circumference of the index finger at the 1st phalanx	
M	Circumference of the index finger at the 3rd phalanx	
N	Circumference of the thumb at the 1st phalanx	
O	Circumference of the thumb at the 2nd phalanx	
Desired length		Fingertips
		Measures in cm
D	Length of the palm surface of the hand	
P	Length of the little finger	<input type="checkbox"/> OPEN <input type="checkbox"/> CLOSED
Q	Length of the ring finger	<input type="checkbox"/> OPEN <input type="checkbox"/> CLOSED
R	Length of the middle finger	<input type="checkbox"/> OPEN <input type="checkbox"/> CLOSED
S	Length of the index finger	<input type="checkbox"/> OPEN <input type="checkbox"/> CLOSED
T	Length from the tip of the index finger to the join with the thumb	
U	Thumb length	<input type="checkbox"/> OPEN <input type="checkbox"/> CLOSED
Z	Length of the sleeve up to 5 cm max	

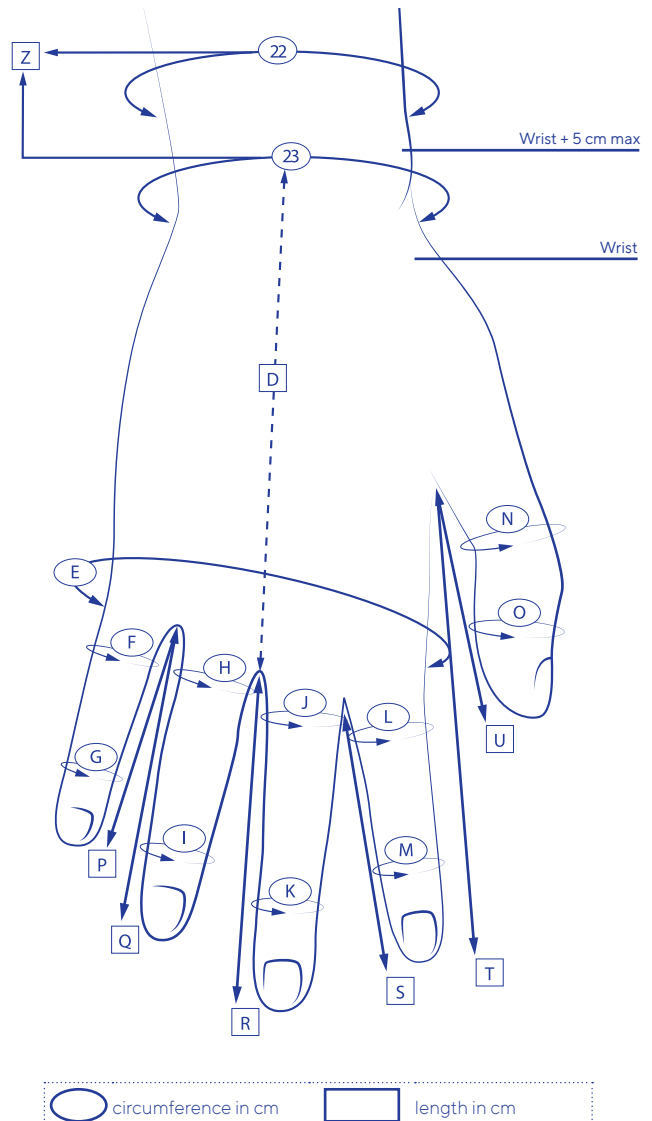

Please contact your regular Thuasne distributor



THUASNE

CICATREX



ARM n°3

ORDER (by default) **QUOTATION** **RENEWAL**

Customer code

Case No. for renewal

1st treatment

Date: Quantity:

RETAILER IDENTIFICATION

Patient's surname:

Patient's first name:

Gender: M F Child Patient's height:

I authorize my health care professional to collect my data and to communicate them to Thuasne company as part of the processing of my made to measure medical device in accordance with Law No 78-17 of 6 January 1978, and European Regulation No 2016/679/EU of 27 April 2016, I have rights including in particular the rights of access, rectification, portability and deletion of my data. I can exercise these rights by contacting the health care professional from whom I ordered my medical device.

PATIENT SIGNATURE

If possible, please enclose photos of the limb to be fitted. Please draw in the contours of the garment on the diagram and cross unnecessary measures.

RIGHT ARM **LEFT ARM**

Fill in one form for each side

Indications

- Major burn (by default)
- EDS (Ehlers-Danlos Syndrome)

Fabrics

- Cicatrex Nature
- Cicatrex Filifine
- Cicatrex AirSkin

	WHITE	BEIGE	BLACK
<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<input type="checkbox"/>	-	<input type="checkbox"/>	<input type="checkbox"/>

Options

Sleeve fastening

- Without anti-slip
- Elastic plain braid 3 cm
- Anti-slip with silicone dots 3 cm
- Silicone men anti-slip 5 cm
- Silicone women anti-slip 5 cm
- Up to the armpit
- Shoulder cover
- Shoulder attachment (bra loop)

Opening

Type:

- Zipper
- Velcro

Dimensions:

cm

Position:

- Join side
- Back of the hand side
- Palm of the hand side

Other

- Compression pad


Dimensions (l x h): cm

Position: to be drawn in

Comments

.....

Heights	Measures in cm
V Length from the wrist to the middle of the widest part of the forearm	
W Length from the wrist to the elbow	
X Length from the wrist to the middle of the widest part of the upper arm	
Y Length from the wrist to the armpit	
H Length from the armpit (g) to the acromion process (tip of the shoulder)	
Circumferences	
C Wrist circumference	
D Circumference at the middle of the widest part of the forearm	
E Elbow circumference	
F Circumference at the middle of the widest part of the upper arm	
G Arm circumference at the armpit	
I Chest circumference from the acromion process and going under the opposite armpit	

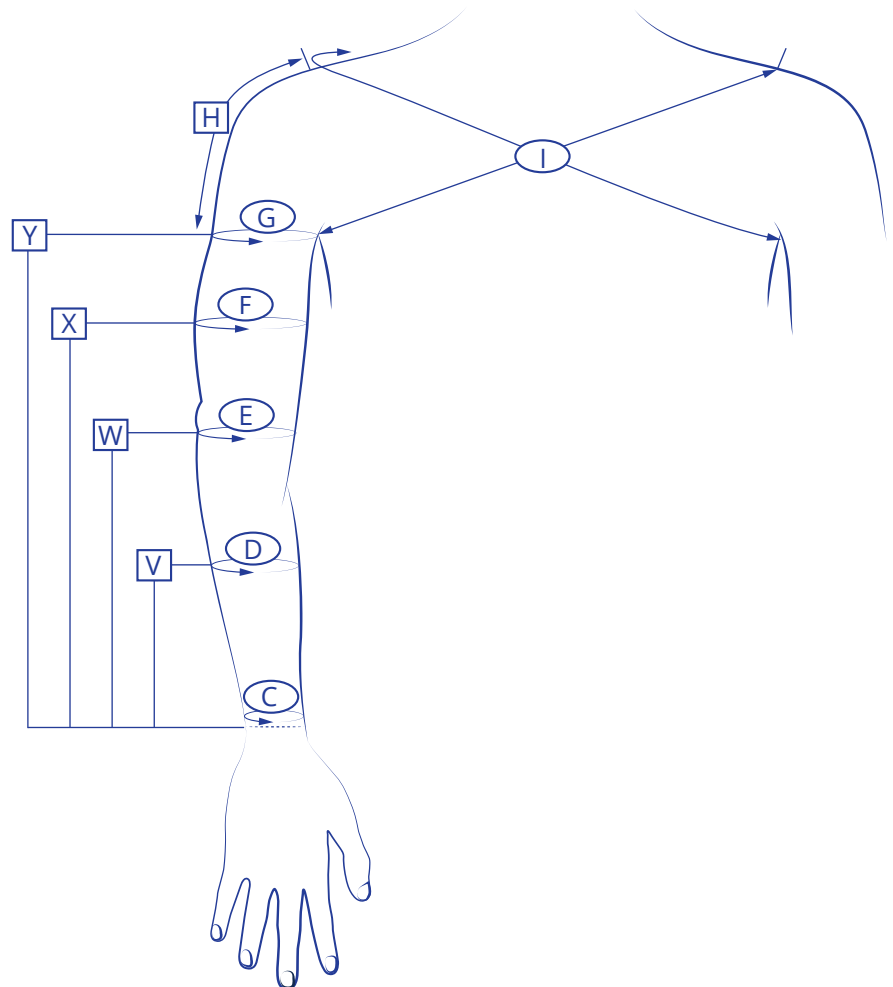

Circumferences in cm Length in cm



THUASNE

CICATREX



LEG n°4a

ORDER (by default) **QUOTATION** **RENEWAL**

Customer code

Case No. for renewal

1st treatment

Date: Quantity:

Patient's surname:

Patient's first name:

Gender: M F Child Patient's height:



I authorize my health care professional to collect my data and to communicate them to Thuasne company as part of the processing of my made to measure medical device in accordance with Law No 78-17 of 6 January 1978, and European Regulation No 2016/679/EU of 27 April 2016, I have rights including in particular the rights of access, rectification, portability and deletion of my data. I can exercise these rights by contacting the health care professional from whom I ordered my medical device.

PATIENT SIGNATURE

RETAILER IDENTIFICATION

L

**If possible, please enclose photos of the limb to be fitted.
Please draw in the contours of the garment on the diagram and cross unnecessary measures.
Please also fill in form No. 4b.**

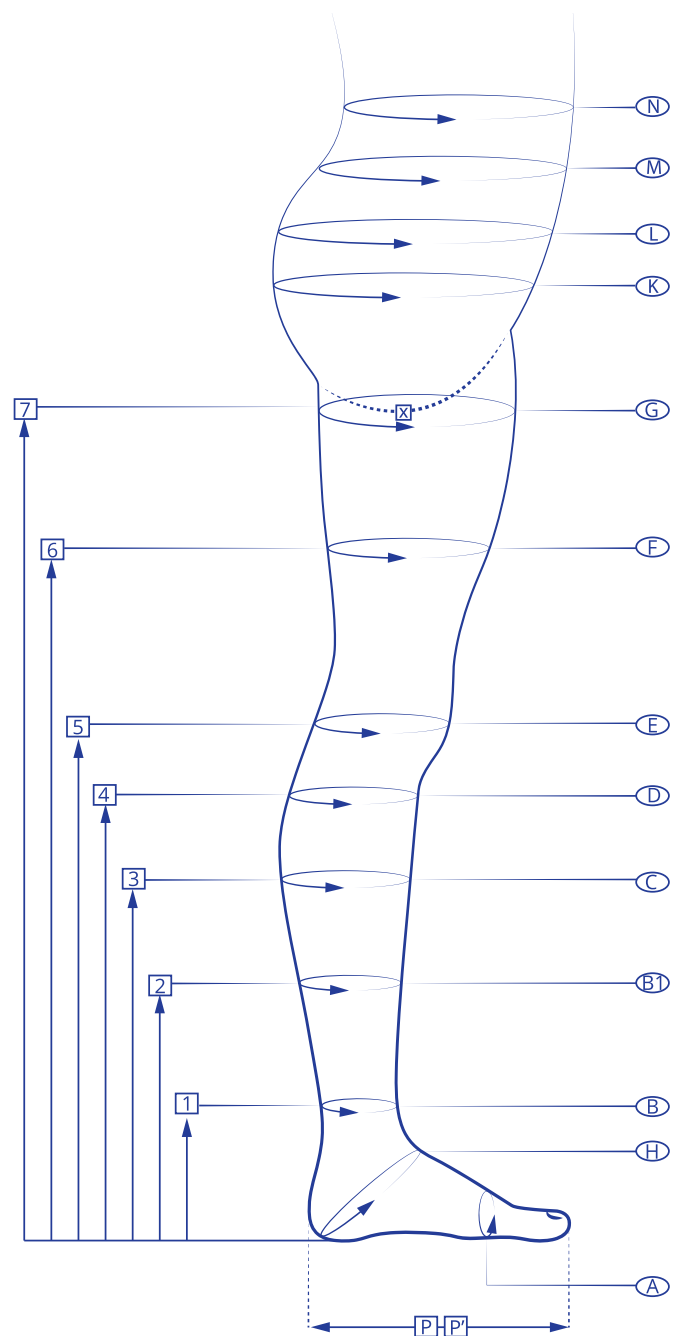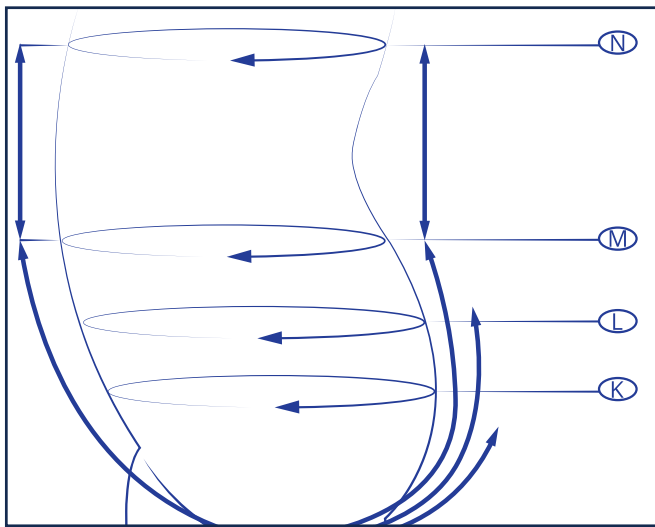

Circumference in cm Length in cm Crotch

Please contact your regular Thuasne distributor



THUASNE

CICATREX



LEG n°4b

ORDER (by default) **QUOTATION** **RENEWAL**

Customer code

Case No. for renewal

1st treatment

Date: Quantity:

Patient's surname:

Patient's first name:

Gender: M F Child Patient's height:

I authorize my health care professional to collect my data and to communicate them to Thuasne company as part of the processing of my made to measure medical device in accordance with Law No 78-17 of 6 January 1978, and European Regulation No 2016/679/EU of 27 April 2016, I have rights including in particular the rights of access, rectification, portability and deletion of my data. I can exercise these rights by contacting the health care professional from whom I ordered my medical device.

PATIENT SIGNATURE

RETAILER IDENTIFICATION

**If possible, please enclose photos of the limb to be fitted.
Please draw in the contours of the garment on the diagram and cross unnecessary measures.
Please also fill in form No. 4a.**

Indications

- Major burn (by default)
- EDS (Ehlers-Danlos Syndrome)

Models

- Below-knee
- Leg only
- Half-tight belt
- Panty

Fabrics

- Cicatrex Nature WHITE BEIGE BLACK
- Cicatrex Filifine
- Cicatrex AirSkin

Options

- Crotch**
- Open
- Hygiene tab
- Closed
- Gusset
- Kangaroo pouch

Velcro fastening (for vest)

- Vest over panty
- Panty over vest

Foot

- Open LEFT RIGHT
- Closed
- Attached
- Separate

Stomach opening

- Type:**
- Zipper cm

- Position:**
- Front Back Right Left

Soft fabric

- Knee LEFT RIGHT
- Heel
- Other:

Leg opening

- Type:**
- Zipper
- Velcro

- Length:
- Position: (to be drawn in) LEFT RIGHT
- Front position
- Back position (panty only)
- Outer position
- Inner position (sock only)

Sock & leg fastening

- Without anti-slip LEFT RIGHT
- Silicone anti-slip for men 5 cm
- Silicone anti-slip for women 5 cm
- Anti-slip with silicone dots 3 cm
- Elastic plain braid 3 cm

Panty & half-tight fastening

- Belt
- Braces
- Length: cm
- Other
- Compression pad
- Dimensions (l x h): cm
- Position: to be drawn in

Comments

RIGHT LEG

- A** 1 Circumference at the head of the metatarsal bones Height from floor at point B
- H** 2 Circumference at the instep going round the heel Height from floor at point B1
- B** 3 Ankle circumference (at the narrowest point) Height from floor at point C
- B1** 4 Ankle circumference at the base of the calf Height from floor at point D
- C** 5 Calf circumference (at the widest part) Height from floor at point E
- D** 6 Circumference below the knee Height from floor at point F
- E** 7 Knee circumference Height from floor at point G
- F** Mid-thigh circumference
- G** Thigh circumference at the gluteal fold
- P** Total inside length of the foot (from the tip of the big toe to the heel)
- P'** Desired length (from the tip of the little toe to the heel)

BRIEFS

- K** Circumference at the middle of the buttocks
- L** Hip circumference
- M** Waist circumference
- N** Abdominal circumference at the finished height desired
- g-m** Front distance: between the crotch and the waist (m)
- m-n** Front distance: between the waist (m) and the finished height of the panty (n)
- g-k** Back distance: between the crotch and the buttocks (k)
- g-l** Back distance: between the crotch and the hips (l)
- g-m'** Back distance: between the crotch and the waist (m)
- m-n'** Back distance: between the waist (m) and the finished height of the panty (n)

LEFT LEG

- A** 1 Circumference at the head of the metatarsal bones Height from floor at point B
- H** 2 Circumference at the instep going round the heel Height from floor at point B1
- B** 3 Ankle circumference (at the narrowest point) Height from floor at point C
- B1** 4 Ankle circumference at the base of the calf Height from floor at point D
- C** 5 Calf circumference (at the widest part) Height from floor at point E
- D** 6 Circumference below the knee Height from floor at point F
- E** 7 Knee circumference Height from floor at point G
- F** Mid-thigh circumference
- G** Thigh circumference at the gluteal fold
- P** Total inside length of the foot (from the tip of the big toe to the heel)
- P'** Desired length (from the tip of the little toe to the heel)



THUASNE

CICATREX



FOOT n°5

ORDER (by default) **QUOTATION** **RENEWAL**

Customer code

Case No. for renewal

1st treatment

Date: Quantity:

Patient's surname:

Patient's first name:

Gender: M F Child Patient's height:

I authorize my health care professional to collect my data and to communicate them to Thuasne company as part of the processing of my made to measure medical device in accordance with Law No 78-17 of 6 January 1978, and European Regulation No 2016/679/EU of 27 April 2016, I have rights including in particular the rights of access, rectification, portability and deletion of my data. I can exercise these rights by contacting the health care professional from whom I ordered my medical device.

PATIENT SIGNATURE

RETAILER IDENTIFICATION

**If possible, please enclose photos of the limb to be fitted.
Please draw in the contours of the garment on the diagram in black felt-tip pen.
Please complete form No. 4 for the rest of the lower limb.
For open toes, please indicate the length desired for each toe.**

RIGHT FOOT **LEFT FOOT**
Fill in one form for each side

Indications

- Major burn (by default)
- EDS (Ehlers-Danlos Syndrome)

Fabrics

- | | | | |
|---|--------------------------|--------------------------|--------------------------|
| | WHITE | BEIGE | BLACK |
| <input type="checkbox"/> Cicatrex Filifine* | <input type="checkbox"/> | <input type="checkbox"/> | <input type="checkbox"/> |
| <input type="checkbox"/> Cicatrex AirSkin** | - | <input type="checkbox"/> | <input type="checkbox"/> |


*Minimum circumferences of 4.6 cm, minimum length of fingers of 2 cm. No closed toes.
**Minimum circumferences of 2.6 cm, minimum length of fingers of 2 cm.

Option

Compression pad
Dimensions (ℓ x h): cm
Position: to be drawn in

Comments

Circumferences		Measures in cm
H	Circumference at the instep going round the heel	
A	Circumference at the head of the metatarsal bones	
C	Circumference of little toe at 1st phalanx	
D	Circumference of little toe at 2nd phalanx	
E	Circumference of the 4th toe at the 1st phalanx	
F	Circumference of the 4th toe at the 2nd phalanx	
G	Circumference of the 3rd toe at the 1st phalanx	
H	Circumference of the 3rd toe at the 2nd phalanx	
I	Circumference of the 2nd toe at the 1st phalanx	
J	Circumference of the 2nd toe at the 2nd phalanx	
K	Circumference of the big toe at the 1st phalanx	
L	Circumference of the big toe at the 2nd phalanx	
Desired lengths of the toe		Measures in cm
M	Length of little toe	<input type="checkbox"/> OPEN <input type="checkbox"/> CLOSED
N	Length of 4th toe	<input type="checkbox"/> OPEN <input type="checkbox"/> CLOSED
O	Length of 3rd toe	<input type="checkbox"/> OPEN <input type="checkbox"/> CLOSED
P	Length of 2nd toe	<input type="checkbox"/> OPEN <input type="checkbox"/> CLOSED
Q	Length of big toe	<input type="checkbox"/> OPEN <input type="checkbox"/> CLOSED
Total length of the foot		Measures in cm
Length of the inside of the foot (from the tip of the big toe to the heel)		
Length of the outside of the foot (from the tip of the little toe to the heel)		

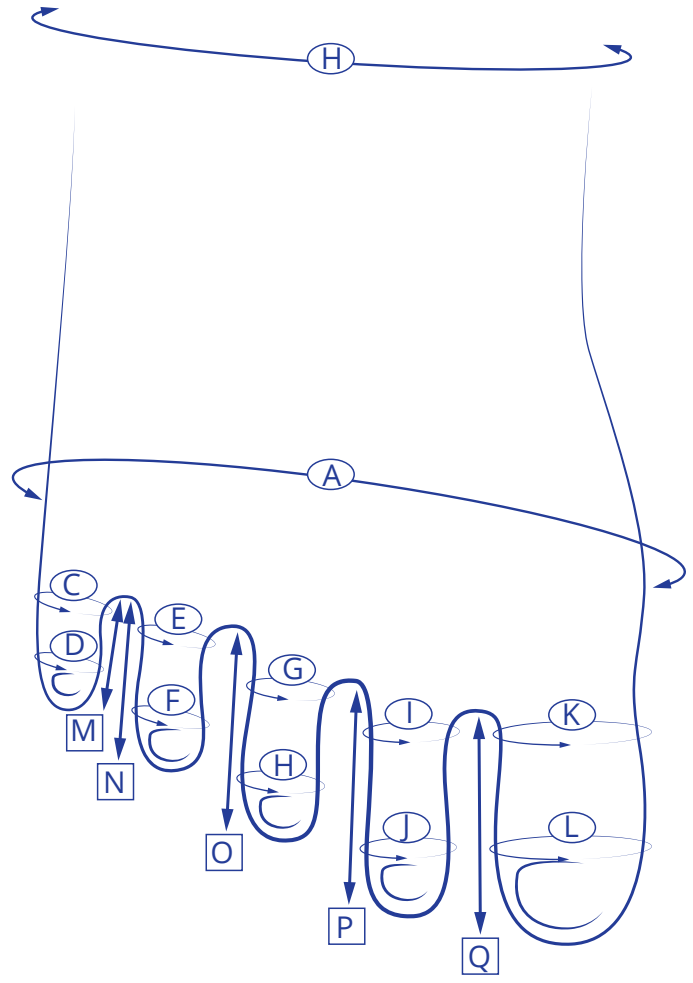

Circumference in cm Length in cm

Please contact your regular Thuasne distributor



THUASNE

CICATREX



HEAD n°6

ORDER (by default) **QUOTATION** **RENEWAL**

Customer code

Case No. for renewal

1st treatment

Date: Quantity:

Patient's surname:

Patient's first name:

Gender: M F Child Patient's height:

I authorize my health care professional to collect my data and to communicate them to Thuasne company as part of the processing of my made to measure medical device in accordance with Law No 78-17 of 6 January 1978, and European Regulation No 2016/679/EU of 27 April 2016, I have rights including in particular the rights of access, rectification, portability and deletion of my data. I can exercise these rights by contacting the health care professional from whom I ordered my medical device.

PATIENT SIGNATURE

RETAILER IDENTIFICATION

**If possible, please enclose photos of the limb to be fitted.
Please draw in the contours of the garment on the diagram and cross unnecessary measures.**

Models

- Hood
- Chin band

Fabrics

- Cicatrex Nature
- Cicatrex AirSkin

WHITE BEIGE BLACK

-

Options

Coverage

- Face UNCOVERED COVERED
- Ridge of nose
- Right ear
- Left ear



Other

- Compression pad REMOVABLE FIXED

Dimensions (ℓ x h): cm
Position: to be drawn in

Comments

A	Horizontal head circumference	
B	Vertical head circumference	
C	Upper neck circumference	
D	Lower neck circumference – end of bandage	
E	Nasal bone/occipital protuberance	
F	Nasal bone/upper neck circumference	
G	Lower lip – tip of the chin	
H	Tip of the chin – upper base of neck	
I	Front neck height	
J	Back neck height	
K	Ear canal – ear canal under the chin	
L	Lateral measurement of nose length	
M	Distance external canthus – ear canal	
1	Mouth width	
2	Nose width	
3	Space between the eyes, inner	
4	Space between the eyes, outer	
5	Length of right ear	
6	Length of left ear	
7	Bottom of ear – bottom of ear above the upper lip	
8	Top of ear – top of ear above the top of the nose	
9	Wing of the nose – wing of the nose above the nasal bone	
10	Nose length	
11	Eye width	

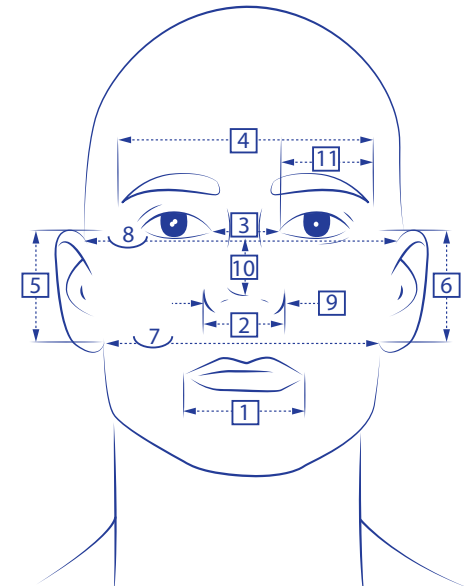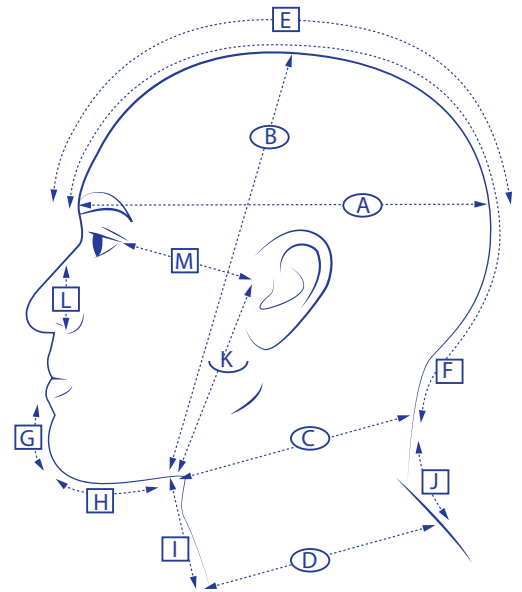

Circumference in cm Length in cm Half-circumference in cm



THUASNE



THUASNE

120 rue Marius AUFAN
92300 Levallois-Perret Cedex – France
Tel. +33 1 41 05 92 92 – Fax +33 1 41 05 89 91
Email : sales.export@thuasne.fr
www.thuasne.com

Thuasne UK Ltd

Unit 4 Orchard Business Center
North Farm Road
Tunbridge Wells - TN2 3XF
United-Kingdom
Tel: 01892 481620 - Fax: 01892 481609
Email: orders@thuasne.co.uk

THUASNE USA

4615 Shepard Street
Bakersfield, CA 93313
United-States
Tel: 1-800-432-3466
Email: CSOrders@thuasneusa.com

Biflex - Indications: Venous insufficiency, simple and complicated varicose veins, reduction of venous and post-traumatic edema, dermatitis/lipodermatosclerosis, varicose ulcers, deep vein thrombosis (prevention and confirmed accident), post-thrombotic syndrome, following sclerotherapy, treatment of lymphedema. Contraindications: PAD (peripheral arterial disease), skin wounds, abscess, infected skin conditions, extra-anatomic bypass, uncontrolled heart failure. **BiflexIdeal®** - Indications: Treatment of lymphedema, simple and complicated varicose veins, following sclerotherapy, following surgery, reduction of post-traumatic edema, varicose ulcers. Contraindications: Do not apply directly to an open wound. **Biplast** - Indications: Traumatology (joint or muscle compression sprain, dislocation, tendinitis), Phlebology (reduction and stabilization of edema), lymphology. Contraindication: Do not apply the product in direct contact with broken skin. **Cicatrex (RAL standard)** - Indications: Scar hypertrophy and keloid scars (excessive scar tissue), Scar prophylaxis from thermal, chemical and electrical injuries, Skin transplants, Reconstructive plastic surgery, Severe dermatological lesions, Treatment of lymphedemas in special cases. Contraindications: Advanced peripheral arterial occlusive disease, Decompensated heart insufficiency, Cardiogenic edema due to heart insufficiency, Myocardial infarction, Lung edema, Wetting dermatoses, Sensitivity disorders, Advanced peripheral neuropathy, Primary chronic polyarthritis. **Cicatrex (ASQUAL standard)** - Indications: Treatment of scars: heat, chemical, electrical burns; prevention of functional or unaesthetic complications of wound healing (keloids, hypertrophy, adhesions); skin grafts; Maintenance of volume reduction in lymphedema. **KitBiflex** - Indications: Venous ulcer of the lower limbs (stage C6 of the CEAP classification); Chronic venous edema; Lymphedema. Contraindications: Arterial pathologies; obliterant arterial disease of the lower limbs (AOMI), arterial ulcer or mixed arterial ulcer with blood predominance (systolic pressure index: IPS 0.6 (according to the High Authority of Health) or > 1.3); Patients with advanced diabetic microangiopathy, phlegmatia coerulea dolens (painful blue phlebitis with arterial compression), skeptical thrombosis; Extra-anatomical bypass surgery; Unbalanced heart failure; Pre-existing or infected dermatological conditions (abscesses, boils, infected dermatitis); Known allergy to any of the components; Advanced peripheral neuropathy; Uncontrolled diabetes. **Lymphatrex Essential** - Indications: Lymphatrex Essential compression devices are indicated in the treatment of primary or secondary lymphedema of the upper or lower limbs. Class 2: additional treatment for the lymphedema maintenance phase. Class 3: during the maintenance phase, after initial reduction of the affected limb, to maintain the reduced volume. Contraindications: Do not use Lymphatrex Essential lower limb compression devices in the event of: known allergies to the materials used, weeping or major dermatological conditions of a limb concerned, phlegmasia cerulea dolens (painful blue phlebitis with arterial compression) of a limb concerned, septic thrombosis of a limb concerned. Do not use Lymphatrex Essential lower limb compression devices in the event of: advanced diabetic microangiopathy (for compression > 30 mmHg), heart failure, peripheral artery disease (PAD) with ankle-brachial pressure index (ABPI) < 0.6. The risk/benefit ratio must be regularly reassessed by a healthcare professional in the event of: PAD with ABPI between 0.6 and 0.9, advanced peripheral neuropathy, weeping or eczematous dermatitis. **Lymphatrex Expert** - Indications: lymphatic diseases (primary and secondary lymphedema of the lower or upper limbs); venous diseases (varicose veins, edema, following phlebological surgery and sclerotherapy, pigmentation, venous eczema, healed ulcer, deep vein thrombosis, severe venous disorders during pregnancy, treatment of superficial and deep vein thrombosis, prevention and treatment of post-thrombotic syndrome). Contraindications: peripheral arterial disease (PAD) with ankle-brachial index (ABI) < 0.6; advanced diabetic microangiopathy (for compression > 30 mmHg); phlegmasia cerulea dolens (painful blue phlebitis with arterial compression); septic thrombosis. The risk/benefit ratio must be regularly re-assessed in the event of: PAD with ABI between 0.6 and 0.9; advanced peripheral neuropathy; exudative or eczematous dermatitis; intolerance to the fibres used. **Lymphatrex** - Indications: Lymphedema, Lipedema, Lipo-lymphedema, Phlebo-lymphedema, Congestion conditions following immobility (arthrogenic congestion syndrome paresis and partial paresis of the extremity). Contraindications: Advanced peripheral arterial occlusive disease, Decompensated heart insufficiency, Cardiogenic edema due to heart insufficiency, Myocardial infarction, Lung edema, Wetting dermatoses, Sensitivity disorders, Advanced peripheral neuropathy, Primary chronic polyarthritis. **MOBIDERM PADS & BANDAGES** - Indications: Acute or chronic edema management (e.g. lymphedema). Contraindications: Skin infection of the limb or acute inflammation, known allergy to components used, septic thrombosis, severe peripheral neuropathy of the limb. Lower limb - PAD (peripheral arterial disease) of lower limbs with ABPI < 0.6, congestive heart failure, advanced diabetic microangiopathy, phlegmasia cerulea dolens, extra-anatomic bypass. Upper limb - pathology of brachial plexus, vasculitis of the extremities. **MOBIDERM AUTOFIT, STANDARD AND MADE-TO-MEASURE GARMENTS** - Indications: Maintenance of volume reduction in lymphedema. Contraindications: Skin infection of the limb or acute inflammation, known allergy to components used, septic thrombosis, severe peripheral neuropathy of the limb. Lower limb - PAD (peripheral arterial disease) of lower limbs with ABPI < 0.6, congestive heart failure, advanced diabetic microangiopathy, phlegmasia cerulea dolens, extra-anatomic bypass. Upper limb - pathology of brachial plexus, vasculitis of the extremities. **VARICO, FOAM BANDAGE, PROTECTIVE UNDERWRAP** - Indications: Protection of cutaneous surfaces to cover. **VENO Elegance** - Indications: Soft indentable lymphedema (stage I), arm edema following breast cancer surgery and radiotherapy, post-surgery and post-traumatic edema. Contraindications: Advanced peripheral artery occlusive disease, decompensated cardiac insufficiency, septic phlebitis, phlegmasia coerulea dolens, weeping dermatoses, compression material intolerance, paresthesia, advanced peripheral neuropathy, primary chronic polyarthritis. Foam bandages and Varico products contain latex.



LYMPHATREX Essential, LYMPHATREX Expert, Cicatrex (ASQUAL standard), MOBIDERM, BIFLEX, Biplast, BIFLEXIDEAL, KITBIFLEX, bandages accessories:

VENO Elegance, LYMPHATREX, Cicatrex (RAL standard):

THUASNE
120 rue Marius AUFAN
92300 Levallois-Perret Cedex - France

Thuasne Deutschland GmbH
Im Steinkamp 12
30938 Burgwedel - Deutschland

Availability of these products might vary from a given country or region to another, as a result of specific local regulatory approval or clearance requirements for sale in such country or region. All the medical devices mentioned on this document are CE marked according to the European council directive 93/42/EEC and its relatives or the Regulation 2017/745 on medical devices, unless specifically identified as "NOT CE MARKED". Products mentioned in this document are CE class I devices. Please contact Thuasne should you need any additional information on devices classification. Please read carefully the instructions for use, indications and contraindications of the product. We reserve the right to modify our products as shown and described in this catalogue, without previous notice. Non-contractual photographs. Please check the availability of products according to the countries.
Last revision date: (2020-07). Ref: 0116 092 164 - 09

