

MOBIDERM & VENOFLEX MEDICAL COMPRESSION SOLUTIONS



The reduction phase

The reduction phase is also known as the intensive decongestive treatment.

Its goal is to achieve maximum reduction of the edema to come as close as possible to the volume of the opposite healthy limb.

The main treatments used are:

- Manual lymphatic drainage (MLD), carried out by a specifically trained physiotherapist, to stimulate and facilitate the circulation of the lymph by promoting the unblocking of the lymph nodes.
- Reducing and draining bandages requiring the use of multi-component bandages which can be adapted to the volume and repositioned at will.

• Coached physical activity under bandages to improve lymph circulation, regain mobility and flexibility.



This phase may vary from one to several weeks, depending on the severity of the lymphedema, and can be renewed later.



Thuasne Protocol

The reduction phase is usually performed

using multi-component bandages⁽¹⁾

The advantage of bandages is that they can be repositioned over and over to ensure the best possible fit to successive variations in edema volume.

There are different sorts of bandages, and it is their combined use that reduces edema.



PROTECT THE SKIN

PROTECTIVE COTTON BANDAGE

In direct contact with the skin, this tubular cotton bandage is used to protect the skin. Made of cotton, it provides soft and lined protection of the skin weakened by the lymphedema.



MOBILIZE THE EDEMA MOBIDERM Bandage

Used under a reducing bandage, the Mobiderm technology mobilizes edematous or indurated subcutaneous tissue.



CONTAIN THE EDEMA BIFLEXIDEAL Short-stretch bandage

A short-stretch bandage (< 100% stretch) as the third layer helps containment of the edema, as well as drainage in case of physical activity (high working pressure)⁽²⁾.



COMPRESS THE EDEMA

BIFLEX Graduated elastic compression bandage

A long-stretch bandage as last layer applies constant pressure, even at rest. **BIFLEX 16+** (light) is recommended for mild to moderate lymphedema. **BIFLEX 17+** (strong) is recommended for moderate to severe lymphedema.



Scan to learn how to bandage with MOBIDERM https://www.youtube.com/watch?v=7-wkbmFEkkk



Compression bandages











Cutaneous protection

PROTECTIVE COTTON BANDAGE*

Protective bandage

Tubular bandage. Composition 100% cotton.

Protection of skin surface under non-adhesive bandage.

Width	Length	Color	Reference
11 cm	4 m	White	010101 01140000

MOBIDERM

MOBIDERM PADS

Dimensions	Designation	Reference
25 cm x 25 cm	Small blocks 5 mm x 5 mm	371001 02502501
25 cm x 1 m	Small blocks 5 mm x 5 mm	371001 03310001
20 cm x 1 m	Big blocks 15 mm x 15 mm	371001 03310002

MOBIDERM BANDAGES

Dimensions	Designation	Reference
10 cm x 3 m	Small blocks 5 mm x 5 mm	372001 01030001
10 cm x 3 m	Big blocks 15 mm x 15 mm	372001 01030002

Containment

BIFLEXIDEAL

Short-stretch compression bandage

A short-stretch bandage (<100% stretch) as the third layer helps with containment of the edema, as well as drainage in case of physical activity (high working pressure).⁽¹⁾

Width	Length	Color	Reference
3 cm			110002 00350000
8 cm	5 m		110002 00850000
10 cm			110002 01050000

BIPLAST

Adhesive short-stretch compression bandage

Herringbone adhesive application. Multidirectional adhesiveness right to the edge of the bandage. Flexible and adjustable. 50.2% maximum stretch. (2)

Width	Length	Color	Reference
3 cm	2 E	\	20000100325000
10 cm	2.5 m	White	200001 01025000

Compression bandages



Compression


BIFLEX+ GRADUATED

Graduated elastic compression bandage

Two-ways elastic bandage. Elastic in length and extensible in width. Repeatable control of the tension exerted thanks to the fitting indicator. Multiple overlappings possible to achieve multiple pressure level.



2 versions	Width	Length	Color	Reference
BIFLEX 16+ (Light)	8 cm	5 m	Beige	160002 00850002
BIFLEX 17+ (Strong) 2 yarns	8 cm	4 m	Beige	170002 00840002

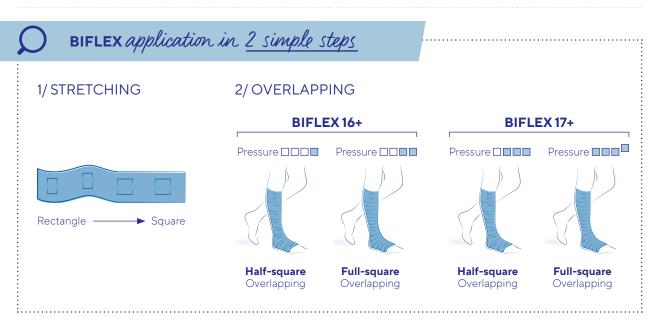



The **maintenance** phase

As soon as a satisfactory volume reduction is achieved, the maintenance phase of the lymphedema starts.

Its role is to stabilize the volume of the reduced limb in the initial phase, or even to continue the volume reduction, but also to avoid relapses and enable the patient to recover normal mobility.

This phase is conducted first with elastic compression garments which will be renewed regularly under the supervision of a healthcare professional.



Worn daily from waking up to bedtime, day-time compression garments ensure both:

- **compression** aimed at easing the circulation of the lymph to the heart, thus preventing the formation of edema.
- containment preventing the edema from recovering more than a certain percentage of volume, irrespective of its intrinsic expansion force.

To optimize treatment efficiency, it is recommended to wear light compression device during rest and night-time.



MOBIDERM garments have been developped to mobilize the edema during rest and night-time. They offer an efficient, easy to use and comfortable alternative to self-bandage.⁽¹⁾

The maintenance phase also includes manual lymphatic drainage at a lesser frequency, as well as the learning of self-massaging and self-bandaging as needed. Lastly, it requires the practice of an adapted physical activity, skin cares and regular medical monitoring.

VENO ELEGANCE

Standard circular knit compression garments



For which patients

VENO ELEGANCE armsleeves are intended for use in the treatment of lymphedema in maintenance phase following a reduction phase.

The product is recommended for patients presenting:

- Mild to moderate lymphedema(1)
- Regular limb shape without skin folds(1)

Principle of action

Compression: to help lymphatic circulation and prevent the formation of edema⁽²⁾

Characteristics

Composition⁽³⁾: polyamide microfiber, elastane

Color: beige

Compression class: class 2 (23 - 32 mmHg)

Innovation in Lymphology Dec 2015 - Page 5
(3) Please refer to the Instructions for Use of the product for the full list of components



⁽¹⁾ SFMV - Compression and lymphoedema of the lower limb Compression Interface Committee - March 2015 - Page 2 (2) SNITEM - Coupé - Compression and restraint: what are the differences?, History and



Models and Options


Armsleeve & Armsleeve with mitten

23 - 32 mmHg



DESIGNATION	Ref Short	Ref Long
Armsleeve with anti-slip		
Small	2002D221102561	2002D221102563
Medium	2002D221202561	2002D221202563
Large	2002D221302561	2002D221302563
Armsleeve with mitten and anti-slip		
Small	2002D221102551	2002D221102553
Medium	2002D221202551	2002D221202553
Large	2002D221302551	2002D221302553

Sizes (in cm)

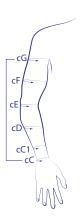

	SMALL	MEDIUM	LARGE					
cG	25 - 29.5	30 - 35.5	36 - 42					
сF	23 - 26.5	27 - 31	31.5 - 36					
cЕ	21 - 24.5	25 - 28.5	29 - 33					
сD	20 - 23.5	24 - 27.5	28 - 32					
cC1	13.5 - 16	16.5 - 19	19.5 - 22.5					
сC	13 - 15.5	16 - 18.5	19 - 22					
Leng	th SHORT							
eG	38 - 44	40 - 46	42 - 48					
Length LONG								
eG	42 - 48	44 - 50	46 - 52					



LYMPHATREX Expert

Made-to-measure patient flat knit compression garments

For which patients?

LYMPHATREX Expert garments are intended for use in the treatment of lymphedema in maintenance phase, following a reduction phase.


The range is recommended for patients presenting:

- Moderate to severe lymphedema(1)
- Non-regular limb shape with skin folds(1)
- Spreading to fingers or toes(1)
- Large limb circumference(1)

Principle of action

Containment: a short-stretch and robust technical knit allows LYMPHATREX Expert garments to apply a high working pressure and a low resting pressure on the limb (2).

of lymphatic circulation and optimization of the drainage during mouvements

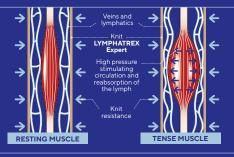

Characteristics

Composition (3): polyamide, elastane



Colors: beige , tanning beige



), black _i

Compression classes:

class 2 (15 - 20 mmHg), class 3 (20 - 36 mmHg), class 4 (> 36 mmHg)







SOFTNESS

- Knitted from **polyamide microfibre yarn**, naturally soft⁽¹⁾.
- Fine weft of knitting and not in direct contact with the skin, in order to keep the softness of the mesh. (1)

COMFORT

- Anatomical adjustment, even at the most sensitive areas such as elbow folds or knee. (2)
- Thermoregulation by double-sided knitting: absorption and then removal of moisture outwards(1).




- Flat seams. (3)
- Sheathing knit, does not penetrate skin folds.

DISCRETION AND DRESS

- Available in timeless and sober colors: black, beige and tanning beige to match different skin carnations. (4)
- Controlled balance between the knitting parameters and the anti-slip device helps with the **proper hold on the limb.** (5)





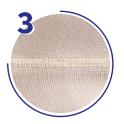






Knit adaptation to limb volume variation

The flat knitted technology allows to vary the number of stitch loop on each row based on the precise patient measurements.

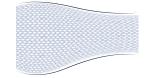




https://mobiderm.com.au/collections/made-to-measure-flat-knit-garments

LYMPHATREX Expert

Made-to-measure patient flat knit compression garments

Models and Options

Upper limb

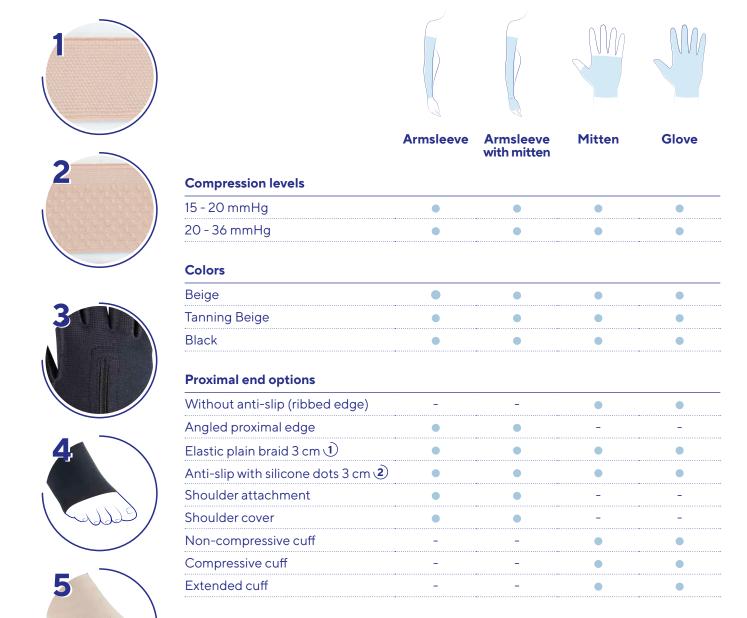

Distal end options

Zipper opening hand top 3

Foam compression pad

Lower limb

Combination with thigh-high















)}^	
	Below- knee	Thigh- high	Half- Tight	Tights	Panty	Toe Cap
Compression levels						
20 - 36 mmHg	•		•	•	•	•
36 - 50 mmHg	•	•	•	•	•	•
Colors						
Beige	•	•	•	•	•	•
Tanning Beige	•	•	•	•	•	•
Black	•	•	•	•	•	•
Proximal end options						
Without anti-slip (ribbed edge)	•	_	-	-	-	-
Elastic plain braid 3 cm ①	•	_	_	-	-	_
Anti-slip with silicone dots 3 cm 2	•	-	-	-	-	-
Anti-slip with silicone dots 5 cm 2	-	•	-	-	-	-
Angled thigh end	-	•	-	-	-	-
Hip cover	_	•	_	-	_	_
Elastic belt 5 cm	_	_	•	•	•	_
Adjustable elastic belt	_	_	•	•	•	_
Without belt	-	_	•	•	•	-
Brief options						
Compressive brief	-	_	•	•	•	-
Non-compressive brief	-	-	•	•	•	-
Hygiene tab	-	-	•	•	•	-
Opening zipper	-	-	•	•	•	-
Opening self-gripping fasteners	-	-	•	•	•	-
Without opening	-	-	•	•	•	-
Distal end options						
Open toe 4	•	•	•	•	-	_
Closed toe	•	•	•	•	-	-
With toes 5	•	•	•	•	_	•
Compression pad	•	•	•	•	-	-
Other options						

MOBIDERM autofit

Standard adjustable mobilizing garments

Description

The **MOBIDERM** autofit garments are designed to mobilize edematous or indurated subcutaneous tissues.

They are intented for the treatment of lymphedema in maintenance phase at rest and night to maintain the volume reduction.

The standard, self-adjustable garment adapts to most of patient's individual body shape and evolution overtime.

They are available in multiple models:

- For upper limb: **MOBIDERM** autofit armsleeve with mitten.
- For lower limb: MOBIDERM autofit below-knee or thigh-high.

For which patients?

MOBIDERM autofit garments are recommended for patients presenting:

- Mild to moderate lymphedema
- Regular or non-regular limb shape
- Stable or evolutive edema






Principle of action

The MOBIDERM autofit garments incorporate MOBIDERM technology that is made up of foam blocks encased between two non-woven bandages.

Due to the difference in pressure it creates between the support area of the blocks and their surrounding area, it helps stimulate the flow of lymphatic fluid and optimizes drainage efficacy(1).

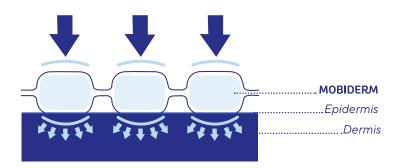

Characteristics

Composition⁽²⁾:

- Outer layer: polyester, polyamide, elastane
- Inner layer along the leg/arm: adhesive non-woven, polyurethane foam (15 x 15 mm blocks)
- Inner layer foot/hand level: adhesive non-woven, polyethylene foam (5 x 5 mm blocks)

Color: beige



Learn how to fit Mobiderm autofit here!



Upper limb: https://www.youtube.com/ watch?v=gcTrtkPIR94



Lower limb: https://www.youtube.com/ watch?v=zv2Qaj59614

MOBIDERM autofit

Standard adjustable mobilizing garments

Upper limb

Several tightening notches

- adaptation to most of patient's individual limb shape
- adjustment marks: precise and repeatable application, control of the edema volume variation

Self-gripping fasteners

• easy and quick set up(1)




Semi-open garment

easy to put on, easy to take off(1)



- helps limiting the risk of edema migration
- the hand can be freed for usual tasks

Sizes

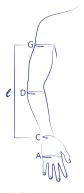

- ℓ : Length from wrist to axillary
- cA: Hand circumference
 cC: Wrist circumference
 cD: Forearm circumference (at
 the widest point)
 cG: Axillary arm circumference

Size	cA (in cm)	cC (in cm)	cD (in cm)	cG (in cm)	Reference Normal version ℓ = 40 - 45 cm	Reference Long version ℓ = 45 - 50 cm
Right	armsleeve					
1	17 - 21	15 - 19	22 - 28	25 - 32	373302 20100261	373302 20100271
2	17 - 21	16 - 20	24 - 30	29 - 36	373302 20200261	373302 20200271
3	18 - 22	17 - 21	26 - 32	33 - 40	373302 20300261	373302 20300271
4	18 - 22	17 - 21	28 - 34	37 - 44	373302 20400261	373302 20400271
5	19 - 23	18 - 22	30 - 36	41 - 48	373302 20500261	373302 20500271
6	20 - 24	19 - 23	31 - 37	45 - 52	373302 20600261	373302 20600271
Left a	rmsleeve					
1	17 - 21	15 - 19	22 - 28	25 - 32	373302 20100262	373302 20100272
2	17 - 21	16 - 20	24 - 30	29 - 36	373302 20200262	373302 20200272
3	18 - 22	17 - 21	26 - 32	33 - 40	373302 20300262	373302 20300272
4	18 - 22	17 - 21	28 - 34	37 - 44	373302 20400262	373302 20400272
5	19 - 23	18 - 22	30 - 36	41 - 48	373302 20500262	373302 20500272
6	20 - 24	19 - 23	31 - 37	45 - 52	373302 20600262	373302 20600272



_ower limb



Several tightening notches

- · adaptation to most of patient's individual limb shape
- adjustment marks: precise and repeatable application, control of the edema volume

Semi open garment

• easy to put on, easy to take off(1)

Self-gripping fasteners

easy and quick set up⁽¹⁾

Double row


- · wide adaptation range
- · allows right or left leg use

Anti-slip sole

· limits slippage when walking



Sizes

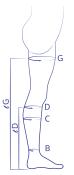

	_	_	_		Reference	(right/left)	
Size	cB (in cm)	cC (in cm)	cD (in cm)	Short ℓ D = 38 - 40 cm	Normal ℓ D = 40 - 42 cm	Long ℓ D = 42 - 44 cm	Xtra Long ℓ D = 44 - 46 cm
1	16 - 24	24 - 35	25 - 36	373401 20100231	37340120100232	373401 20100233	373401 20100234
2	20 - 28	32 - 43	32 - 43	37340120200231	37340120200232	37340120200233	37340120200234
3	24 - 32	40 - 51	39 - 50	37340120300231	37340120300232	37340120300233	37340120300234
4	28 - 36	48 - 59	46 - 57	37340120400231	37340120400232	37340120400233	37340120400234

Thigh-high

Below-knee

				Reference (right/left)			
cB (in cm)	cC (in cm)	cG (in cm)	Short ℓ G = 67 - 72 cm	Normal ℓ G = 72 - 77 cm	Long ℓ G = 77 - 82 cm	Xtra Long ℓ G = 82 - 87 cm	
16 - 24	24 - 35	46 - 57	37350120100231	37350120100232	37350120100233	37350120100234	
20 - 28	32 - 43	53 - 65	37350120200231	37350120200232	37350120200233	37350120200234	
24 - 32	40 - 51	61 - 72	37350120300231	37350120300232	37350120300233	37350120300234	
28 - 36	48 - 59	68 - 80	37350120400231	37350120400232	37350120400233	37350120400234	
	20 - 28 24 - 32	20 - 28 32 - 43 24 - 32 40 - 51	20 - 28 32 - 43 53 - 65 24 - 32 40 - 51 61 - 72	16-24 24-35 46-57 37350120100231 20-28 32-43 53-65 37350120200231 24-32 40-51 61-72 37350120300231	cB (in cm) cC (in cm) cG (in cm) Short (e = 67 - 72 cm) Normal (e = 72 - 77 cm) 16 - 24 24 - 35 46 - 57 37350120100231 37350120100232 20 - 28 32 - 43 53 - 65 37350120200231 37350120200232 24 - 32 40 - 51 61 - 72 37350120300231 37350120300232	cB (in cm) cC (in cm) cG (in cm) Short (e = 67 - 72 cm) Normal (e = 77 - 82 cm) Long (e = 77 - 82 cm) 16 - 24 24 - 35 46 - 57 37350120100231 37350120100232 37350120100233 20 - 28 32 - 43 53 - 65 37350120200231 37350120200232 37350120200233 24 - 32 40 - 51 61 - 72 37350120300231 37350120300232 37350120300233 28 - 36 48 - 59 68 - 80 37350120400231 37350120400232 37350120400233	

ℓD: Floor-to-knee height (measurement taken below

the bend of the knee, back side)

G: Floor-to-thigh height (measurement at the gluteal fold)

B: Ankle circumference (at the narrowest point)

cC: Calf circumference (at the widest point)
cD: Below knee circumference (on the head
of the fibula)
cG: Thigh circumference (at the widest point)



STANDARD MOBIDERM GARMENTS

Standard mobilizing garments

Description

The MOBIDERM standard garments are designed to mobilize edematous or indurated subcutaneous tissues. They are intended for the treatment of lymphedema in maintenance phase at rest and for night to maintain the volume reduction.

They are available in 3 models: armsleeve, mitten or glove.

For which patients?

MOBIDERM standard garments are recommended for patients presenting:

- Mild to moderate lymphedema
- Regular limb shape
- Stable lymphedema volume

Characteristics

Composition(1):

- Outer layer: polyamide, elastane
- Armsleeve inner layer: adhesive non-woven, polyurethane foam (15 x 15 mm blocks)
- Mitten and glove inner layer: adhesive non-woven, polyethylene foam (5 x 5 mm blocks)

Color: beige





-					
А	rm	si	е	e١	ve

	•	_	•	Reference	(right/left)		
Size	cC (in cm)	cE (in cm)	cG (in cm)	Normal ℓ = 40 - 45 cm	Long ℓ=45-50 cm		
1	14 - 16	22 - 25	25 - 29	373002 20100232	373002 20100233		
1+	14 - 10	24 - 27	28 - 32	373002 40100232	373002 40100233		
2	16 - 18	25 - 28	29 - 33	373002 20200232	373002 20200233		
2+	10 - 18	27 - 30	31 - 35	373002 40200232	373002 40200233		
3	18 - 20	28 - 31	32 - 36	373002 20300232	373002 20300233		
3+	10 - 20	30 - 33	35 - 39	373002 40300232	373002 40300233		
4	20 - 22	31 - 34	35 - 39	373002 20400232	373002 20400233		
4+	20 - 22	33 - 36	38 - 42	373002 40400232	373002 40400233		
5	22 - 24	34 - 37	38 - 42	373002 20500232	373002 20500233		
5+	ZZ - Z4	36 - 39	41 - 45	373002 40500232	373002 40500233		



A: Hand circumference (head of metacarpal bones) C: Wrist circumference E: Elbow circumference G: Axillary arm circumference





Glove

C:	сC	сA	Reference				
Size	(in cm)	(in cm)	right	left			
1	14 - 16	15 - 17	373201 20100201	373201 20100202			
2	16 - 18	17 - 19	373201 20200201	373201 20200202			
3	17 - 19	19 - 21	373201 20300201	373201 20300202			
4	19 - 21	21 - 23	373201 20400201	373201 20400202			
5	20 - 22	23 - 25	373201 20500201	373201 20500202			
6	21 - 23	25 - 27	373201 20600201	373201 20600202			

Mitten

Size	сC	cA	Reference
Size	(in cm) (in cm		right/left
1	14 - 16	15 - 17	373101 20100299
2	16 - 18	17 - 19	373101 20200299
3	17 - 19	19 - 21	373101 20300299
4	19 - 21	21 - 23	373101 20400299
5	20 - 22	23 - 25	373101 20500299
6	21 - 23	25 - 27	373101 20600299

MADE-TO-MEASURE PATIENT MOBIDERM GARMENTS



Description

The MOBIDERM made-to-measure patient garments are designed to mobilize edematous or indurated subcutaneous tissues.

They are intended for the treatment of lymphedema in maintenance phase for day and night to maintain the volume reduction.

They are available in multiple models:

- For upper limb: armsleeve 1, armsleeve with mitten and without thumb 2 or armsleeve with mitten and thumb 3
- For lower limb: below-knee 4 or thigh-high 5



For which patients

MOBIDERM made-to-measure patient garments are recommended for patients

- Moderate to severe lymphedema
- Non-regular limb shape
- Out of standard sizes
- · Stable lymphedema volume



Characteristics

Composition(1):

- Outer layer: polyamide, elastane
- Inner layer: adhesive non-woven, polyurethane foam (15 x 15 mm blocks) or polyethylene foam (5 x 5 mm blocks)













MOBIDERM INTIMATE BRA

For which patients

The Mobiderm Intimate Bra is recommended for patients presenting:

- Acute or chronic breast and/or chest wall edema
- Chronic infiltration after reconstruction

Benefits

- **Ready to use:** compressing lymph bra with mobilising Mobiderm pads (2 pieces) that are integrated, removable and ready for immediate use
- **Self management:** easy to put on, hook and zip front closure, softly lined integrated pockets for breast form
- Discreet color: light nude color, not visible even under light clothes

Features

- Adaptable to most body shapes: several sizes available
- Flexible and thin: thanks to the physical properties of the materials: polyamide, elastane⁽¹⁾
- A turnkey solution: bra + pad
- Possibility of inserting the pad (or an external breast prosthesis) inside the bra or removing it according to the patient's needs
- Easy to put on and take off: thanks to the opening on the front of the bra

Color: nude



Compression class

• Light compression 5-10 mmHg



Chest

The bra is intended for women who have had breast cancer surgery, to reduce swelling and prevent the appearance of lymphedema.

After breast plastic surgery, the Mobiderm Intimate bra has a form-stabilizing effect(1) and provides support through compression and drainage of edema⁽²⁾⁽³⁾ that may develop after breast cancer treatment.

- The bra exerts increased pressure⁽⁴⁾ on the tissue areas affected by edemas. This improves lymphatic drainage⁽³⁾.
- Compression in the scar tissue area minimizes the development of hematomas⁽⁵⁾.





Front view Rear view

Sizes

Underbust size INT	Size	68-72 70	73-77 75	78-82 80	83-87 85	88-92 90	93-97 95	98-102 100	103-107 105	108-112 110	113-117 115
		X	S	9	5	1	4	I	L	Х	L
	AA/A	80 - 89 T571301201AAA24		90 - 99 T571301202AAA24		100 - 109 T571301203AAA24		112 - 119 T571301204AAA24		120 - 129 T571301205AAA24	
Cup size	B/C	84 - 93 T5713012010BC24		94 - 103 T5713012020BC24		104 - 113 T5713012030BC24		116 - 123 T5713012040BC24		124 - 133 T5713012050BC24	
	D/E	88 - 97 T5713012010DE24		98 - 107 T5713012020DE24		108 - 117 T5713012030DE24		120 - 127 T5713012040DE24		128 - 137 T5713012050DE24	



Discover the video tutorial with donning instructions for Mobiderm Intimate Bra on our Thuasne YouTube channel!



https://mobiderm.com.au/ collections/pelvic-andgenital-lymphoedema/ products/mobidermintimate-bra

(1) Due to its origin; the Mobiderm Intimate bra is based on the Cara compression bra.
(2) Thanks to the pad.
(3) I. Quéré, E. Presles, M. Coupé, S. Vignes, L. Vaiillant, D. Eveno, S. Laporte, A. Leizorovicz, Investigateurs étude POLIT - Prospective multicenter.
Observational study of lymphoedema therapy: POLIT study – Journal des maladies vasculaires 2014) 39, 256-263
(4) Refer to CE marking
(5) Refer to CE marking data

MOBIDERM INTIMATE SHORTS

Compression and mobilization shorts Treatment of pelvic and genital lymphedema

For which patients

Mobiderm Intimate shorts is intended for patients with:

- · Pelvic and/or genital edema
- Mild to severe lymphedema
- · Stable or progressive edema
- Regular or irregular morphology

Benefits

- Improve the quality of life*: can be worn throughout the day and night and the product is discreet
- Easy to use: the product is ready to wear, all-in-1 product with removable pad, and it's easy to use for 3 out of 4 patients*

Composition

- Compression shorts: polyamide, elastane, polyester
- Mobiderm mobilizing genital pad small blocks: polyester, polyamide, acrylic glue, polyethylene foam

Sizes

cK		cK (cm)	cG (cm)	Ť
cG	1	88 - 94	45 - 54	3111790041429
\ /	2	95 - 101	48 - 58	3111790041436
	3	102 - 108	51 - 62	3111790041443
	4	109 - 115	54 - 66	3111790041450
\	5	116 - 122	57 - 69	3111790041467
	6	123 - 129	60 - 72	3111790041474

cK		cK (cm)	cG (cm)	*
cG	1	88 - 94	45 - 54	3111790041368
\ /	2	95 - 101	48 - 58	3111790041375
	3	102 - 108	51 - 62	3111790041382
	4	109 - 115	54 - 66	3111790041399
	5	116 - 122	57 - 69	3111790041405
1	6	123 - 129	60 - 72	3111790041412

cK (cm): Pelvic circumference cG(cm): Upper thigh circumference



Pelvic and genital area



Features:

Conform:

- Soft fabric⁽¹⁾⁽²⁾ for more comfort
- Flat seams to limit skin reactions

Easy donning⁽³⁾:

Easy to put in and out - elastic fabrics

Easy measuring: only 2 measurements to take and only one size table.

Mobiderm pad:

- Covered with soft⁽¹⁾⁽²⁾ fabric for more comfort
- Small blocks for more adaptability^{(4)*}
- 2 Mobiderm pads versions to adapt various anatomies: 1 men version, to be placed in the pocket and around the testicles

1 women version, to be placed in the shorts inner pocket







- (1): CC1508 Mobiderm Panty Validation Plan (2019)
- (2): CC1508 Rational formulation (2021)
- (3): CC1508 Usability Test Report Summative Evaluation (2020)
- (4): Please refer to internal EC data
- *The contact surface is reduced, optimisation of the number of blocks.



Discover all available video tutorials Mobiderm Intimate fitting instructions **on our Thuasne YouTube channel!**

*OLYMPY study: Exploratory study to assess the clinical benefits of wearing the MOBIDERM Intimate Short in patients with pelvic and/or genital lymphedema. - 2021 32 patients - n° 10-RCB: 2020-A00247-32; primary objective: assessment of the effect of the MOBIDERM Intimate short, after 12 weeks, on an overall impression of change in their quality of life in relation to PL/GL



https://mobiderm.com.au/collections/pelvic-and-genital-lymphoedema/products/intimate-short-for-pelvic-and-genital-lymphoedema



FOR HER VENOFLEX KOKOON ABSOLU

Comfort and softness for cocooning moments

Description

15 - 20 mmHg

- The semi-opaque range
- · Broadest Venoflex palette of sizes and colours
- Soft mesh in opaque microfiber
- · Flat seams
- · Reinforced heel and foot arch
- Instep shaping
- Anatomic calf shape
- · Machine washable

For which patients?

- Those who desire an opaque finish to their hosiery
- Casual style
- Those requiring a flexible and soft material 40



SOCKS

5220 01



OPEN-TOF SOCKS 5220 01



SOCKS CALF+/-5220 01



TIGHTS 5223 01



MATERNITY & FULL HIPS **TIGHTS**



STOCKINGS 5225 02



5225 02



STOCKINGS 5225 02



15 - 20 mmHg 15 - 20 mmHg 15 - 20 mmHg

5224 01

15 - 20 mmHg 15 - 20 mmHg 15 - 20 mmHg

































FOR HIM VENOFLEX FAST AIR

Rapid moisture transpiration and refreshing effect (2)

Description

15 - 20 mmHg

- An original combination of fibers suitable for all life situations, including the most extreme
- BREEZE: soft fiber with refreshing effect*
- AQUARIUS: High-performance hydrophilic fiber*
- Patented FAST technology inside
- Technical sole
- Yarn composition: Polyamide, elastane
- Machine washable 40

For which patients?

- Those requiring hard-wearing stockings
- Elegant and urban style
- Those wishing to wear comfortable and breathable socks



SOCKS

6120 N3



THUASNE / 25

15 - 20 mmHg

http://www.nilit.com: NILIT BREEZE: soft fiber with cooling effect / NILIT AQUARIUS: high performance hydrophilic fiber (1) Soft mesh composition - (2) As a result of the composition of the BREEZE and AQUARIUS fibers

VENOFLEX

VENOFLEX devices are designed to be used on the lower limbs by adults meeting the indications outlined below. They provide compression, applying pressure to the limb and thereby aiding venous return.

thereby didning verious return.

INDICATIONS:

- Varicose veins (≥ 3 mm)
- Following sclerotherapy or surgery for varicose veins (≥ 3 mm)
- · Chronic venous oedema
- Trophic disorder (pigmentation, eczema, venous lipodermatosclerosis or atrophie blanche)
- · Healed venous ulcer
- Chronic venous disease associated with pregnancy or the post-partum period
- Prevention and/or treatment of venous thrombosis (lower limb) and post-thrombotic syndrome
- Lower limb lymphoedema (maintenance phase)

Compression level: 20 - 36 mmHg

INDICATIONS:

- Primary and secondary varicosis, varices during pregnancy, after sclerotherapy or venous surgery.
- Superficial thrombophlebitis and condition after healed phlebitis, deep vein thrombosis, condition arter thrombosis, post thrombotic syndrome, thrombosis prophylaxis.
- Chronic venous insufficiency (CVI) of stages I to III according to Widmer and C1S-C6 according to CEAP, ulcer prevention, venous insufficiency, angiodysplasia.
- Lymphedema, edema in pregnancy, post-traumatic edema, postoperative edema, lipoedema, lipo-lymphedema, phlebo-lymphatic edema, congestive conditions as a result of immobility (arthrogenous congestion syndrome, pareses nd partial pareses of a limb).

Compression level: 23 - 32 mmHg



FOR HER VENOFLEX KOKOON ABSOLU

Comfort and softness for cocooning moments

Description

20 -36 mmHg

- The semi-opaque range
- Broadest Venoflex palette of sizes and colours
- Soft mesh in opaque microfiber
- Flat seams
- · Reinforced heel and foot arch
- · Instep shaping
- · Anatomic calf shape
- Machine washable 40

For which patients?

- Those who desire an opaque finish to their hosiery
- · Casual style
- · Those requiring a flexible and soft material











5230 01

OPEN-TOE SOCKS 5230 01

STOCKINGS 5235 01

5233 01

20 - 36 mmHg 20 - 36 mmHg 20 - 36 mmHg 20 - 36 mmHg























Discover more about Venoflex here! www.venoflex.com.au

FOR HER VENOFLEX MICRO

The opaque range

Description

23 - 32 mmHg

- Comfortable and soft thanks to the silky soft yarn
- · Skin-friendly, hypoallergenic material
- Machine washable at 40

For which patients?

 Those looking for elegant and feminine style





30021B



30023E



STOCKINGS STOCKING+



TIGHTS

30025B



MATERNITY & FULL HIPS **TIGHTS**

30027B

23 - 32 mmHg 23 - 32 mmHg 23 - 32 mmHg 23 - 32 mmHg 23 - 32 mmHg

30023H













THUASNE /

WINGS **FOR YOUR HEALTH**

Indications

Indications:

MOBIDERM pad & bandages: treatment of acute or chronic edema (for example: lymphedema). MOBIDERM garments: maintenance of the reduction in lymphedema swelling. MOBIDERM INITIMATE: treatment of pelvic and genital lymphedema. MOBIDERM autofit: maintenance of the reduction in lymphedema swelling. COTTON BANDAGE: For skin protection and immobilisation in multicomponent bandaging. BIFLEXIDEAL: Treatment of edema (traumatic or chronic). Treatment of lymphedema. Chronic venous insufficiency (as an alternative to compression stockings). Following sclerotherapy or varicose vein surgery. Treatment of venous ulcers (with ABI > 0.8 or under close medical supervision for an ABI between 0.5 and 0.8). Symptomatic treatment of deep or superficial acute venous thronosis. Prevention of post-thrombotic syndrome: alternative to compression stockings for the treatment of edema of the legs or svelling during pregnancy. Treatment of symptoms related to soft tissue injuries.

BIFLEX: Alternative to compression stockings for the treatment of edema of the legs or svelling during pregnancy. Treatment of symptoms related to soft tissue injuries.

BIFLEX: Alternative to compression stockings for the treatment of edema of the legs or svelling during pregnancy. Treatment of symptoms related to soft tissue injuries.

BIFLEX: Alternative to compression stockings for trophic disorders (dermatitis/lipodermatosclerosis). Venous leg ulcers (multitype bandaging, elastic bandage for patients with reduced mobility). Symptomatic treatment of acute superficial and deep venous thromosis. Alternative to compression or antithrombotic stockings. Post-thrombotic syndrome prevention: alternative to compression stockings for high-risk patients and prevention or surgery of varicose veins as an alternative to compression stockings for high-risk patients. Lymphedema prevention: alternative to compression stockings for trophic disorders (dermatitis/lipodermatosclerosis). Venous leg ulcers (multitype bandaging, elastic bandages for patients with redu Class 3: during the maintenance phase, after initial reduction of the affected limb, to maintain the reduced volume. MOBIDERM INTIMATE BRA: Acute or chronic edemas of the breast and/or the breast wall. Post-operative edemas or edemas after radio therapy (only on intact skin). Chronic infiltration after reconstruction, remaining chronic breast wall edemas. MOBIDERM INTIMATE SHORT: Treatment of pelvic and genital lymphedema. VENOFLEX KOKOON ABSOLU & FAST*IAIR: 15-20 mmHg/Class 2: heavy legs, restless legs, evening oedema, telangiectasia or reticular veins (< 3 mm), following sclerotherapy for telangiectasia or reticular veins (< 3 mm), following sclerotherapy or surgery for varicose veins (≥ 3 mm), varicose veins (≥ 3 mm), prevention of venous thrombosis in pregnant women or post-partum, prevention of venous thrombosis during long journeys, treatment of lymphoedema (maintenance phase). 20-36 mmHg/Class 3: varicose veins (≥ 3 mm), following sclerotherapy or surgery for varicose veins (≥ 3 mm), chronic venous oedema, trophic disorders (pigmentation, eczema), chronic trophic disorders (lipodermatosclerosis or atrophic blanche), healed venous ulcer, chronic venous disease associated with pregnancy or the post-partum period, prevention and/or treatment of venous thrombosis (lower limb) and post-thrombotic syndrome, lower limb lymphoedema (maintenance phase). VENOFLEX MICRO: Primary and secondary varicosis, varices during pregnancy, after sclerotherapy or venous surgery. Superficial thrombosis and condition after healed phlebitis, deep vein thrombosis, condition arter thrombosis, postthrombotic syndrome, thrombosis prophylaxis. Chronic venous insufficiency (CVI) of stages I to III according to Widmer and CIS-C6 according to CEAP, ulcer prevention, venous insufficiency, angiodysplasia. Lymphedema, edema in pregnancy, post-traumatic edema, postoperative edema, lipoedema, lipoedema, phlebo-lymphatic edema, congestive conditions as a result of immobility (arthrogenous congestion syndrome, pareses and partial pareses of a l

Availability of these products might vary from a given country or region to another, as a result of specific local regulatory approval or clearance requirements for sale in such country or region. The medical devices mentioned on this document are CE marked according to the Regulation 2017/745 on medical devices. Products mentioned in this document are CE class I devices (except for the cotton bandage which is a Medical Device accessory). Please contact Thuasne should you need any additional information on devices classification. Please read carefully the instructions for use, indications and contraindications of the product. We reserve the right to modify our products as shown and described, without previous notice. Non-contractual photographs. Please check the availability of products according to the countries. Ref.: 2302049 - Last revision date: 03-2023. Photos: Studio Caterin - Krakenimages/Shutterstock.com.



Thuasne France

120 rue Marius Aufan 92300 Levallois-Perret Cedex - France www.thuasne.com sales.export@thuasne.fr

Thuasne Deutschland Im Steinkamp 12, 30938 Burgwedel Deutschland

Thuasne Pacific Ptv Ltd Unit 6, 640-680 Geelong Rd Brooklyn Victoria 3012 Australia 0432 847 576 info@thuasne.com.au mobiderm.com.au Follow MOBIDERM Australia on Facebook