- ① Wash hands and face thoroughly before using I-LID 'N LASH[®]. Apply a moist warm compress to closed eyelid in order to loosen encrusted debris and secretions.
- ② Open the container and remove one I-LID 'N LASH[®] wipe. When finished, close the container tightly.
- ③ Gently rub eyelids and eyelash roots with the I-LID 'N LASH[®] pre-soaked wipe in a circular motion until all debris is removed. (Please refer to Figure 1) If you are using I-LID 'N LASH[®] Pump, simply add some gel to a cotton pad and apply to the eye in the same way.
- ④ If necessary, clean the area around the lid margins more thoroughly by wiping in short strokes in the direction of the eyelashes, starting at the roots and going away from the eye for both lower and upper eyelids. (Please refer to Figure 2)

I-LID 'N LASH[®] does not need to be rinsed-off. The remaining gel will continue to hydrate the skin.

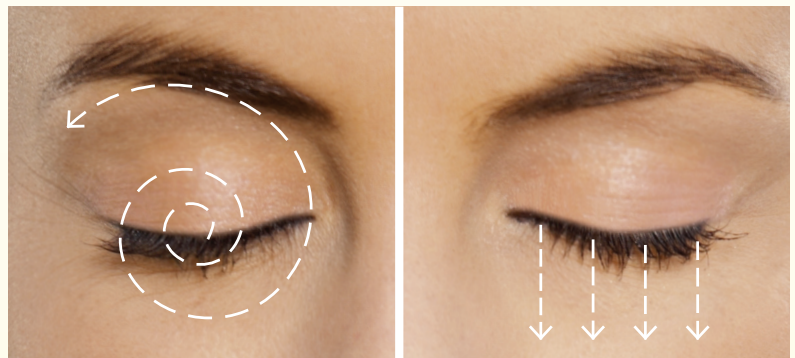
Figure 1

Figure 2

- ⑤ Repeat steps 2 through 4 for the second eye.

I-LID 'N LASH[®] can be used daily as a lid and lash cleanser to remove ocular debris.

(Do not use on broken or abraded skin. Not to be used by children under 3 years of age)

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