

# Instructions for use

*i-Lid 'n Lash*  
**PRO**

1. Wash the area around the patient's eyes before applying i-Lid 'n Lash® Pro.
2. Place an anesthetic drop in the eye before the treatment.
3. Dispense a generous portion of i-Lid 'n Lash® Pro on a clean wipe or the palm of your hand. Note that upon first use the pump may need to be primed before any gel comes out.
4. Instruct the patient to close the eye lightly and to keep it closed for the duration of the treatment.
5. Gently apply the gel using your finger or a cotton swab on the lid margin up to the brow line (please refer to Figure 1).

6. If necessary, clean the area around the lid margins and roots of the eyelash more thoroughly by wiping in short strokes in the direction of the eyelashes, e.g. starting at the roots and going away from the eye for both lower and upper eyelashes.
7. If necessary, add more gel of i-Lid 'n Lash® Pro during any step of the process.

8. Allow the gel to stay on the area around the eye and the lid margins, while keeping the eye shut, for a period of 10 minutes for optimal results.

9. Wipe off any excess gel from around the eye with a clean tissue.

10. Repeat steps 2 through 8 for the second eye.


11. Rinse off any residual gel with warm water. If the patient experiences stinging, rinse the eye and surrounding area thoroughly with water.

12. Repeat this process if any crusty residue remains on the lash roots or lashes.

13. Send the patient home with a jar of the i-Lid 'n Lash® Plus, and depending on the severity of the condition, use once or twice daily.

Start here for Step 5

For Step 6: use motion as below

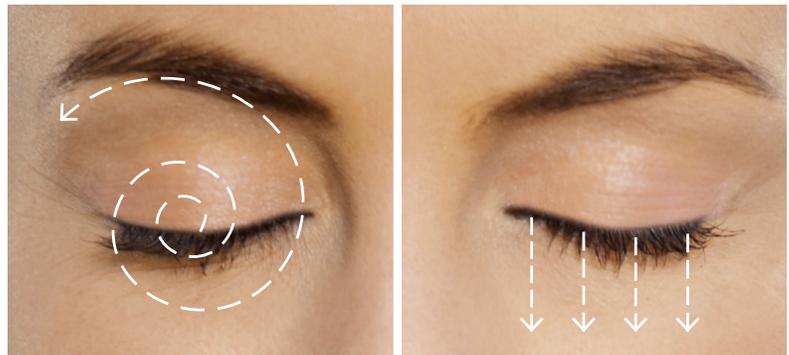

**Fig. 1:** Apply the i-Lid 'n Lash® Pro gel in a circular and spiral motion starting at the roots of the lashes and then out towards the lids and eyebrows.