


# Instructions for use



1. Wash hands and face thoroughly before using i-Lid 'n Lash<sup>®</sup> Plus.  
i-Lid 'n Lash<sup>®</sup> Plus may be used to remove any residual makeup around the eyes and the face.
2. Place a drop of i-Drop<sup>®</sup> Pur or i-Drop<sup>®</sup> Pur GEL on the eye to provide a protective barrier.
3. Open the container and remove one i-Lid 'n Lash<sup>®</sup> Plus wipe. When finished, close the container tightly.
4. Before applying the i-Lid 'n Lash<sup>®</sup> Plus, close the eye lightly, without squinting, and keep it closed until the application is complete.

5. Gently apply the gel from the wipe in a circular motion and expanding outward spiral starting at the eyelash roots (please refer to Figure 1).
6. If necessary, clean the area around the lid margins and roots of the eyelash more thoroughly by wiping in short strokes in the direction of the eyelashes (e.g. starting at the roots and going away from the eye for both lower and upper eyelashes).
7. If necessary, use a new i-Lid 'n Lash<sup>®</sup> Plus wipe during any step of the process.

Start here for Step 5

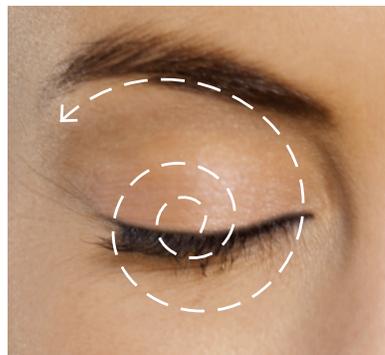

For Step 6: use motion as below



**Fig. 1:** Apply the i-Lid 'n Lash<sup>®</sup> Plus wipe in a circular and spiral motion starting at the roots of the lashes and then out towards the eyebrows and face.

8. Allow the gel to stay on the area around the eye and the lid margins, while keeping the eye gently shut, for a period of 30 seconds to 1 minute.
9. Repeat steps 2 through 8 for the second eye.
10. If you experience stinging, rinse the eye and surrounding area thoroughly with water.
11. Repeat this process if any crusty residue remains on the lash roots or lashes.
12. Perform this procedure once or twice a day for at least four weeks.