


Cleaning Procedure: Stabilizers and Knee Belts

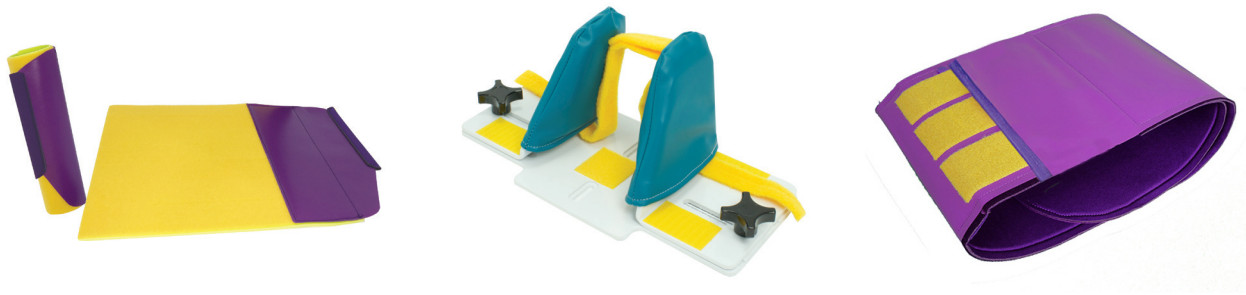

Applies to Rainbow® Arm and Leg Stabilizers (RAS), Rainbow® Knee Belts (RKB) and Rainbow® Head Stabilizer **not including the vinyl cushions* (RHII, RHIC, RHIA)

When cleaning the Rainbow® Head Stabilizer, remove vinyl head cushions from side panels and clean the cushions separately.

Apply disinfecting spray or wipes following the manufacturer's directions, then wipe dry. Do not allow disinfecting agent to remain on the plastic or foam, as this may degrade the material and reduce the life of the product.