Mesh Nebulizer

User Manual

Model No. YS35

User Manual(EN).....	1-18
Bedienungsanleitung(DE).....	19-38
Manuale Utente(IT).....	39-58
Manuel d'utilisation(FR).....	59-78
Manual de usuario(ES).....	79-98

Contents

Easy@Home YS35 Mesh Nebulizer Overview	1
Intended Use	1
Contraindications	2
Nebulizer Components	2
Instructions for Use	3
Cleaning and Disinfection	7
Storage and Maintenance	9
Safety Warning	10
Electromagnetic Compatibility	12
Warranty.....	15
Troubleshooting	16
Technical Specifications	17
Explanation of Symbols	18

Easy@Home YS35 Mesh Nebulizer Overview

Thank you for purchasing Easy@Home YS35 mesh nebulizer. Please read the manual carefully to make sure that you can operate it correctly and safely. Please store the manual in an easily accessible place you will remember after reading it for later reading and referencing.

The nebulizer is designed for respiratory disease treatment and respiratory tract infection prevention. The nebulizer is energized by a certain circuit frequency vibration to make a piezoelectric ceramic vibrate harmony that causes high-speed vibration of metal mesh. Liquid medicine will be quickly moved through micro mesh holes of metal mesh plate to become countless micro atomizing particles. This will transfer by inhalation treatment using masks or mouthpieces to the respiratory system.

Intended Use

The nebulizer is designed for respiratory disease treatment and respiratory tract infection prevention. It is widely used in improving and alleviating symptoms of diseases of the respiratory systems, such as cold, fever, bronchitis, asthma, pneumonia, pneumonic mycosis, and pulmonary heart disease. It can also be used for throat care and treatment of chronic allergic rhinitis, acute laryngitis, acute pharyngitis, acute tonsillitis, and other ear, nose, and throat disorders.

The nebulizer should be used with a face mask, and the face mask's connection tube diameter should be $20.0 \pm 0.1\text{mm}$, so the face mask can be connected to the nebulizer tightly.

The nebulizer is suitable for adults, the elderly, infants, and children. The nebulizer can be used in a household environment, and it complies with the requirement of IEC 60601-1-11.

Contraindications

- The nebulizer cannot be applied to anesthesia breathing systems and ventilator systems.
- Infants and children must use this nebulizer only under adult supervision.
- Pentamidine is not applicable to this nebulizer.

Nebulizer Components

Package Contents

- | | | |
|------------------|----------------|-----------------|
| • Main unit*1 | • Adult mask*1 | • Carry pouch*1 |
| • Medicine cup*1 | • Child mask*1 | • User manual*1 |
| • Mouthpiece*1 | • USB cable*1 | • Rope*2 |

Main unit

Medicine cup

Mouthpiece

Rope

Adult Mask

Child Mask

USB cable

Carry pouch

Figure 1 Components

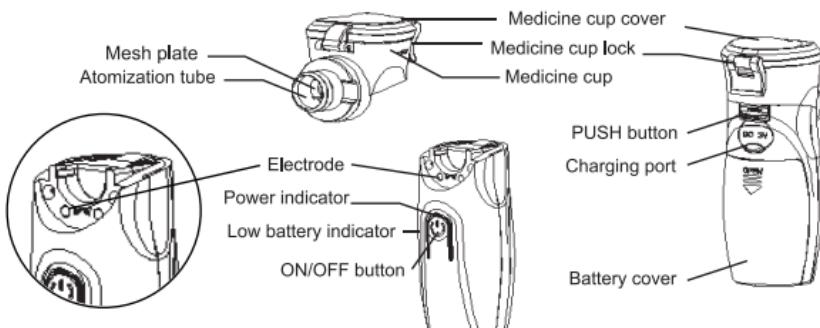
Figure 2 Detailed Components

Instructions for Use

1. Power Supply

This nebulizer comes with a built-in rechargeable battery and USB cable (included). Charge the nebulizer with an adaptor (not included, normal smartphone charger can be used). Please make sure to charge the nebulizer fully before first use. It takes 1-1.5 hours for a full charge.

Low Power Indication and Charging

The LED light will flash to indicate low battery. Turn off the nebulizer and charge the battery immediately. The red LED light will keep flashing during charging and will become solid red when the charge is completed.

NOTE:

- Adaptor not included.
- Use an adaptor that meets electrical safety standard.

Adaptor technical parameter reference:

Output: DC 3 ~ 5V, 1 ~ 2A

2. Fitting the parts of the nebulizer

It is recommended that only one mask should be used by one person. For hygiene reasons, the nebulizer and its accessories must be cleaned before and after each use and disinfected after the last treatment of the day or before a prolonged period of no use. See the "Cleaning and Disinfection" section.

Holding the main unit in one hand and the medicine cup in the other, push them together (on the top of the main unit) until you hear a click. (Figure 3)

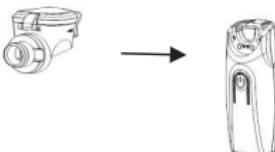
Figure 3 Fitting the parts of the nebulizer

⚠ NOTE:

- If the medicine cup is not fitted in place, the electrodes may fail to conduct and the nebulizer will not work.
- Please keep the electrodes clean for good contact and conductivity (see the "Cleaning and Disinfection" section), otherwise the nebulizer may not work.

3. Filling the cup with medicinal liquid

- Holding the main unit in one hand, use the other to release the clasp of the lid on the medicine cup;
- Pour the medicinal liquid into the medicine cup (Fig. 4) The amount of liquid in the cup must not be less than 0.5 ml, not more than 8 ml.
- Close the lid and fasten the clasp.
- Do not shake the nebulizer or turn it upside-down, which could cause spillage of the medicinal liquid.

Figure 4 Filling the medicinal liquid

⚠ NOTE:

- While using the nebulizer, sufficient contact should be maintained between the medicinal liquid and the mesh, to avoid the mesh rupture due to air shock.

4. Fitting the accessories and Beginning Use

- Fit the nebulizer with the appropriate accessory: use the mouthpiece for treatments to be administered to the throat, and the child or adult masks for treatments to be administered through the nasal cavity or to the throat (Figure 5).

Figure 5 Fitting the accessories

- Push START/STOP button once the accessory is in place, then the power indicator light (Blue Light) will light up and nebulized medicinal liquid will be released from the nebulizer.
 - Keep calm, relaxed and sit upright, with your hand holding the nebulizer well. Take slow deep breaths to ensure more effective treatment:
 - 1) When using the mouthpiece, put the mouthpiece flat end into the mouth, lips slightly closed, so that the liquid medicine is targeted to the affected area.
 - 2) When using a mask, put the mask slightly close to face, covering nose and mouth:
Treat nose cavity by deep inhaling through the nose, exhaling through the mouth. Treat throat by inhaling through the mouth, exhaling through the nose.
- *You can make alternate use of the mouthpiece and the mask in one treatment.

 **NOTE:**

- After assembly and before use, pour 0.9% sterile saline solution into the medicine cup, then push START/STOP to check if it works properly.
- To use the mouthpiece correctly, the liquid should not be directed towards the tongue.
- Do not swallow the liquid, make sure it reaches the intended area.
- Poor posture and uneven breathing may lead to potential obstruction of respiratory passages.
- After 20 minutes using the nebulizer, the time indicator (Yellow) will flicker continuously, and the nebulizer powers off automatically.
- When the medicinal liquid level is low, tilt the nebulizer toward you from the START/STOP side and collect the medicinal liquid that has accumulated on the mesh plate to make use of all the residual liquid.
- Please avoid using the nebulizer with the medicine cup empty to prevent the mesh plate from breaking.
- When the Low battery indicator (Red) flashing, it indicates the battery is running out. Charge the battery as soon as possible.
- Do not shake the nebulizer violently during use, otherwise it will cause the device to accidental power off.

5. Turn Off The Power

- At the end of the treatment, press the START/STOP button to turn off the power, the blue Power Indicator will switch off.

6. Disassembling the Nebulizer After Using

- Remove the mouthpiece or mask.
- Hold the main unit in one hand, press the PUSH button on the back (shown in figure 6); Hold the Medicine cup in the other hand, remove the Medicine cup from its recess by sliding it forwards;

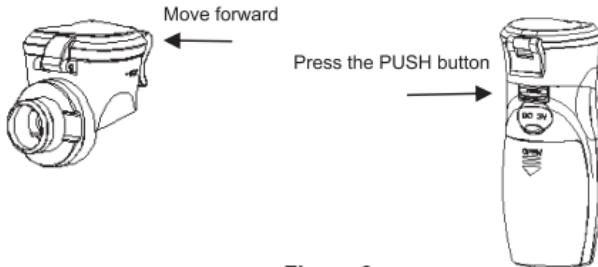
Figure 6

- Open the lid, dispose all the residual liquid in the medicine cup, and promptly clean the nebulizer. Then allow the parts to air dry and store the nebulizer in a clean, well-ventilated, and dry place.

Cleaning and Disinfection

- Make sure the nebulizer is switched off after use.
- After each use, discard the residual medicine, and remove the cup attachment from the main unit to clean and disinfect thoroughly. Dry the parts immediately after washing.
- Do not store the main unit, medicine cup and accessories when wet or damp.

1. Cleaning and sanitizing the medicine cup

- Open the medicine cup and discard any leftover medicinal liquid.
- Pour a small amount of hot distilled water (30°C-60°C) into the medicine cup and close the lid.
- Press the START/ STOP button for 5 seconds, the blue, yellow, and red indicator light will turn on simultaneously.
- After switching the nebulizer to cleaning mode as above, nebulize the distilled water for 1-2 minutes then the residual medicine will be eliminated.
- Finally, rinse the medicine cup and lid with distilled water. Carefully remove any excess water with a clean, soft cloth or leave the parts to dry naturally in a clean place.

 **NOTE:**

- Use medical gauze or cotton balls for cleaning and disinfecting operations.
- Do not rub with paper or other cloths, to avoid compromising the correct operation of the nebulizer.
- Do not clean using sharp objects or touch the mesh.
- If the mesh is clogged, please pour 2ml of hot water (the water temperature is recommended not to exceed 80°C, and pay attention to avoid scalding) or acetic acid solution (the acetic acid solution should be prepared by mixing 1 equal part white vinegar with 3 equal parts water) into the medicine cup first, and press the [Start/Stop] button to start the nebulizer until hot water or acetic acid solution is exhausted. Turn off the power, remove the cup from the main unit, then soak the medicine cup in hot water or acetic acid solution for about 3 minutes, and then take out the medicine cup and rinse it with distilled water.
- Do not use household bleach or other aggressive substances to clean the nebulizer, only the solution indicated above is allowed.

2. Cleaning and sanitizing the main unit

- Gently wipe the main unit with 75% medical alcohol and wipe off excess water with a soft, clean cloth.
- Clean the electrodes with a soft, clean cloth.

Caution:

- Do not wipe the main unit with other volatile liquid, such as benzene or diluents.
- Do not rinse or immerse the main unit in any liquid.

3. Cleaning and sanitizing the mouthpiece and masks

- Remove the mouthpiece or masks;
- Rinse the mouthpiece and mask with distilled water for 2 ~ 3 minutes (Water Temperature ≤50°C) ;
- Dry well with medical gauze;
- Wipe the mask and mouthpiece with 75% medical alcohol at least 3 minutes to disinfect, and air dry to store.

 **NOTE:**

- Clean and disinfect all the parts.

Storage and Maintenance

1. Storage Conditions:

- Environmental Temperature: - 20°C ~ 55°C;
- Relative Humidity: ≤ 80% R.H.
- Air Pressure: 70.0 ~ 106.0 kPa;
- Other requirements: No direct sunlight, Ventilation, No corrosive gas, No heating nebulizer, or Open fire.

Storage Instructions:

- The product's validity is advised as 3 years.
- Clean and disinfect the nebulizer after each use. Store the nebulizer in the storage pouch provided, when not in use.

2. Working Conditions

- Environmental Temperature: 10°C ~ 40°C;
- Relative Humidity: ≤ 80% R.H. No condensation;
- Air Pressure: 86.0 ~ 106.0 kPa

Maintenance Instructions

- Operate the nebulizer in normal work conditions.
- Do not operate the nebulizer near a heating nebulizer or open fire. Do not dry the nebulizer and its accessories in the microwave oven or stove, nor with a hair dryer, etc.
- Don't make the nebulizer and its accessories contact corrosive liquids or gases.
- Refer to 'Troubleshooting' section for solutions to issues or contact the manufacturer or distributor if issues continue.

 **WARNING:** Users are strictly prohibited from disassembling, repairing, or altering the nebulizer parts by themselves.

Safety Warning

1. Safety Information

- Do not use or store the nebulizer in humid places. Use the nebulizer within the temperature and humidity ranges included in "Working Conditions".
- Read this instruction manual carefully before using the nebulizer.
- Accessories mentioned in the instruction manual are intended for use with this nebulizer only.
- Treat the nebulizer with care, protect it from violent shocks, such as dropping on the floor.
- DO NOT try to disassemble, repair or replace the nebulizer components yourself.
- DO NOT operate the nebulizer or touch the adaptor or power switch with wet hands.
- Do not immerse the main unit in water or any other liquid, avoid splashing water or other liquid onto the main unit or AC adaptor.
- To prevent damage to the nebulizer, do not carry or leave the medicine cup filled with medicinal liquid or distilled water in the nebulizer when not in use.
- Dispose of the nebulizer, components, and optional accessories according to local law or regulations. Unlawful disposal may cause environmental pollution.
- This nebulizer is a medical device. Consult your physician for users wearing the heart pacemaker.
- This nebulizer is approved for human use only.

2. Care for the Nebulizer

- Do not use the nebulizer with ester, fat or suspended particles, including herbal medicine. It is suggested to use the standard nebulizing medicine in accordance with your physician's advice.
- The nebulizer nebulizes for 20 minutes each time. The amount of Medicine is 2 ~ 4 ml; however, it must be always used as indicated by a doctor.
- The nebulizer is a personal medical equipment. It must be utilized for its intended use only. Any other use is to be considered improper, therefore, potentially dangerous.
- Do not touch the mesh with your hand or any other objects.
- Keep the nebulizer out of the reach of babies and unsupervised children.
- Keep the nebulizer at least 3.5 meters from other electrical products with an energy consumption higher than 2W, to avoid electromagnetic interference.

Electromagnetic Compatibility

Guidance and manufacturer's declaration - electromagnetic emission		
Mesh Nebulizer is intended for use in the electromagnetic environment specified below. The customer or the user of Mesh Nebulizer should assure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment-guidance
RF emissions GB4824	Group 1	Mesh Nebulizer uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions GB4824 Class B	Class B	
Harmonic emissions GB 17625.1	N/A	Mesh Nebulizer is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Voltage fluctuations/ flicker emissions GB 17625.2	N/A	

Guidance and manufacturer's declaration -electromagnetic immunity			
Mesh Nebulizer is intended for use in the electromagnetic environment specified below. The customer or the user of Mesh Nebulizer should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment-guidance
Electrostatic discharge(ESD) GT/B 17626.2	±6KV contact ±8KV air	±6KV contact ±8KV air	Floor should be wood, concrete, or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%

Electrostatic Transient/burst GB/T 17626.4	$\pm 2\text{KV}$ for power supply lines $\pm 1\text{KV}$ for input/output lines	N/A	Mains power quality should be that of a typical commercial or hospital environment
Surge GB/T 17626.5	$\pm 1\text{KV}$ differential mode $\pm 2\text{KV}$ common mode	N/A	Mains power quality should be that of a typical commercial or hospital environment
Voltage dips, short interruption, and voltage variations on power supply input lines GB/T 17626.11	<5%UT, (>95% dip in UT) for 0.5cycle 40%UT, (60% dip in UT) For 5 cycles 70%UT, (30% dip in UT) For 25 cycles <5%UT, (>95% dip in UT) For 5sec	N/A	Mains power quality should be that of a typical commercial or hospital environment. If the user of the Mesh Nebulizer requires continued operation during power mains interruptions, it is recommended that the Mesh Nebulizer be powered from an uninterruptible power supply or a battery
Power frequency (50/60Hz) magnetic field GB/T 17626.8	3A/m	3A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment
NOTE: UT is the a.c. mains voltage prior to application of the test level			

Guidance and manufacturer's declaration -electromagnetic immunity			
The Mesh Nebulizer is intended for use in the electromagnetic environment specified below. The customer or the user of Mesh Nebulizer should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment-guidance
Conducted RF GB/T 17626.6	3V/m 150kHz to 80MHz	N/A	<p>Portable and mobile RF communications equipment should be used no closer to any part of the Mesh Nebulizer, including cables, than the recommended separation distance calculates from the equation applicable to the frequency of the transmitter</p> <p>Recommended Separation Distance</p> $d=1.2\sqrt{P}$ $d=1.2\sqrt{P} \quad 80 \text{ MHz} \sim 800 \text{ MHz}$ $d=2.3\sqrt{P} \quad 800 \text{ MHz} \sim 2.5 \text{ GHz}$ <p>Where P is the maximum output power rating of the transmitter in watts (w) according to the transmitter manufacturer and d is the recommended separation distance in meters (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range.</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol: </p>
Radiated RF GB/T 17626.3	3V/m 80MHz to 2.5GHz	3V/m	
<p>NOTE1 At 80MHz and 800MHz, the higher frequency range applies</p> <p>NOTE2 These guidelines may not apply in all situations. Electromagnetic is affected by absorption and reflection from structures, objects, and people.</p>			
<p>a. Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcasts cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to the Mesh Nebulizer being used exceeds the applicable RF compliance level above, the Mesh Nebulizer should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting the Mesh Nebulizer.</p> <p>b. Over the frequency range 150kHz to 80MHz, field strengths should be less than 3V/m.</p>			

Recommended separation distance between Portable and mobile RF communications equipment and the Mesh Nebulizer			
Rated maximum output of transmitter W	Separation distance according to frequency of transmitter/m		
	150KHz~80MHz $d=1.2\sqrt{P}$	80 MHz~ 800 MHz $d=1.2\sqrt{P}$	800 MHz~2.5 GHz $d=2.3\sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above the recommended separation distance in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (w) according to transmitter manufacturer.

NOTE1 At 80MHz and 800MHz, the separation distance for the higher frequency range applies.

NOTE2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects, and people.

Warranty

1. This product is warranted for manufacturing defects for one year from the date of retail purchase. It does not cover damages or wear-and-tear resulting from an accident, theft, misuse, abuse, commercial use, or an unauthorized adjustment or repair of the product.
2. Please direct all returns to the place of the original receipt, and you may be asked to provide proof of purchase. If you have any questions about this nebulizer, please feel free to contact our customer service via email: service@healthcare-manager.com.

Troubleshooting

If you encounter any issues when using the nebulizer, please cross-check the following table first.

PROBLEM	CAUSE	SOLUTION
The nebulizer is on but nebulizes weakly	The medicine cup is not properly installed.	Make sure the medicine cup is properly installed.
	The medicine cup is not clean.	Clean the medicine cup.
	The liquid medicine level is low.	Tilt the main unit slightly towards you, otherwise add liquid medicine.
	The liquid medicine is not in full contact with the mesh plate.	Tilt the main unit slightly towards you, otherwise add liquid medicine.
The power indicator flashes for 1 second and then turns off	The medicine cup is not properly fitted.	Make sure the medicine cup is properly fitted.
	The medicine cup is not filled with medicine.	Fill the medicine cup with medicine liquid.
	The electrodes in the connection part of the medicine cup and the main unit are not clean.	Clean the electrodes with a soft, clean cloth.
	The liquid medicine level is low.	Tilt the main unit slightly towards you, otherwise add liquid medicine.
	The liquid medicine is not in full contact with the mesh plate.	Tilt the main unit slightly towards you, otherwise add liquid medicine.
The power indicator does not light, and the nebulizer does not nebulize	The battery is extremely low.	Charge and then restart.
	The power AC adaptor (purchased separately) is not properly connected to the nebulizer.	Reconnect correctly and restart.
	Low Battery indicator flashes, battery power shortage or exhaustion.	Charge and then restart.
The nebulizer automatically shuts down in use	The medicine cup is not properly fitted.	Make sure the medicine cup is properly fitted.
	Shake or vibrate the nebulizer during use.	Hold the nebulizer firmly.
	Medicine cup malfunction.	Replace the medicine cup before using the nebulizer again.
Liquid medicine is running out but the nebulizer cannot automatically power off	The usage time does not reach 20 minutes.	Press the power button manually to turn off the nebulizer.
The medicine of medicine cup is leaking	The medicine cup is not closed tightly with lock, broken or damaged.	Close the medicine cup tightly with lock. Or replace the medicine cup before using the nebulizer again.

* If you have followed the above tips trying to fix your device, but the nebulizer still cannot work as normal, please contact the seller.

Technical Specifications

Name	Easy@Home Mesh Nebulizer
Model	YS35
Dimensions	45mm(L)×63mm(W)×123mm(H)
Power Source	Lithium battery nominal voltage 3.7V ±20%, AC adaptor (Input: 100 ~ 240V, 50/60 Hz, Output: 3V ~ 5V DC, 1 ~ 2A) (not included)
Safety Grade	BF
Power Consumption	1.5W+15%
Vibration Frequency	120KHz±10%
Nebulization Rate	≥0.2mL/min
Medication Temperature	≤41°C
Working Voice	≤50dB(A)
Particle Size	5μm±25%
Medication Capacity	8mL maximum, 0.5mL minimum
Continuous use time after full charge	Continuous spraying for more than 1 hour (if used for 20 minutes per day, it can be used for more than 3 days)
Operating Environment	Environmental temperature:10°C ~ 40°C, Relative humidity: ≤80% RH, No Coagulation Air Pressure: (86.0 ~ 106.0) kPa
Storage Environment	Environmental temperature: -20°C ~ 55°C Relative Humidity: ≤80%R.H. Air Pressure: 70.0 ~ 106.0 kPa
Accessory	Main unit, medicine cup, mouthpiece, adult mask, child mask, user manual, USB cable, carry pouch

Explanation of Symbols

	Caution		CE marking
	Power indicator		Manufacturer
	Low battery indicator		Authorized Representative in the European Community
	Type BF applied part		WEEE (Waste Electrical and Electronic Equipment)
IP22	Degree of protection against ingress of water		Date of manufacture
	Recyclable		Serial number
	Read the user manual before use		Batch code

Healthcare-Manager.com

Questions or comments?

Please contact us via email:

service@healthcare-manager.com

Distributed by: **Easy Healthcare Corporation**

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Distribuido por: **Easy Healthcare Corporation**