Ovulation Test

EN -

REF EZW.S

INTENDED USE

Easy@Home ovulation test is a qualitative test that is designed to help predict when a woman will experience a surge in luteinizing hormone (LH). The test detects the surge in LH to identify the most fertile days of a woman's menstrual cycle.

For in vitro diagnostic use.

For self-testing use.

HOW DOES IT WORK?

The LH (luteinizing hormone) in the urine of most women will increase dramatically in the middle of the menstrual cycle. The LH increase triggers ovulation, which is when a woman's ovaries releases an egg. WHO experts state that LH testing is a reliable way to detect ovulation. The LH surge occurs about 24 to 48 hours before you actually ovulate. A positive LH test result may indicate that you are entering your most fertile period of your cycle.

The ovulation test contains antibodies which specifically react with LH. When the strip is immersed in a urine specimen capillary action carries the specimen to migrate along the membrane. When the luteinizing hormone (LH) in the urine reaches the test zone, a colored line will appear in the test zone. The control zone also works as a reference line for you to find the LH peak time when the test line is as dark or darker than the control line.

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