Ovulation Test

INTENDED USE
Easy@Home ovulation test is a qualitative test that is designed to help predict when a woman will experience a surge in luteinizing hormone (LH). The test detects the surge in LH to identify the most fertile days of a woman's menstrual cycle.

For *in-vitro* diagnostic use.

For self-testing only.

HOW DOES IT WORK?
The LH (luteinizing hormone) in the urine of most women will increase dramatically in the middle of the menstrual cycle. The LH increase triggers ovulation, which is when a woman's ovaries release an egg. WHO experts state that LH testing is a reliable way to detect ovulation. The LH surge occurs about 24 to 48 hours before you actually ovulate. A positive LH test result may indicate that you are entering your most fertile period of your cycle.

The ovulation test contains antibodies which specifically react with LH. When the strip is immersed in a urine specimen capillary action carries the specimen to migrate along the membrane. When the luteinizing hormone (LH) in the urine reaches the test zone, a colored line will appear in the test zone.

If the test has been performed properly, a colored line will appear at the control zone. The control line also works as a reference line for you to find the LH peak time when the test line is as dark or darker than the control line.

CONTENTS OF THE TEST KIT

- One pouch containing a test strip and a desiccant. The desiccant is for storage purpose only and is not used in the test.
- Instructions for use.

WHAT ELSE DO YOU NEED?

- A clean, dry, plastic or glass container to collect the urine.
- Timer (watch, clock or smartphone)

PRECAUTIONS

- This kit is for external use only. Do not swallow.
- Do not use the test kit beyond expiration date.
- Do not use the kit if the test pouch is punctured or not fully sealed.
- Keep out of the reach of children.
- Discard any first day of ovulation test strip cannot be used more than once.
- Please follow local regulations to discard used tests.

STORAGE AND STABILITY

- Store at 4°C to 30°C (39°F to 86°F) in the sealed pouch up to the expiration date.
- The test should be used immediately after opening.
- Keep away from sunlight, moisture and heat.
- DO NOT FREEZE

COLLECTION AND STORAGE OF SPECIMENS

WHEN TO COLLECT URINE FOR THE TEST?
Any urine specimen is appropriate for ovulation testing.

HOW TO COLLECT THE URINE SAMPLE?
This test is designed to have urine collected in a clean container just prior to testing. Urine may be collected in any clean, dry, plastic or glass container.

WHEN TO START TESTING?

The length of the menstrual cycle is the duration for your first menstrual bleeding day to the day before the next bleeding begins. Determine the length of your menstrual cycle by counting. Count the first day of bleeding as cycle day 1. Place the test strip below to determine when you should start testing. If you do not know your cycle length, start testing from cycle day 5 until the LH surge has been detected.

Example: Your most fertile days may vary from cycle to cycle.

▼ Cycle Day
5 6 7 8 9 10 11 12 13 14 15 16
Start Testing

Day of Cycle

21 days	day 5
22 days	day 6
23 days	day 7
24 days	day 8
25 days	day 9
26 days	day 10
27 days	day 11
28 days	day 12
29 days	day 13
30 days	day 14
31 days	day 15
32 days	day 16
33 days	day 17
34 days	day 18
35 days	day 19
36 days	day 20
37 days	day 21
38 days	day 22
39 days	day 23
40 days	day 24

Example: If your cycle is normally 26 days, according to the Cycle Chart, testing should begin on day 10 of your cycle. To determine day 10, count 10 days from the first day of your menstrual bleeding (cycle day 1). For example, if your first day of menstrual bleeding was January 3, then day 10 would be January 12. You would start testing for ovulation on January 12 and continue until the LH surge has been detected.

Sample calendar

1	2	3	4	5	6	7
8	9	10	11	12	13	14
15	16	17	18	19	20	21
22	23	24	25	26	27	28

3+ = The first day of menstrual bleeding
12+ = The day to begin testing (day 10)

HOW TO READ THE RESULTS?

- Remove the test strip from the sealed pouch.
- Dip the white absorbent end into the urine sample up to the black 'Max' line until you see the dye begins to rise into the testing window (Est. time: 5-10 seconds).
- Lay the test strip flat on a clean, dry, and non-absorbent surface.
- Read the results at 5 minutes after immersion or after any excess dye has completely cleared the testing window. Do not read after 5 minutes.

The test strip and urine sample should be at room temperature (59-86°F or 15-30°C) before performing the test to ensure accurate results.

HOW TO READ THE RESULTS?

Positive (LH Surge)
If two color lines are visible, and the test line is as dark or darker than the control line, you will probably ovulate in the next 24-48 hours. If trying to get pregnant, the best time to have intercourse is after 24 but before 48 hours. Repeat with a new test strip. If test still fails, please contact us for assistance.

Negative (No LH Surge)
Only one color line appears in the control zone, or the test line appears but is lighter than the control line. This means there is no LH surge.

Invalid
No visible lines at all, or there is a visible line only in the test zone but not in the control zone.

Repeat with a new test strip. If test still fails, please contact us for assistance.

LIMITATIONS

- The tests results are more accurate when the test procedures are precisely followed.
- This test may not be used as a form of birth control.
- Consult a doctor if you experience irregular or unusually long cycles.
- This test is a qualitative screening assay. It is not designed to determine the quantitative concentration of LH.
- The device is only suitable for human urine test.
- Unknown specimen or interfering substances in the urine may cause false result.
- Some drugs for infertility treatment, such as 17 β -Estradiol, Progesterone and Estradiol Valerate, may cause false result.
- Unclean test device may cause false result.

PERFORMANCE CHARACTERISTICS

SENSITIVITY AND SPECIFICITY
The Easy@Home Ovulation Test will detect hLH in human urine at the level higher than 25 mIU/ml. The test has been calibrated against the W.H.O. Standard (2nd WHO International Standard (IS) 80/552). No cross-reactance when there is hFSH (200 mIU/ml) or hTSH (250 mIU/L) in specimen.

INTERFERING SUBSTANCE
The following potentially interfering substances were added to LH negative and positive specimens.

Acetaminophen	20mg/dl	Glucosic acid	20mg/dl
Acetylsalicylic acid	20mg/dl	Glucose	20mg/dl
Ascorbic acid	20mg/dl	Hemoglobin	20mg/dl
Atropine	20mg/dl	Tetracycline	20mg/dl
Caffeine	20mg/dl	Ampicillin	20mg/dl

None of the substances at the concentration tested interfered in the assay.

QUESTIONS AND ANSWERS

- Is it a concern if there is a big difference between the control lines of two different testing periods? Is this a concern?**
No. Variations in the color of the control line will not affect the result. It is only effective to compare the color of the test line to that of the control line of the same test on the day the test is performed.
- How long should I continue to perform the test?**
Continue testing for at least 5 days or until the LH surge has been detected.
- A pink background color and vertical streaking appeared in the result area during the testing period. Is this a concern?**
No. Each urine sample will vary in its chemical makeup, as will the humidity of the air in the testing chamber. Such variations in physical conditions can cause the vertical streaking and/or the pink-rose background color, but will not affect the results. As long as the control line appears within 5 minutes, the test is working properly.
- Does alcohol or common medications affect the test?**
No, but you should consult your physician if you are taking any medication.
- Once I see a positive result, when is the best time to have intercourse?**
Ovulation is likely to occur within 24-48 hours. This is your most fertile time. Sexual intercourse should be performed within this time frame.
- I am now using the basal body temperature method (BBT). Does this test replace BBT?**
The elevated basal body temperature primarily indicates that ovulation has already occurred. This test indicates that ovulation is about to occur within 24 to 48 hours.
- I have received a positive result and had intercourse during these fertile days. I have not become pregnant. What shall I do?**
Many factors can contribute to your inability to become pregnant. Often you may need to use the test kit for 3-4 months. You and your partner should consult your physician if pregnancy is not achieved after 3-4 months.
- Will the amount of liquid I drink affect the result?**
Yes. We suggest that you limit your fluid intake for about two hours before you collect your urine. Heavy intake of fluids prior to testing will dilute the hormone in your urine.
- Can I use my first morning urine for testing?**
Unlike pregnancy testing, the FMU (first morning urine) is NOT recommended for use for ovulation testing. Luteinizing hormone (LH) levels usually start to rise in the early morning and may not be easily detectable in the first morning urine. The ideal time to collect a urine sample is from 10 a.m. to 8 p.m., although testing can safely be performed at any time.

NOTE: It is suggested to test twice a day when it is close to the LH surge day so that you won't miss it; the LH surge can be very short for some women. If you choose to test twice a day, you may test once around noon and another time in the early evening. We recommend collecting the urine specimen at about the same time each day.

BIBLIOGRAPHY

1. Siervest LL, Dubois CA. "Validation signs of ovulation: Do women who think they know, really know?" Department of Anthropology, Machmer Hall, UMass Amherst, Amherst, Massachusetts. Am J Hum Biol. 2005 Apr; 22:1713-30-320
2. Ruiz Anguas J, Carballo Mondragon E, Anta Jaen E, Diaz Calvillo J, Kably Ambe A. "Usefulness of luteinizing hormone levels to determine the oocyte quality." Ginecol Obstet Mex. 2005 Jan;73:28-35

MEANING OF SYMBOLS ON PACKAGE

Keep away from sunlight, Use by date, Lot#, Authorized representative in the European Community, Temperature limit, Confirmed sufficient for cap-test, Recycle, Do not reuse, Keep dry, In-vitro diagnostic medical device, CE marking of conformity, Catalogue number, Manufacturer.

QUESTIONS OR COMMENTS?
Please contact us via e-mail: support@premom.com
Easy Healthcare Corporation
360 Shore Dr., Burr Ridge, IL, USA 60527
Made in China

Scan the code to get the manual in more languages

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