Pregnancy Test

- EN -

REF EZW1-S-10/EZW1-S-20/EZW1-S-25/EZW1-S-40/
EZW1-S-50/EZW1-S-60/EZW1-S-80/EZW1-S-100

INTENDED USE

Easy@Home pregnancy test is an immunochromatographic assay designed for the rapid determination of human chorionic gonadotropin (hCG) in urine. The test is used to obtain a visual qualitative result. *In vitro* self-testing use only.

HOW DOES IT WORK?

Human chorionic gonadotropin (hCG) is a hormone produced by the developing placenta shortly after conception and secreted into the urine. The pregnancy test contains antibodies that specifically react with this hormone.

When the strip is immersed in a urine specimen, capillary action carries the specimen to migrate along the membrane. When hCG in the sample reaches the Test Zone of the membrane, it will form a colored line. The absence of this colored line suggests a negative result. To serve as a procedure control, a colored line will appear at the Control Zone if the test has been performed properly.

CONTENT OF THE TEST KIT

- One pouch containing a test strip and a desiccant. The desiccant is for storage purposes only and is not used in the test.
- Instructions for use.

WHAT ELSE DO YOU NEED?

- A clean, dry, plastic or glass container to collect the urine.
- Timer (watch, clock or smartphone)

PRECAUTIONS

- This kit is for external use only. Do not swallow.
- Do not use the test kit beyond expiration date.
- Do not use if the test pouch is punctured or not fully sealed.
- Keep out of the reach of children.
- Discard after first use. The test strip cannot be used more than once.
- Urine specimens may be infectious; Ensure proper handling and disposal of all used reaction devices as per the local statute.

STORAGE AND STABILITY

- Store at 2°C to 30°C (35.6°F to 86°F) in the sealed pouch up to the expiration date.
- The test should be used immediately after opening.
- Keep away from sunlight, moisture and heat.
- DO NOT FREEZE.**

COLLECTION AND STORAGE OF SPECIMENS

WHEN TO COLLECT URINE FOR THE TEST?


Any urine specimen is appropriate for pregnancy testing. However, the first morning urine specimen is preferred because of its highest concentration of hCG.

HOW TO COLLECT THE URINE SAMPLE?

This test is designed to have urine collected in a clean container just prior to testing. Urine may be collected in any clean, dry, plastic or glass container.

HOW TO DO THE TEST?

- Remove the test strip from the sealed pouch.
- Dip the white absorbent tip of the test into the urine up to the black "MAX" line and keep the tip in urine until dye rises into the results window. (Est. time: 5-10 seconds)
- Lay the test strip flat on a clean, dry, and non-absorbent surface.
- Read the results in 5 minutes after immersion. Do not read the result after more than 5 minutes.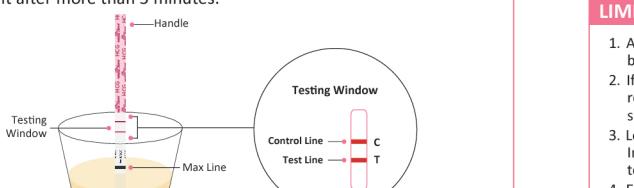

The test strip and urine have to be at room temperature 15-30°C (59-86°F) for testing.

HOW TO READ THE RESULTS?

Positive(Pregnant)

Two distinct color lines appear in the Control and Test Zones. One line should be in the control zone (C) and another line should be in the test zone (T).

Negative(Not pregnant)

Only one color line appears in the control zone (C). No apparent Rosy Pink line in the test zone(T).

Invalid

No visible lines at all, or there is a visible line only in the test zone (T), but not in the control zone (C). Repeat with a new test strip. If test still fails, please contact us for assistance.

Note: The color intensity of the test lines may vary since different stages of pregnancy have different concentrations of hCG hormone. If the color of the test lines is faint, it is recommended to repeat the test in 48 hours.

LIMITATIONS

As it is with any diagnostic procedure, a confirmed pregnancy diagnosis should only be made by a physician after evaluating all clinical and laboratory findings.

If a urine sample is very dilute (ie, low concentration) it may not contain a representative level of hCG to be detected, another urine specimen should be collected 48 hours later and tested.

Low concentration of hCG in a very early pregnancy can give a negative result. In this case, another specimen should be obtained at least 48 hours later and tested.

Elevated levels of hCG can be caused by a few conditions other than pregnancy. Therefore, the presence of hCG in urine specimen should not be used to diagnose pregnancy unless these conditions have been ruled out.

A normal pregnancy can not be distinguished from an ectopic pregnancy based on hCG levels alone. Spontaneous miscarriage may cause confusion in interpreting test results.

Sterility treatments, based on hCG, may cause false results. Consult your physician.

PERFORMANCE CHARACTERISTICS

Sensitivity:

The Pregnancy Test detects hCG in the urine at a concentration of 10 mIU/mL or greater.

Specificity:

The test has been standardized to the W.H.O. International Standard. The addition of LH (300 mIU/mL), FSH (1,000 mIU/mL) and TSH (1,000 µU/mL) to negative (0 mIU/mL hCG) and positive (10 mIU/mL hCG) specimens showed no cross-reactivity.

QUESTIONS AND ANSWERS

1 How soon after I suspect that I am pregnant can I take the test?

You can test your urine as early as the first day you miss your period. You can perform the test anytime of the day; however, if you are pregnant, first morning urine contains the most pregnancy hormone.

2 Do I have to test with first morning urine?

Although you can test any time of the day, your first morning urine specimen is usually the most concentrated of the day and would have the most hCG in it.

3 How accurate is the test?

A clinical evaluation was conducted comparing the results obtained using the pregnancy test to another commercially available urine measurement HCG test. The consumer clinical trial included 216 urine specimens: both assays identified 76 positive and 140 negative results. The results demonstrated >99% overall accuracy of the pregnancy test when compared to the other urine membrane hCG test.

4 Which factors may affect the test result?

Drugs which contain hCG (such as Pregnyl, Profasi, Pergonal, APL) can give a false positive result. Alcohol, oral contraceptives,

painkillers, antibiotics or hormone therapies that do not contain hCG should not affect the test result.

5 What should I do if the result shows that I am pregnant?

It means that your urine contains hCG and you are probably pregnant. See your doctor to confirm that you are pregnant and to discuss the steps you should take.

6 Does the result mean anything other than normal pregnancy if it shows that I am pregnant?

A number of medical conditions other than pregnancy, including, ovarian cyst or ectopic pregnancy (pregnancy outside the uterus) can cause elevated levels through normal pregnancy. J. Obstet. Gynecol. 1976; 126(6): 678-680.

7 What should I do if the result shows that I am not pregnant?

It means that no hCG has been detected in your urine and probably you are not pregnant. If you do not start your period within a week of its due date, repeat the test with a new test. If you receive the same result after repeating the test and you still do not get your period, you should see your doctor.

8 How do I know that the test was run properly?

The appearance of a red line in the control zone (C) tells you that you followed the test procedure properly and the proper amount of urine was absorbed.

Please take the following steps to increase your chances for a healthy pregnancy and your baby.

1. Use the Pregnancy test to detect pregnancy when your period is late. You can begin better prenatal care as soon as you learn of your pregnancy.

2. If you get a positive result, it is advisable to visit your doctor immediately.

3. Maintain a well-balanced diet, stop smoking, and reduce your intake of alcohol.

BIBLIOGRAPHY


- Batzer FR. Hormonal evaluation of early pregnancy. Fertil. Steril. 1980; 34(1): 1-13
- Catt KJ, ML Dufau, JL Vautrait. Appearance of hCG in pregnancy plasma following the time of implantation of the blastocyst. J. Clin. Endocrinol. Metab. 1975; 80(2): 537-540
- Braunstein GD, J. Raso, H. Danzer, D. Adler, ME Wade. Serum human chorionic gonadotropin levels throughout normal pregnancy. Ann. J. Obstet. Gynecol. 1976; 126(6): 678-680.
- Lenton EA, LM Neal. R. Sulaiman Plasma concentration of chorionogonadotropin after implantation. Obstet. Gynecol. 1977; 50(2): 172-181
- Braunstein GD, JL Vautrait, PP Carbonne, CT Ross. Ectopic production of human chorionogonadotropin by neoplasias. Ann. Intern. Med. 1973; 78(1): 39-45

MEANING OF SYMBOLS ON PACKAGE

Keep away from sunlight	Temperature limit	IVD	CE marking of conformity
Use-by date	Contains sufficient for one test	REF	Catalogue number
Lot	Batch code	Manufacturer	
Recyclable	Do not re-use	Keep dry	
Authorized representative in the European Community	Consult instructions for use or consult electronic instructions for use		

QUESTIONS OR COMMENTS?

Please contact us via email: support@premom.com
Easy Healthcare Corporation
360 Shore Dr., Burr Ridge, IL USA 60527
Made in China

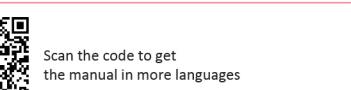Scan the code to get the manual in more languages

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