
Smart Basal Thermometer

BT-A31

User Manual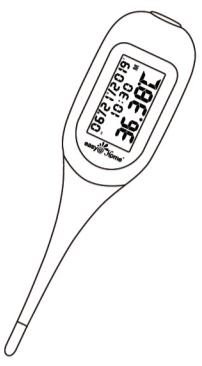

easy@Home®

- EN -

Electronic Clinical Thermometer

CONTENTS

1 INTENDED USE	01
2 CONTENTS	01
3 WHY BT-A31?	01
4 SET UP AND USE	01
5 DATA STORAGE	02
6 CHANGE THE BATTERY	03
7 TROUBLESHOOTING	03
8 TECHNICAL SPECIFICATIONS	04
9 MAINTENANCE & CLEANING	04
10 WARRANTY	05
11 WORKING & STORAGE CONDITIONS	05
12 PRECAUTIONS	06
13 EMC/FCC INFORMATION	06
14 WARNINGS	12
15 DISPOSAL OF THE DEVICE	12
16 EXPLANATION OF SYMBOLS	13

1. INTENDED USE

The digital thermometer model No. BT-A31 is intended for the measurement and monitoring of human body temperature by doctor or consumers in the hospital or at home.

2. CONTENTS

- 1 Thermometer
- 1 User Manual

3. WHY BT-A31?

- Upon setting up, data auto sync to free app Premom.
- Large and backlit LCD display helps you read results easily.
- 1/100th High-precision with memory recall of last 30 readings.

4. SET UP AND USE

- ① Refer the quick guide of Premom App to set up the App before BBT measurement.

② Take your temperature


Step1: Keep the smart thermometer near your bed. Make sure to take your basal body temperature each morning at the same time for the best results.

6. CHANGE THE BATTERY

Please replace the battery when the low-battery symbol appears on the right of the LCD display. Remove the battery cover and replace it with a battery of the same battery type CR2032H. Make sure the “+” sign faces up. Always dispose of used batteries according to local laws and regulations.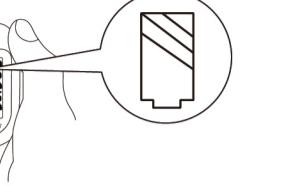

8. TECHNICAL SPECIFICATIONS

Product name	Electronic Clinical Thermometer
Model	BT-A31
Size	13.2cm x 2.92cm x 1.39cm
Transmission distance	9m
Display range	-32.00°C ~ 42.99°C
Accuracy	±0.1°C (from 35.0°C to 39.0°C) ±0.2°C (from 32.0°C to 35.0°C, and from 39.0°C to 42.99°C)
Battery	CR2032H
Receiver	Apple Phone: iPhone5+ iOS8.0+ Android Phone: Android 4.4+
Battery life	About 3 months
Operating conditions	Temperature: 5°C to 40°C
Weight	About 20g (without battery)

10. WARRANTY

1. This product is warranted from manufacturing defects for one year from the date of retail purchase. It does not cover damages or wear resulting from an accident, misuse, commercial use, or an unauthorized adjustment or repair of the product.

2. Please direct all returns to the place of original receipt, and you may be asked to provide proof of purchase. To find the customer service menu, please visit our official website at www.premom.com.

11. WORKING & STORAGE CONDITIONS

Working environment temperature: 5°C ~ 40°C
Working humidity: 15% ~ 85%
Working atmosphere pressure: 86kPa ~ 106kPa
Storage environment temperature: -20°C~55°C
Storage humidity: ≤85%
Storage atmosphere pressure: 86kPa ~ 106kPa

12. PRECAUTIONS

1. Do not bite, bend, drop or take apart the thermometer.
2. Keep the device out of the reach of children/pets to avoid inhalation or swallowing of small parts.
3. Do not expose it to direct sunlight, high temperature and moisture.
4. Not intended to be sterilized. Prevent saliva or cleaning solution from penetrating the display window.
5. Clean the thermometer after or before use with soft cloth and cleaning solution listed below and store in the plastic case. Recommend to wipe with wet water soaked cloth for home use.
6. If LCD shows the symbol or LCD shows unclear, it means the battery has run out. Please replace the battery within 10 minutes.

13. EMC/FCC INFORMATION

1. This product needs special precautions regarding EMC and needs to be installed and put into service according to the EMC information provided, and this unit can be affected by portable and mobile RF communications equipment.
2. Do not use a mobile phone or other devices that emit electromagnetic fields, near the unit. This may result in incorrect operation of the unit.
3. Caution: This unit has been thoroughly tested and inspected to assure proper performance and operation!
4. Caution: This machine should not be used adjacent to or stacked with other equipment and that if adjacent or stacked use is necessary, this machine should be observed to verify normal operation in the configuration in which it will be used.

Check BT-A31 User Manual
in more languages

Guidance and manufacturer's declaration – electromagnetic emission

The BT-A31 is intended for use in the electromagnetic environment specified below.

The customer or the user of the BT-A31 should assure that it is used in such an environment.

EMISSION TEST	COMPLIANCE	ELECTROMAGNETIC ENVIRONMENT - GUIDANCE
RF emission CISPR 11	Group 1	The BT-A31 uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.

RF emission CISPR 11	Class B	The BT-A31 is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
----------------------	---------	---

Guidance and manufacturer's declaration – electromagnetic immunity

The BT-A31 is intended for use in the electromagnetic environment specified below.

The customer or the user of the BT-A31 should assure that it is used in such an environment.

IMMUNITY TEST	IEC 60061 TEST LEVEL	COMPLIANCE LEVEL	ELECTROMAGNETIC ENVIRONMENT - GUIDANCE
Conducted RF	3 Vrms IEC 61000-4-6	Not Applicable	Portable and mobile RF communications equipment should be used no closer to any part of the BT-A31, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.
Power frequency (50Hz/60Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

NOTE: UT is the a.c. mains voltage prior to application of the test level.

Guidance and manufacturer's declaration – electromagnetic immunity

The BT-A31 is intended for use in the electromagnetic environment specified below.

The customer or the user of the BT-A31 should assure that it is used in such an environment.

IMMUNITY TEST	IEC 60061 TEST LEVEL	COMPLIANCE LEVEL	ELECTROMAGNETIC ENVIRONMENT - GUIDANCE
Radiated RF	3 Vrms IEC 61000-4-3	3 Vrms	Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer, and d is the recommended separation distance in metres (m).

NOTE 1: At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

a: Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, CB radios, shortwave broadcast, and other licensed and non-licensed radio services, as well as from non-radio sources such as power lines, medical devices, street lighting, radar, television, and other communication equipment.

b: Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

Recommended separation distances between portable and mobile RF communications equipment and the BT-A31

The BT-A31 is intended for use in an electromagnetic environment in which RF disturbances are controlled. The customer or the user of the BT-A31 can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the BT-A31 as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter (W)	Separation distance according to frequency of transmitter (m)		
	150 kHz to 80 MHz $d = 1.2\sqrt{P}$	80 MHz to 800 MHz $d = 1.2\sqrt{P}$	800 MHz to 2.5 GHz $d = 2.3\sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1: At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

FCC ID: 2ADNBQTA41CNBT. This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

(1) This device may not cause harmful interference; (2) This device must accept any interference received, including interference that may cause undesired operation.

14. WARNINGS

1. No servicing/maintenance while the thermometer is in use.

2. Not for use in OXYGEN RICH ENVIRONMENT.

3. Before every use, check the device. Do not use the device or an electrode if it is damaged in any way. The continuous use of a damaged unit may cause injury, improper results, or serious danger.

4. Do not open or repair the device by yourself.

5. The housing materials of the thermometer are ABS and TPE. Be careful of potential allergic reactions to these materials.

6. The typical service life of the new and unused batteries is 100 hours for continuous operation.

7. Protection against electric shock: Internally powered ME equipment.

8. Protection against harmful ingress of water or particulate matter: IP22. However, for the sake of accuracy and safety, please refrain from fully immersing the device in water or any other liquid.

9. Keep new and used batteries out of the reach of children, to seek immediate medical attention if a battery is ingested, and to follow any other consensus medical advice.

10. Choking Hazard-Small parts not for children under 3 years or any individuals who have a tendency to place inedible object in their mouths.

15. DISPOSAL OF THE DEVICE

Once the product life has ended or his components do not work anymore their disposal should be carried out according to the current regulations.

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