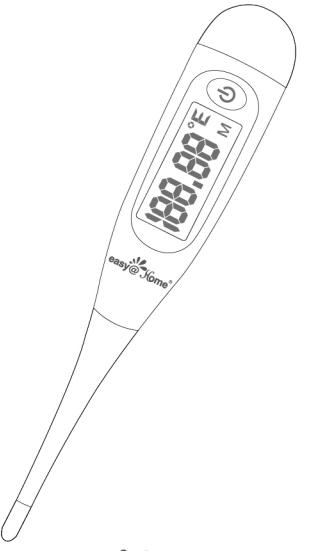
Digital Basal Thermometer

BT-A23

User Manual

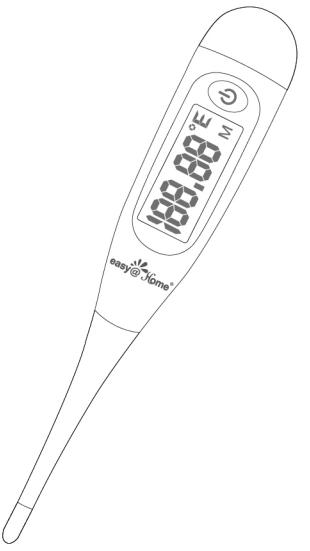
easy@Home®

Powered by premOM APP
Electronic Clinical Thermometer
- EN -

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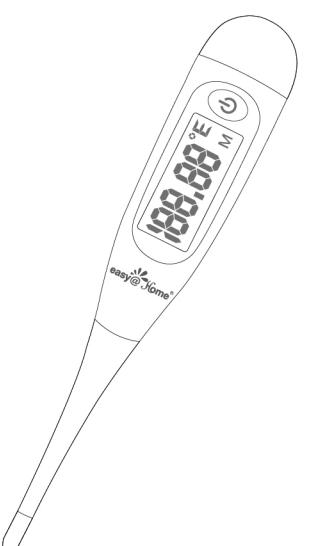
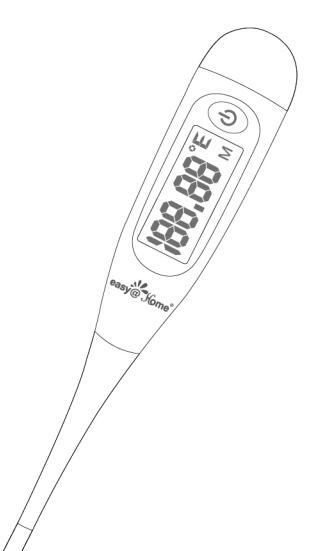
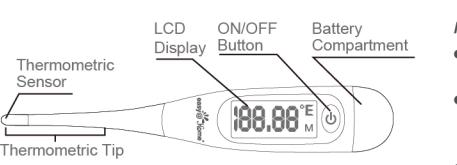
The Digital Basal Thermometer (BT-A23) is used for the measurement of basal body temperature (BBT).

What is basal body temperature, and why do you need a digital basal thermometer?

Basal body temperature is your temperature when you are fully at rest. It is taken immediately upon awakening before any activity. Unlike traditional thermometers, basal body thermometers can measure with increased accuracy to easily detect your fertile window. The BT-A23 Digital Basal Thermometer is sensitive enough to measure subtle changes in body temperature with high precision of 1/100th and takes about 90 seconds to finish a reading when used properly.

Note: It is important to record your temperature every day to identify your BBT trend.

How does your BBT help you pinpoint ovulation? What do you need to know?

Your BBT temps will likely surge right after ovulation, with a 0.2°C – 0.6°C or more shift. The free Premom app can automatically chart your reading inputs and draw your coverline, a visual tool to help you identify your BBT shift. And with Premom you will have unlimited measurements in the cloud.

Note: Users MUST manually enter temperature in the Premom app with this non-Bluetooth enabled model.

How Does The Premom App Help?

1. Tap the button "Log BBT" on the main screen.
2. Choose "Manual" to log your BBT.
3. Tap the degree $^{\circ}$ area to enter your current numerical BBT and tap "Save".

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Note:

- To change the time, tap the current time and scroll to the previous time and press save.
- If you would like to log the temp from a previous day, simply tap the arrow down on the top of the screen, then choose the date and tap "Done".
- If you would like to edit or delete a logged temp, just tap charts on the bottom of the main home screen. Tap "BBT" on the top of that next screen. After you access the table, swipe left and it will show you "Edit" or "Del" to proceed for a given temp.

NOTE: As we are constantly improving and updating the Premom app to offer the best user experience for our customers, the interface may be different from the illustrations included but all features mentioned should be still available and similar.

What's Included:

- Thermometer
- Manual
- Protective case

Operating Instructions

1. Open battery chamber cover, remove the insulation plastic sheet before use.

2. Disinfect the probe before using with 70% ethyl alcohol on a damp cloth.

3. Press the ON/OFF button to switch the device on.

A short beep will sound indicating that the thermometer is operational. Initially, 188.88 (the calibrator number) appears for 2 seconds, followed by your last temperature reading for two seconds; finally, dotted lines will appear. When a flashing "°F" or "°C" is displayed, the thermometer is ready for use.

4. Place the thermometer tip under the tongue and close your mouth.

The thermometer tip should remain in constant contact with the tissue under the tongue, and the mouth should remain closed during measuring. Avoid eating or drinking just before measuring the temperature, as this will result in an inaccurate temperature reading. The average measuring time is about 90 seconds.

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