Pregnancy Test

User Manual

REF PM1-M3-3/ PM1-M3-5

English

For Self-testing

INTENDED USE

Premom pregnancy test (midstream) is an immunochromatographic assay designed for the rapid determination of human chorionic gonadotropin (hCG) in urine. The test is used to obtain a visual qualitative result. *For in vitro* self-testing use only.

PRINCIPLE

Human chorionic gonadotropin (hCG) is a hormone, produced by the developing placenta shortly after the conception and secreted into the urine. The pregnancy test contains antibodies which specifically react with this hormone. When the test is inserted to a urine specimen, capillary action carries the specimen to migrate along the membrane. When hCG in the sample reaches the Test Zone region of the membrane, it will form a colored line. Absence of this colored line suggests a negative result. To serve as a procedure control, a colored line will appear at the control zone region if the test has been performed properly.

MATERIALS SUPPLIED

- PM1-M3-3/ PM1-M3-5 3/5 Test Midstream (individually wrapped). One pouch contains a test midstream and a desiccant. The desiccant is for storage purposes only and is not used in the test procedures.
- One instruction for use.

MATERIAL REQUIRED BUT NOT PROVIDED

- Clock or Timer
- Specimen collection containers.

PRECAUTIONS

- This kit is for external use only. Do not swallow.
- Discard after first use. The pregnancy test cannot be used more than once.
- Do not use the test kit beyond expiry date.
- Do not use the kit if the pouch is punctured or is not well sealed.
- Keep out of the reach of children.
- Urine specimens may be infectious; Ensure proper handling and disposal of all used reaction devices as per the local statute.

STORAGE AND STABILITY

- Store at 2°C - 30°C in the sealed pouch up to the expiration date.
- DO NOT FREEZE.
- The test should be used immediately after opening.

SPECIMEN COLLECTION

Any urine specimen is appropriate for pregnancy testing but the first morning urine specimen is optimal because of its highest concentration of hCG.

TEST PROCEDURE

- Remove the midstream test from the foil pouch.
- Remove the Cap and hold the midstream test by the Handle with the exposed Absorbent Tip pointing downward directly into your urine stream for at least 10 seconds until it is thoroughly wet. See the illustration below.

Note: Do not urinate on the Result Window. If you prefer, you can urinate into a clean and dry container, then dip only the Absorbent Tip of the midstream test into the urine for at least 10 seconds.
- After removing the midstream test from your urine, immediately replace the Cap over the Absorbent Tip, lay the midstream test on a flat surface with the result window facing upwards, and then begin timing.
- As the test begins to work, you may notice a light red flow moving across the Result Window. Read the result in 5 minutes. Do not read the result after 5 minutes.

STORAGE AND STABILITY

- Store at 2°C - 30°C in the sealed pouch up to the expiration date.
- DO NOT FREEZE.
- The test should be used immediately after opening.

SPECIMEN COLLECTION

Any urine specimen is appropriate for pregnancy testing but the first morning urine specimen is optimal because of its highest concentration of hCG.

TEST PROCEDURE

- Remove the midstream test from the foil pouch.
- Remove the Cap and hold the midstream test by the Handle with the exposed Absorbent Tip pointing downward directly into your urine stream for at least 10 seconds until it is thoroughly wet. See the illustration below.

Note: Do not urinate on the Result Window. If you prefer, you can urinate into a clean and dry container, then dip only the Absorbent Tip of the midstream test into the urine for at least 10 seconds.
- After removing the midstream test from your urine, immediately replace the Cap over the Absorbent Tip, lay the midstream test on a flat surface with the result window facing upwards, and then begin timing.
- As the test begins to work, you may notice a light red flow moving across the Result Window. Read the result in 5 minutes. Do not read the result after 5 minutes.

PERFORMANCE CHARACTERISTICS

A layer users evaluation was conducted comparing the results obtained using the HCG Pregnancy Rapid Test (Urine) to professional users. The study included 60 urine specimens, and layer users and professional users identified 30 negative and 30 positive results. The results demonstrated an identical agreement between layer users and professional users.

Layer users vs professional users

Tests (tester)	Total Evaluated Samples	Correct Positive Results	Correct Negative Results
hCG Test by Layer Users	60	30	30
Positive Agreement	/	30/30=100%	/
Negative Agreement	/	/	30/30=100%
Total Agreement	(30+30)/60=100%	/	/
hCG Test by Professional users	60	30	30
Positive Agreement	/	30/30=100%	/
Negative Agreement	/	/	30/30=100%
Total Agreement	(30+30)/60=100%	/	/

Sensitivity:
The HCG Pregnancy Rapid Test detects hCG in the urine at a concentration of 10 mIU/ml or greater.

Specificity:
The test has been standardized to the W.H.O. International Standard. The addition of LH (300 mIU/ml), FSH (1,000 mIU/ml) and TSH(1,000µIU/ml) to negative (0 mIU/ml hCG) and positive (10 mIU/ml hCG) specimens showed no cross-reactivity.

INTERPRETATION OF RESULTS

Positive

The presence of two red lines in both the test window (T) and control window (C) means you are probably pregnant. One line may be lighter than the other. They do not have to match.

Negative

The presence of one red line in the control window (C) means you are probably not pregnant.

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Invalid

The result is invalid if no red line appears in the control window (C), even if a line appears in the test window (T). You should repeat the test with a new midstream test.

LIMITATIONS

- As it is with any diagnostic procedure, a confirmed pregnancy diagnosis should only be made by a physician after evaluating all clinical and laboratory findings.
- If a urine sample is too dilute (ie, low specific gravity) it may not contain a representative level of hCG. If pregnancy is still suspected, another urine specimen should be collected 48 hours later and tested.
- Low concentration of hCG in a very early pregnancy can give a negative result. In this case, another specimen should be obtained at least 48 hours later and tested.
- Elevated levels of hCG can be caused by a few conditions other than pregnancy. Therefore, the presence of hCG in urine specimen should not be used to diagnose pregnancy unless these conditions have been ruled out.
- A normal pregnancy can not be distinguished from an ectopic pregnancy based on hCG levels alone. Also, spontaneous miscarriage may cause confusion in interpreting test results.
- Sterility treatments, based on hCG, may cause false results. Consult your physician.

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PREPAREDATION AND STABILITY
<ol style="list-style-type: none">Store at 2°C - 30°C in the sealed bag until the expiration date. DO NOT FREEZE. The test should be used immediately after opening.
COLLECTING THE SAMPLE
<p>Any urine sample is appropriate for the pregnancy test, but the first morning urine is optimal because of its higher concentration of hCG.</p>
TESTING PROCEDURE

POSITIVE RESULT
<p>The presence of two red lines in the test window (T) and the control window (C) indicates that you are probably pregnant. One line may be clearer than the other, but both must be present.</p>
NEGATIVE RESULT
<p>The presence of one red line in the control window (C) indicates that you are probably not pregnant.</p>

PERFORMANCE CHARACTERISTICS
<p>An user evaluation was conducted under layer-users to compare the results obtained with the hCG Pregnancy Rapid Test (Urine) to professional users. The study included 60 specimens of urine, and layer-users and professional users identified 30 negative and 30 positive results. The results showed an identical agreement between layer-users and professional users.</p> <p>Layer-user vs professional users</p>

Sensitivity: The hCG Pregnancy Rapid Test detects hCG in urine with a concentration of 10 mIU/mL or higher.

Specificity: The test is standardized according to the WHO. International Standard. The addition of LH (300 mIU/mL), FSH (1000 mIU/mL) and TSH(1000 µIU/mL) to negative (0 mIU/ml hCG) and positive (10 mIU/ml hCG) samples shows no cross-reactivity.

Questions and answers

1. Q: How soon after I suspect I'm pregnant can I take the test?
A: You can test your urine as early as the first day that you miss your period. You can test at any time of the day; however, if you are pregnant, the first morning urine contains the most pregnancy hormones.

2. Q: Do I need to use first morning urine for the test?
A: While you can test at any time of the day, your first morning urine sample is normally the most sensitive for detecting pregnancy.

3. Q: How accurate is the test?
A: A clinical evaluation was conducted to compare the results obtained with the pregnancy test compared to another commercially available membrane hCG test. The clinical trial involved 216 urine samples: both professional users identified 76 positive and 140 negative results. The results showed a general accuracy of >99% of the pregnancy test in comparison with the other membrane hCG test.

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