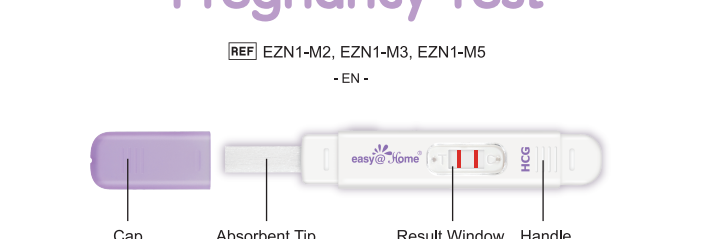
Pregnancy Test

REF EZN1-M2, EZN1-M3, EZN1-M5
-EN-

Cap Absorbent Tip Result Window Handle

INTENDED USE

Easy@Home Pregnancy Test is a rapid pregnancy test, which you can easily carry out yourself. It detects the presence of human chorionic gonadotropin (hCG), which appears in urine very early during pregnancy.

Although the hCG levels in normal early pregnant women are variable, easy@Home Pregnancy Test is capable of detecting pregnancy as early as 1 day after the first missed menses.

For *in vitro* diagnostic use only. For Self-testing.

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When the test is inserted to a urine specimen, capillary action carries the specimen to migrate along the membrane. When hCG in the sample reaches the Test Zone region of the membrane, it will form a colored line. The absence of this colored line suggests a negative result.

To serve as a procedure control, a colored line will appear at the control zone region if the test has been performed properly.

MATERIALS SUPPLIED

- One pouch containing a midstream and a desiccant. The desiccant is for storage purpose only and is not used in the test procedures.
- One user manual.

MATERIAL REQUIRED BUT NOT PROVIDED

- Clock or Timer
- Specimen collection containers.

PRECAUTIONS

- This kit is for external use only. Do not swallow.
- Discard after first use. The test strip cannot be used more than once.
- Store between 2°C and 30°C (35.6°F - 86°F)
- Keep out of the reach of children.
- Do not re-use.
- Do not use test kit beyond expiry date.
- Do not use the kit if the pouch is punctured or not well sealed.

STORAGE AND STABILITY

- Store at 2°C to 30°C (35.6°F - 86°F) in the sealed pouch up to the expiration date.
- Keep away from direct sunlight, moisture and heat.
- DO NOT FREEZE.
- The test should be used immediately after opening.

SPECIMEN COLLECTION

- Any urine specimen is appropriate for pregnancy testing but the first morning urine specimen is optimal because of its highest concentration of hCG.
- Urine may be collected in any clean, dry, plastic or glass container.

TEST PROCEDURE

Step 1. To begin testing, open the sealed pouch by tearing along the notch. Remove the test device from the pouch.

Step 2. Remove the cap of the test device, put it to the back of the test device and expose the absorbent tip.

PRECAUTIONS

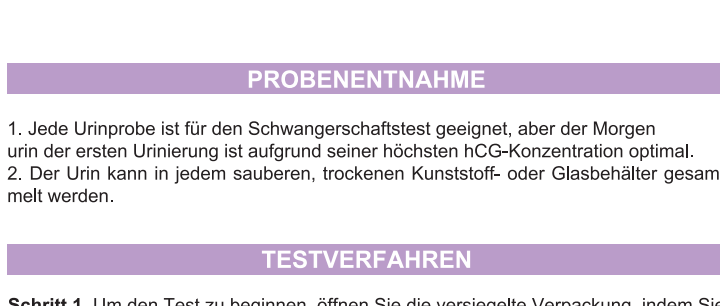
- This kit is for external use only. Do not swallow.
- Discard after first use. The test strip cannot be used more than once.
- Store between 2°C and 30°C (35.6°F - 86°F)
- Keep out of the reach of children.
- Do not re-use.
- Do not use test kit beyond expiry date.
- Do not use the kit if the pouch is punctured or not well sealed.

STORAGE AND STABILITY

- Store at 2°C to 30°C (35.6°F - 86°F) in the sealed pouch up to the expiration date.
- Keep away from direct sunlight, moisture and heat.
- DO NOT FREEZE.
- The test should be used immediately after opening.

PRINCIPLE

Human chorionic gonadotropin (hCG) is a hormone, produced by the developing placenta shortly after the conception and secreted into the urine. The pregnancy test contains antibodies which specifically react with this hormone.

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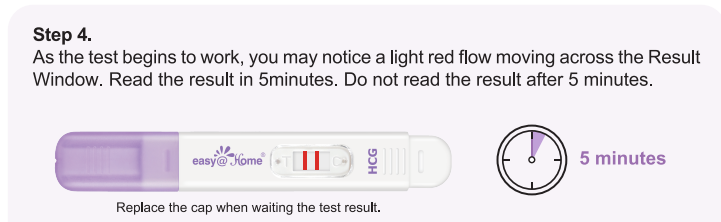
Step 4. As the test begins to work, you may notice a light red flow moving across the Result Window. Read the result in 5minutes. Do not read the result after 5 minutes.


Replace the cap when waiting the test result.

INTERPRETATION OF RESULTS

Positive (Pregnant) : Distinct color bands appear in the Control and Test Zones. It indicates that you are pregnant. The color intensity of the test bands may vary since different stages of pregnancy have different concentrations of hCG hormone.

Negative (Not Pregnant) : No one color band appears in the Control Zone. No apparent band in the Test Zone. It indicates that no pregnancy has been detected.

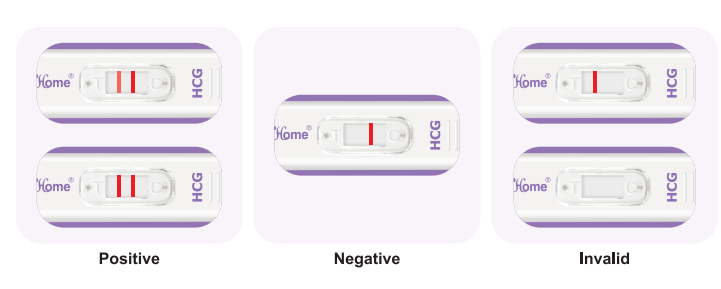Invalid: No visible band at all, or there is a visible band only in the Test Zone and not in the Control Zone. Repeat with a new midstream. If test still fails, please contact the distributor or the store, where you bought the product, with the lot number.

Positive Negative Invalid

NOTE: If the color in the Test Zone is weak, it is recommended that the test be repeated in 48 hours.

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LIMITATIONS

- As it is with any diagnostic procedure, a confirmed pregnancy diagnosis should only be made by a physician after evaluating all clinical and laboratory findings.
- If a urine sample is too dilute (ie, low specific gravity) it may not contain a representative level of hCG. If pregnancy is still suspected, another urine specimen should be collected 48 hours later and tested.
- Low concentration of hCG in a very early pregnancy can give a negative result. In this case, another specimen should be obtained at least 48 hours later and tested.
- Elevated levels of hCG can be caused by a few conditions other than pregnancy. Therefore, the presence of hCG in urine specimen should not be used to diagnose pregnancy unless these conditions have been ruled out.
- A normal pregnancy can not be distinguished from an ectopic pregnancy based on hCG levels alone. Also, spontaneous miscarriage may cause confusion in interpreting test results.
- Sterility treatments, based on hCG, may cause false results. Consult your physician.

PERFORMANCE CHARACTERISTICS

A layer users evaluation was conducted comparing the results obtained using the Pregnancy Test to professional users. The study included 60 urine specimens, and layer users and professional users identified 30 negative and 30 positive results. The results demonstrated an identical agreement between layer users and professional users.


Layer users vs professional users

Tests (tester)	Total Evaluated Samples	Correct Positive Results	Correct Negative Results
hCG Test by Layer users	60	30	30
Positive Agreement	/	30/30=100%	/
Negative Agreement	/	/	30/30=100%
Total Agreement	(30+30)/60=100%	/	/
hCG Test by Professional users	60	30	30
Positive Agreement	/	30/30=100%	/
Negative Agreement	/	/	30/30=100%
Total Agreement	(30+30)/60=100%	/	/

Sensitivity: The Pregnancy Test detects hCG in the urine at a concentration of 10 mIU/ml or greater.

Specificity: The test has been standardized to the W.H.O. International Standard. The addition of LH (300 mIU/ml),FSH (1,000 mIU/ml) and TSH(1,000µIU/ml) to negative (0 mIU/ml hCG) and positive (10 mIU/ml hCG) specimens showed no cross-reactivity.

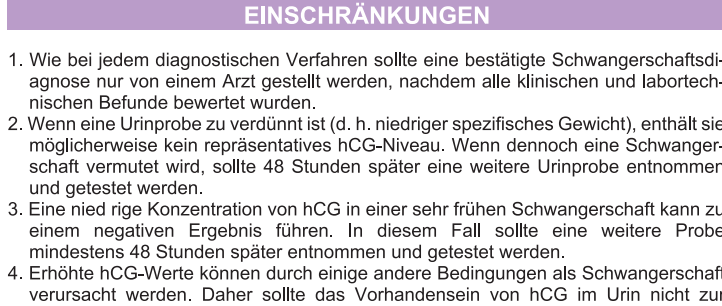Do not allow urine to contact with parts of the test midstream other than the absorbent tip.

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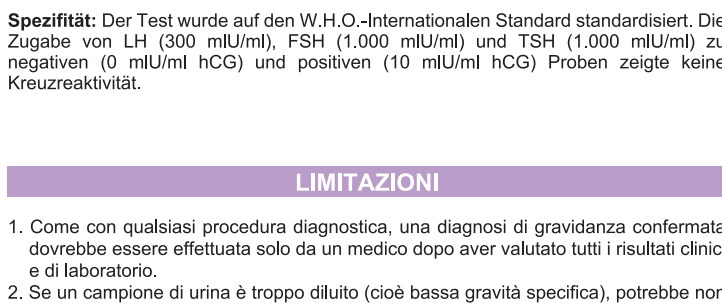
QUESTIONS AND ANSWERS

Q: There is much difference between the control lines of two tests. Is this a concern?
A: No. Variations in the color of the control band will not affect the test result.

Q: Can test results be interpreted after more than five minutes?
A: No. Test results must be read in 5 minutes. Though a positive result should not change for several days, a negative result may change to a false positive within minutes after the end of the testing period, which would not be an accurate reading. It is always best to read the results at the 5 minute testing period and then discard the test to avoid confusion.

Q: I have waited longer than 5 minutes, and still there are no lines in the result window. What does this mean?
A: There should always be at least one pink line in the result window at 5 minutes. If not, the test is invalid. The invalid results are normally caused by 3 major reasons: not placing absorbent tip in urine for enough 10 seconds, urine is insufficient or the urine permeates through the inner part of the test device. You should retest with another test and make sure place the absorbent tip in urine for 10 seconds meantime make sure urine is sufficient and avoid urinating or dripping beyond the scope of absorbent pad. If you still see the invalid result, contact the vendor.

Q: What factors may affect the test result?
A: Drugs containing hCG can give a false positive result. Alcohol, oral contraceptives, painkillers, antibiotics, or hormone therapies that do not contain hCG should not affect the test result.

Q: What should I do if the result shows that I am pregnant?
A: Consult your physician. They should confirm your pregnancy after evaluating all clinical and laboratory findings. Indeed, in some rare conditions, the test may give a false positive, result. They will also give you further information and advise on your pregnancy.

Q: What should I do if the result shows that I am not pregnant?
A: If you have reasons to believe that you are pregnant (e.g. no menstrual bleeding), you can repeat the test a few days later. If the test is still negative, consult your physician.

BIBLIOGRAPHY

- Batzler FR. Hormonal evaluation of early pregnancy, Fertil. Steril. 1980; 34(1): 1-13
- Catt KJ, ML Dufau, JL Vaitukaitis. Appearance of hCG in pregnancy plasma following the initiation of implantation of the blastocyste. J. Clin. Endocrinol. Metab.1975; 40(3): 537-540
- Braunstein GD, J Rasor, H. Danzer, D Adler, ME Wade Serum human chorionic gonadotropin levels throughout normal pregnancy. Am. J. Obstet. Gynecol. 1976; 126(6): 678-681
- Lenton EA, LM Neal, R Sulaiman Plasma concentration of human chorionic gonado-

tropin from the time of implantation until the second week of pregnancy, Fertil. Steril. 1982; 37(6): 773-778

- Steier JA, P Bergsjö, OL Myking Human chorionic gonadotropin in maternal plasma after induced abortion, spontaneous abortion and removed ectopic pregnancy. Obstet. Gynecol. 1984; 64(3): 391-394
- Dawood MY, BB Saxena, R Landesman Human chorionic gonadotropin and its subunits in hydraliform mole and choriocarcinoma. Obstet. Gynecol. 1977; 50(2): 172-181
- Braunstein GD, JL Vaitukaitis, PP Carbone, GT Ross Ectopic production of human chorionic gonadotropin by neoplasms, Ann. Intern Med. 1973; 78(1): 39-45

INDEX OF SYMBOLS			
	Keep away from sunlight		For in vitro diagnostic use only
	Temperature limit of 2°C–30°C (35.6°F - 86°F)		Contains sufficient for n- tests
	Manufacturer		Consult instructions for use
	Do not reuse		Authorized representative in the European Community
	Recyclable		Use-by date
	Batch code		CE marking of conformity
	Catalog number		

Questions or comments?
Please contact us via email: support@premcom.com
Distributed by **Easy Healthcare Corporation**
360 Shore Dr., Burr Ridge, IL USA 60527
To learn more, please visit us at www.premcom.com

Made in China

Scan the code to get the manual in more languages

3. Braunstein GD, J Rasor, H. Danzer, D Adler, ME Wade. Seruminspiegelung des humanen Choriongonadotropins während einer normalen Schwangerschaft, Am. J. Obstet. Gynecol. 1976; 126(6): 678-681.

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