

Reynard Neutral detergent wipes.

Unparalleled equipment compatibility.

An alcohol and fragrance-free solution to effectively cleanse hard surfaces and equipment with no risk of tainting. Widely used in the health sector for cleaning medical equipment, mattress covers, operating theatres and other areas where harsh chemicals need to be avoided.

Points to remember:

- Use one wipe per piece of equipment or surface
- Wipe from clean to dirty
- Clean from top to bottom
- Always reseal the pack after use

Step 1: Turn the lid in a clockwise motion to remove.



Step 2: Cut two crossing slits into the foil seal, approximately 3cm in length each.



Step 3: Take hold of the wipe in the centre of the roll and pull through the cross.



Step 4: Thread the cloth through the dispensing hole in the lid.



Step 5: Reattach the lid to the canister.



Step 6: Pull the required number of wipes through the dispensing hole.



Step 7: Pull sharply in a downward motion to detach the wipe(s) from the roll, ensuring the top of the next cloth is presented



Step 8: Always reseal the lid to keep product moist.



outside the dispensing hole for the next use.

Please note: we recommend following hospital patient cleaning protocols, including the use of P.P.E. where appropriate.

Product code	RHS203
Wipes per canister	160
Canisters per carton	12



Reynard is reassurance.

