



hadley girl

Replaceable filters offer an additional layer of protection, and while the mask doesn't include a filter, they're easy to make yourself using our printable template.

MAKING A FILTER FOR YOUR MASK

- 1. Cut out the template filter corresponding to the size of your mask**
- 2. Trace the shape on a filter material, see below for suggestions**
- 3. Cut filter material**
- 4. Insert your filter through the side pocket, and use fingers to gently open the filter and smooth inside the mask.**

WHAT CAN I USE FOR A FILTER?

The best option to use is any kind of HEPA filter. A few of our suggested options are vacuum cleaner bags, air conditioner filters, air purifier filters. Even tea towels, felt, and coffee filters will work in a pinch. Look for these

materials at hardware stores near you or online for delivery.

***For your safety, please ensure that your vacuum cleaner bag does not contain fiberglass before using as a filter.**

HOW OFTEN CAN I USE THE FILTER?

Just once. Remember to remove and dispose of the filter between uses.

After disposal of the filter, place mask in a washing machine on a high temperature setting