

WAX BLASTER MD.[®]

Maximum Defense

Ear Irrigation & Cleaning Kit

Frequently Asked Questions

How do you use WAX BLASTER MD?

View our instructional video [HERE](#). WAX BLASTER MD is simple to use. Simply fill the bottle with warm water, (using water that is too hot or too cold may result in nausea, vertigo, dizziness, or discomfort) pour in one packet of EAR CLEAN MD, shake well, and hold the rinse basin under your ear and spray the solution until you have completely rinsed out your ear canal.

What is WAX BLASTER MD used for?

WAX BLASTER MD is an ear irrigation device that provides a strong rinse designed to remove and clean away earwax and other debris in the ear canal. For optimal results, use after administering EARWAX MD[®] ear cleaning drops.

Is WAX BLASTER MD safe for use?

Yes, the WAX BLASTER MD is designed to safely and efficiently rinse the ear canal and clean away stubborn wax.

How often can you use WAX BLASTER MD?

The frequency by which you should clean your ears varies from person to person, as it depends on the type of earwax you have, and the amount of buildup. Consult your doctor or audiologist for a recommendation on how often to clean your ears.

Will WAX BLASTER MD remove all my earwax?

It is suggested to use WAX BLASTER MD as a rinsing system after properly using EARWAX MD. EARWAX MD works quickly to dissolve the wax and WAX BLASTER MD rinses it away. You may have to flush the ear with warm water 10 or more times to fully remove the wax from the ear canal.

