
HEY YOU, PUT DOWN THE COTTON SWAB.

Although it's tempting to use a cotton swab, bobby pin, or even car keys to scrape out unwanted wax, these methods, including ear candling, are extremely dangerous. We designed WAX BLASTER MD® Ear Irrigation and Cleaning Kit to clear your ears so you can hear those voices in your head again. Whether you are keeping up with your routine ear cleaning or unclogging an excess in buildup, WAX BLASTER MD is gentle enough to use weekly and strong enough to blast away unwanted wax and debris.

CONTENTS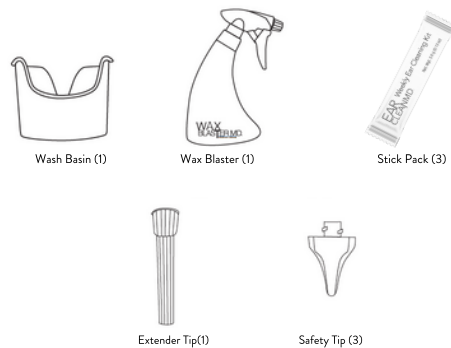

WARNINGS

- Consult your healthcare provider before use.
- Do not use if you know or suspect you have an ear infection, discharge from the ear, a perforation (hole) of the eardrum, or if you have pain in the ear canal.
- Using WAX BLASTER MD should not cause pain in the ear canal. Feeling the pressure of the spray is normal, but if you feel pain you should stop immediately and consult your healthcare provider.
- Only warm water should be used to rinse the ear canal. Using cold or hot water may cause discomfort, often in the form of dizziness, vertigo, or nausea.
- Discard the green safety tip after use.

It is recommended to wash and completely dry the WAX BLASTER MD before storage.

Visit www.EARcareMD.com for more information about ear care and to view the instructional video.

WAX BLASTER MD®

Maximum Defense
Ear Irrigation & Cleaning Kit

INSTRUCTIONS FOR USE

For routine cleaning:

Use WAX BLASTER MD paired with EAR CLEAN™ (included). Fill the WAX BLASTER MD bottle with 10 ounces of warm water and one packet of EAR CLEAN MD. Using water that is too hot or too cold may cause discomfort to the ear canal. Attach the sprayer to the bottle and shake to dissolve. For best results, dance while shaking the bottle. One 10-ounce bottle should be enough for both ears. Unless you're a basset hound, in which case see our pet product. EAR CLEAN MD moisturizes the ear canals while gently cleansing them free of wax, dirt, and debris.

For clogged ears or for a deeper clean:

Use EARWAX MD® earwax removal drops (sold separately) before using WAX BLASTER MD. EARWAX MD will dissolve the wax, making the ear rinsing process a breeze. After using EARWAX MD, fill the WAX BLASTER MD bottle with 10 ounces of warm water and one packet of EAR CLEAN MD (included). Using water that is too hot or too cold may cause discomfort to the ear canal. Attach the sprayer to the bottle and shake to dissolve. For best results, dance while shaking the bottle. One 10-ounce bottle should be enough for both ears. Unless you're a basset hound, in which case see our pet product.

For more information and to sign up for our newsletter, please visit www.EARcareMD.com

FOLLOW @EOSERAINC ON SOCIAL MEDIA

ASSEMBLY INSTRUCTIONS




1. Now that you have attached the sprayer to the bottle, remove the end cap on the sprayer. Attach the enclosed extender. (Figures A and B) Attach one of the green disposable safety tips to the end of the extender. (Figure C) Reusing safety tips is not recommended due to potential contamination. Replacement safety tips are available on our website at www.EARcareMD.com.

2. Once fully assembled, spray the bottle into the sink several times to prime the bottle. (Figure D) It may take 8-10 pumps to fully prime the bottle.

3. Place the included wash basin under the ear to catch the warm water rinse exiting the ear canal. (Figure E)

4. Place the safety tip into the outer ear canal and rapidly pump the warm solution into the ear canal to flush out the wax. (Figure F)

If there is heavy wax buildup, multiple rinses may be required.

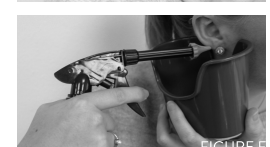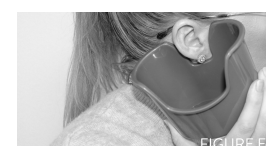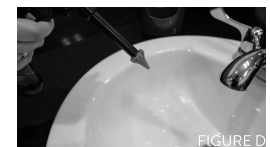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