141mm

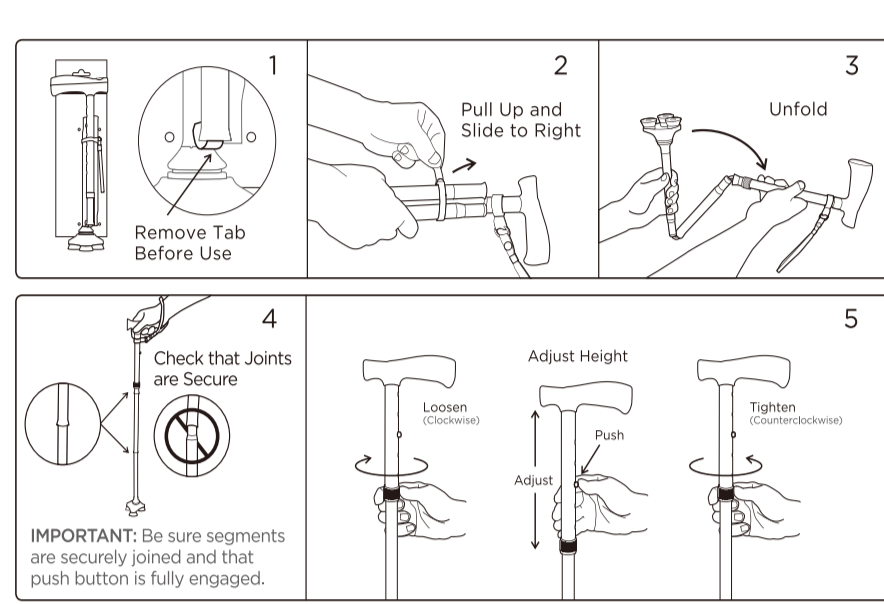
INSTRUCTIONS AND SAFETY PRECAUTIONS

WARNING: To minimize the risk of injury from improper and unsafe use of this cane, read these instructions and safety precautions before using. Also, go to HurryCane.com and view the top and tripod videos.


Intended Use and Limitations: This cane is a walking aid and is not intended to absorb the full weight of the user. Not to be used by persons heavier than 350 lbs (159 kg). Not intended for use on ice or other slippery surfaces as it has the potential to slip.

Safety Precautions: Consult a physician before using any assisted mobility device. Do not use unless individual segments are securely joined together and push button is fully engaged. When assembling individual segments, keep fingers away to avoid being pinched. Do not stretch the internal elastic cord more than necessary to assemble or disassemble the segments. Do not use cane if elastic cord is cut, frayed, or damaged. Applying downward pressure on the base top (stepping on the base) while lifting up on the handle can cause the cane segments to separate and the cane to collapse, possibly causing you to fall. If the handle or bottom base becomes loose, stop using cane immediately and contact HurryCane customer service.

SETUP INSTRUCTIONS Read before setting up and using.

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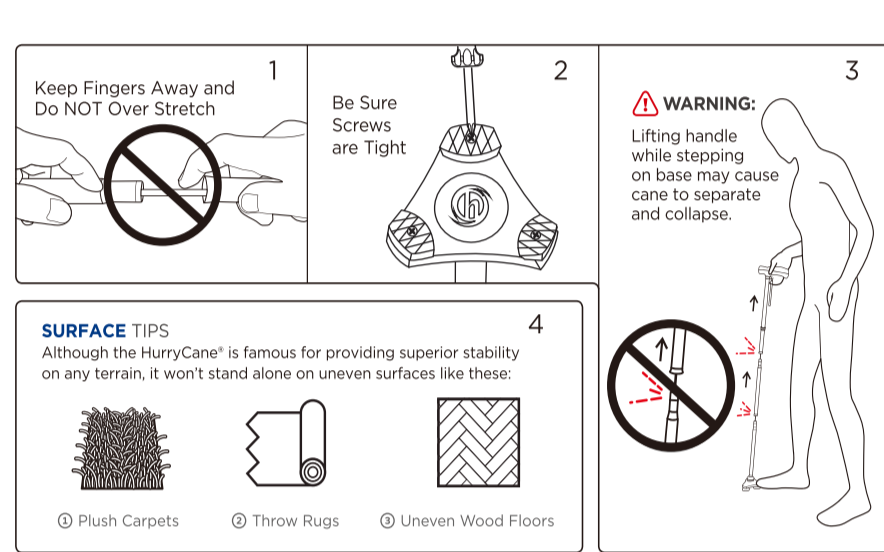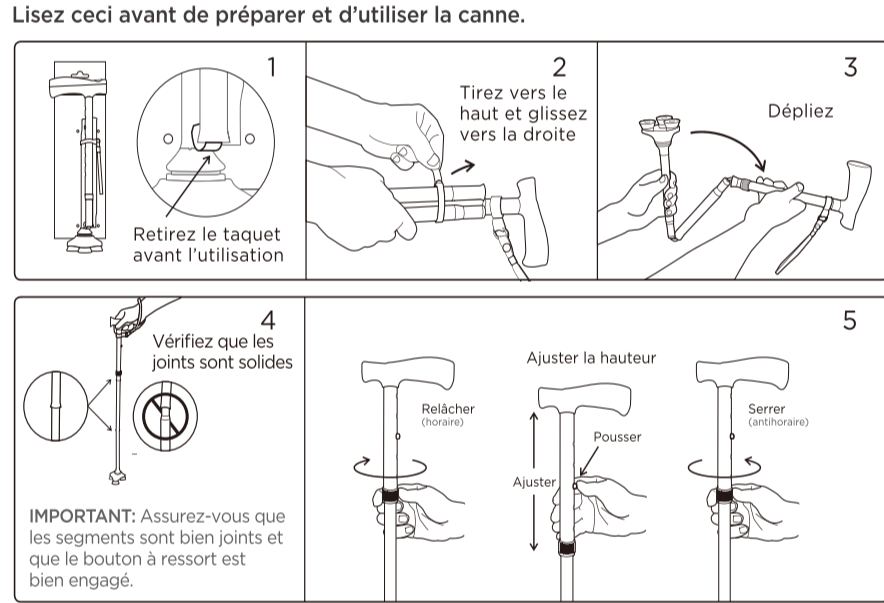
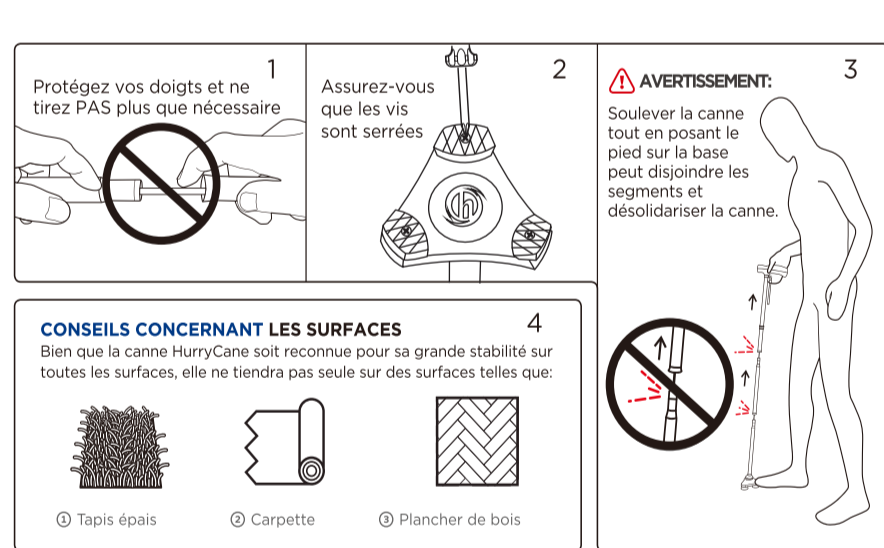
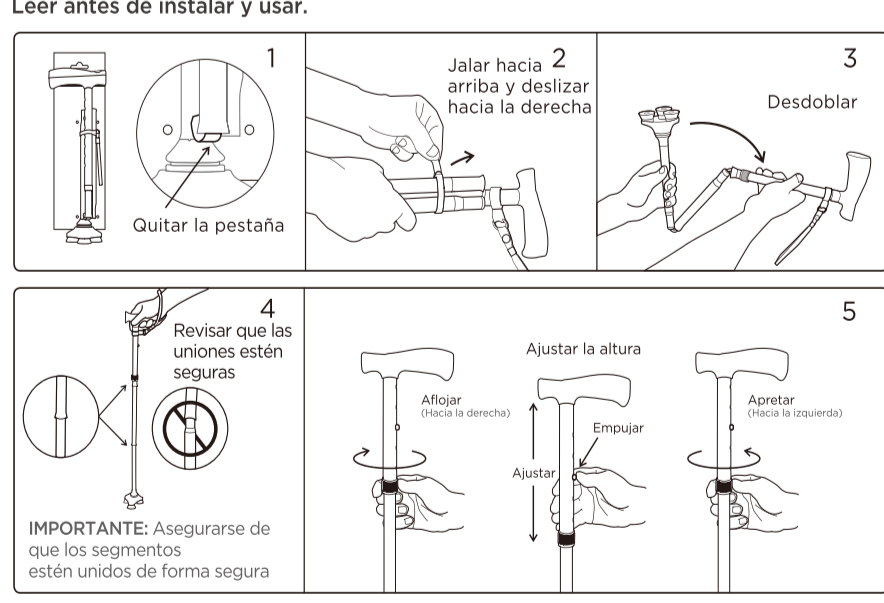
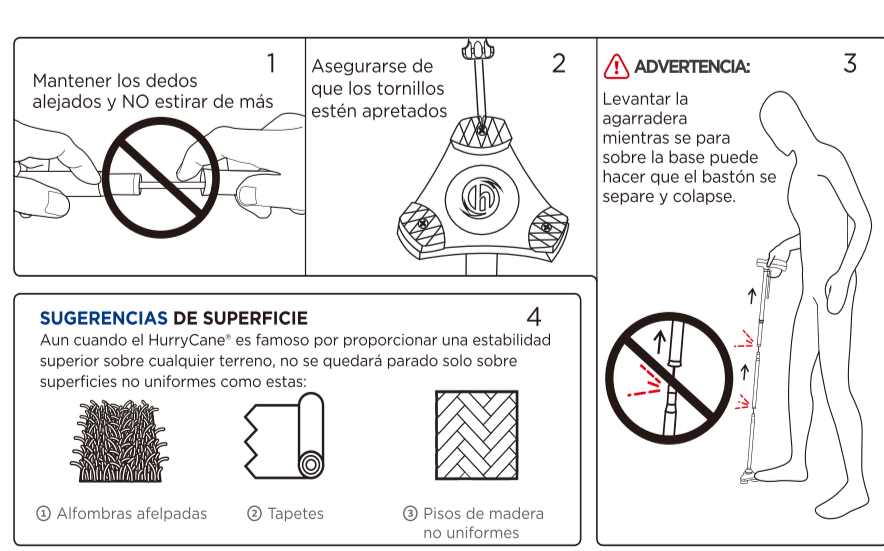
MAINTENANCE

- Check elastic cord before every use. Do not use cane if cord is cut, frayed or damaged. Periodically check the screws on the feet to be sure they are tight. Replace the feet every six months. See HurryCane.com to purchase new feet. Enhance your mobility at HurryCane.com for official HurryCane™ accessories.


PreserveTech™

Drive PreserveTech™ products are manufactured with an antimicrobial agent that helps to minimize the growth of stain and odor-causing bacteria and mold on treated surfaces. Treated surfaces on this product include the handgrip. Antimicrobial properties are built in to protect the products. These products do not protect users or others against bacteria, viruses, germs or other disease organisms.

Table with 2 columns: SPECIFICATIONS and Comments, Feedback, or Questions. Includes details like FDA cleared, Eight adjustable heights, and contact information.


RESET THE FLEX

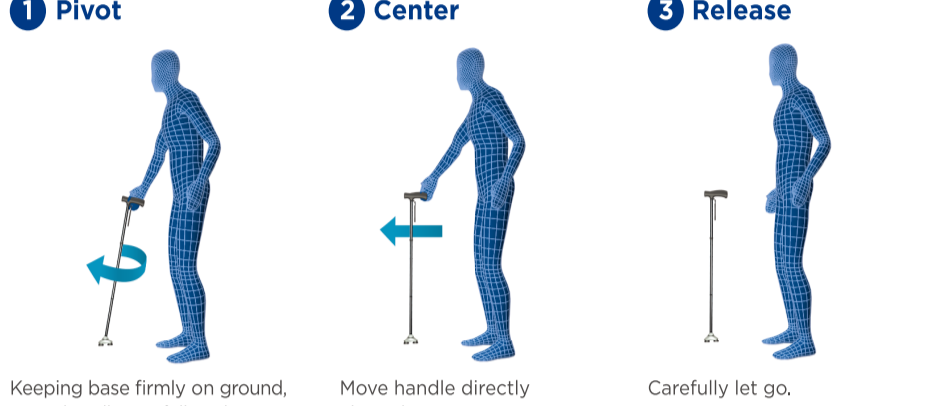The pivoting base of a new HurryCane™ is designed to have less flexibility initially but will adapt to the user's movement over time. As the cane adapts to your movements, your HurryCane™ may need to have its flex reset when you want the cane to stand beside you without holding on to it. Simply reset the flex™ by following these 3 easy steps.

NOTE: If the cane falls over, you can raise it easily by putting your foot on the base and pressing down.

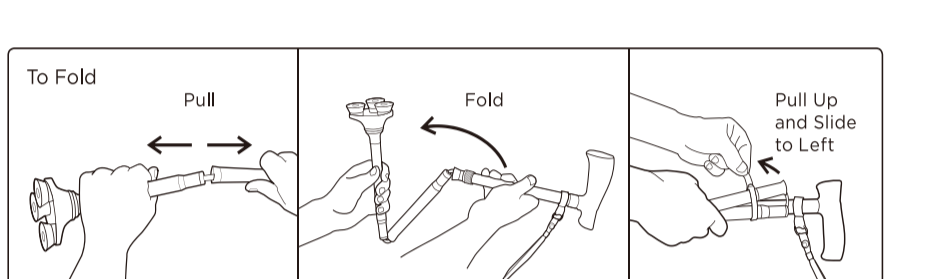STORAGE INSTRUCTIONS

HURRY CANE

© 2022 HurryCane, LLC. 99 Sawney Boulevard | Fort Washington, WI 53033. HurryCane.com | Made in China. USA A.L.S. For more information, visit www.themedical.com/canestaff

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Table with 2 columns: CARACTÉRISTIQUES and Commentaires, rétroaction ou questions. Includes details like FDA approved, Number of heights, and contact information.

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