

# INSTRUCTIONS AND SAFETY PRECAUTIONS

**WARNING:** To minimize the risk of injury from improper and unsafe use of this cane, read these instructions and safety precautions before using. Also, go to [HurryCane.com](https://www.hurrycane.com) and view the tips and tricks videos.

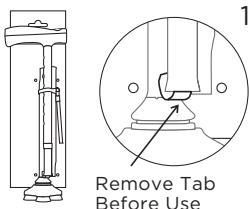
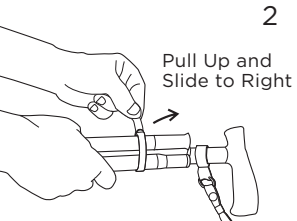
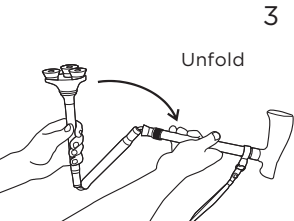
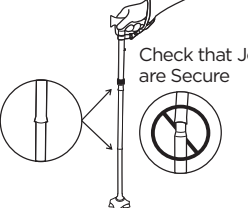
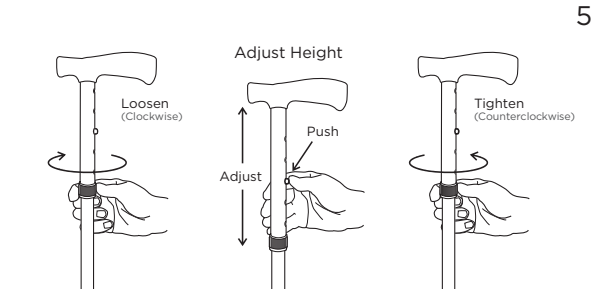
## Intended Use and Limitations:

- This cane is a walking aid and is not intended to absorb the full weight of the user.
- Not to be used by persons heavier than 350 lbs (158 kg).
- Not intended for use on ice or other slippery surfaces as it has the potential to slip.

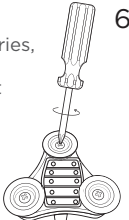
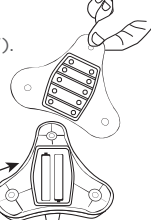
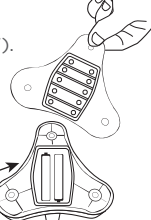

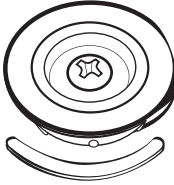
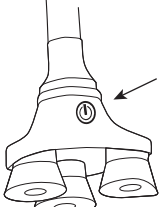
## Safety Precautions:

- Consult a physician before using any assisted mobility device.
- Do not use unless individual segments are securely joined together and push button is fully engaged.
- When assembling individual segments, keep fingers away to avoid being pinched.
- Do not stretch the internal elastic cord more than necessary to assemble or disassemble the segments. Do not use cane if elastic cord is cut, frayed, or damaged.
- Applying downward pressure on the base (e.g. stepping on the base) while lifting up on the handle can cause the cane segments to separate and the cane to collapse, possibly causing you to fall.
- If the handle or bottom base becomes loose, stop using cane immediately and contact HurryCane customer service.

## SETUP INSTRUCTIONS Read before setting up and using.

 <p>1</p> <p>Remove Tab Before Use</p>	 <p>2</p> <p>Pull Up and Slide to Right</p>	 <p>3</p> <p>Unfold</p>
 <p>4</p> <p>Check that Joints are Secure</p> <p><b>IMPORTANT:</b> Be sure segments are securely joined and that push button is fully engaged.</p>	 <p>5</p> <p>Adjust Height</p> <p>Loosen (Clockwise)</p> <p>Adjust</p> <p>Push</p> <p>Tighten (Counterclockwise)</p>	

## SETUP INSTRUCTIONS Two (2) AAA Batteries are Required For Use

 <p>6A) To install or replace the batteries, first unscrew and remove each foot (Figure 6).</p> <p>Figure 6</p>	 <p>6B) Remove the clear battery cover (Figure 7). *Note: The clear battery cover can be difficult to remove by hand. You can use a screw driver to gently prop up the cover (Figure 8).</p>  <p>6C) Insert two (2) AAA batteries.</p> <p>Figure 7</p>	 <p>6D) Place the clear battery cover back on the cane base.</p> <p>Figure 8</p>
 <p>6E) Put the feet back on the base. Orient them so the gap in the foot matches up with the base plate rib (Figure 9). Tighten the screws.</p> <p>Figure 9</p>	 <p>6F) Turn the light on or off by touching the power sensor (Figure 10).</p> <p>Figure 10</p>	

# SAFETY PRECAUTIONS

1

Keep Fingers Away and Do NOT Over Stretch

2

Be Sure Screws are Tight

3

**WARNING:**

Lifting handle while stepping on base may cause cane to separate and collapse.

**SURFACE TIPS**

Although the HurryCane® is famous for providing superior stability on any terrain, it won't stand alone on uneven surfaces like these:

1 Plush Carpets

2 Throw Rugs

3 Uneven Wood Floors

## MAINTENANCE

- Check elastic cord before every use. Do not use cane if cord is cut, frayed, or damaged.
- Periodically check the screws on the feet to be sure they are tight.
- Replace the feet every six months. See HurryCane.com to purchase new feet.
- Enhance your mobility at HurryCane.com for official HurryCane® accessories.
- Replace batteries (AAA) as needed.

## STORAGE INSTRUCTIONS

To Fold

Pull

Fold

Pull Up and Slide to Left

### SPECIFICATIONS

- FDA-listed
- Eight adjustable heights:  
From 30 1/2" to 37 1/2"
- Folded size: 13 3/4"

### Comments, Feedback, or Questions

Toll-Free Number: 1-800-933-1136  
E-mail: [succes@hurrycane.com](mailto:succes@hurrycane.com)  
[HurryCane.com](http://HurryCane.com)  
Like us on Facebook: [facebook.com/AllTerrainCane](https://facebook.com/AllTerrainCane)

## RESET THE FLEX

The pivoting base of a new HurryCane® is designed to have less flexibility initially but will adapt to the user's movement over time. As the cane adapts to your movements, your HurryCane® may need to have its flex reset when you want the cane to stand beside you without holding on to it. Simply 'reset the flex' by following these 3 easy steps:

1 Pivot

Keeping base firmly on ground, move handle in a full circle.

2 Center

Move handle directly above base.

3 Release

Carefully let go.

**NOTE:** If the cane falls over, you can raise it easily by putting your foot on the base and pressing down.