



Endure Bandage Scissors

INSTRUCTION OF USE

Preparation: Ensure the Endure Bandage Scissors are properly sterilized before use, especially if they will be used in a sterile environment or near open wounds.

Grip and Handling: Hold the Endure scissors using the ergonomically designed handles for a firm and comfortable grip. This is crucial for precision and control during use.

Cutting Technique: The scissors feature a blunt tip and angled blades for safety and efficiency. When cutting bandages or gauze, place the blunt tip under the bandage to avoid injuring the skin. The angled blades facilitate a smooth cutting motion.

Application: The Endure Bandage Scissors are ideal for cutting through different types of medical dressings including bandages, tapes, and gauze. They are designed to cut close to the skin without causing harm.

Cleaning and Maintenance: After each use, clean and disinfect the scissors according to standard medical equipment cleaning protocols to maintain hygiene and functionality.

Storage: Store the scissors in a clean, dry place to prevent corrosion or damage.

Inspection: Regularly inspect the scissors for signs of wear or damage, especially on the cutting edges and joint.

Endure Industries, Inc.
45 N Fulton St
Homer, NY 13077
www.endureind.com
sales@endureind.com