



Endure Surgical Skin Staple Removers

INSTRUCTION OF USE

Preparation:

Ensure your hands and the surgical area are properly sterilized. Use gloves to maintain sterility.

Verification:

Confirm that the Endure Staple Remover is in sterile packaging before use.

Technique:

Gently insert the lower jaw of the staple remover under the staple to be removed. Squeeze the handles gently and evenly until the staple is deformed and lifted from the skin.

Removal:

Carefully extract the staple from the skin and dispose of it in a sharps container.

Post-Removal Care:

Inspect the wound for any signs of infection or incomplete staple removal.

Disposal:

Dispose of the used staple remover in an appropriate medical waste container.

Endure Industries, Inc.
45 N Fulton St
Homer, NY 13077
www.endureind.com
sales@endureind.com