

**POWER  
SOCCER  
SHOP.COM**



**PS FORCE  
SOCCER GUARD**




**ASSEMBLY INSTRUCTIONS**

**POWER SOCCER SHOP, INC.  
11841 243 AVE.  
ZIMMERMAN, MN 55398  
763-856-2044 x.204**


# POWER SOCCER SHOP.COM



2. Use clamps and two 2½" bolts to attach each side panel to the wheelchair's leg rest. The smaller holes on the side panel are for the standard clamps while the larger holes are for the stainless steel clamps. Secure with one washer and hex nut for each bolt.

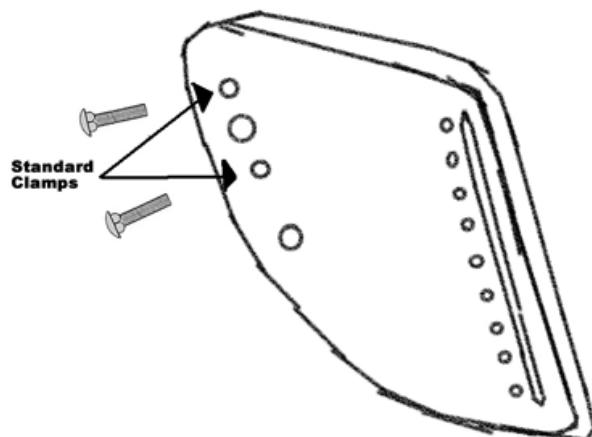

3. Attach 1 side panel to Right Front Panel. Insert one 1½" bolt in the row of holes in the side panel to attach the guard at the correct height (2 -5 inches from the ground). Insert a second 1½" bolt into the long slot on the side panel to adjust the angle of the guard (flat front, approximately 90 degrees from the floor). Secure with one washer and hex nut for each bolt.

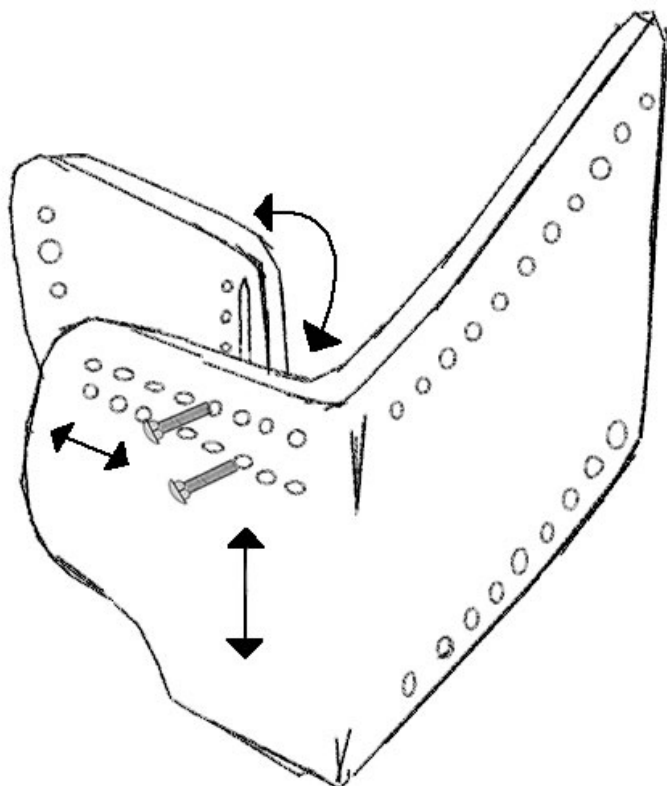


4. Attach second side panel to Left Front Panel.

# POWER SOCCER SHOP.COM



5. Select width of guard. Insert four  $1\frac{1}{2}$ " bolts to attach Left Front Panel to Right Front Panel. Secure with one washer and hex nut for each bolt.

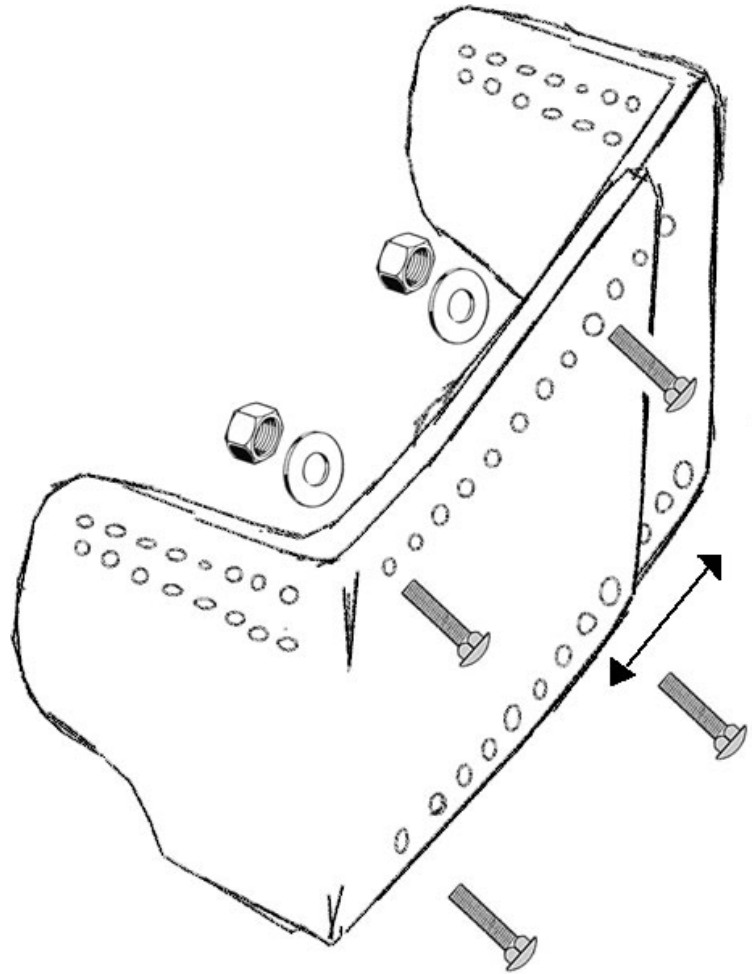

# POWER SOCCER SHOP.COM

