



**FROM WHEELCHAIR TO SLED IN SECONDS**

## WHEELBLADES S

Safety and mobility with a wheelchair – even in winter on snow and ice? It's no problem with Wheelblades S. Thanks to the easy-to-adjust binding, this practical aid can be attached to any conventional wheelchair. With the patented quick-release mechanism, you can switch from wheels to runners with ease in just seconds. The broad contact surface of the Wheelblades optimally distributes the weight on the ground, thus preventing the small front wheels from sinking into the snow. The runners move along the ground with very little friction, and even larger irregularities in the terrain can be easily navigated. To ensure that the tip of the ski always

points forward, the binding was deliberately positioned in the front part of the ski. Stability while in motion is provided by two channels on the bottom of the Wheelblades that compress the snow and let you glide as if on rails.

**Adjustment range:** 2.0 – 6.0 cm

**Suitable for:** wheelchairs

**Available:** in pairs



## TECHNICAL SPECIFICATIONS

### Sophisticated technology

Patented binding with universal adjustment capability for wheel diameters of 8.0 – 20 cm and wheel widths of 2.0 – 6.0 cm.

### Durability

Stainless steel provides protection against corrosion due to moisture and salt.

### Robustness

Glass fiber-reinforced plastic components ensure maximum strength even at low temperatures.

### Secure base

Channels and grooves on the bottom of the runner provide stability and hold the line while in motion.



1. Place the Wheelblades S on the floor.



2. Tilt the wheelchair slightly.



3. Place the front wheels in the binding.



4. Close the quick-release astener and off you go.

## ABOUT WHEELBLADES



Unlike many products, the Wheelblades product range was developed primarily out of the sheer necessity to overcome barriers and not just to make a profit. After a sporting accident in 2000, inventor Patrick Mayer has had to live with incomplete paraplegia and relies on crutches and wheelchairs. He knows all about the limitations and problems of everyday life firsthand. As a passionate winter sports enthusiast, landscape photographer and family man, he wants maximum mobility and flexibility on any surface, in any weather and at any time of year. To achieve these goals, Patrick founded Wheelblades GmbH – a young and innovative company dedicated to significantly improving the mobility of people with a walking disability.

Together with expert partners from Switzerland and Germany, Wheelblades GmbH develops and produces products for people with disabilities, enabling them to experience mobility, freedom and joy even off the beaten track.

### Contact

Wheelblades GmbH  
Untere Industrie 10  
CH-7304 Maienfeld

**MOBILE** +41 78 898 74 11

**MAIL** [info@wheelblades.ch](mailto:info@wheelblades.ch)

**WEB** [www.wheelblades.com](http://www.wheelblades.com)

