

# Relaxa-Back



The Relaxa-Back will give support when getting onto the belt and lowering into the bath.



When the Relaxa-Back is moved to the back of the bath, it can be used as a head rest.



**Australian Distributor**  
Gong Life Care Solutions  
P.O Box 7, Fairy Meadow, NSW - 2519  
[www.gonglcs.com.au](http://www.gonglcs.com.au)  
[info@gonglcs.com.au](mailto:info@gonglcs.com.au)

**FOR FURTHER INFORMATION PLEASE CONTACT**

**CR**  
**MANUFACTURING**

**C.R. Manufacturing Ltd**

Frederick Street, Waddesdon, Aylesbury, Bucks HP18 0LU  
Tel: (01296) 655001 Fax: (01296) 658441  
[crmanufacturing@btconnect.com](mailto:crmanufacturing@btconnect.com) [www.crmanufacturing.co.uk](http://www.crmanufacturing.co.uk)

The company policy is one of continuous improvement and reserves the right to change the design and specification without notice.

**RELAX WITH A** **Relaxa**<sup>TM</sup>  
**BATHLIFT**

- Easy to fit
- Battery powered NOT mains Electricity
- Audible alarm when battery is low
- Remote control as standard
- Thermometer for water temperature
- Does not obstruct the bath
- Provides independence for those that find the bath difficult to enter or leave

  
Quality  
Made in the UK

# HOW DOES IT WORK

Simply push in the re-chargeable battery.



1



5

Sit on the belt and by touching a button, be gently lowered into the bath.



2

Feed out the belt and slip on belt sleeve.

Relax & enjoy the luxury of a good soak.



6



3

Locate the belt on the floor mounting bar and tension.



4



7

Touch a button and be gently raised out of the bath.



Audible alarm + Graphics when battery is low.



8

In addition there is a remote hand control and a temperature probe to register the water temperature and avoid scalding.