



# COMFORT CHAIR

## User Guide

Code: ANI-DAYCOMCA1



## DAYS Comfort Chair

The ultimate in comfort and mobility, the Comfort Chair is an easy care solution for elderly or special needs seating. Cushioned, adjustable air cell support has been incorporated into the design. The four lateral support wings fold away for easy transferring of the patient. This chair has a reclinable seat along with an adjustable back rest and calf rest, and extending foot rests.

### Features

- Adjustable backrest (90° - 170°)
- Adjustable calf rest
- Extending foot rests
- Reclinable seat
- Air system for pressure relief
- Full steel frame
- Three lockable castors and one steering castor (non-marking)
- Breathable polyurethane cover (fire retardant)
- Replaceable & easy to clean cover
- Four folding wings
- Standard dining tray
- Magazine pocket
- 10 years warranty on frame


### Specifications

- |                  |             |                   |                   |
|------------------|-------------|-------------------|-------------------|
| • Seat Height    | 63cm        | • Table Size      | 40 x 40.5 x 2cm   |
| • Seat Width     | 51cm        | • Back Reclining  | 102° - 136°       |
| • Seat Depth     | 50cm        | • Seat Tilting    | 0° - 18°          |
| • Overall Width  | 73cm        | • Leg Elevating   | 70° down to 5° up |
| • Overall Length | 135 - 188cm | • Weight of Chair | 43.5kg            |
| • Table to Floor | 88cm        | • Max User Weight | 120kg             |
| • Table Slide    | 23cm        |                   |                   |

### Seat Adjustment




### Leg Rest Adjustment

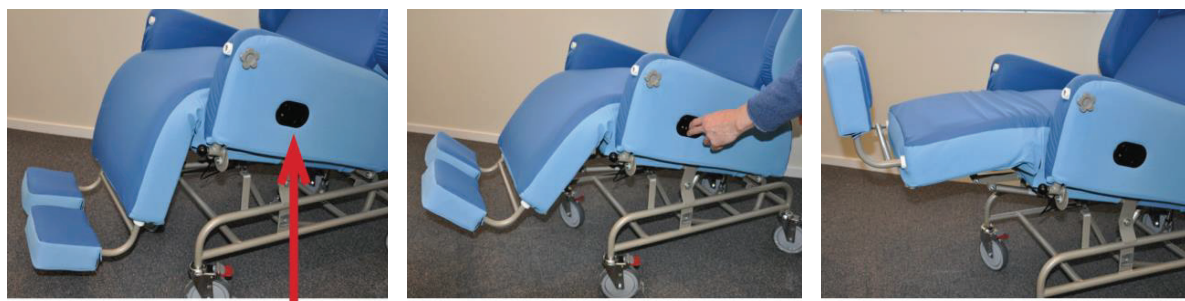

Trigger for leg rest elevation

## Head Support

Adjustment for head support

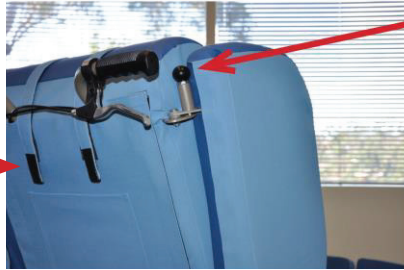

Knob to adjust lateral supports

## Lateral Support Positions



## Side Supports



Knob for lowering side supports



Side support lowered

## Air Cell



Adjustable air cell, located on right side of client seat. The air cells are inflated at the factory to a standard operating pressure. The level of air is adjustable by inflation and deflation via the air valves. Adjust the air for comfort, stability and clinical requirements.

## Foot Rest




Knob for foot rest adjustment



Foot rest extended

## Tray

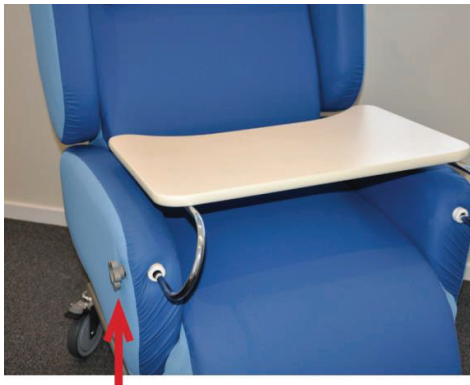

Knob for tray adjustment and locking



Storage position of tray

## Neck Support



## Castors



Locking Castor



Directional Castor (for steering)

## Intended Use

---

The intended use of the DAYS Comfort Chair is as a chair for elderly or special needs seating. The unit is intended for use by qualified health professionals within a clinical setting.

## Precautions and Warnings

---

- After moving the chair into position, ensure wheel lock is applied before transferring user into position.
- To avoid injury, keep patient and operator limbs clear of mechanisms / moving parts, especially when in motion.
- **WARNING:** Do not modify this equipment without authorisation of the Manufacturer.
- **WARNING:** Do not sit on the foot section; ensure user transfers on to the chair on the seat section with side support lowered.

## Warranty

---

A manufacturer's warranty of 10 years applies to the frame and 1 year on other parts when used in accordance with the user instructions and when the product has been properly maintained. Warranties do not extend to products damaged by any abuse, accident or improper use or normal wear and tear.

**UNDER NO CIRCUMSTANCES** - Shall PERFORMANCE HEALTH or their agents or dealers be liable in any manner whatsoever for any compensation or damages to any person occasioned by this chair for any loss, injury or any damage occasioned by or as a result of the misuse or abuse of this chair.

Any express or applied conditions, statements or warranty, statutory or otherwise is hereby excluded.

## Upholstery

---

The breathable polyurethane cover can be removed and washed in warm soapy water or wiped down using disinfectant wipes. It is recommended that all upholstery panels be cleaned weekly with warm soapy water and then wiped clean with a damp cloth to rinse off any soapy residue. This will preserve the Upholstery and increase its life span.

## Preventative Maintenance

---

It is recommended that a program of regular and appropriate Preventative Maintenance, be instituted for this equipment to confirm continued operator and patient safety.