



Etac Swift

Shower stool/chair

 **etac**[®]



Etac Swift

shower stool & shower chair

For the past 20 years, users around the world have relied on Etac Swift shower stools and chairs to help them feel safe and confident in the bathroom. Developed in cooperation with rehab professionals, industrial designers and users, Swift shower stools and chairs are designed to ensure wellbeing, support and comfort.

After 40 years of creating possibilities in the home healthcare and rehabilitation markets, we pride ourselves on a unique range of high-quality products. With its award-winning design and carefully crafted features, **the Swift Shower chair is a true Etac original!**



Designed for safe and sustainable use

A truly great product will not only perform well for the primary user in its intended environment. It should offer practical functionality throughout the product lifetime. The Swift shower stool/chair comes in a convenient flatpack for easy storage, shipment and carrying. The product itself is very easy to assemble without the need for any tools.

At Etac, products are designed with safety, hygiene and sustainability in mind. The Swift shower stool/chair is thoroughly tested in accordance with European and international standards and withstands rigorous use and reconditioning. Its estimated lifetime is 10 years and it comes with a 5-year warranty.

Discover the benefits

With sturdy height-adjustable legs and detachable back and armrests that gently click into place, the Swift shower stool/chair has been designed for safe use and easy adaptation.



Etac Swift shower stool



Etac Swift shower stool with armrests

Easily adapted

Detachable, modular back and armrests that gently click into place without the need for tools.

Sturdy grip

Without armrests, the side openings enable a stable grip and offer excellent support.

Non-slip surface

The surface structure ensures a safe, non-slip seat even when wet.

Height adjustable

Sturdy legs with a convenient built-in height adjustment.

Smart drainage


Specially-designed perforations that allow excess water to quickly drain away.

Flexible seat

Reversible seat with a generous recess.

Dependably stable

Responsive, non-slip ferrules that provide stability on any floor.

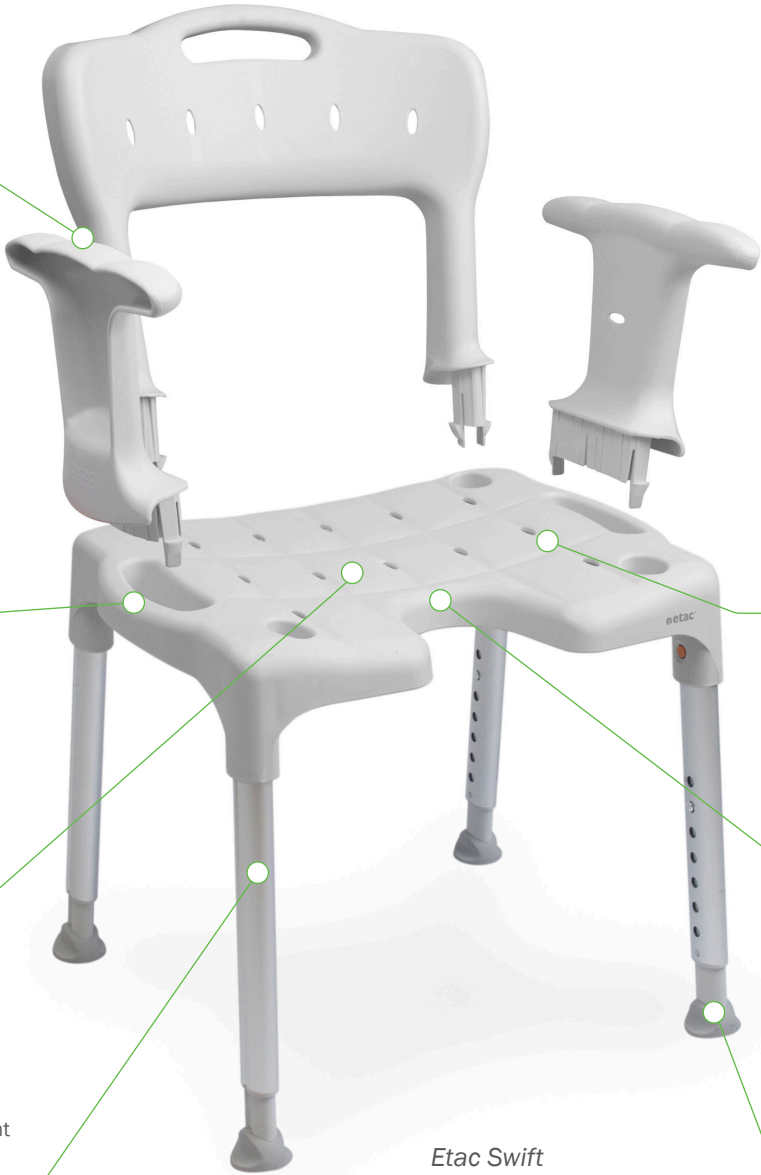

Etac Swift shower chair

Etac Swift shower stool/chair

A shower chair with many options.

One Swift - three options

- The basic version of Swift comes without armrests. The side openings enable a stable grip and provide excellent support.
- By adding armrests, you get extra support when sitting down and standing up.
- By adding both armrest and backrest, you get a Swift shower chair, for a comfortable and safe shower experience.



Two height options

Swift shower stool/chairs are available in two different sizes, both of which are fully adjustable. The lower model can be adjusted into four different height positions and the higher model can be adjusted into seven positions.


The height-adjustable function can also be used to tilt the Swift shower chair slightly forward. This is particularly useful for anyone who has difficulty getting up and down from a seated position.



Etac Swift

	Grey	Green	Blue
- shower stool	81701410	81701010	81701510
- shower chair with back and armrests	81701430	81701030	81701530
- shower stool Low	81701450		
- shower chair Low w. back and armrests	81701460		

Colours

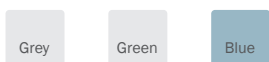

Dimensions

Total height with backrest: 80 - 95 cm (31½ - 37½")
 Total height Swift Low with backrest: 72.5 - 80 cm (28½ - 31½")
 Total width, stool: 54 cm (21¼")
 Total width, with armrests: 54.5 cm (21½")
 Total depth: 50 cm (19¾")
 Total depth Swift Low: 46.5 cm (18")
 Width between armrests: 45 cm (17¾")
 Seat height: 42 - 57 cm (16½ - 22½")
 Seat height Swift Low: 34.5 - 42 cm (13½ - 16½")
 Seat area: 54 x 41cm (21¼ x 16¼")

Weight

Swift shower stool: 3.1 kg (6.8 lbs)
 Swift Low shower stool: 2.9 kg (6.4 lbs)
 Swift shower chair: 4.7 kg (10.4 lbs)
 Swift Low shower chair: 4.5 kg (8.8 lbs)

Materials

Seat: Polypropylene
 Legs: Aluminium, polyamide
 Ferrules: TPE
 Seat pads: Polyethylene


Maintenance



Clean / disinfect the product using a standard non-abrasive cleaning agent with a pH between 5 and 9, or with a 70% disinfectant solution. The product can be safely cleaned in a cabinet washer-disinfector at 85 °C for 3 minutes.



Features

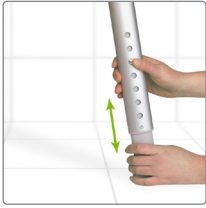

Height adjustable

It is easy to adjust the height. Just click and press.




Dependably stable

Responsive, non-slip ferrules adapt to the floor and provide stability on the most uneven of surfaces.



Easily adapted

Modular design with detachable back and armrests that gently click into place without the need for tools. Easy to customise for different user needs.

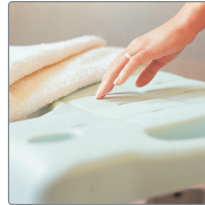

Ease of access

A reversible seat with a generous recess and a smooth surface provides good access and comfort.





Accessories



Backrest

Detachable, modular backrest that gently clicks into place without the need for tools.

Grey84006181
 Green 84006182
 Blue..... 84006183



Pair of armrests

Detachable, modular armrests that gently click into place without the need for tools.

Grey 84006184
 Green 84006185
 Blue..... 84006186



Soap dish

Attaches to the side of the seat. Perfect place to store shower gel or shampoo. Also has a built-in shower holder.

Grey 80209266
 Green81701140
 Blue81701340



Seat pad

The non-slip pads provide an instant warm feeling. Sitting is more comfortable and not as slippery. Easy to wipe down and simple to click into place.

Grey 84005073



Backrest pad

The non-slip pads provide an instant warm feeling. Sitting is more comfortable and not as slippery. Easy to wipe down and simple to click into place.

Grey 84005072



Lightweight and portable, the Swift shower stool /chair comes in a convenient flatpack carrycase.



Etac is a world-leading developer and provider of ergonomic assistive devices and patient handling equipment. Our heart lies in the solutions that optimise quality of life for the individual, their family and caregivers.

For the latest news and continuously updated product information – visit: www.etac.com

Etac AB
Kista Science Tower
SE-164 51 Kista, Sweden
Tel +46 371 - 58 73 30
info@etac.se www.etac.com


Creating Possibilities