

Limbguard™

Pat # 6679193


Patented protection after below knee amputation.

SIZING INFORMATION



Small	Medium	Large	X-Large
12.5" - 16.5"	14" - 18"	15.5" - 19.5"	19.5" - 22.5"

- 1) Take the distal measurement to determine the size that is needed for fitting. The Limbguard can be cut and customized to the patient on site.
- 2) The Limbguard is fit directly over the surgical dressing, or with a **Limbsock™**. Wrap the neoprene sleeve securely on limb proximally and distally. The Limbguard should accommodate the residuum via the floating anterior shell and the gills on the posterior aspect.
- 3) After initial fitting, the Limbguard may be removed at any time for inspection of the limb. Multiple layers of the **Limbsock™** are used to accommodate volume changes (edema-related or variability in the dressing thickness).

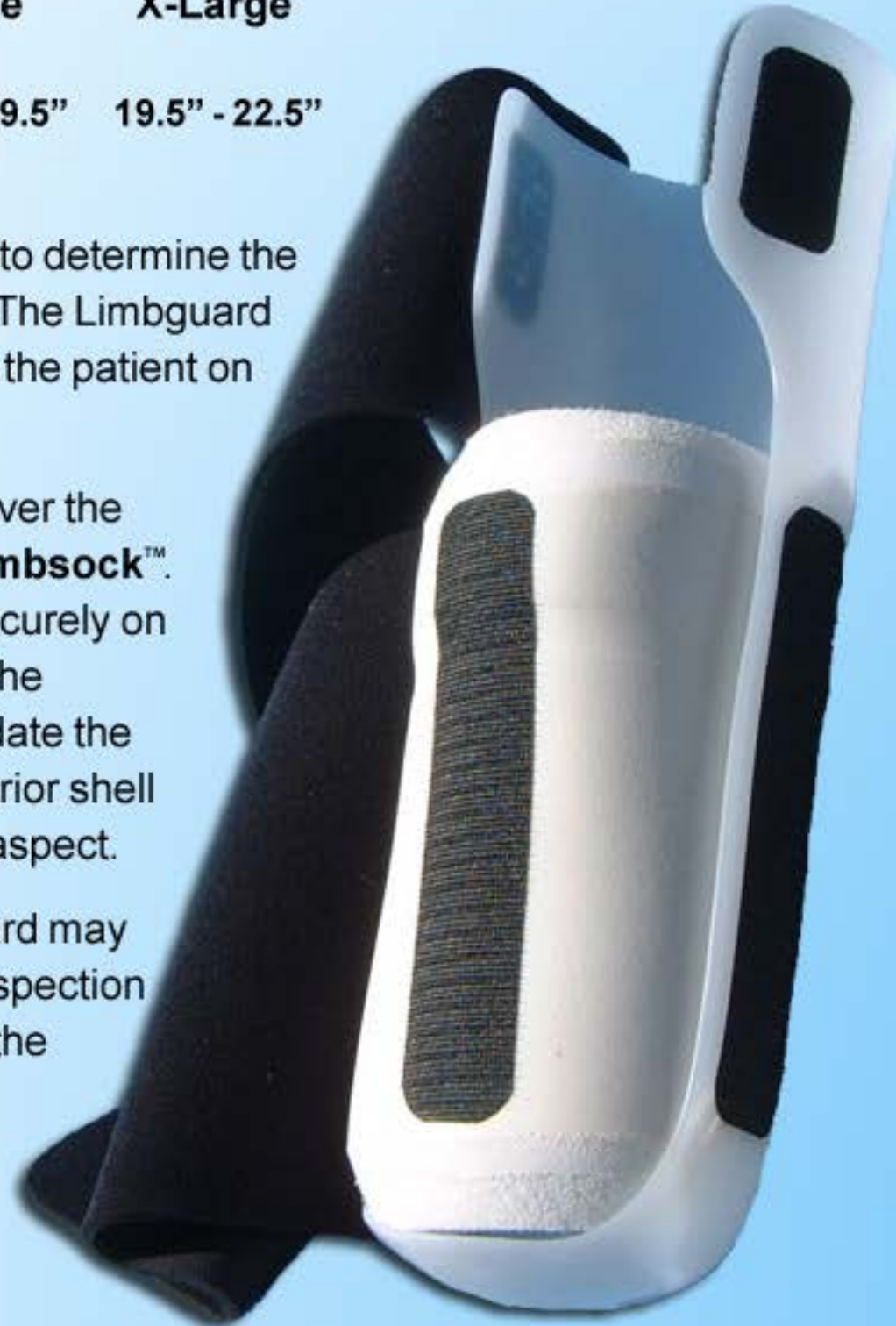

OPTIONS:

Limbsock™: universal size, multi-ply sock for skin / dressing protection.

Shrinker socks: Once the Limbguard has been discontinued, a shrinker should be applied as indicated.

Limb Suspension device and Waist Belt: May be applied to the Limbguard to aid in suspension as indicated.



Limbsock™