




ANIMAL ORTHO CARE

Embrace

TOE-UP SUPPORT

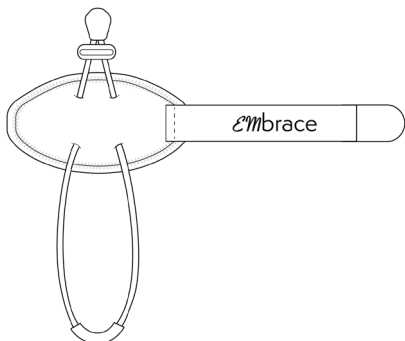

INSTRUCTIONS FOR USE

BEFORE USING THIS SUPPORT, PLEASE READ THE ENTIRE INSTRUCTIONS FOR USE. CORRECT APPLICATION IS VITAL TO THE PROPER USE OF THIS SUPPORT.

INTENDED USE

This support provides a solution to help reduce knuckling of the rear paws that occur as a result of neurological issues. It provides the stimulation of proprioceptive nerves. Knuckling is described as dragging or walking on the tops of the paws.

WARNINGS AND PRECAUTIONS

1. DO NOT leave your pet unattended while wearing this support.
2. Consult your veterinarian for recommended wear time.
3. It is recommended the support be worn during periods of activity. Remove the support when the dog is resting.

WARRANTY

Our warranty covers all manufacturing defects in materials and workmanship for a period of six months. This warranty does not cover support failure due to normal wear and tear or improper use.

ADDRESS:

Caerus Corp
1251 Red Fox Road
Arden Hills, MN 55112, USA

C009-469 Rev A
Caerus Corp © 2019

EMAIL:

care@aocpet.com

WEBSITE:


www.aocpet.com

PHONE:


651-440-9321

FITTING INSTRUCTIONS

1. Unhook the Velcro® strap. Fully loosen elastic the cord using the cord stop.

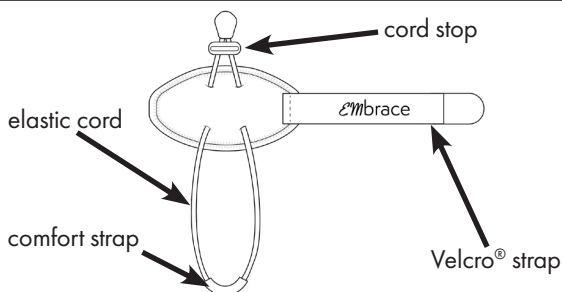

2. Place the elastic cord around the middle two toes, ensuring that the comfort pad runs underneath and through the toes.
3. Wrap the support around the leg directly above the hock joint and secure in place using the Velcro® strap.

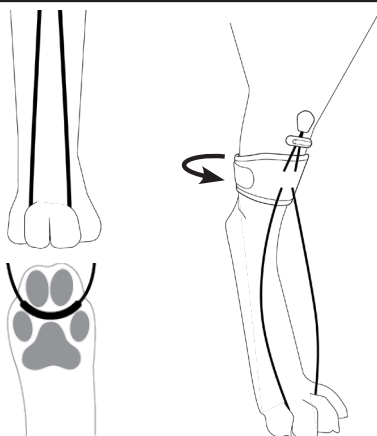

4. Cinch the elastic cord with the cord stop until taut.
5. Check to make sure the support is snug enough so that it does not slip below the hock. If a pen fits between the support and the leg, it is not tight enough.
6. Verify the elastic cord is cinched tight enough to prevent knuckling. If the foot is still dragging upon use, tighten a bit more. Tighten in small increments to avoid over-tightening.



WEAR & USAGE

1. Do not over-tighten the cord or Velcro® strap. This can cause irritation and limit mobility.
2. DO NOT leave your pet unattended while wearing this support.
3. This support is not meant for continuous long-term wear. Please consult your veterinarian for recommended wear time.
4. Machine wash cold, gentle cycle. Lay flat to dry.