

RECOMMENDED USAGE

This device uses Pulsed Electromagnetic Field (PEMF) to promote your pet's natural healing processes. PEMF technology is subsensory, meaning the therapy cannot be felt against the skin. For optimal results, we recommend completing one two-hour treatment every day for at least two weeks. Consult your veterinarian to determine if your pet's individual treatment plan will differ. While it is most common to see improvements after two weeks, some pet owners may see results right away. You may notice improvements in mood, activity, behavior, and ease of movement. After seeing improvements with your pets, we recommend using the EMBED Drift System on an as-needed basis.



ANIMAL ORTHO CARE

Animal Ortho Care
1251 Red Fox Road
Arden Hills, MN 55112, USA

Email: care@aocpet.com

Website: aocpet.com

CARE & USE

EMpower Device(s) & Cap(s)

Dampen a cloth with isopropyl alcohol and wipe clean.

Do NOT wash the EMpower Device(s) or immerse in liquid.

Microvelvet Cover & Mattress Cover

Machine washable, cold on gentle wipeable with warm soapy water.

Tumble dry on low.

Do NOT iron.


Do NOT wash the mattress inserts.

WARRANTY

Our warranty covers all manufacturing defects in materials and workmanship for a period of six months. This warranty does not cover device failure due to normal wear or improper use.

WARNINGS AND PRECAUTIONS

Do NOT allow the EMpower Device to get wet. Do NOT allow your dog to chew on the EMpower Device. The EMpower Device can be fatal if swallowed. Seek emergency medical attention if this occurs.

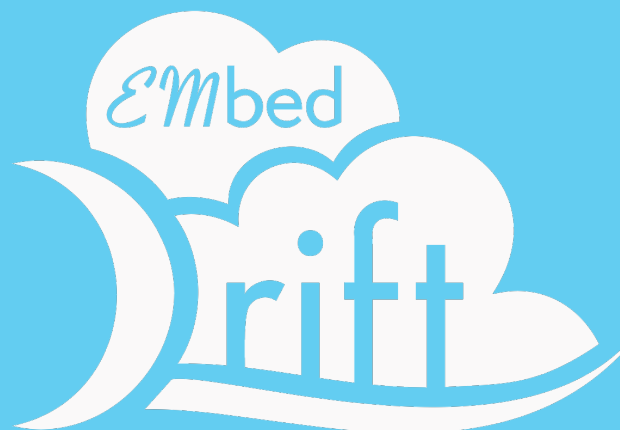

INSTRUCTIONS FOR USE

Before using this product, please read the entire instructions for use.

INTENDED USE

The EMBED Drift System is used for the treatment of chronic and acute pain and inflammation associated with joint and soft tissue trauma, arthritis, and Intervertebral Disc Disease (IVDD).

TECHNICAL SPECIFICATIONS

Output Signal

27.12MHz

Power

3.7v DC lithium-ion rechargeable battery

Compliance


IEC 60601-1

IEC 60601-2

PACKAGE COMPONENTS

QUANTITY

SM	LG	ID	DESCRIPTION
1	1	A	EMbed Drift Pet Bed Cover
1	1	B	EMbed Drift Pet Bed Mattress
1	2	C	EMpower Device Cap(s)
1	2	D	EMpower Device(s)
-	-	E	Power Button
-	-	F	Charging Port
-	-	G	Indicator Light
1	2	-	AC Charger(s) (not shown)

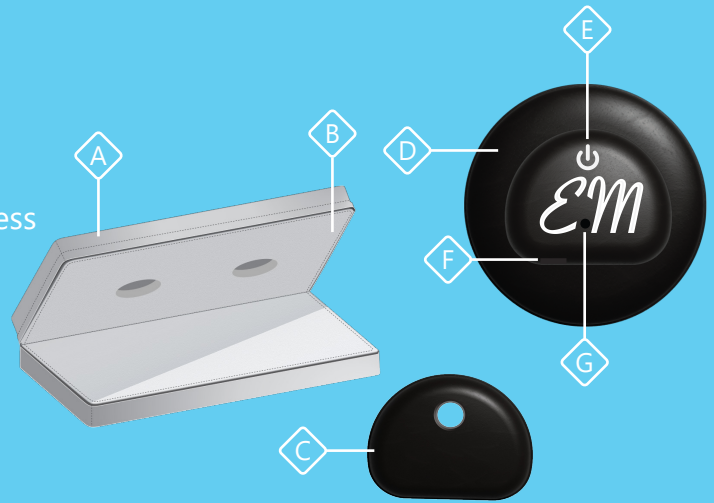

CHARGING

1. Plug the USB connector of the AC adapter into the USB port on the EMpower Device(s).
2. Plug the AC adapter into an electrical outlet.
3. A red light on the EMpower Device(s) will indicate that the Device(s) is charging. (H)
4. A full charge takes about 1.5 hours.
5. When the Device(s) is fully charged, the red light will turn off.



OPERATING INSTRUCTIONS

1. Before using, please charge the EMpower Device(s). See charging instructions. NOTE: Once fully charged, the Device(s) are good for **at least** two treatments.
2. Unzip the EMbed Drift bed cover. (I)
3. Open the mattress and lay it flat. (J)
4. Affix the EMpower Device(s) by placing one into each hole on the mattress. Ensure the hook and loop attachments are secure. (K)
5. Press the power button to turn on the EMpower Device(s). A blue light will indicate that the Device(s) is on (L). Place the Device Cap(s) over the power button(s). (M)
6. Fold the mattress closed. Zip the cover closed. (N)
7. Place or guide your pet onto the pet bed and encourage them to rest there for the entire 2-hour treatment.

