



ANIMAL ORTHO CARE

RECOMMENDED USAGE

EM technology is subsensory, meaning the therapy cannot be felt against the skin. We recommend you consult with your veterinarian to determine the best treatment plan. For optimal results, we recommend completing one two-hour treatment every day for at least two weeks.

It is most common to see improvements after two weeks, however some pet owners may see results right away. You may notice improvements in mood, increased levels of activity, changes in behavior, and ease of movement. After seeing improvements with your pet, we recommend using the EMpower device on an as-needed basis.

CHARGING

1. Plug the USB connector of the AC charger into the USB port on the EMpower device.
2. Plug the AC charger into an electrical outlet.
3. A red light on the EMpower device will indicate that the device is charging.
4. A full charge takes about 1.5 hours.
5. When the device is fully charged, the red light will turn off.

WARRANTY

Our warranty covers all manufacturing defects in materials and workmanship for a period of six months. This warranty does not cover device failure due to normal wear and tear or improper use.

TECHNICAL SPECIFICATIONS

OUTPUT SIGNAL

27.12 MHz

POWER

3.7v DC lithium ion rechargeable battery

COMPLIANCE

IEC 60601-1

IEC 60601-1-2


MANUFACTURED BY:
Animal Ortho Care

EMAIL:
care@aocpet.com

WEBSITE:
www.aocpet.com

PHONE:
651-440-9321

EMpower device

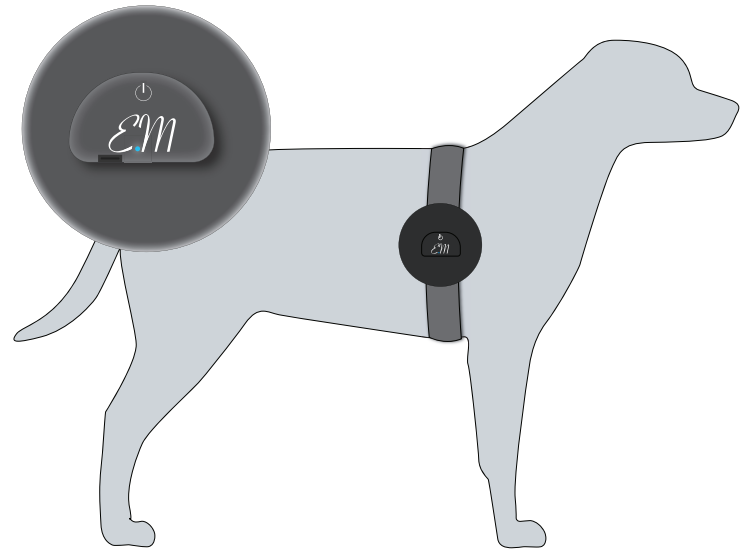

INSTRUCTIONS FOR USE

BEFORE USING THIS DEVICE, PLEASE READ THE ENTIRE INSTRUCTIONS FOR USE. CORRECT APPLICATION IS VITAL TO THE PROPER USE OF THIS DEVICE.

INTENDED USE


The EMpower device is used on animals for the treatment of chronic and acute pain and inflammation associated with joints and soft tissue trauma, arthritis, hip dysplasia and Intervertebral Disc Disease (IVDD).

WARNINGS AND PRECAUTIONS


1. **DO NOT** allow device to get wet.
2. **DO NOT** leave dog unattended while wearing device.
3. **DO NOT** allow your dog to chew on device. The EMpower device can be fatal if swallowed. Seek emergency medical assistance if this occurs.

COMPONENTS


| QTY | ID | DESCRIPTION |
|-----|----|------------------------|
| 1 | A | EMpower Device |
| | B | Power Button |
| | C | Indicator Light |
| | D | Charging Port |
| 1 | E | AC Charger (not shown) |
| 1 | F | Long Strap |
| 1 | G | Short Strap |
| 1 | H | Strap Connector Pad |
| 1 | I | Buckle Cover |

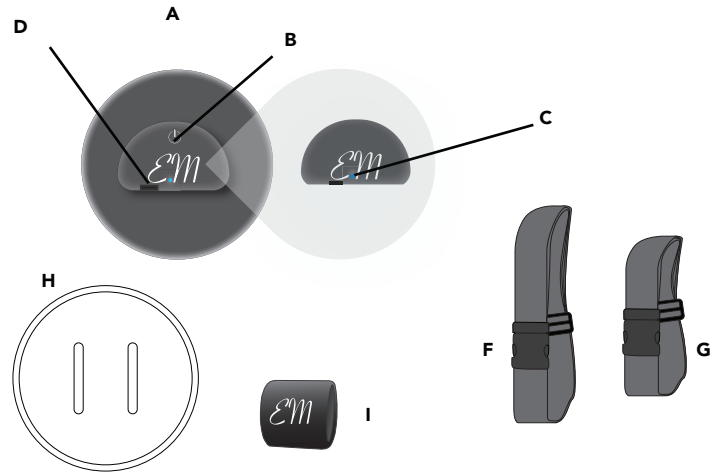


INSTRUCTIONS

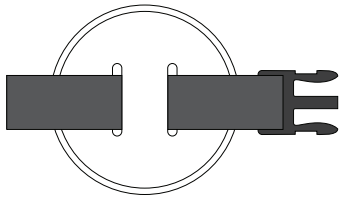


1. Open the strap and run it through the slits of the strap connector pad. Use smaller or larger strap based on the treatment area and the size of your dog.

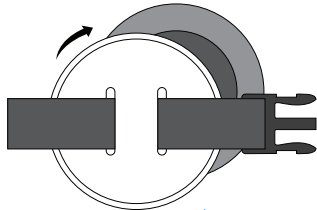


2. Attach strap connector pad to the EMpower Device via hook and loop.

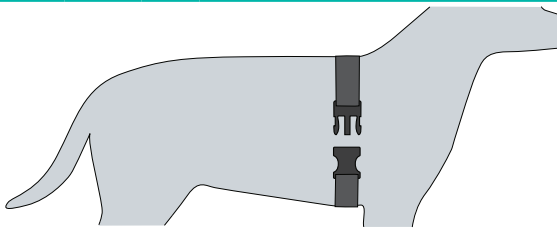

3. Wrap elastic strap around pet, making sure the silicone grip is facing towards the body. Use the adjuster to tighten or loosen the adjustment of the strap.

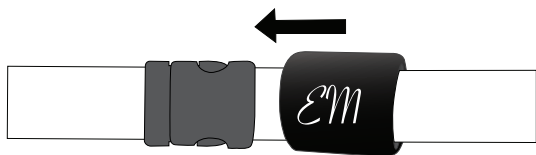

4. Align the Buckle Cover over the buckle.

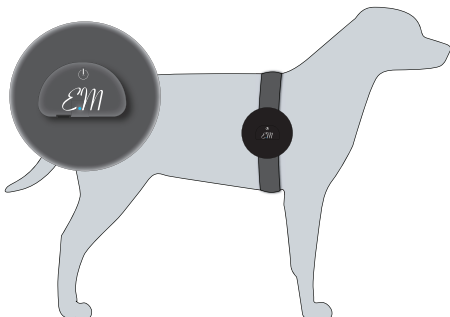

5. Press the Power Button to begin treatment. The LED will turn on, indicating treatment has started. The treatment will automatically complete after two hours, and the LED will turn off.