



ANIMAL ORTHO CARE

# EMbed

## PAIN RELIEF SYSTEM

### INSTRUCTIONS FOR USE

**BEFORE USING THIS SUPPORT, PLEASE READ THE ENTIRE INSTRUCTIONS FOR USE. CORRECT APPLICATION IS VITAL TO THE PROPER USE OF THIS SUPPORT.**

#### INTENDED USE

The EMpower Device, when used with the EMbed is used for the treatment of chronic and acute pain and inflammation associated with joint and soft tissue trauma, arthritis, and Intervertebral Disc Disease (IVDD).

#### WARNINGS AND PRECAUTIONS

Advice for wearing is advised from a veterinary care provider. Complete wear and care instructions are found on our website at: [www.aocpet.com](http://www.aocpet.com).

#### WARRANTY

Our warranty covers all manufacturing defects in materials and workmanship for a period of one year. This warranty does not cover support failure due to normal wear and tear or improper use.

**ADDRESS:**

Caerus Corp  
1251 Red Fox Road  
Arden Hills, MN 55112, USA

**EMAIL:**

[care@aocpet.com](mailto:care@aocpet.com)

**WEBSITE:**

[www.aocpet.com](http://www.aocpet.com)

**PHONE:**

651-440-9321

**FAX:**

651-207-8896


## FITTING INSTRUCTIONS

1. Before using, please review the instructions included with the EMpower Device.
2. With the dog in a laying position, drape the blanket over the body to fully covering the desired treatment area.



3. Attach the EMpower Device to the blanket directly above the desired treatment site. The treatment site will vary depending on your dog's condition.
4. Turn the EMpower Device and leave on for the full duration of the recommended two-hour treatment.

NOTE: EMpower Device may be attached to the underside of the EMBED and placed in a dog bed or area where the dog rests or sleeps.

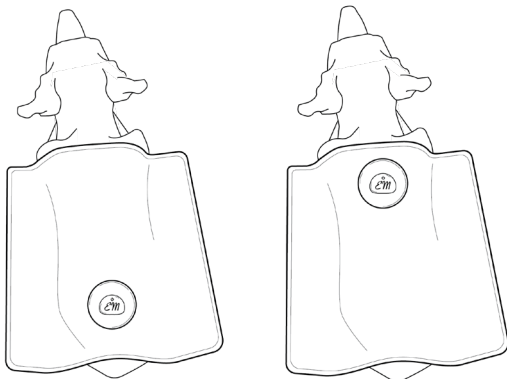

## WEAR & USAGE

5. DO NOT leave your pet unattended while using the device.
6. Machine wash cold, on gentle, and lay flat to dry.