




ANIMAL ORTHO CARE

Embrace

LIFT SUPPORT

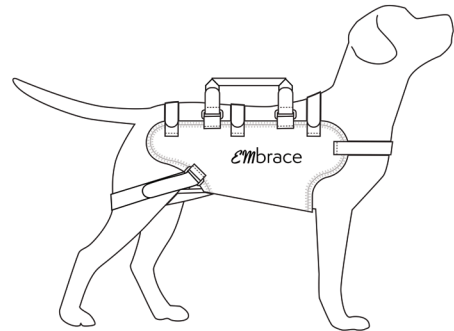

INSTRUCTIONS FOR USE

BEFORE USING THIS SUPPORT, PLEASE READ THE ENTIRE INSTRUCTIONS FOR USE. CORRECT APPLICATION IS VITAL TO THE PROPER USE OF THIS SUPPORT.

INTENDED USE

Stabilization of back (via compression), carrying, partial load bearing, i.e. helping up stairs, over curbs, into vehicle, etc.

WARNINGS AND PRECAUTIONS

1. DO NOT leave your pet unattended while wearing this support, when metal stays are inserted.
2. Consult your veterinarian for recommended wear time.
3. If treating an acute injury, avoid strenuous exercise, including running and jumping.
4. It is recommended the support be worn during periods of activity. Remove the support when the dog is resting.
5. Do not carry dog for extended periods of time. If dog shows signs of irritation, stop carrying.

WARRANTY

Our warranty covers all manufacturing defects in materials and workmanship for a period of six months. This warranty does not cover support failure due to normal wear and tear or improper use.

ADDRESS:

Caerus Corp
1251 Red Fox Road
Arden Hills, MN 55112, USA

C009-930 Rev A
Caerus Corp © 2020

EMAIL:

care@aocpet.com


WEBSITE:

www.aocpet.com

PHONE:

651-440-9321


Consult your veterinarian for recommended wear time.

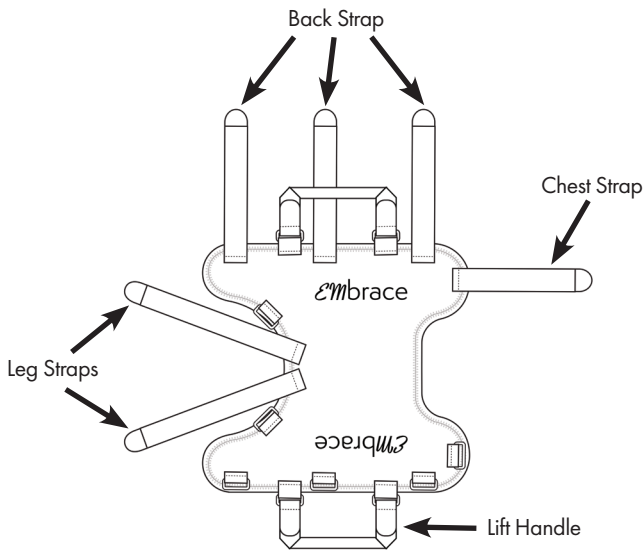

WEAR & USAGE

- Break in period consists of wearing a half hour on/off the first day, increasing daily as tolerance improves.
- Please consult your veterinarian for recommended wear time.
- **DO NOT** leave your pet unattended while wearing this support, when metal stays are inserted.
- **DO NOT** over-tighten Velcro® straps. This can cause irritation and limit mobility.
- Machine wash cold, gentle cycle, and lay flat to dry.
- All supports are compatible for use with our EMpower Device.

FITTING INSTRUCTIONS

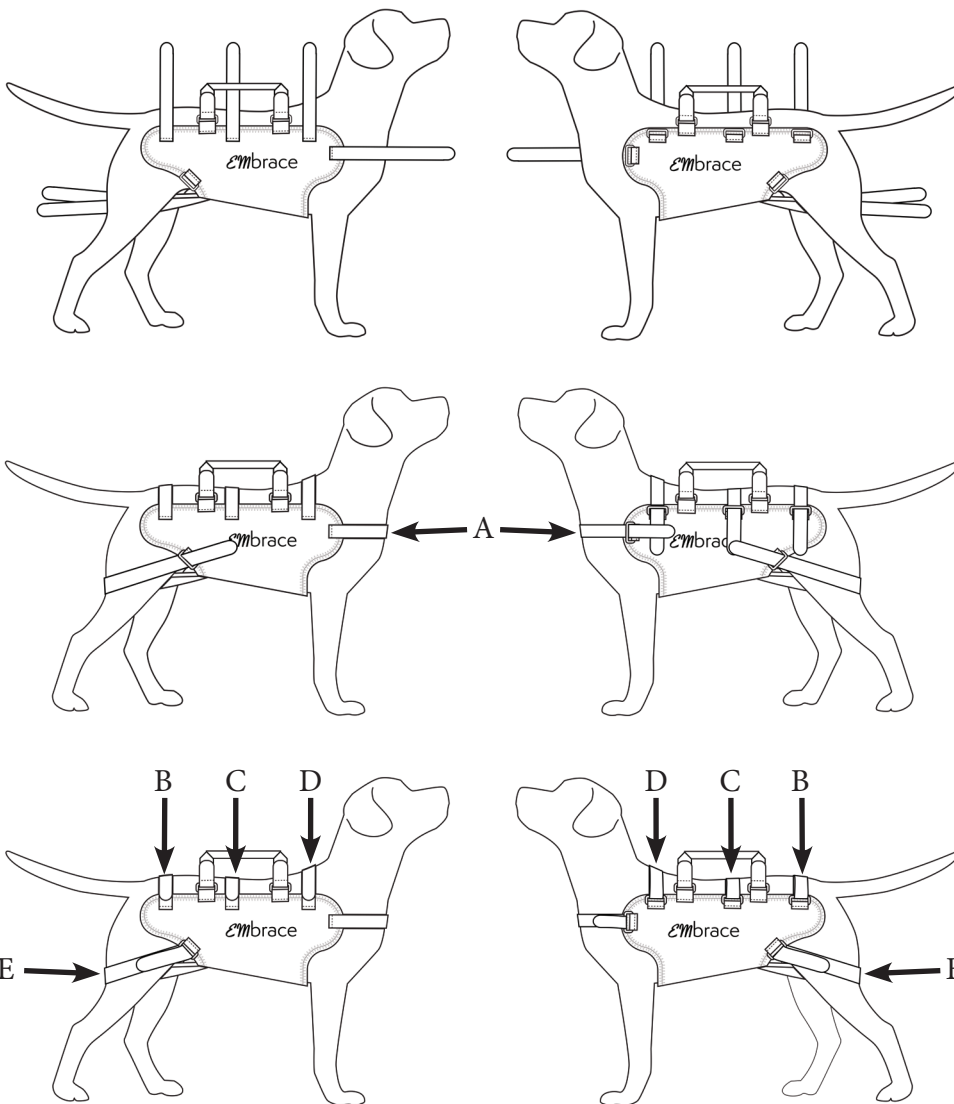

1. Unhook all Velcro® straps to open the support
2. Place the support underneath the dog, bringing front side up against the armpits, and secure the chest strap (A).
3. Wrap the three top Velcro® straps over the back and secure, starting from the shoulders and moving to the tail (B-D).
4. Wrap the back leg straps behind and around the legs and through the plastic loop to secure (E). These should be very snug for carrying purposes, as they support the dog's rear end.
5. Ensure the support is snug enough to not slip into misalignment or allow any slipping while carrying.
6. Check regularly for areas of irritation. If there are areas of irritation, adjust the support for a better fit.

CARE

- Hand wash only, lay flat to dry.
- If signs of wear and tear appear on the support during the warranty period, discontinue use and contact us at: care@aocpet.com