


FITTING INSTRUCTIONS

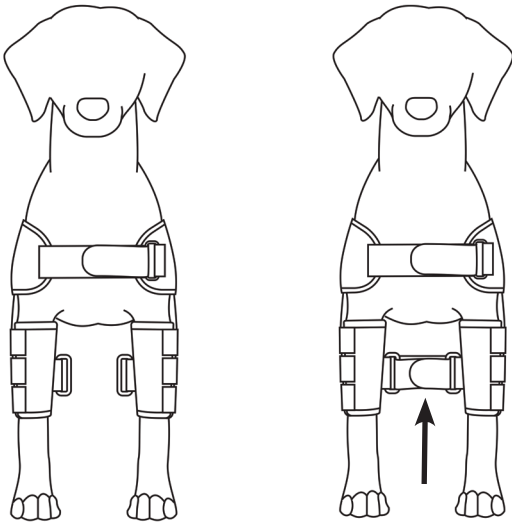

1. If desired, the hobble strap can be wrapped through the plastic loops on the inside of each leg wrap to limit range of motion of the front legs.
2. Ensure the support is snug enough to not slip into misalignment with the joint. If a pen will fit between the leg and the support, it is not tight enough.
3. Check regularly for areas of irritation. If there are areas of irritation, adjust the support for a better fit or reduce wear time.

CARE


- For spot cleaning, use a damp cloth to wipe the support.
- If the support gets soiled, machine wash cold on gentle.
- After washing the shoulder support, lay flat to air dry. Allow to dry thoroughly before reuse.
- If signs of manufacturing defects or malfunction appear on the support during the warranty period, discontinue use and contact us at care@aocpet.com



ANIMAL ORTHO CARE

EMbrace

SHOULDER SUPPORT

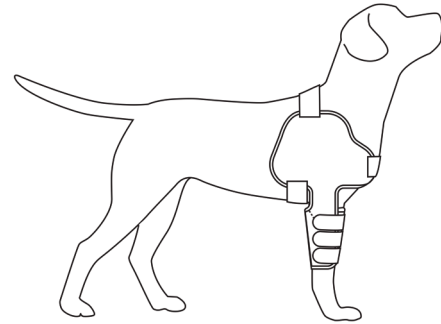

INSTRUCTIONS FOR USE

BEFORE USING THIS SUPPORT, PLEASE READ THE ENTIRE INSTRUCTIONS FOR USE. CORRECT APPLICATION IS VITAL TO THE PROPER USE OF THIS SUPPORT.

INTENDED USE

The EMbrace Shoulder Support is used for the stabilization of shoulder (via compression), protection of the front legs from hygroma or licking, front leg range of motion control.

WARNINGS AND PRECAUTIONS

1. DO NOT leave your pet unattended while wearing this support.
2. Consult your veterinarian for recommended wear time.
3. If treating an acute injury, avoid strenuous exercise, including running and jumping.
4. It is recommended the support be worn during periods of activity. Remove the support when the dog is resting.

WARRANTY

Our warranty covers all manufacturing defects in materials and workmanship for a period of six months. This warranty does not cover support failure due to normal wear and tear or improper use.

ADDRESS:
Caerus Corp
1251 Red Fox Road
Arden Hills, MN 55112, USA


C009-931 Rev A
Caerus Corp © 2020

EMAIL:
care@aocpet.com

WEBSITE:
www.aocpet.com

PHONE:
651-440-9321


Consult your veterinarian for recommended wear time.

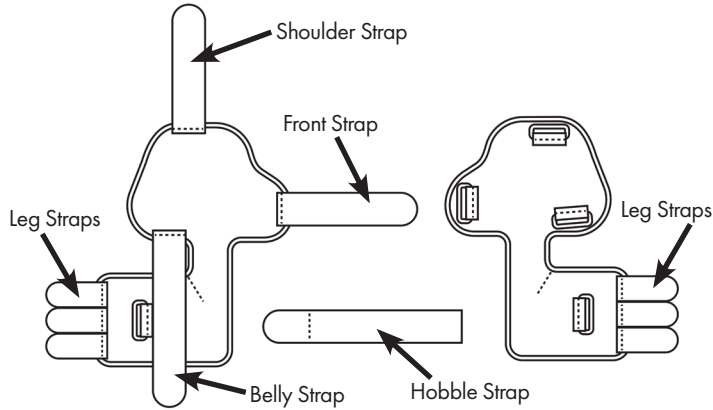


WEAR & USAGE

- Break in period consists of wearing a half hour on/off the first day, increasing daily as tolerance improves.
- Please consult your veterinarian for recommended wear time.
- DO NOT leave your pet unattended while wearing this support.
- DO NOT over-tighten Velcro® straps. This can cause irritation and limit mobility.
- All supports are compatible for use with our EMpower Device.


FITTING INSTRUCTIONS

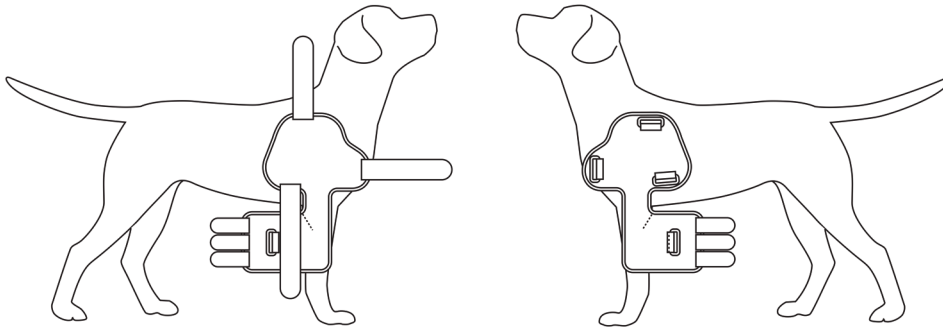


1. Unhook the Velcro® straps to open the support.
2. Align both sides of the shoulder plate over both of the dog's shoulders.

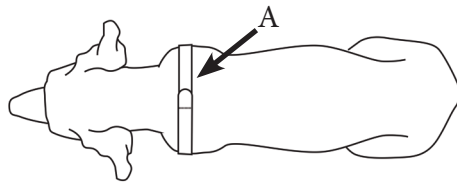

3. Connect the top Velcro® strap over the shoulders, allowing the device to drape over the shoulders.

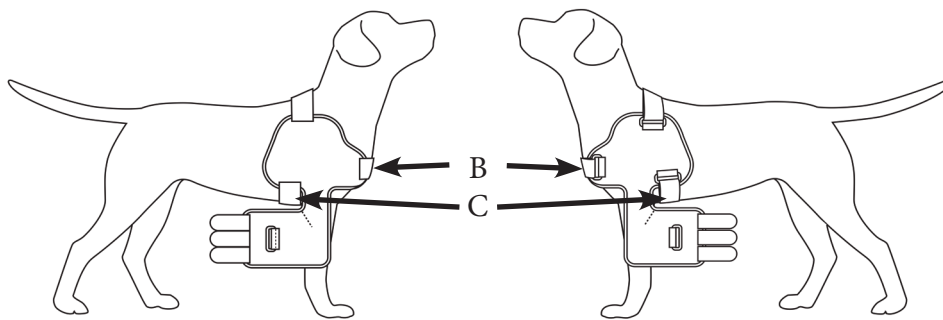

4. Connect the front strap around the chest. Then, connect the bottom strap under the chest, ensuring no rubbing at the armpits.
5. Once secured around the dog, the shoulder plate straps can be adjusted to optimize location of the device over both shoulders.

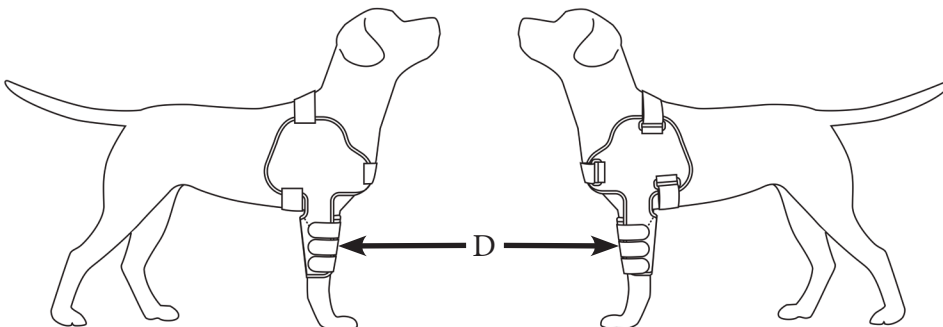

6. Wrap each leg wrap around the top of the leg, ensuring the elbow seam and notch align with the bend in the elbow.
7. Secure each leg wrap around the leg. Note that once the leg wraps are secure for the first wear, the device can be slid on and off the legs for ease of use.

CONTINUED ON REVERSE...